

ULTRA 16 XRAY VERIFICATION FET MODE SELECTIONS - 16 & 8 SLICE MODE

A12 D12	A11 D11	A10 D10	A9 D9	A8 D8	A7 D7	A6 D6	A5 D5	A4 D4	A3 D3	A2 D2	A1 D1	B1 D1	B2 D2	B3 D3	B4 D4	B5 D5	B6 D6	B7 D7	B8 D8	B9 D9	B10 D10	B11 D11	B12 D12	Slice mode	FET mode	FET LED	FET 1 LED	FET 2 LED	FET 3 LED	FET 4 LED	FET 5 LED	FET 6 LED			
									3	2	1	1	2	3										CAL 1	4	ON	ON	OFF	ON	ON	ON	ON	ON		
									3	2	1	1	2	3											CAL 1	4	ON	ON	OFF	ON	ON	ON	ON	ON	
						4	3	2	1				1	2	3	4									CAL 2	40	ON	ON	ON	OFF	ON	ON	OFF	ON	
						4	3	2	1				1	2	3	4									CAL 2	40	ON	ON	ON	OFF	ON	ON	OFF	ON	
					4	3	2	1						1	2	3	4								CAL 3	41	OFF	ON	ON	OFF	ON	ON	OFF	ON	
					4	3	2	1						1	2	3	4								CAL 3	41	OFF	ON	ON	OFF	ON	ON	OFF	ON	
					4	3	2	1							1	2	3	4							CAL 4	42	ON	OFF	ON	OFF	ON	OFF	ON	OFF	
					4	3	2	1							1	2	3	4							CAL 4	42	ON	OFF	ON	OFF	ON	OFF	ON	OFF	
				4	3	2	1								1	2	3	4							CAL 5	43	OFF	OFF	ON	OFF	ON	OFF	ON	OFF	
				4	3	2	1								1	2	3	4							CAL 5	43	OFF	OFF	ON	OFF	ON	OFF	ON	OFF	
			4	3	2	1										1	2	3	4						CAL 6	44	ON	ON	OFF	OFF	ON	OFF	ON	OFF	
			4	3	2	1										1	2	3	4						CAL 6	44	ON	ON	OFF	OFF	ON	OFF	ON	OFF	
		4	3	2	1											1	2	3	4						CAL 7	45	OFF	ON	OFF	OFF	OFF	ON	OFF	ON	
		4	3	2	1											1	2	3	4						CAL 7	45	OFF	ON	OFF	OFF	OFF	ON	OFF	ON	
	4	3	2	1													1	2	3	4					CAL 8	46	ON	OFF	OFF	OFF	OFF	ON	OFF	ON	
	4	3	2	1													1	2	3	4					CAL 8	46	ON	OFF	OFF	OFF	OFF	ON	OFF	ON	
4	3	2	1																	1	2	3	4		CAL 9	47	OFF	OFF	OFF	OFF	OFF	ON	OFF	ON	
4	3	2	1																	1	2	3	4		CAL 9	47	OFF	OFF	OFF	OFF	OFF	ON	OFF	ON	
3	2	1									4	4												1	2	3	CAL 10	48	ON	ON	ON	ON	OFF	ON	OFF
3	2	1									4	4												1	2	3	CAL 10	48	ON	ON	ON	ON	OFF	ON	OFF
2	1									4	3	3	4											1	2	CAL 11	49	OFF	ON	ON	ON	ON	OFF	ON	OFF
2	1									4	3	3	4											1	2	CAL 11	49	OFF	ON	ON	ON	ON	OFF	ON	OFF
1									4	3	2	2	3	4										1	CAL 12	50	ON	OFF	ON	ON	ON	OFF	ON	OFF	
1									4	3	2	2	3	4										1	CAL 12	50	ON	OFF	ON	ON	ON	OFF	ON	OFF	
			2	1	1														1	1	2				TRK 3	39	OFF	OFF	OFF	ON	ON	ON	OFF	ON	OFF
			2	1	1														1	1	2				TRK 3	39	OFF	OFF	OFF	ON	ON	ON	OFF	ON	OFF