GE Healthcare

BrightSpeed and BrightSpeed Select Series 5 Inch MOD Drive Option Installation Service Documentation

5162230

Rev 2

© 2005-2007, General Electric Company,
All Rights Reserved.

LEGAL NOTES

TRADEMARKS

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

COPYRIGHTS

All Material, Copyright © 2005-2007 by General Electric Company, Inc. All rights reserved.

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WARNING

(EN)

- THIS SERVICE MANUAL IS AVAILABLE IN ENGLISH ONLY.
- IF A CUSTOMER'S SERVICE PROVIDER REQUIRES A LANGUAGE OTHER THAN ENGLISH, IT IS THE CUSTOMER'S RESPONSIBILITY TO PROVIDE TRANSLATION SERVICES.
- DO NOT ATTEMPT TO SERVICE THE EQUIPMENT UNLESS THIS SERVICE MANUAL HAS BEEN CONSULTED AND IS UNDERSTOOD.
- FAILURE TO HEED THIS WARNING MAY RESULT IN INJURY TO THE SERVICE PROVIDER, OPERATOR OR PATIENT FROM ELECTRIC SHOCK, MECHANICAL OR OTHER HAZARDS.

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DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent write "Damage In Shipment" on ALL copies of the freight or express bill BEFORE delivery is accepted or "signed for" by a GE representative or hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

Call Traffic and Transportation, Milwaukee, WI (262) 785 5052 or 8*323 5052 immediately after damage is found. At this time be ready to supply name of carrier, delivery date, consignee name, freight or express bill number, item damaged and extent of damage.

Complete instructions regarding claim procedure are found in Section S of the Policy And Procedures Bulletins.

14 July 1993

CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

LITHIUM BATTERY CAUTIONARY STATEMENTS

CAUTION
Risk of
Explosion

Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.

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OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the GEMS CQA Process to report all omissions, errors, and defects in this publication.

Revision History

Revision	Date	Reason for change
1	11/20/05	Initial release.
2	6/4/07	Chapter 2: updated installation procedure for Software 07BW08.x and 07MW11.x

Table of Contents

Chapter 1	
Introduction	19
Section 1.0	
Flowchart	19
Section 2.0	
Material List	20
Chapter 2	
Procedures	21
Section 1.0 Preparation	21
Section 2.0 MOD Drive Jumper Setting	21
Section 3.0 MOD Drive Installation	22
Section 4.0 Image Archive/Restore Test	25
Chapter 3	
Final Check	27
Section 1.0	
Attaching The Rating Plate	27

Chapter 1


Introduction

This option enables BrightSpeed Series to install the MOD on the systems to increase a storage capacity. This manual describes the installation procedures.

Section 1.0

Flowchart

The Illustration below shows the main flow of the installation procedures of these options.

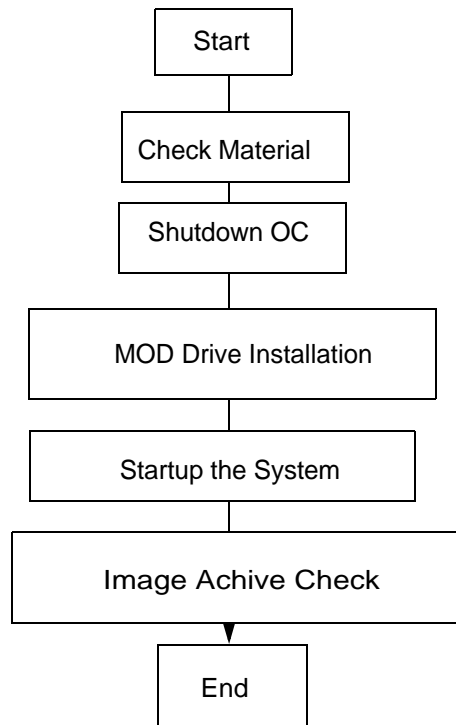Section 2.0 Material List

5 Inch MOD Drive Option Kit (5161555)

ITEM	PART NO.	DESCRIPTION	QTY.
1	2318598	SONY MOD SMO-F551-SD	1
2	2214164	5.25~ 2.3GB 512 BYTES/SECTOR MOD	1
3	5162230	5 inch MOD Drive Option Installation Manual	1
4	N9306HR	SCREW TP 3X6 FE ZN	4
5	5159518	Rating plate of 5 inch MOD	1


Chapter 2 Procedures

Section 1.0 Preparation

Verify that the contents of this option kit are correct according to the Material List (see Chapter1, Section 2).


Section 2.0 MOD Drive Jumper Setting

Check the MOD Assy jumper configuration. GE specific settings are shown as below.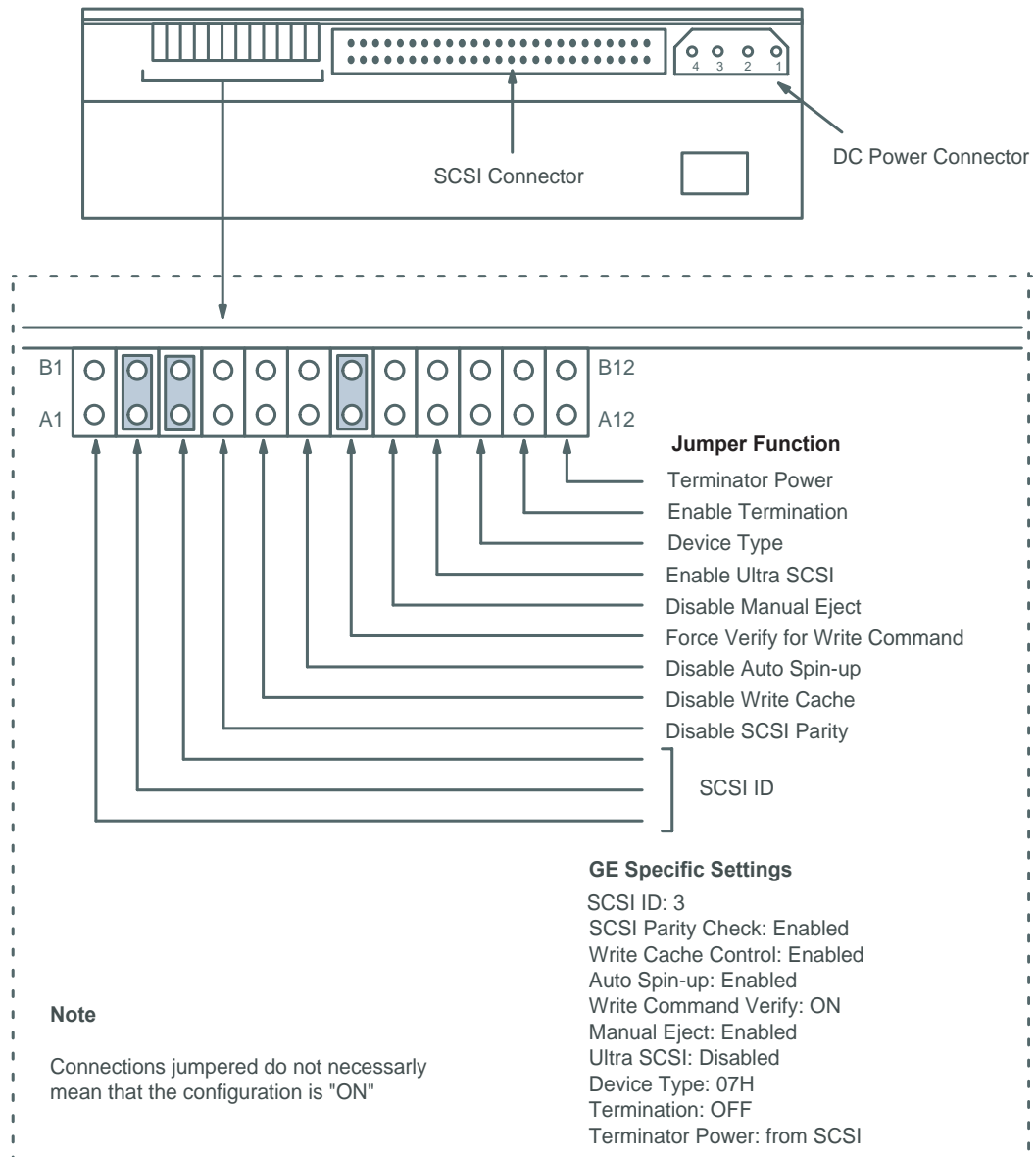


Section 3.0 MOD Drive Installation


- 1.) Shutdown system and power OFF the Operator Console.
- NOTICE: Operator Console must be shutdown before installing the MOD Drive.**
- 2.) Disconnect all the cables from the rear of the SCSI Tower.
- 3.) Remove the SCSI Tower cover by unscrewing the three (3) nuts on the rear side.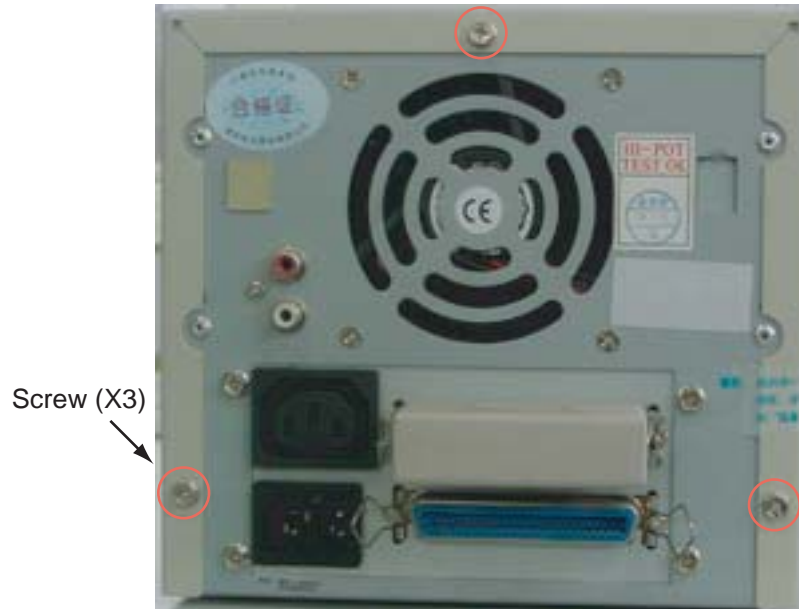

- 4.) Remove the cosmetic cover by pushing it from inside.

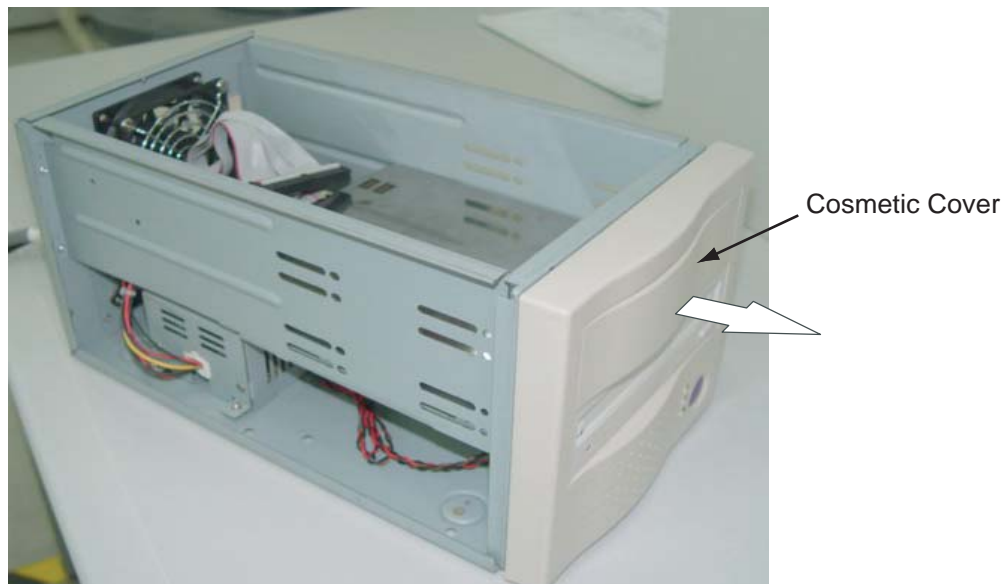
- 5.) Push the MOD Drive into the tower.
- 6.) Secure the MOD Drive using four (4) screws.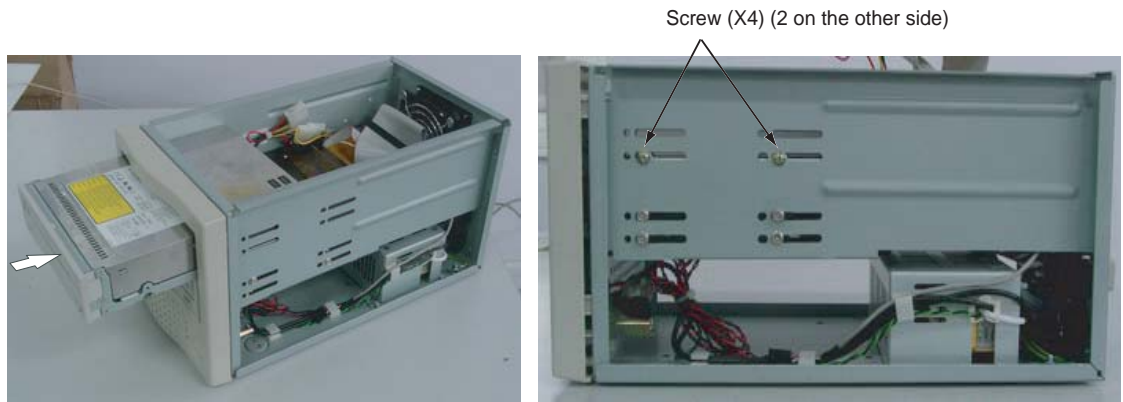

- 7.) Connect the SCSI Cable and Power Cable as illustrated below. Pay attention to the NOTICE. **NOTICE: The SCSI Cable must be twisted. The red edge of the SCSI Cable must be FAR from the Power Cable.**

Top View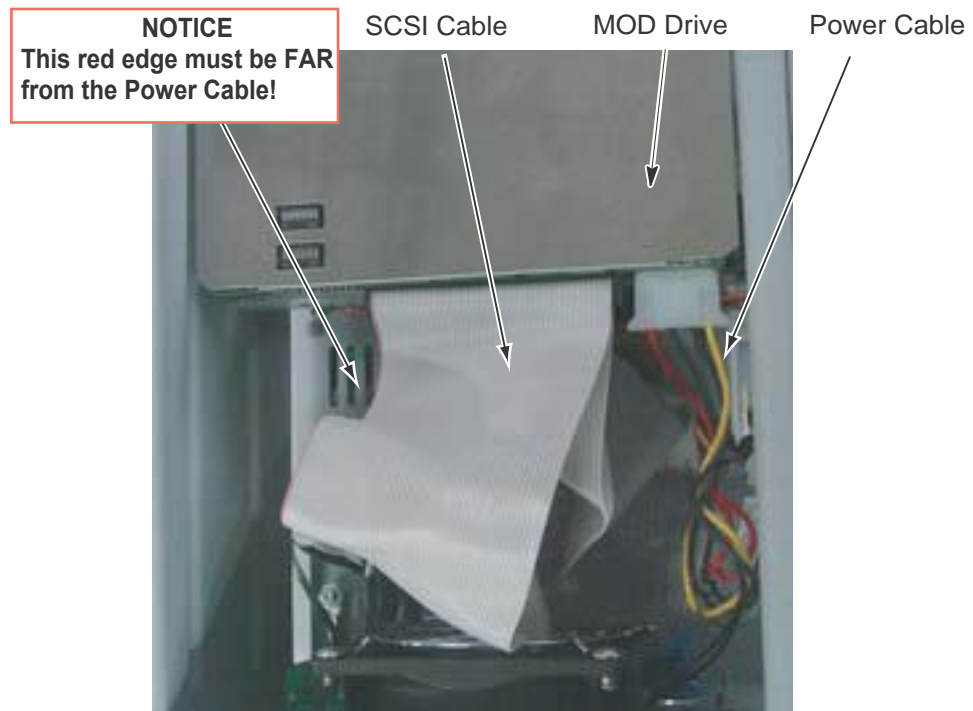

- 8.) Install the removed parts in the reverse order of removal.
- 9.) Power on the Operator Console.

10.) The following steps are required only for system version: 07BW08.x for BrightSpeed Select series and 07MW11.x for BrightSpeed series.

- 1.) Application Shutdown from CSD menu.
- 2.) Open a Unix shell window and execute the followings:

```
{ctuser@hostname } su -  
password: #bigguy  
[ctuser@hostname ] /usr/g/scripts/dvd_search -partition 1 -  
device_type disk -node_name mod -device_name SMO-F551  
[ctuser@hostname ] logout  
{ctuser@hostname }
```

- 3.) Close the window
- 4.) Reboot the system.

Section 4.0 Image Archive/Restore Test

Perform an image archive/restore test.

- 1.) Insert a MOD disk in the MOD drive.
- 2.) Select **Image work -> Browser/Archive/Save examinations**. Ensure images can be saved in the MOD.
- 3.) Restore images from the MOD to the Console by selecting **Image work -> Browser/Archive/Restore**. Ensure images can be restored normally.

Chapter 3


Final Check

Section 1.0

Attaching The Rating Plate

Attach the provided rating plate to the following location (in the label area left to the rating plate of the OC).

OC Rear Side

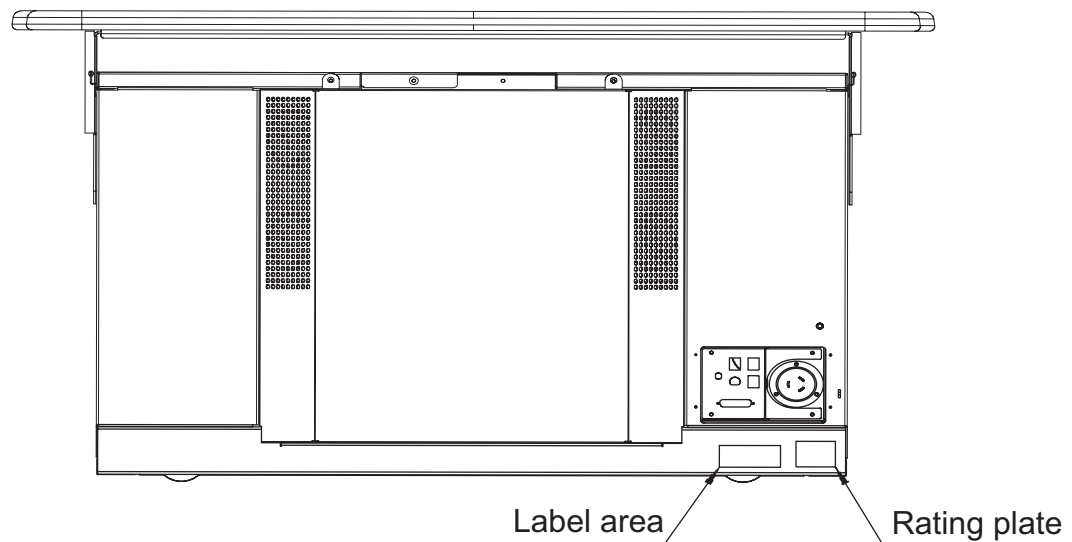3000 N. Grandview Blvd.,
Waukesha, WI 53188 U.S.A.

www.gehealthcare.com

Imagination at work