GE Healthcare

Accessory Interface Hardware Installation Manual

OPERATING DOCUMENTATION

5317058-1EN

Rev 8

© 2020 General Electric Company.

All rights reserved.

LEGAL NOTES

TRADEMARKS

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

COPYRIGHTS

All Material Copyrighted (c) 2020 by the GE Company, All rights reserved.

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DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent, have notation "damage in shipment" written on all copies of the freight or express bill before delivery is accepted or "signed for" by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 6.
- Fill out the GIQ workflow for any items missing, damaged, OBF/FOI for in process installs:
http://supportcentral.ge.com/ProcessMaps/form_new_request.asp?prod_id=268679&form_id=573167&node_id=1916016&map_id=&reference_id=&reference_type

Contact your local service coordinator for more information on this process.

CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

LITHIUM BATTERY CAUTIONARY STATEMENTS

CAUTION

Risk of Explosion.

Danger of explosion if battery is incorrectly replaced.

Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.

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OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the Healthcare PQR Process to report all omissions, errors, and defects in this publication.

Revision History

Revision	Date	Reason for change
8	June 09, 2020	Add Optima CT540 in Prerequisites. Update the Material List.
7	June 07, 2011	Corrected an error in writing.
6	May 16, 2011	RoHS Flag conversion.

Page Left Intentionally Blank

Table of Contents

Preface

Publication Conventions	17
Section 1.0	
Safety & Hazard Information	17
1.1 Text and Character Representation.....	17
1.2 Graphical Representation	18
Section 2.0	
Publication Conventions	18
2.1 General Paragraph and Character Styles.....	18
2.2 Page Layout.....	19
2.3 Computer Screen Output/Input Text Character Styles	19
2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys).....	20

Chapter 1

Accessory Interface Hardware Installation	21
Section 1.0	
Introduction	21
1.1 Overview.....	21
1.2 Prerequisites.....	21
Section 2.0	
Material List	22
Section 3.0	
Installation Procedure.....	24
3.1 Base Cover Removal.....	24
3.2 Install LAN Switch.....	25
3.3 Install IPC and Accessory Panel.....	26
3.4 Install Cable Harness.....	29
3.5 IPC - Injector Control Cable (5169456) Installation	34
Section 4.0	
Finish Installation.....	35
4.1 LED Inspection of IPC Board.....	35
4.2 LED Inspection of LAN Switch.....	36
4.3 Finalization.....	36

Page Left Intentionally Blank

Preface

Publication Conventions

Purpose: Standardize conventions for representing information in a uniform way of communicating information to a reader in a consistent manner. Conventions are used so that the reader can easily recognize the actions or decisions that must be made. There are a number of character and paragraph styles used in this publication to accomplish this task. Please become familiar with them before proceeding forward.

Section 1.0 Safety & Hazard Information

1.1 Text and Character Representation

Within this publication, different paragraph and character styles have been used to indicate potential hazards. Paragraph prefixes, such as hazard, caution, danger and warning, are used to identify important safety information. Text (Hazard) styles are applied to the paragraph contents that is applicable to each specific safety statement. Words describe the type of potential hazard that may be encountered and are placed immediately before the paragraph it modifies. Safety information will normally include:

- Type of potential Hazard
- Nature of potential injury
- Causative condition
- How to avoid or correct the causative condition

EXAMPLES OF HAZARD STATEMENTS USED

It's important that you read and understand hazard statements, and not just ignore them.

**DANGER
SEVERE
PERSONAL
INJURY**

DANGER IS USED WHEN A HAZARD EXISTS WHICH WILL CAUSE SEVERE PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:

- **ELECTROCUTION**
- **CRUSHING**
- **RADIATION**

**WARNING
SERIOUS
PERSONAL
INJURY**

WARNING IS USED WHEN A HAZARD EXISTS WHICH COULD OR CAN CAUSE SERIOUS PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:

- **POTENTIAL FOR SHOCK**
- **EXPOSED WIRES**
- **FAILURE TO TAG AND LOCKOUT SYSTEM POWER COULD ALLOW FOR UN-COMMANDED MOTION.**

**CAUTION
Minor
Personal
Injury**


Caution is used when a hazard exists which can or could cause minor injury to self or others if instructions are ignored. They include for example:

- **Loss of critical patient data**
- **Crush or pinch points**
- **Sharp objects**

- NOTICE** Notice is used when a hazard is present that can cause property damage but has absolutely no personal injury risk. They can include:
- Property Damage Only**
- Disk drive will be destroyed
 - Internal mechanical damage, such as to the x-ray tube
 - Coasting the rotor through resonance.

1.2 Graphical Representation

Important information will always be preceded by the exclamation point  contained within a triangle, as seen throughout the Preface chapter. In addition to text, several different graphical icons (symbols) may be used to make you aware of specific types of hazards that could possibly cause harm.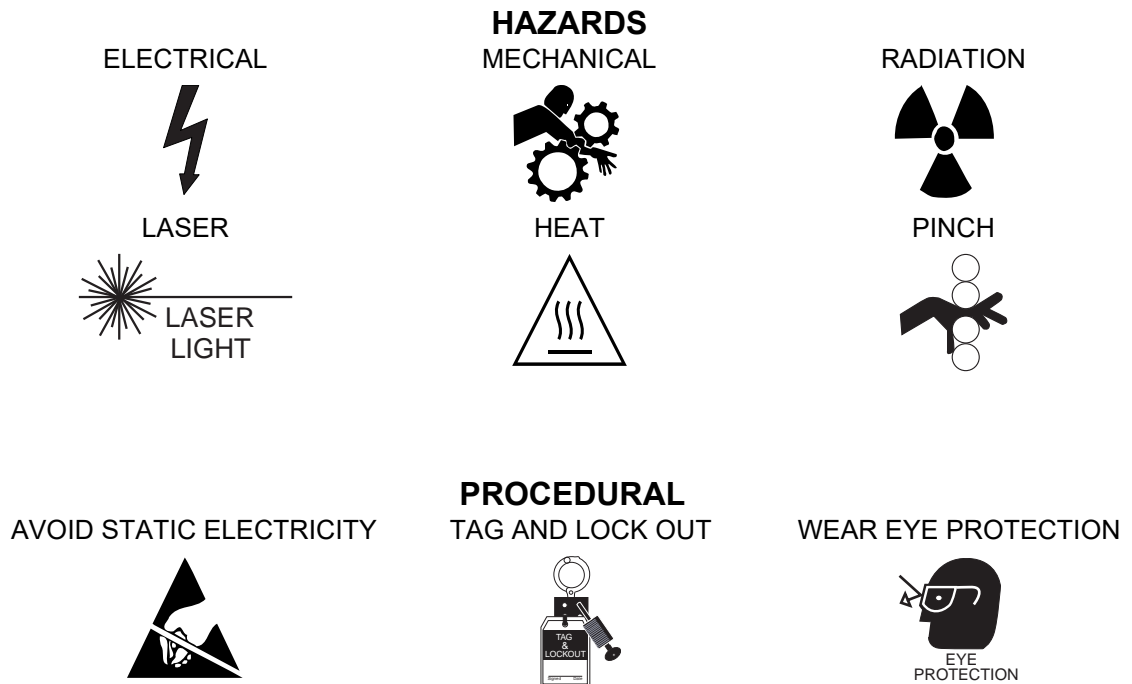

Section 2.0 Publication Conventions

2.1 General Paragraph and Character Styles

Prefixes are used to highlight important non-safety related information. Paragraph prefixes (such as purpose, example, comment or note) are used to identify important but non-safety related information. Text styles are also applied to text within each paragraph modified by the specific prefix.

EXAMPLES OF PREFIXES USED FOR GENERAL INFORMATION


Purpose: Introduces and provides meaning as to the information contained within the chapter, section or subsection, Such as used at the beginning this chapter for example.

Note: Conveys information that should be considered important to the reader.

Example: Used to make the reader aware that the paragraph(s) that follow are examples of information possibly stated previously.

Comment: Represents “additional” information that may or may not be relevant to your situation.

2.2 Page Layout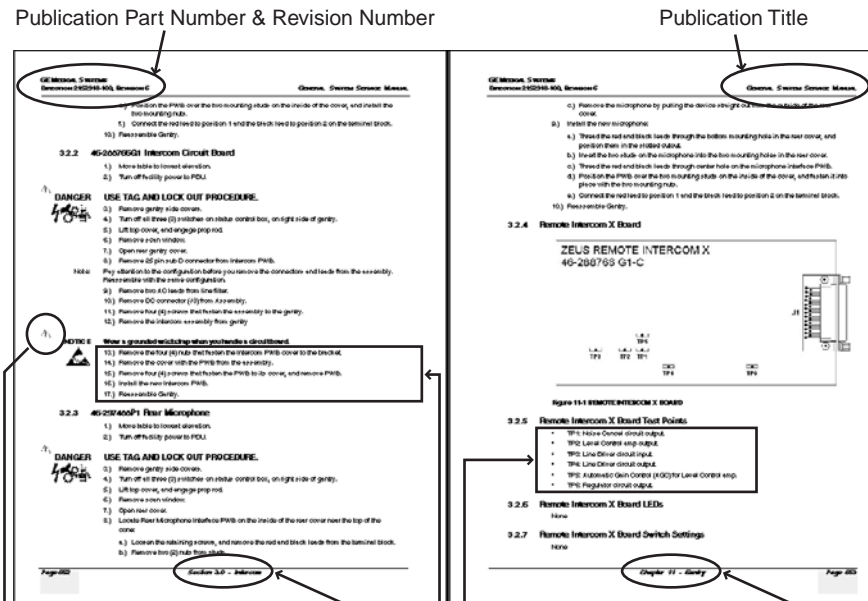

The current section and its title are always shown in the footer of the left (even) page.

An exclamation point in a triangle is used to indicate important information to the user.

Paragraphs preceded by **Alphanumeric** characters (e.g. numbers) contain information that must be followed in a **specific order**.

The current chapter and its title are always shown in the footer of the right (odd) page.

Paragraphs preceded by a **symbol** (e.g. bullets) contain information that has **no specific order**.

Headers and footers in this publication are designed to allow you to quickly identify your location. The document’s part number and revision number appears in every header on every page. Odd numbered page footers indicate the current chapter, its title and current page number. Even page footers show the current section and its title, as well current page number.

2.3 Computer Screen Output/Input Text Character Styles

Within this publication, mono-spaced character styles (fonts) are used to indicate computer text that’s either screen input and output. Mono spaced fonts, such as courier, are used to indicate text direction. When you type at your keyboard, you are generating computer input. Occasionally you will see the math operators “greater than” and “less than” symbols used to indicate the start and finish of variable output. When reading text generated by the computer, you are reading it as computer generated output. In addition to direction, characters are italicized (e.g. *italics*) to indicate information specific to your system or site.

Example: This paragraph’s fonts represents computer generated screen “fixed” output. It’s output is fixed from the sense that it does not vary from application to application. It’s the most commonly used style used to indicate filenames, paths and text that does not change from system to system. The character style used is a fixed width such as courier.

Example: *This paragraph's font represents computer screen output that is "variable". Its used to represent output that varies from application to application or system to system. Variable output is sometimes found placed between greater than and lesser than operators for clarification. For example: <variable_ouput> or <3.45.120.3>. In both cases, the < and > operators are not part of the actual input.*

Example: **This paragraph's font represents fixed input. It's computer input that is typed-in via the keyboard. Typed input that does not vary from application to application or system to system. Fixed text the user is required to supply as input. For example: cd /usr/3p**

Example: *This paragraph's font represents computer input that can vary from application to application or system to system. With variable text, the user is required to supply system dependent input or information. Variable input sometimes is placed between greater-than and less-than operators. For example: <variable_input>. In these cases, the (<>) operators would be dropped prior to input. For example: ypcat hosts | grep <3.45.120.3> would be typed into the computer as ypcat hosts | grep 3.45.120.3 without the greater-than and less-than operators.*

2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)

Different character styles are used to indicate actions requiring the reader to press either a hard or soft button, switch or key. Physical hardware, such as buttons and switches, are called hard keys because they are hard wired or mechanical in nature. A keyboard or on/off switch would be a hard key. Software or computer generated buttons are called soft keys because they are software generated. Software driven menu buttons are an example of such keys. Soft and hard keys are represented differently in this publication.

Example: A power switch **ON/OFF** or a keyboard key like **ENTER** is indicated by applying a character style that uses both over and under-lined bold text that is bold. This is a hard key.

Example: Whereas the computer MENU button that you would click with your mouse or touch with your hand uses over and under-lined regular text. This is a soft key.

END OF SECTION

Chapter 1

Accessory Interface Hardware Installation

Section 1.0

Introduction

1.1 Overview

The Accessory Interface Hardware option provides interface for the following options.

- Cardiac Gating
- Advantage 4D Respiratory Gating
- Smart Step
- Xtream Injector


The Accessory Interface Hardware option is shipped ready for immediate installation. The following installation procedure describes how to install that hardware on the Gantry.

Refer to [Table 1-1](#) for a listing of the parts in the kit.

1.2 Prerequisites

- BrightSpeed and BrightSpeed Select Gantry with New Power Pan (5272458)
- Optima CT540

Section 2.0 Material List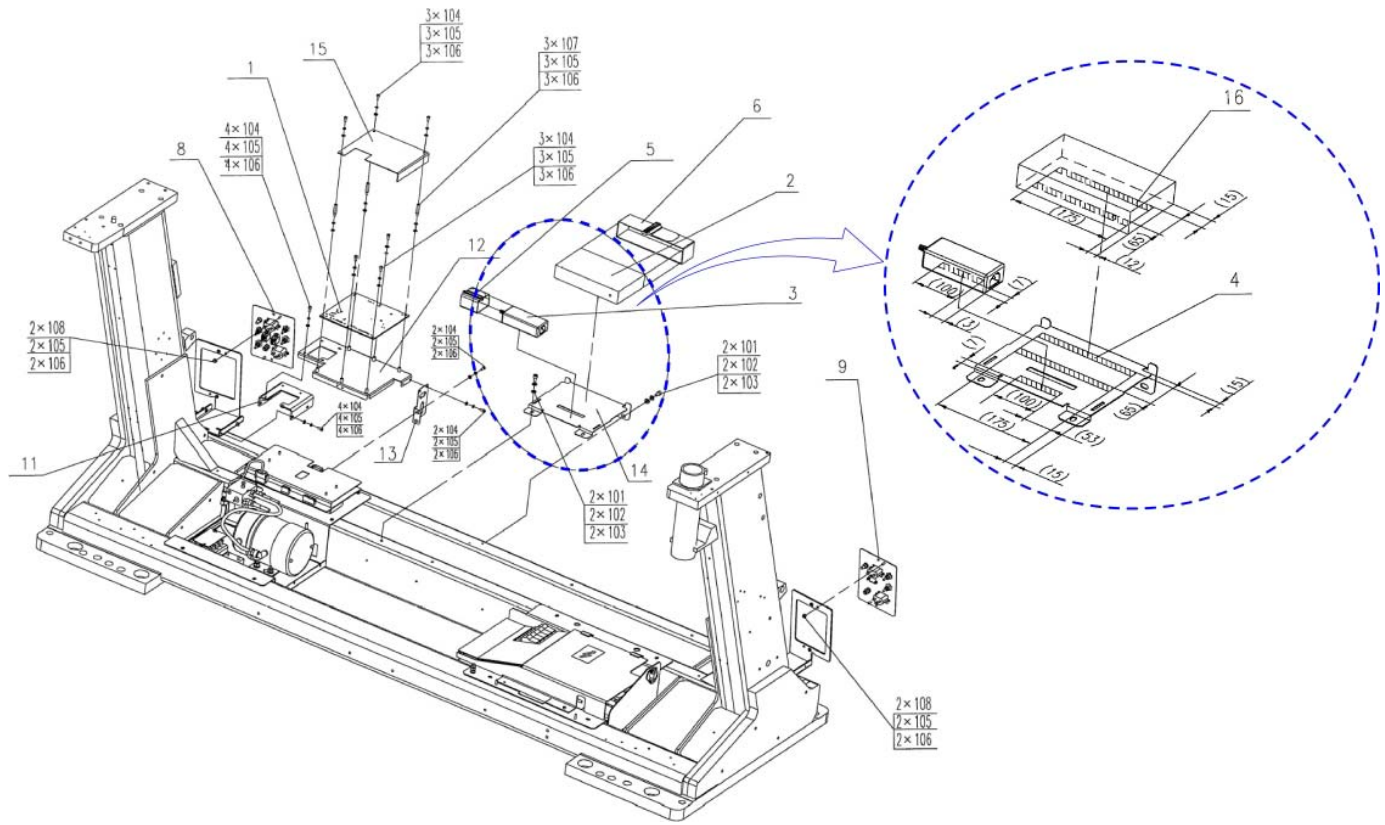

Mark	Part Number	Quantity	Description
1	5136002-20	1.0	Integrated Peripheral Controller PWA
2	5264954-2	1.0	8-port Ethernet Switch hub, with NO power adaptor included
3	5136650	1.0	ups-120vac to 12vdc,
4	46-136246P18	2.0	1~ WIDE, WHT, ADH BACK, VELCRO HOOK 75 FT PER RL
5	2315482	1.0	VELCRO TAPE LENGTH IS 320
6	2315482-3	1.0	VELCRO TAPE LENGTH IS 830
7	5316429	1.0	BSD V3 Cable Harness Assembly
8	5306106	1.0	Accessory Panel LH ASM, CT HD
9	5306107	1.0	Accessory Panel RH ASM, CT HD
10	5306201	1.0	Cable, Cardiac Pulmonary, CT HD
11	5309358	1.0	BSD V3 IPC MNT brkt cushion
12	5309359	1.0	BSD V3 IPC MNT brkt
13	5309361	1.0	BSD V3 connector MNT brkt
14	5309641	1.0	BSD V3 LAN Switch MNT BRKT
15	5304664	1.0	Cover IPC Board, Kitty Hawk
16	46-136246P16	2.0	1~ WIDE, WHT, ADH BACK, VELCRO LOOP 75 FT PER RL

Table 1-1 5317931 Material List

Mark	Part Number	Quantity	Description
17	46-208758P3	20.0	7.31~ X .184~ SELF-LOCKING CABLE TIE 1/16~ TO 1-3/4~ WIRE BUNDLE RANGE. 50LB TENSILE STRENGTH, NYLON MATR~L.
101	46-328417P12	4.0	SCREW HEXAGON SOCKET 6 MM 12 MM ZINC PLATED, STEEL
102	46-328432P3	4.0	WASHER LOCK - SPRING 6.1 MM 11.8 MM ZINC PLATED, SPRING STEEL
103	46-328431P2	4.0	WASHER PLAIN - LARGE 6.4 MM 18 MM ZINC PLATED, STEEL
104	46-328417P3	18.0	SCREW HEXAGON SOCKET 4 MM 10 MM ZINC PLATED, STEEL
105	46-328432P2	25.0	WASHER LOCK - SPRING 4.1 MM 7.6 MM ZINC PLATED, SPRING STEEL
106	46-328430P2	25.0	WASHER PLAIN - NORMAL 4.3 MM 9 MM ZINC PLATED, STEEL
107	5214786-52	3.0	STANDOFF-SPACER MALE-FEMALE M4-0.7 X 15mm Long, STUD RoHS Stud
108	46-328425P1	4.0	NUT HEXAGON 4 MM
109	46-221417P4	2.0	FEMALE SCREWLOCKS FOR HD CONNECTORS. 4-40 THREAD, 0.470 OVERALL LENGTH YELLOW CHROMATE, TWO PER KIT. 0.312 BOTTOM, 0.094 PANEL THK. MAX KIT INCLUDES NUTS AND WASHERS See drawing for approved vendor
200	5317058-1EN	1	Accessory Interface Hardware Installation Manual

Table 1-1 5317931 Material List

Section 3.0 Installation Procedure

DANGER RISK OF ELECTRIC SHOCK. USE PROPER LOCKOUT/TAGOUT PROCEDURES.

3.1 Base Cover Removal

- 1.) Remove the two gantry side covers. Refer to your System Service Manual for any needed instruction.
- 2.) Turn **OFF** all 3 switches (Axial Drive, HVDC, 120 VAC) on the service switch panel.
- 3.) Shutdown system software and turn off the console power switch under the console table top.
- 4.) Remove all system power at the main disconnect (A1) panel. Perform proper Lockout/Tagout power control procedures.
- 5.) Remove both left and right Gantry side base covers (items 6 & 7, respectively, in [Figure 1-1](#)) and then both rear base covers (items 8 & 9, in [Figure 1-1](#)).

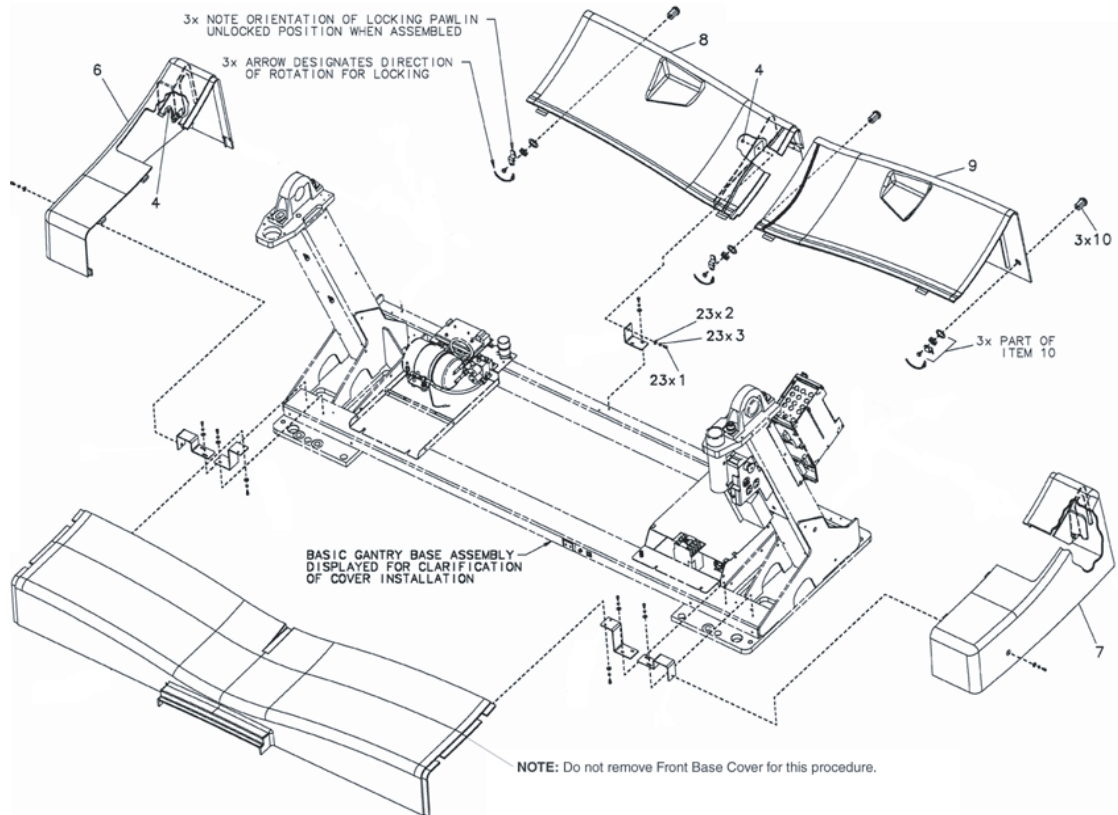
Figure 1-1 Gantry Base Covers

3.2 Install LAN Switch

- 1.) Attach VELCRO Tape to the LAN switch and the mounting bracket as shown in illustration below.

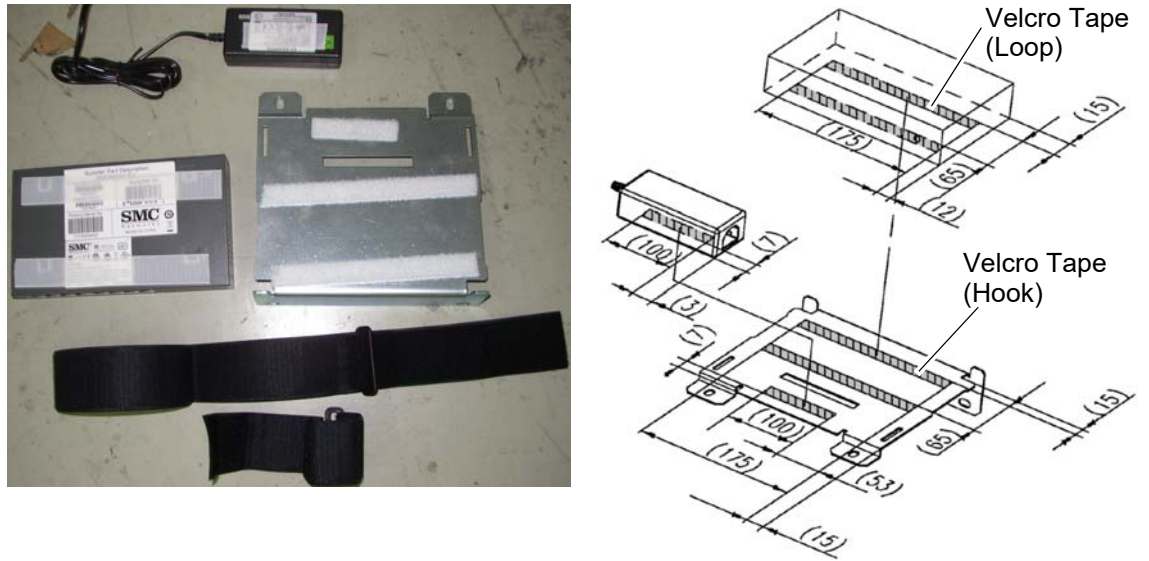
Figure 1-2 Velcro Tape Attachment

- 2.) Fix the LAN switch and the AC/DC adapter to the mounting bracket with long and short Velcro tapes.

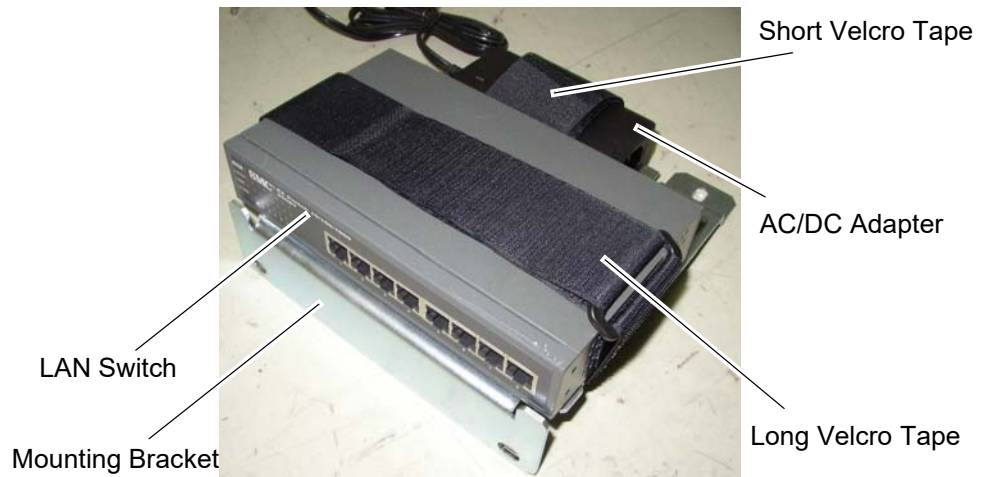
Figure 1-3 LAN Switch Assembly

3.) Install the LAN switch assembly to the Gantry rear base.

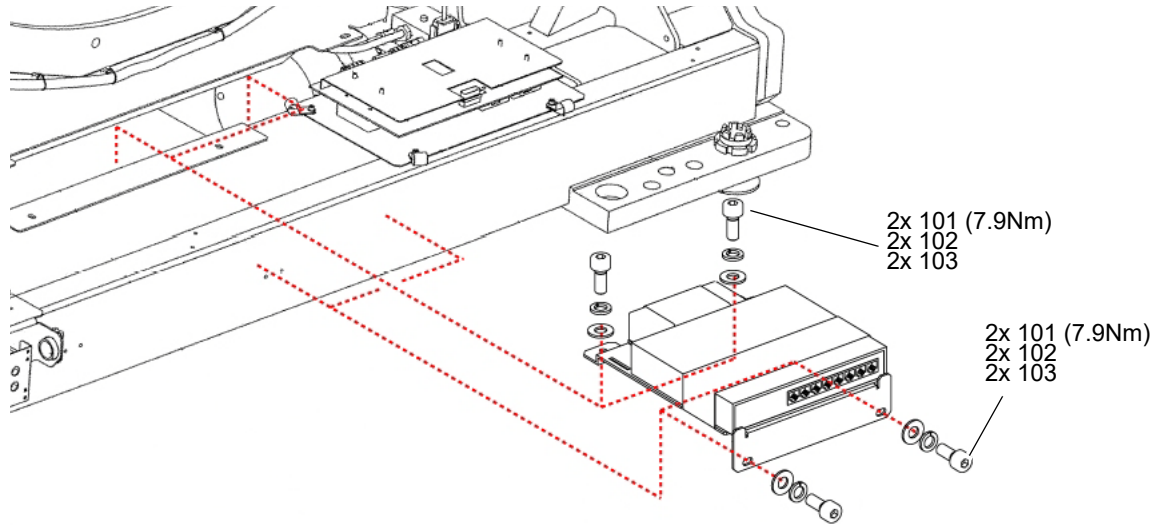
Figure 1-4 LAN Switch Assembly Installation

3.3 Install IPC and Accessory Panel

1.) Assemble the IPC and the left accessory panel.

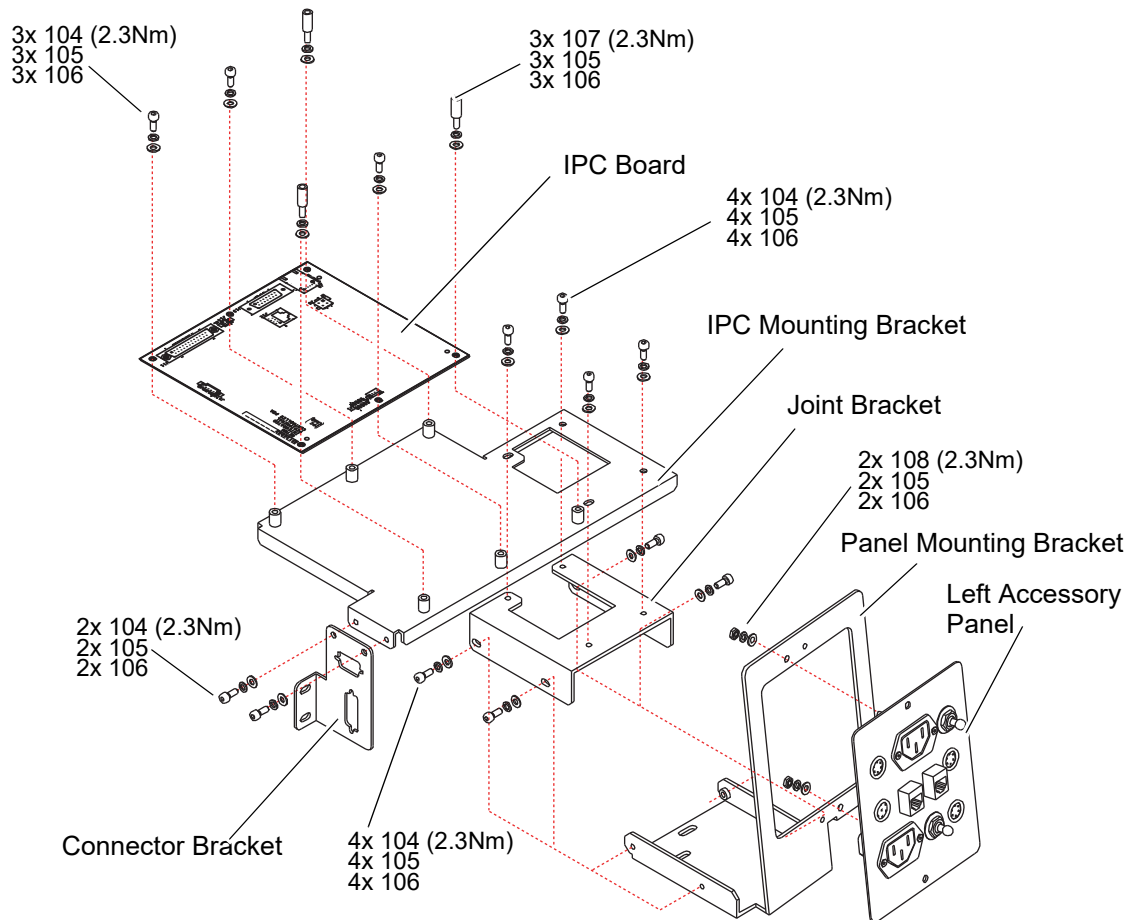
Figure 1-5 IPC Board and Left Accessory Panel

2.) Install the IPC assembly and left panel to the Gantry rear base.

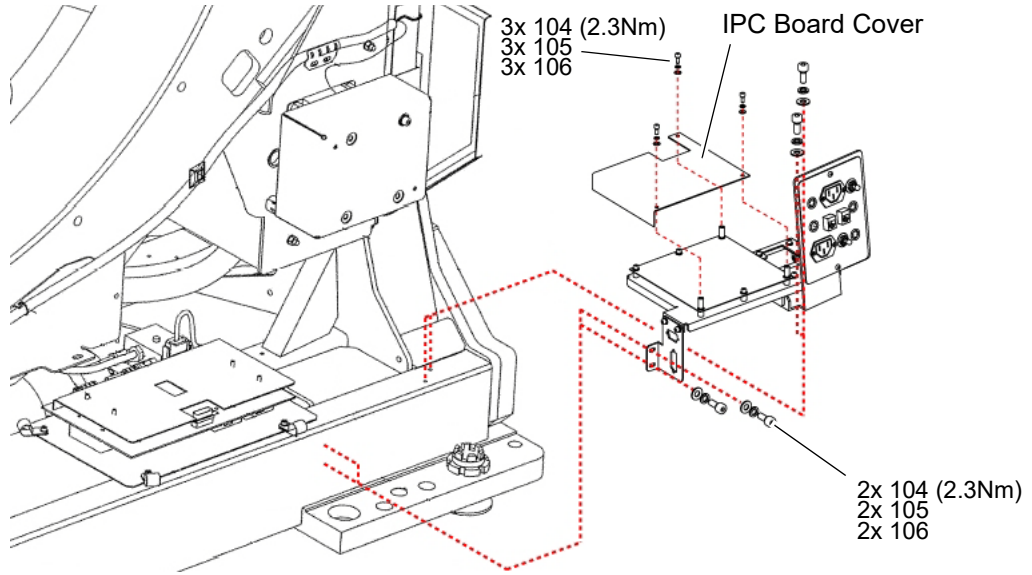
Figure 1-6 IPC and Left Accessory Panel Installation

3.) Connect 5306201 cable (Connector Bracket <-> IPC, J11 <-> Left Accessory Panel).

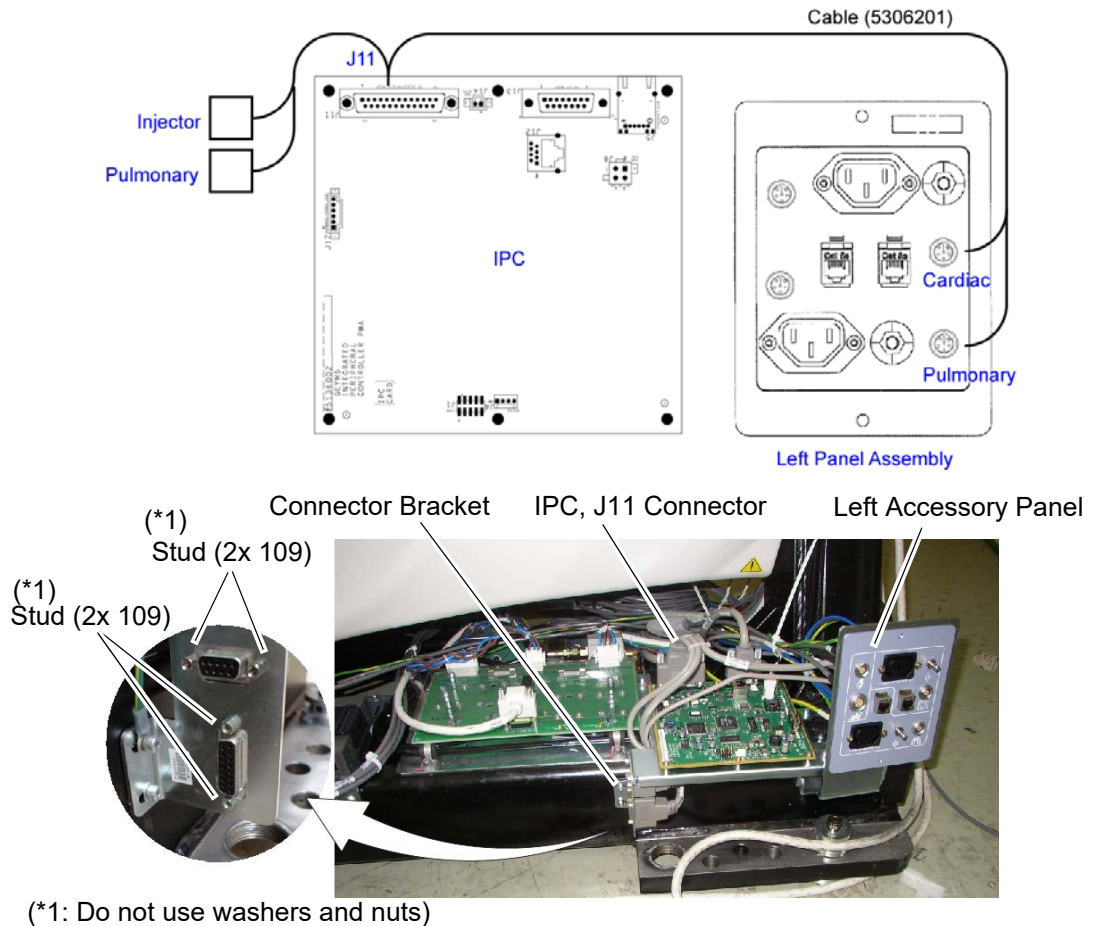
Figure 1-7 Cable (5306201) Connection

4.) Assemble the right accessory panel, and install it to the Gantry rear base.

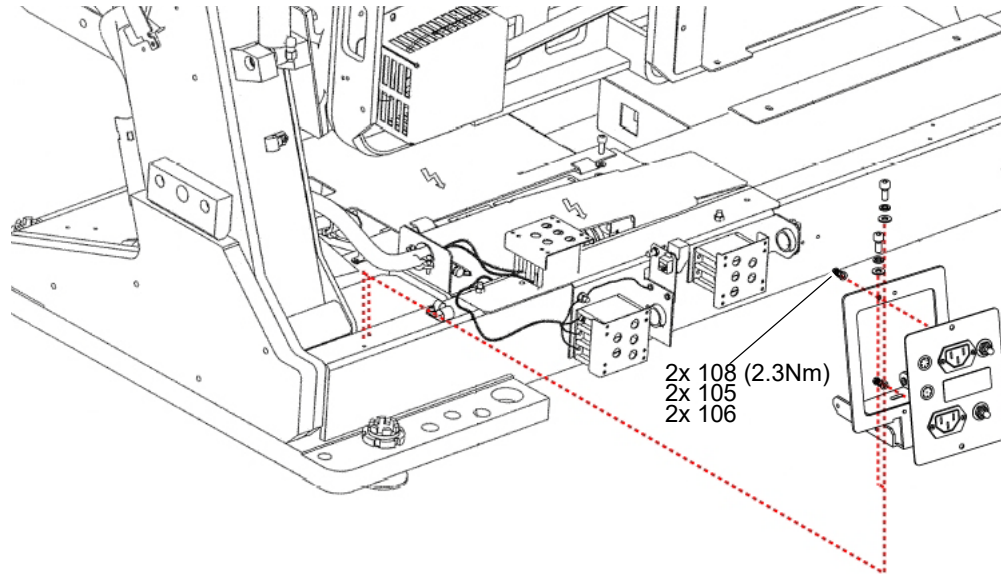
Figure 1-8 Right Accessory Panel Installation

3.4 Install Cable Harness

- 1.) Carefully route the cable harness as shown in illustration [Figure 1-9](#) and [Figure 1-10](#).

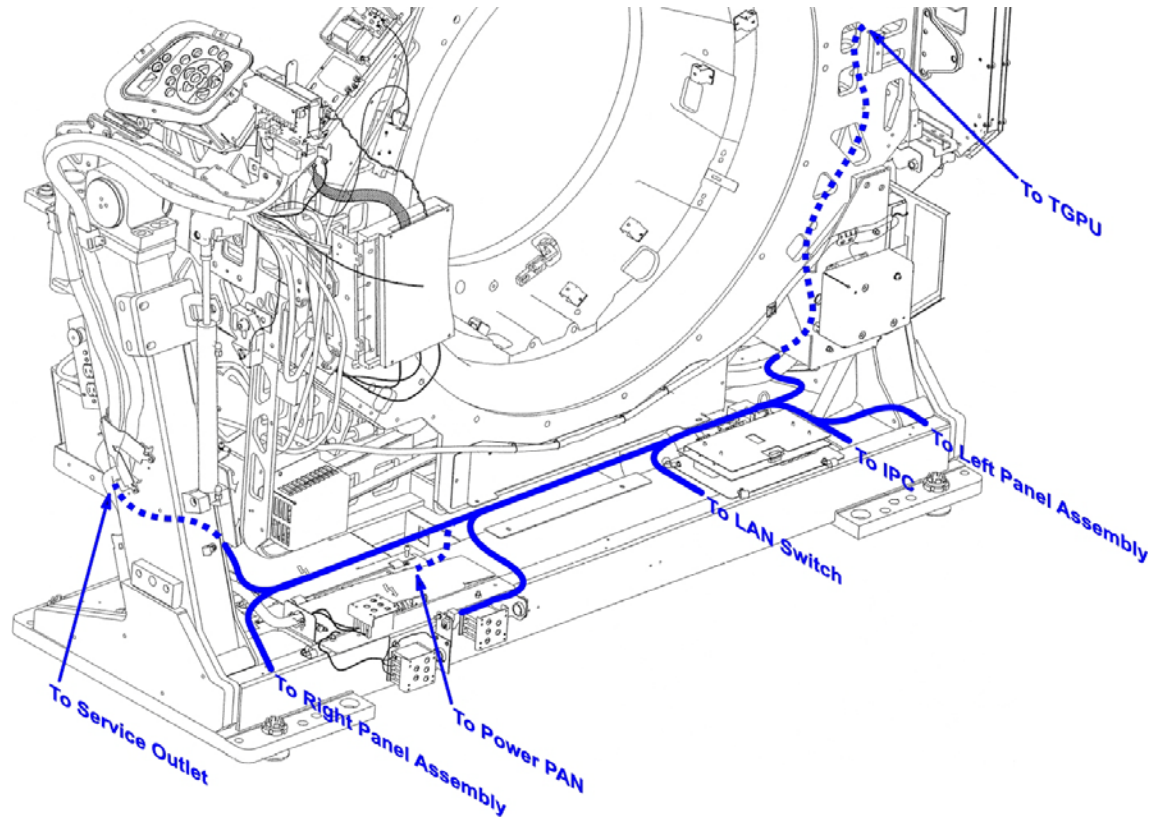
Figure 1-9 Cable Wiring

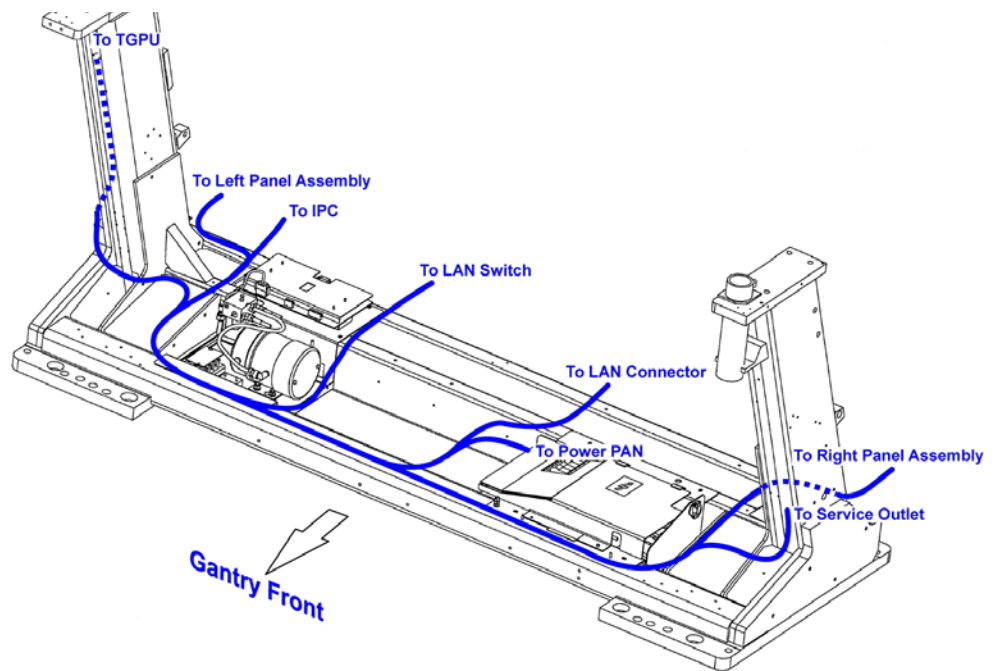
Figure 1-10 Cable Routing

- 2.) Connect all cable connectors of cable harness as shown in [Figure 1-11](#).
 (Refer to [Figure 1-12](#) through [Figure 1-16](#) for further information)

Cable Harness Assembly (5316429) Connection

Item	Part No.	Description	Cable Connection
2	5189456	Cable Asm - IPC power	IPC, J6 <-> TGP, J1 <-> Power Connector
4	5169215	cable, cardiac-pulmonary_PathFinder	IPC, J13 <-> TGPU, J22
5	2286150-2	Smart step cable for BSD Delight Lite.	R-Panel, J2, J3 <-> TGPU, J27 <-> L-Panel, J2, J3
7	5265804-2	Cable Asm-Network Switch Power, BSD V3	Service Outlet <-> Harness Outlet <-> LAN Switch
8	5306481	Power Cable, IPC Accessory Panel, VCT HD	R-Panel <-> Power Pan <-> L-Panel
10	5193969-9	Cable-LAN, BSD Delight Lite	Power Pan <-> LAN Switch 3
12	5193969-8	Cable-LAN, BSD Delight Lite	IPC, J5 <-> LAN Switch 2
13	5193967-2	TGPU J5 to LAN Switch Port1, BSD Delight Lite	TGPU, J5 <-> LAN Switch 1

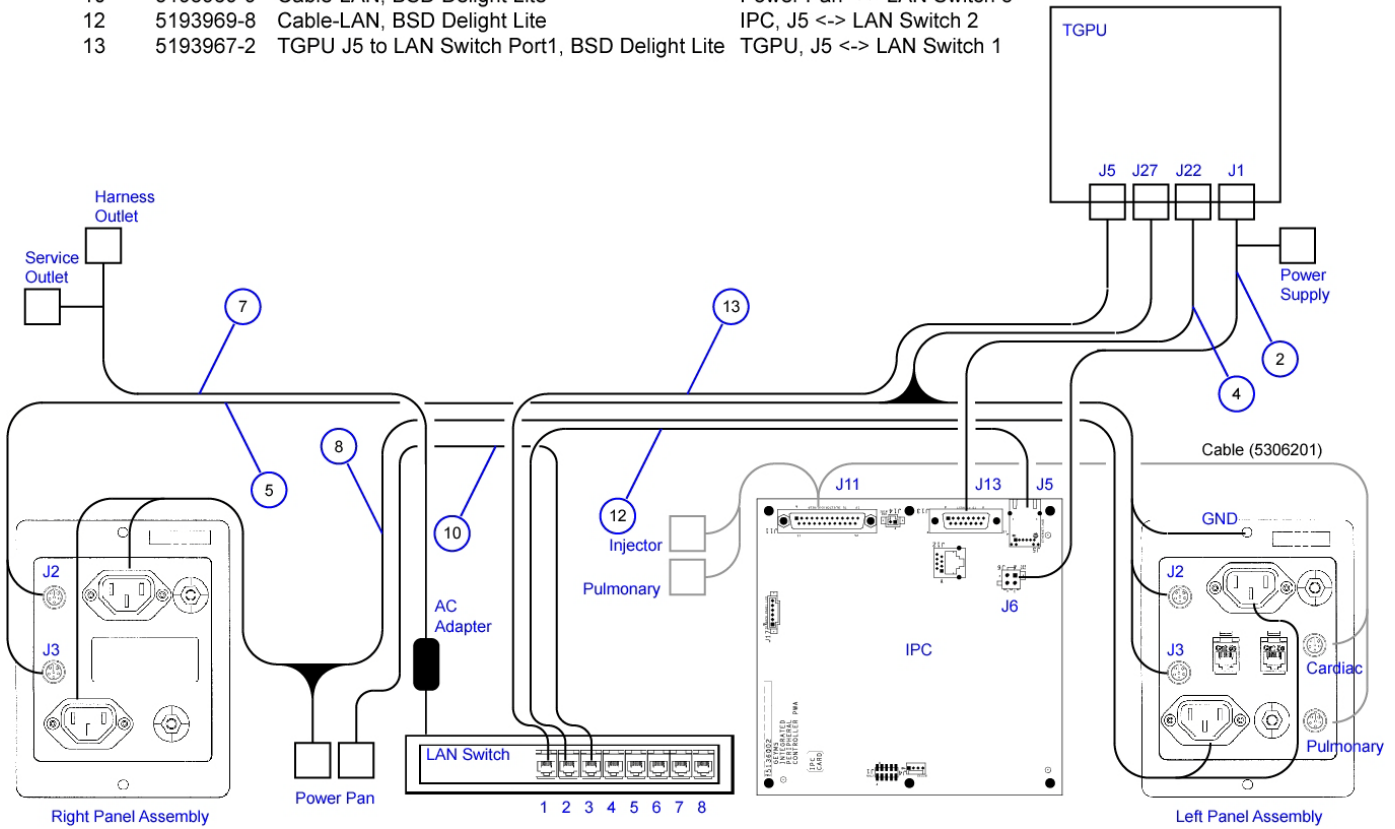
Figure 1-11 Cable Connections


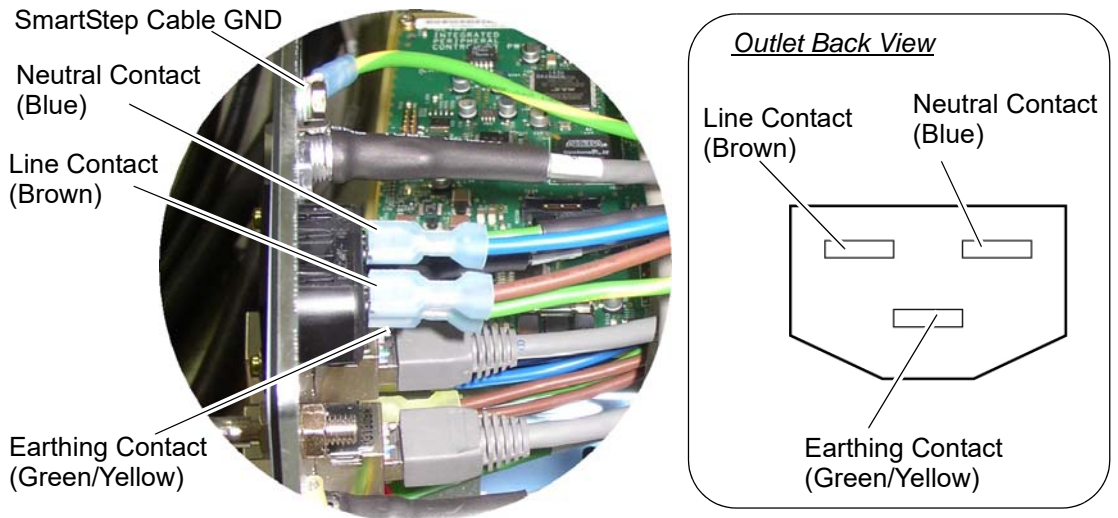
Figure 1-12 Outlet Connection

Disconnect the power cable connector from TGPU, then connect *Item2* cable connector to it.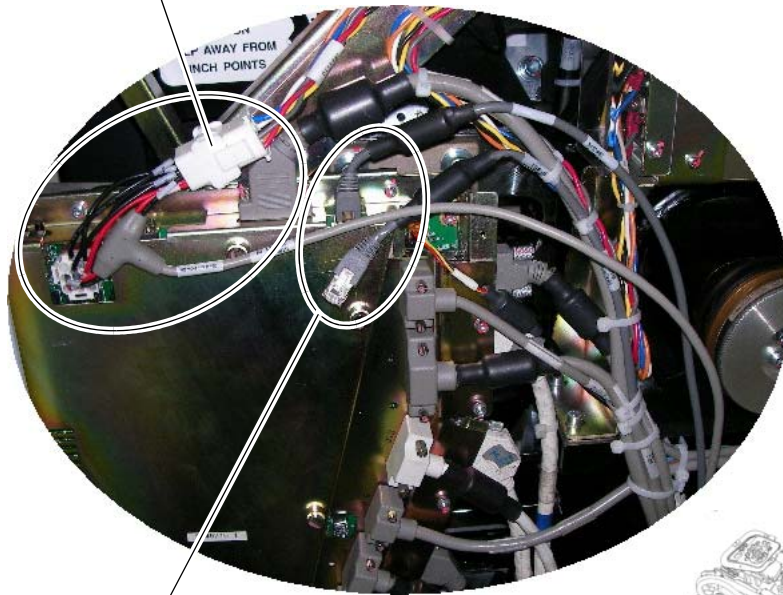

Disconnect the LAN cable connector from TGPU, then connect the *Item13* cable connector to TGPU.

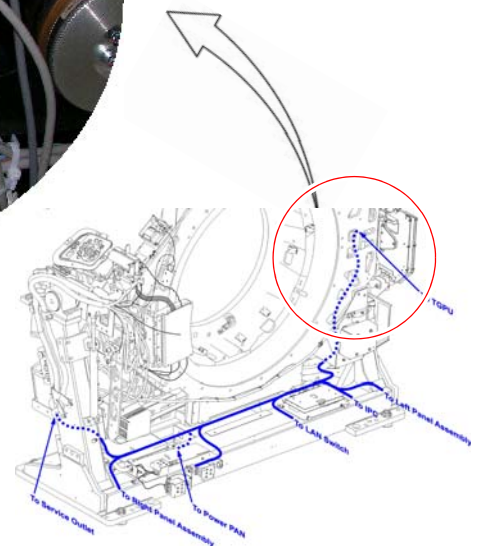
Figure 1-13 TGPU Cable Connection

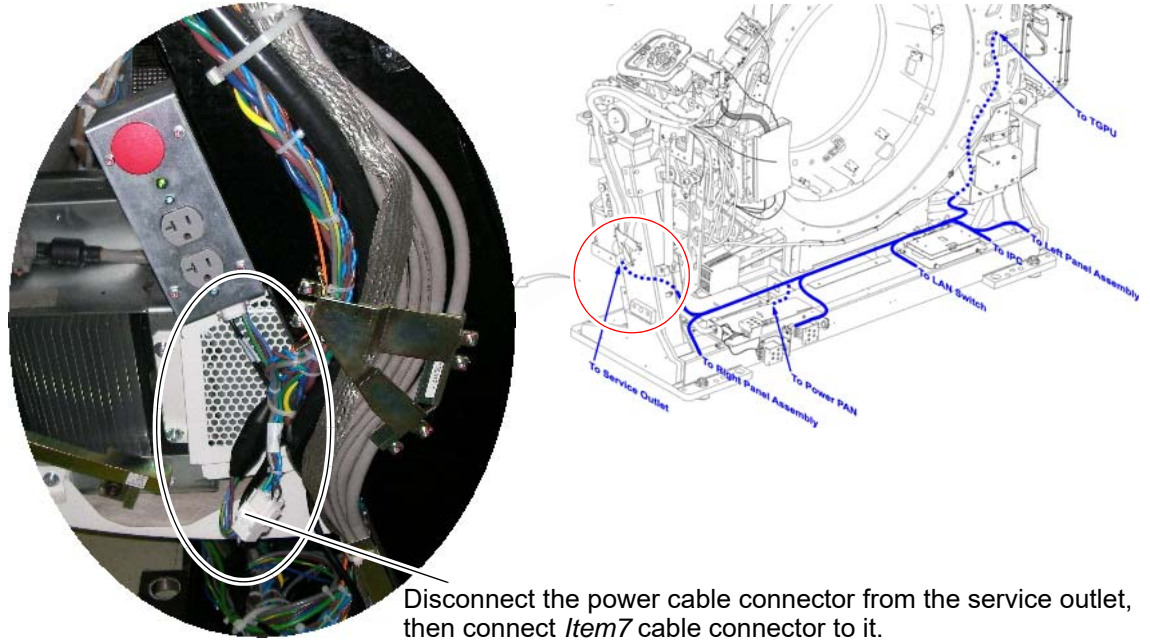
Figure 1-14 Service Outlet Cable Connection

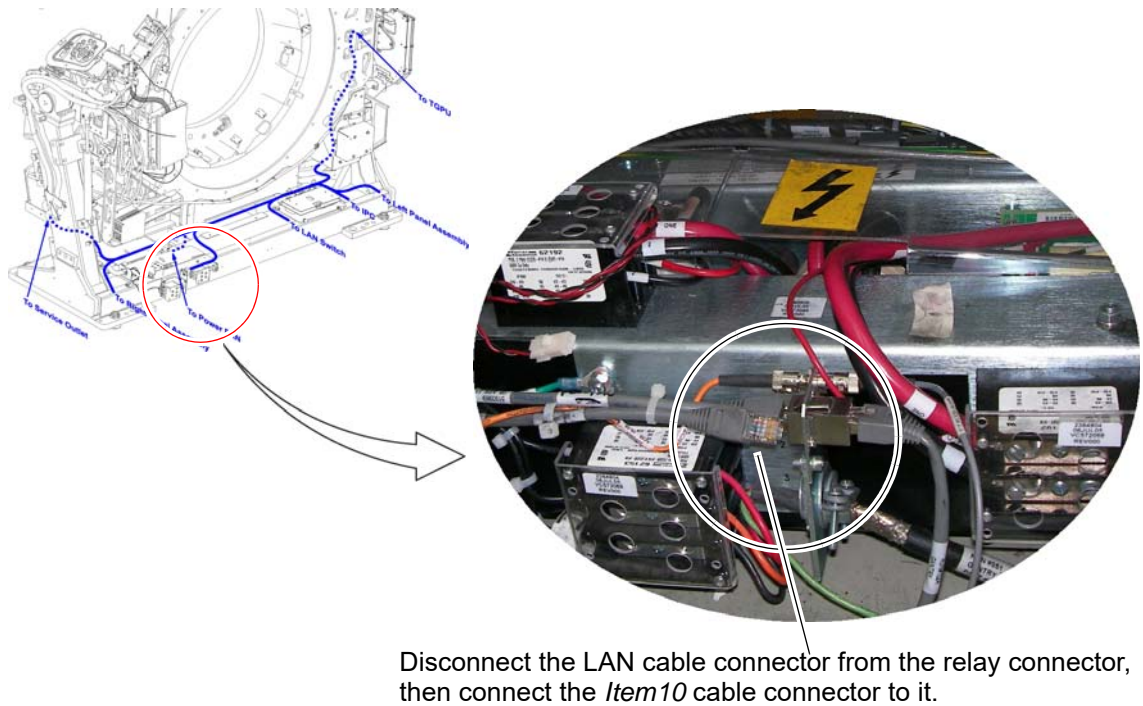
Figure 1-15 Power PAN LAN Cable Connection

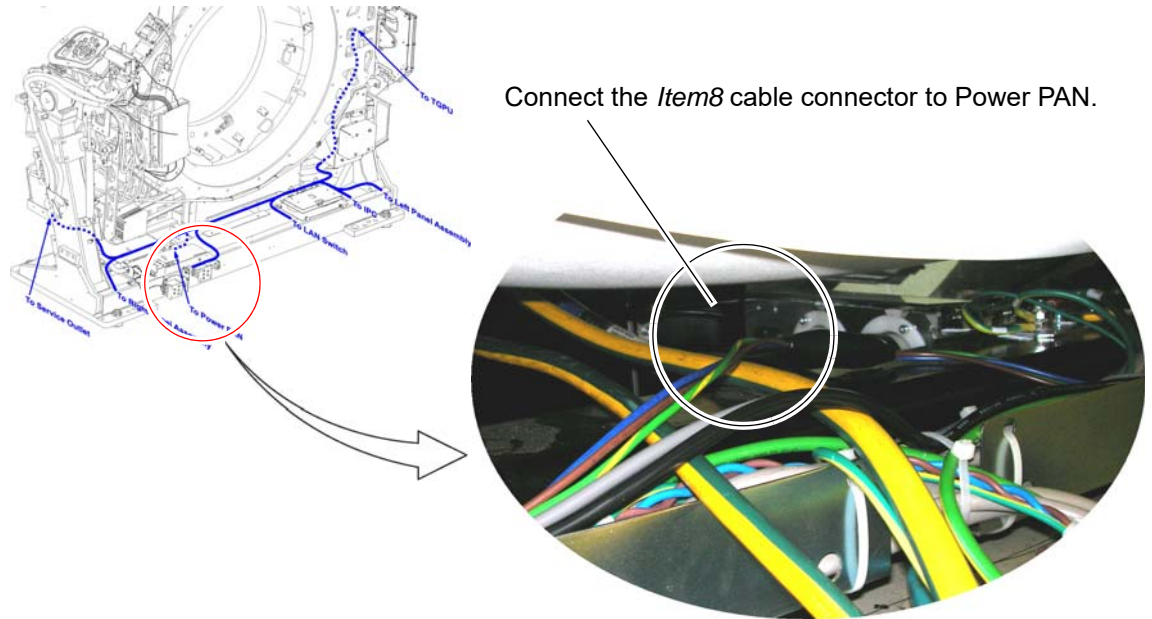
Figure 1-16 Power PAN Cable Connection

Figure 1-17 Cable Connection to LAN Switch

3.) Secure the cables with tie wraps.

Note: If you encounter the following issues when installing the cable harness (P/N 5316429) to the Gantry, cut some tie-wraps and route the cables appropriately.

- Harness interferes with Gantry Left Base Cover thus cover can not be installed.
- Too much force is applied to the LAN switch connectors when Gantry Rear Base Cover is installed.

4.) Restore power to the system.

3.5 IPC - Injector Control Cable (5169456) Installation

- 1.) Route the IPC-Injector Control cable (5169456) between Gantry (IPC) and Injector as shown in Figure 1-18.

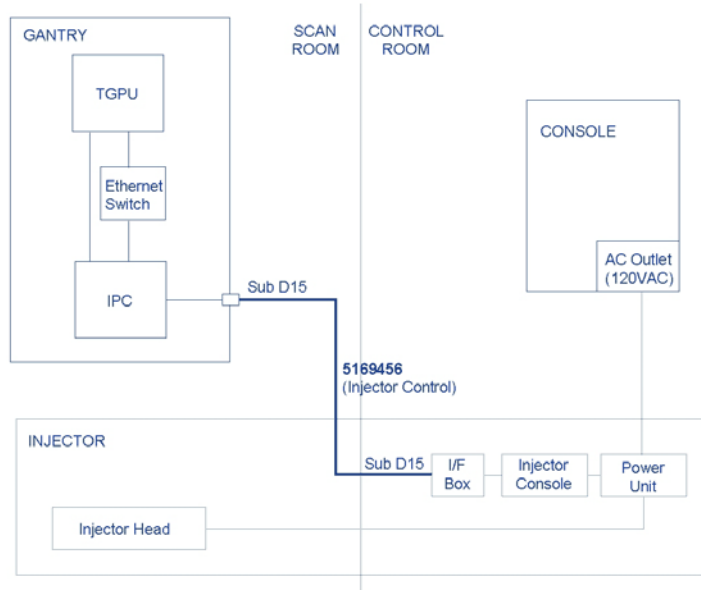
Figure 1-18 IPC - Injector Control Cable Connection

Note: Do not connect the cable to the IPC unless other end of the cable is connected to the Injector.

Section 4.0

Finish Installation

4.1 LED Inspection of IPC Board

- To make sure the IPC board is functioning, an LED inspection should be made on the board. The LED configuration on the board should be as follows with power on but no options connected to the options interface panels:

LED	Color	Label	Status (without options connected)
DS1	Green	24V_PWR	On Solid
DS2	Green	PHY_FW_LED	On Solid
DS3	Green	HRTBT	Off (Blinking when Injector option is installed)
DS4	Green	FW_LED_0	Off
DS5	Green	FW_LED_1	Off
DS6	Green	FW_ERROR	Off
DS7	Green	24V_FUSE	On Solid
DS8	Green	RESET	On Solid
DS9	Yellow	RESET	Off
DS10	Green	R_PULSE_FILTERED	Off
DS11	Green	RESP_PULSE_FILTERED	Off
DS12	Green	R_PULSE_UNFILTERED	Off
DS13	Green	RESP_PULSE_UNFILTERED	Off
DS14	Yellow	FPGA_LOAD	Off
DS15	Green	FPGA_LOAD	On Solid
DS16	Yellow	B/A_CODE	Off
DS17	Green	B/A_CODE	On Solid
DS18	Green	+1.2V_OK	On Solid
Ethernet	Orange/Green	Ethernet-Hub	Blinking with network activity

Table 1-2 IPC Board LED Status

www.gehealthcare.com

Imagination at work