



Section 1: LSI SCSI BIOS Settings


NOTE: HP DEFAULT LSI BIOS SETTINGS ARE ASSUMED. ONLY THE GEHC XW8200 CT/MR CHANGES ARE SHOWN. IF YOU ARE UNSURE ABOUT THE STATE OF CURRENT SETTINGS, SET THE DEFAULTS FIRST USING THE LSI GLOBAL PROPERTIES MENU.

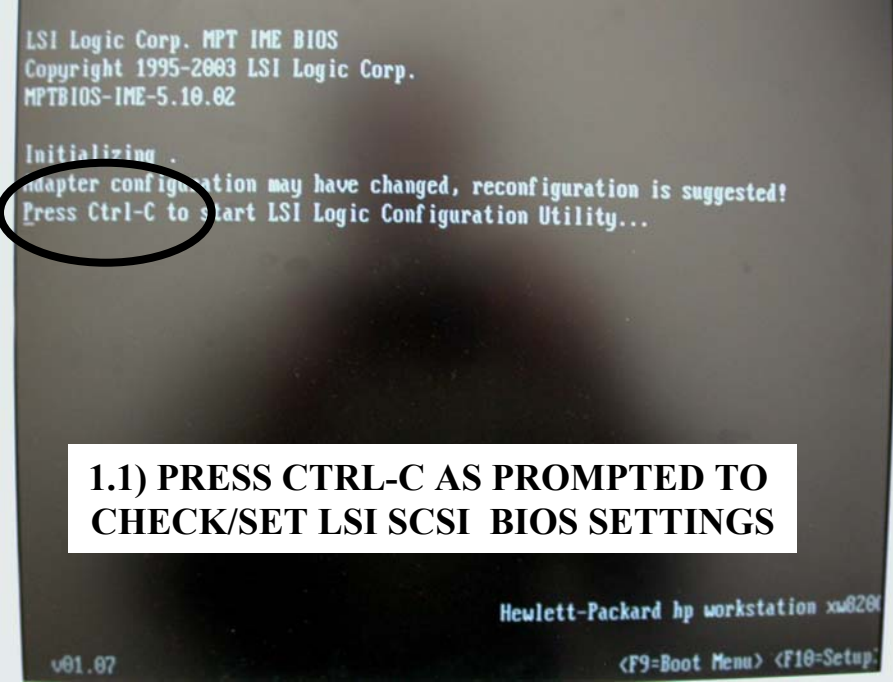


1.1) PRESS CTRL-C AS PROMPTED TO CHECK/SET LSI SCSI BIOS SETTINGS

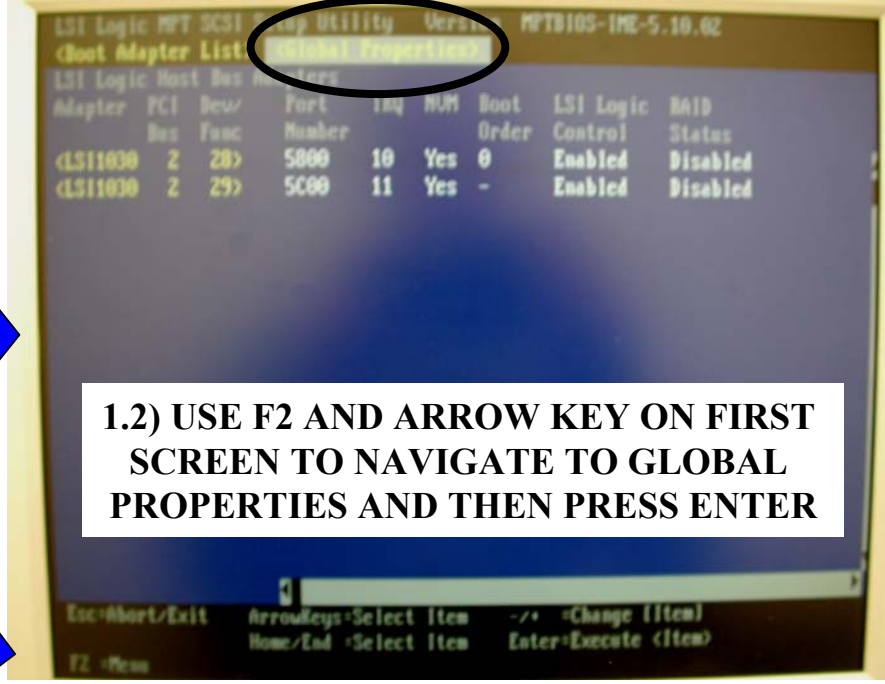

1.2) USE F2 AND ARROW KEY ON FIRST SCREEN TO NAVIGATE TO GLOBAL PROPERTIES AND THEN PRESS ENTER

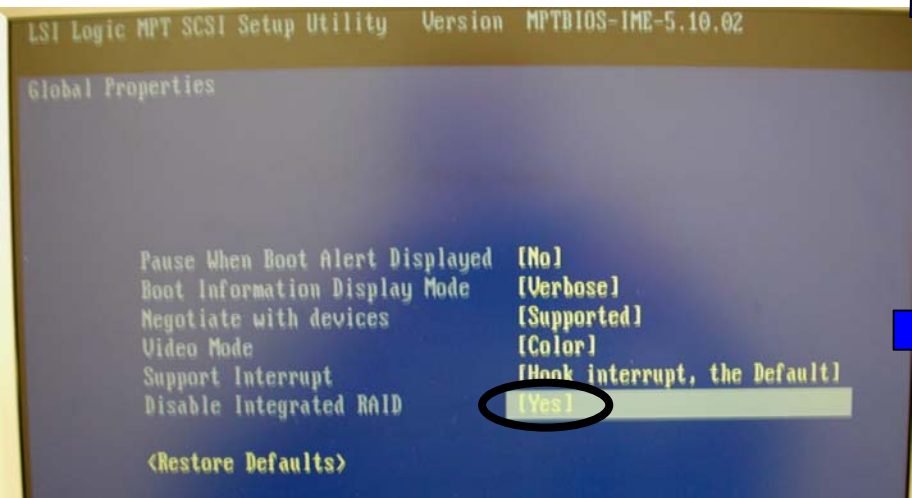

1.3) DISABLE LSI INTEGRATED RAID FEATURE AND THEN PRESS KEY TO EXIT SCREEN

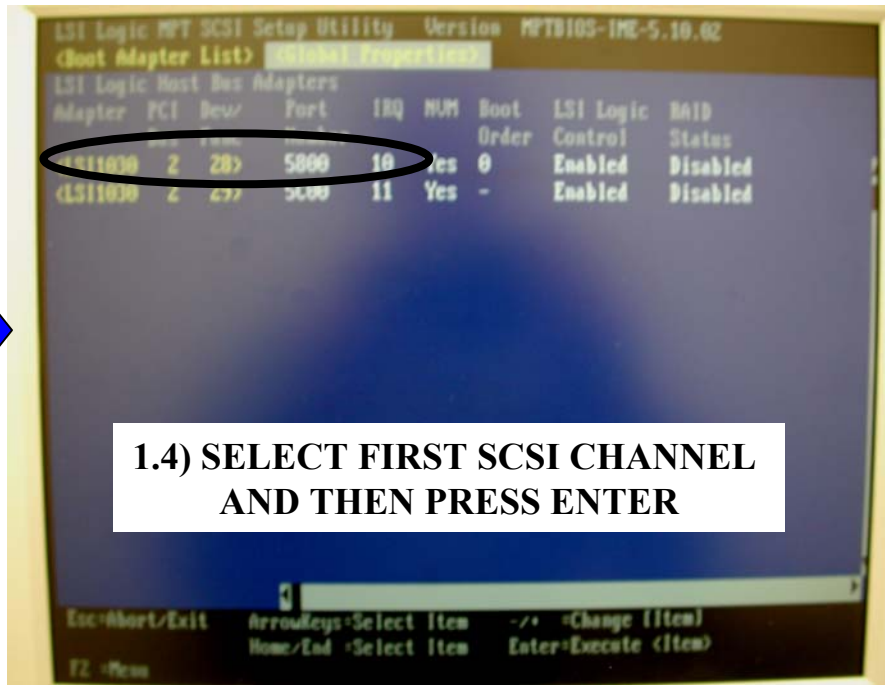

1.4) SELECT FIRST SCSI CHANNEL AND THEN PRESS ENTER

Adapter Properties

Adapter	PCI Bus	Dev/Func
LSI1030	2	28

<Device Properties>

<RAID Properties> <Synchronize Whole Mirror>
 Host SCSI ID [7]
 SCSI Bus Scan Order [Low to High (0..Max)]
 Removable Media Support [None]
 CHS Mapping [SCSI Plug and Play Mapping]
 Spinup Delay (Secs) [2]
 Secondary Cluster Server [No]
 Termination Control [Auto]
 <Restore Defaults>

1.5) PRESS ENTER TO CHECK/SET DEVICE PROPERTIES FOR LSI SCSI CHANNEL 0

Esc=Abort/Exit ArrowKeys=Select Item -/+ =Change [Item]
 Home/End =Select Item Enter=Execute <Item>

Device Properties

SCSI ID	Device Identifier	Restore Defaults	MT/Sec	MB/Sec	Data Width	Scan ID
0	SEAGATE ST336753LW	HPS2 <Defaults>	[160]	320	[16]	[Yes]
1	-	<Defaults>	[160]	320	[16]	[Yes]
2	-	<Defaults>	[160]	320	[16]	[Yes]
3	-	<Defaults>	[160]	320	[16]	[Yes]
4	-	<Defaults>	[160]	320	[16]	[Yes]
5	-	<Defaults>	[160]	320	[16]	[Yes]
6	-	<Defaults>	[160]	320	[16]	[Yes]
7	-	<Defaults>	[160]	320	[16]	[Yes]
8	-	<Defaults>	[160]	320	[16]	[No]
9	-	<Defaults>	[160]	320	[16]	[No]
10	-	<Defaults>	[160]	320	[16]	[No]
11	-	<Defaults>	[160]	320	[16]	[No]
12	-	<Defaults>	[160]	320	[16]	[No]
13	-	<Defaults>	[160]	320	[16]	[No]
14	-	<Defaults>	[160]	320	[16]	[No]
15	-	<Defaults>	[160]	320	[16]	[No]

1.6) USE ARROW/SPACE KEYS TO DISABLE SCAN ID'S 8-15 AS SHOWN AND THEN PRESS ESCAPE KEY TO EXIT SCREEN

Adapter and/or device property changes have been made.
 <Cancel Exit>
 Exit the Configuration Utility
 <Save changes then exit this menu>
 <Discard changes then exit this menu>

1.7) SELECT SAVE CHANGES AS SHOWN AND THEN PRESS ENTER TO SAVE SETTINGS

Esc=Abort/Exit ArrowKeys=Select Item -/+ =Change [Item]





<Boot Adapter List> <Global Properties>

LSI Logic Host Bus Adapters

Adapter	PCI Bus	Dev/Func	Port Number	IRQ	NUM	Boot Order	LSI Logic Control	RAID Status
<LSI1030	2	28	5800	10	Yes	0	Enabled	Disabled
<LSI1030	2	29	5C00	11	Yes		Enabled	Disabled

1.8) SELECT SECOND SCSI CHANNEL AND THEN PRESS ENTER



Esc=Abort/Exit ArrowKeys=Select Item -/+ =Change [Item]
 Enter=Execute <Item>

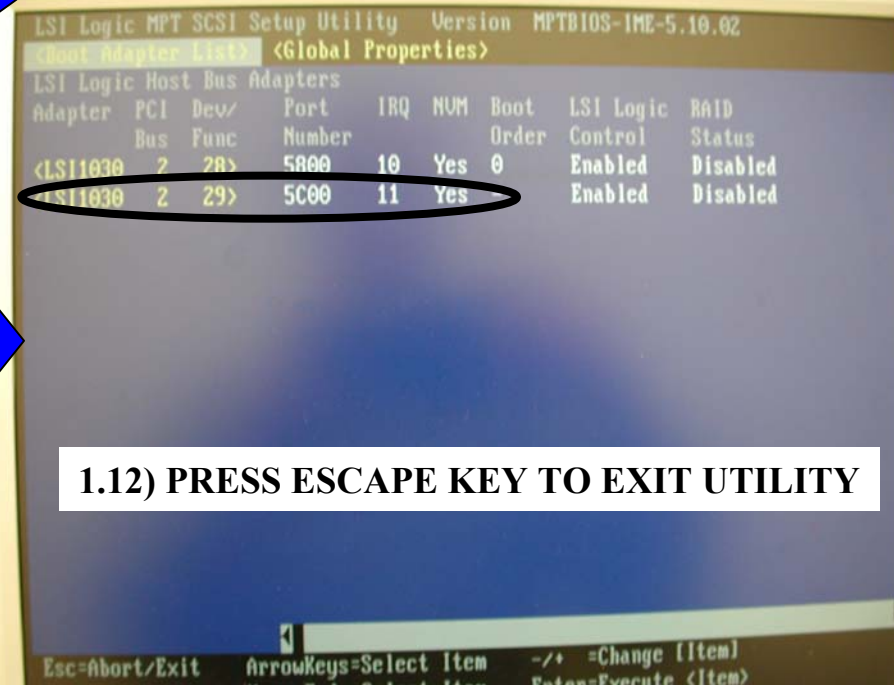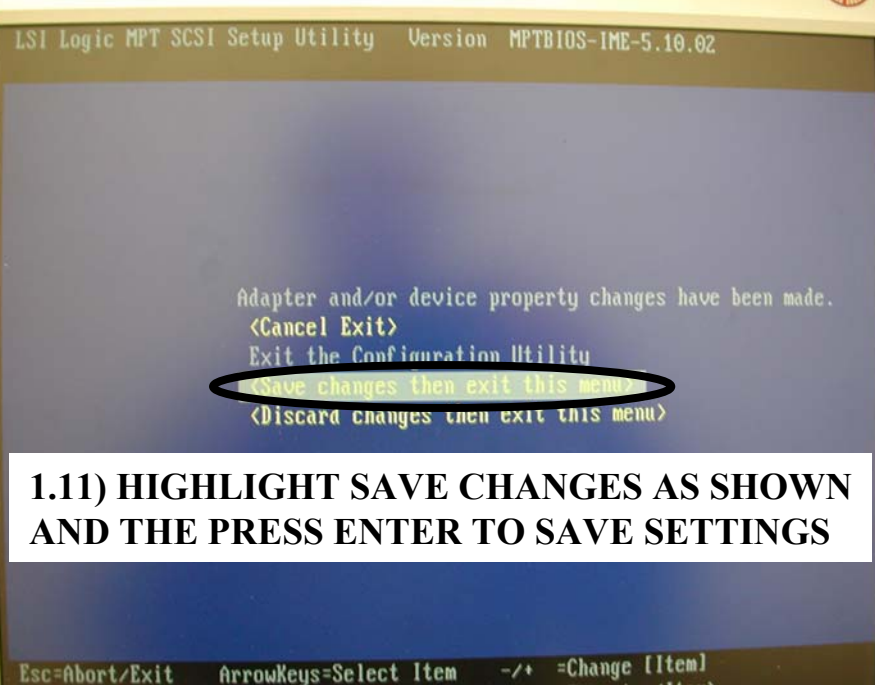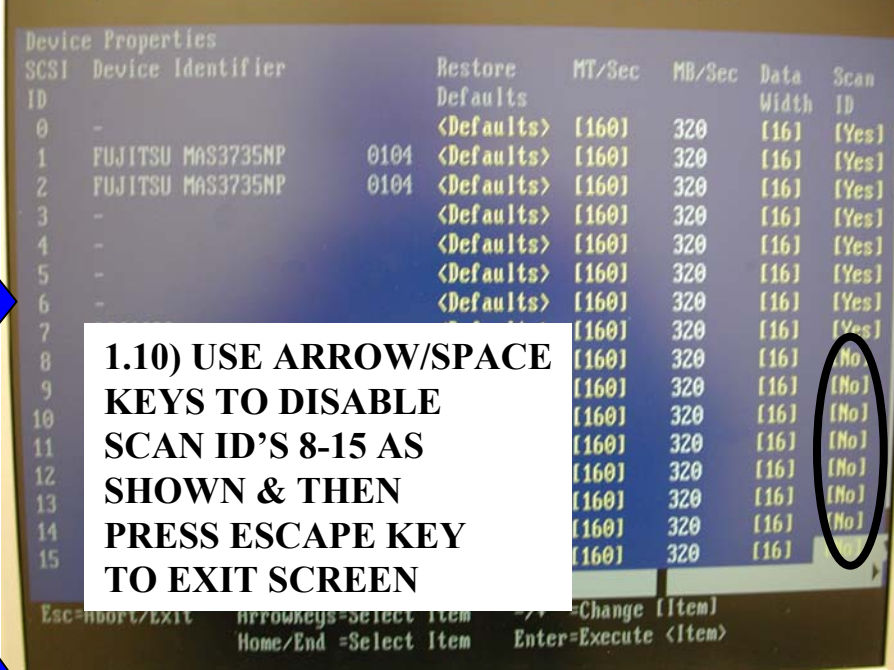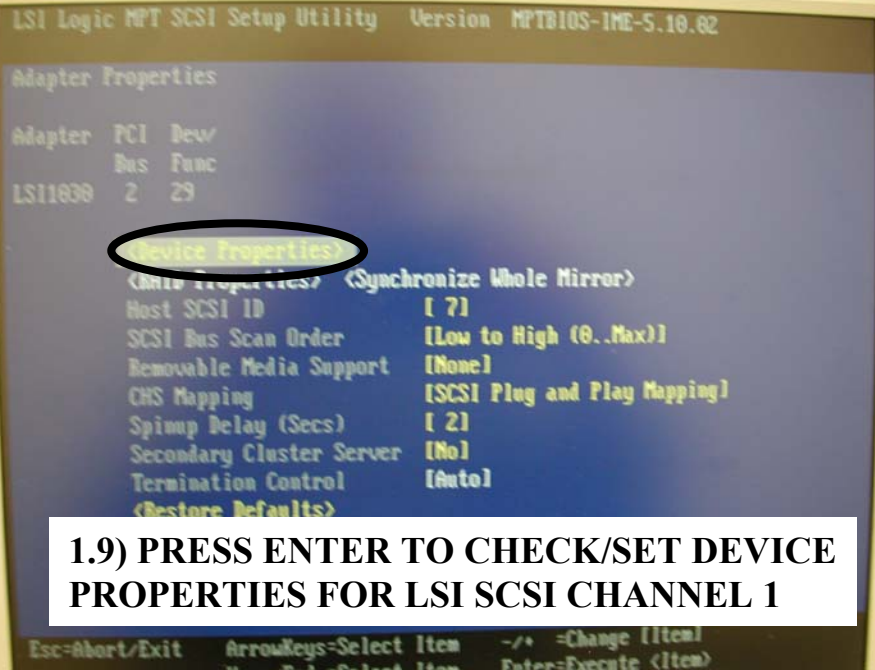


Section 2 – System BIOS Settings

NOTE: HP DEFAULT BIOS SETTINGS ARE ASSUMED. ONLY THE GEHC XW8200 CT/MR CHANGES ARE SHOWN. IF YOU ARE UNSURE ABOUT THE STATE OF CURRENT SETTINGS, SET DEFAULTS FIRST USING THE MAIN BIOS MENU OPTION.


- English
- Dansk
- Suomi
- Français
- Deutsch
- Italiano
- 日本語
- Nederlands
- Norsk
- Português
- Svenska
- Español

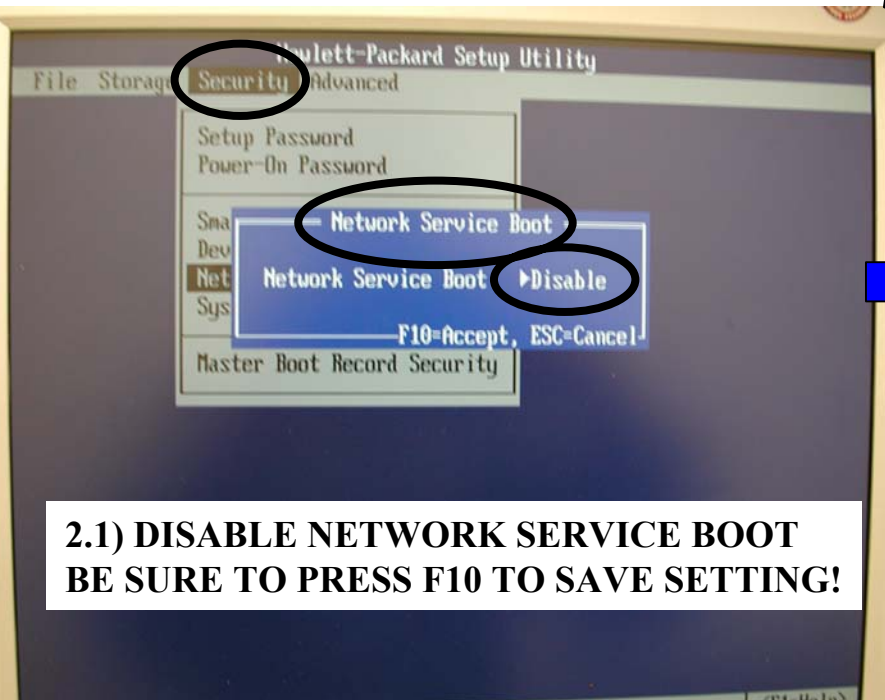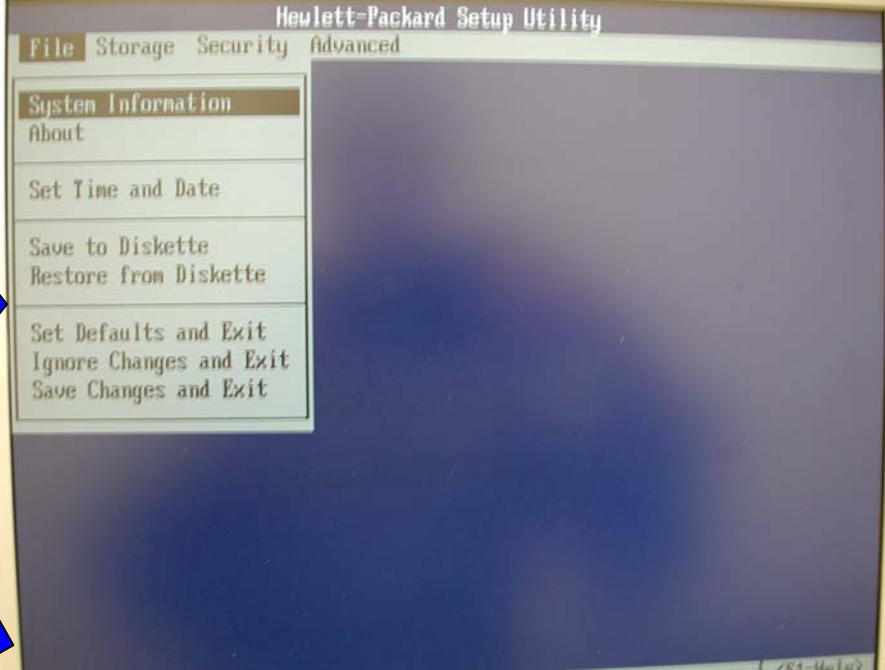


**2.1) DISABLE NETWORK SERVICE BOOT
BE SURE TO PRESS F10 TO SAVE SETTING!**

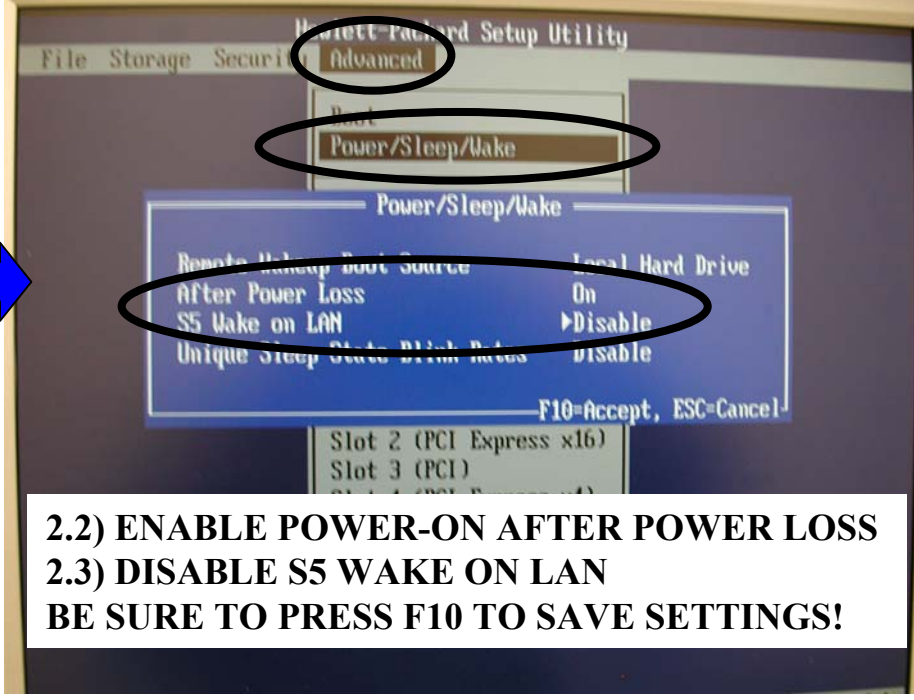


**2.2) ENABLE POWER-ON AFTER POWER LOSS
2.3) DISABLE S5 WAKE ON LAN
BE SURE TO PRESS F10 TO SAVE SETTINGS!**

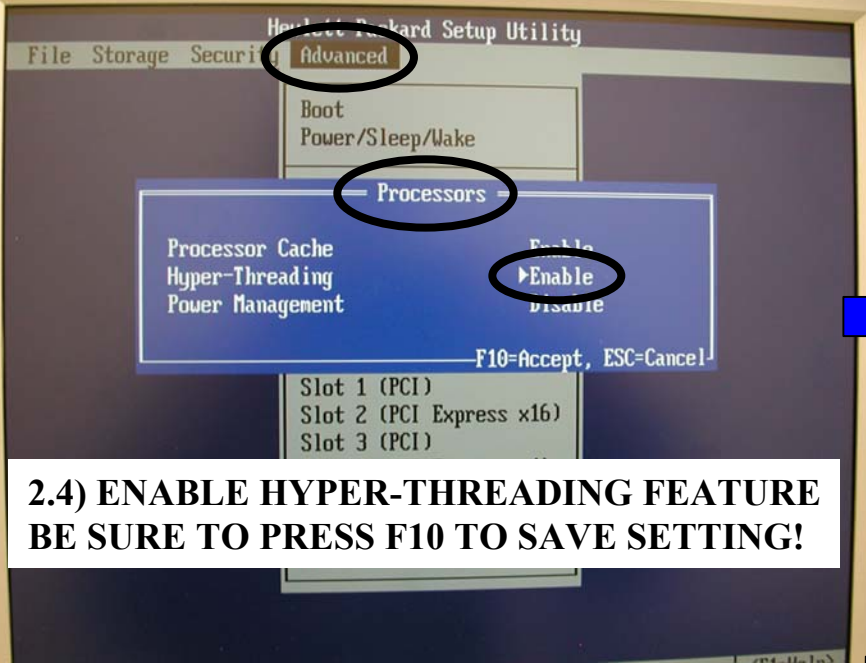


**2.4) ENABLE HYPER-THREADING FEATURE
BE SURE TO PRESS F10 TO SAVE SETTING!**

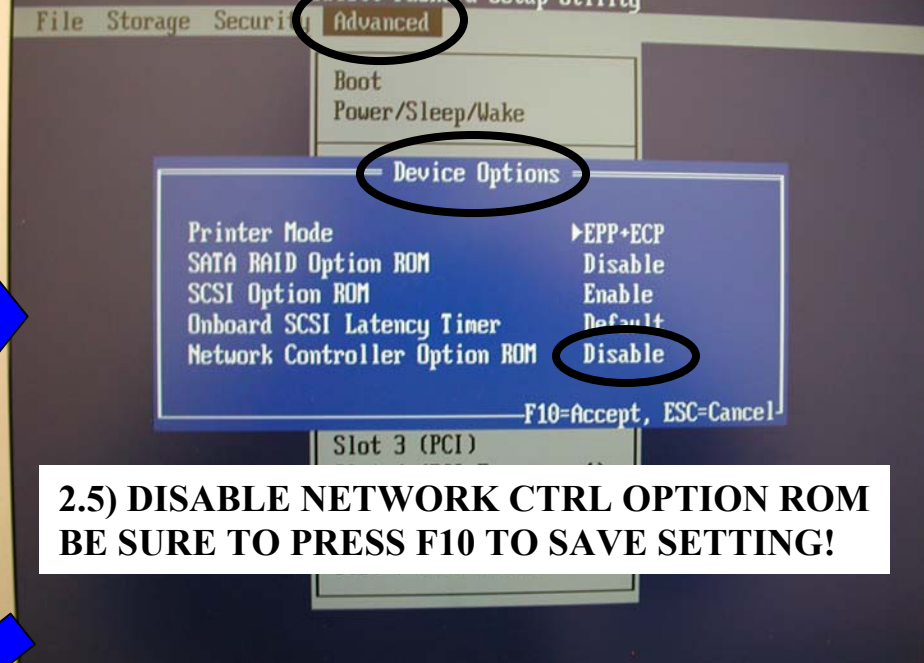





**2.5) DISABLE NETWORK CTRL OPTION ROM
BE SURE TO PRESS F10 TO SAVE SETTING!**

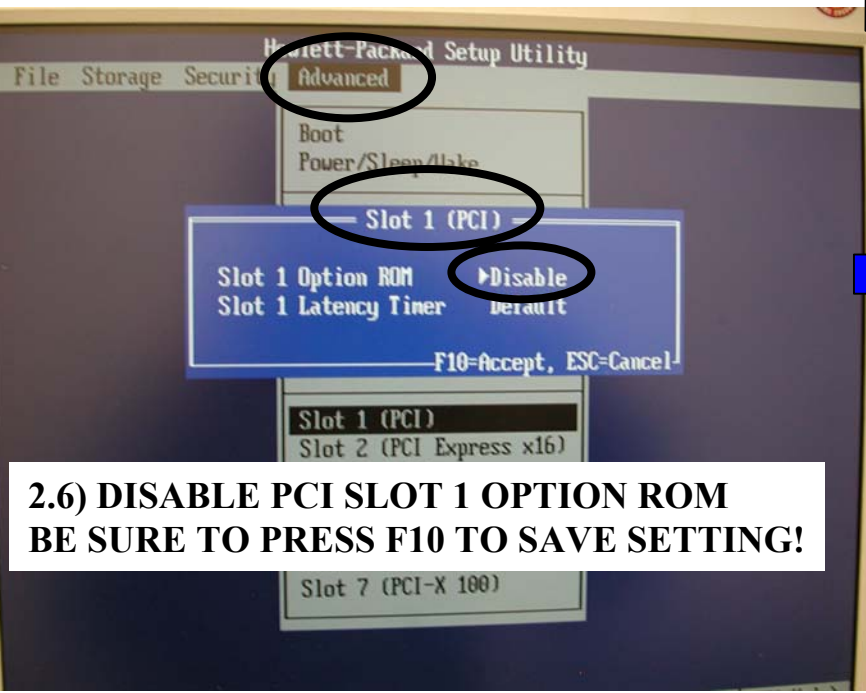

**2.6) DISABLE PCI SLOT 1 OPTION ROM
BE SURE TO PRESS F10 TO SAVE SETTING!**

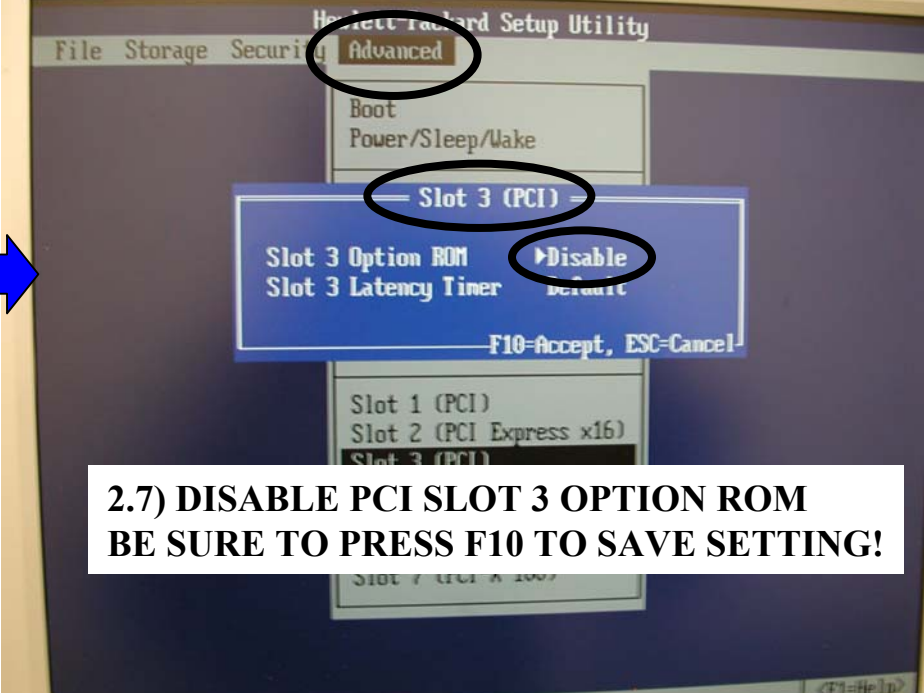

**2.7) DISABLE PCI SLOT 3 OPTION ROM
BE SURE TO PRESS F10 TO SAVE SETTING!**

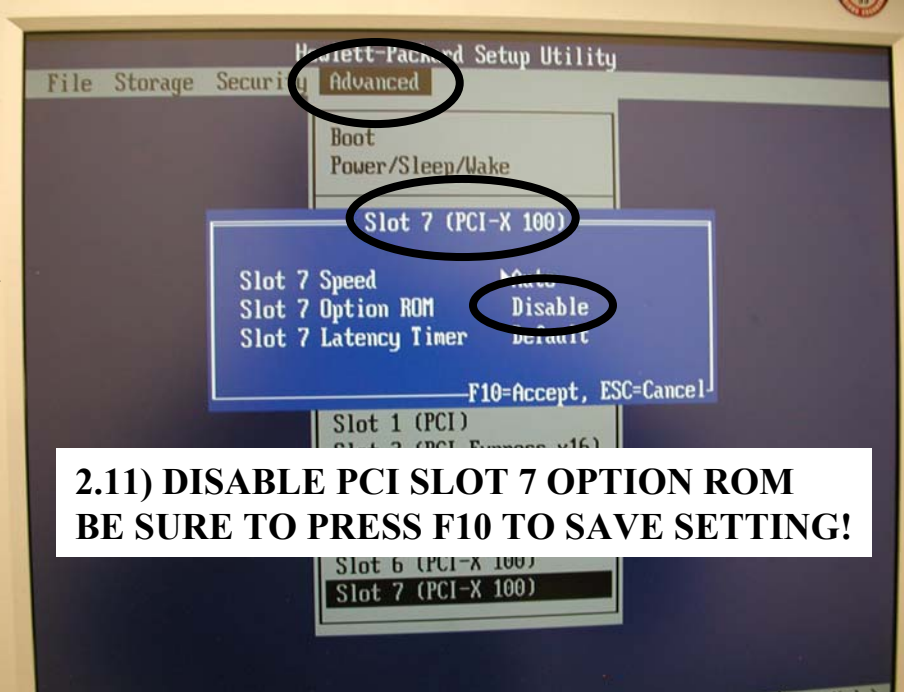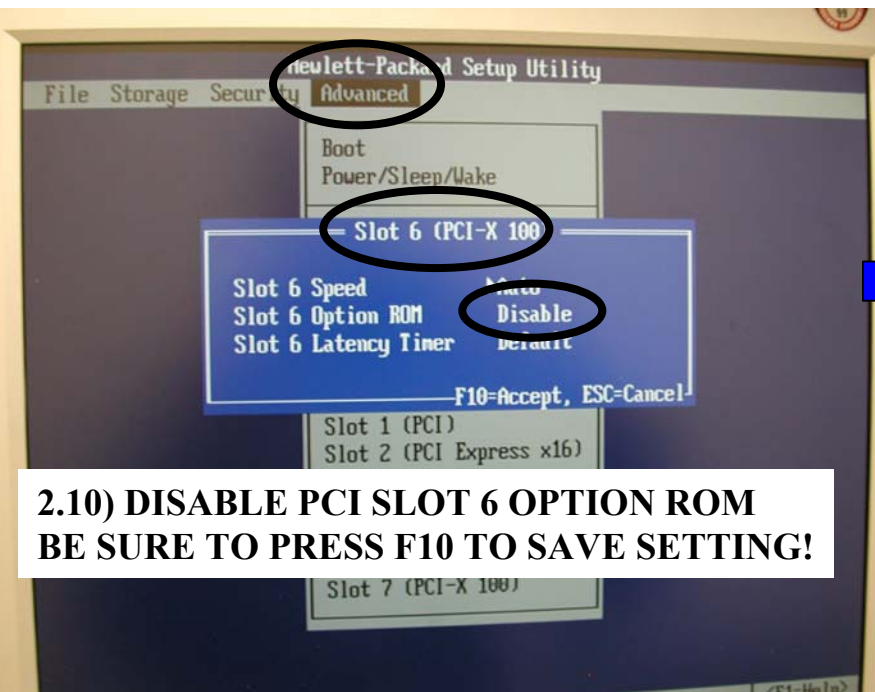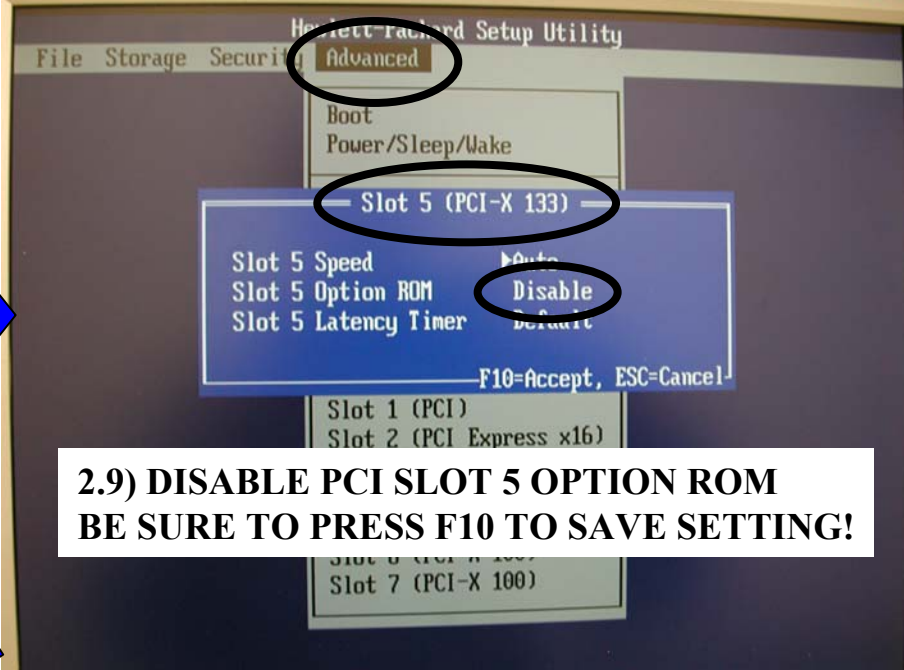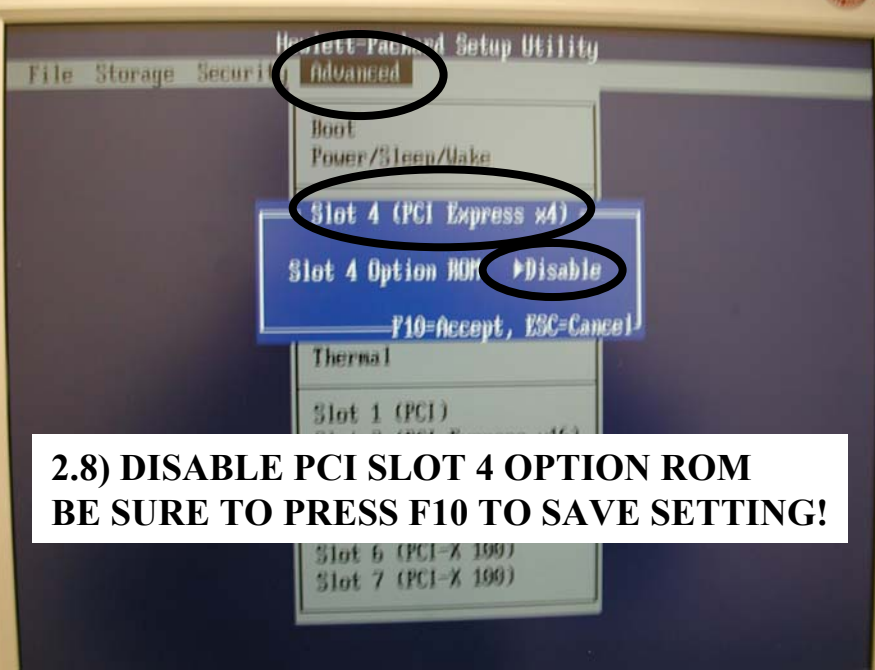

Heulett-Packard Setup Utility

File Storage Security Advanced

System Information
About

Set Time and Date

Save to Diskette
Restore from Diskette

Set Defaults and Exit
Ignore Changes and Exit
Save Changes and Exit

**2.12 SAVE CHANGES AND EXIT
BE SURE TO PRESS F10 WHEN PROMPTED
TO SAVE ALL SETTING CHANGES MADE!**