GE Medical Systems

**Technical
Publication**

**5499527-1-199
5505454**

Revision 001

Quick Starting Guide & Anti-Seismic Kit Mounting Instruction



EIZO LCD Monitor 19"

RadiForce RX150 GE and RadiForce RX150-S GE
RadiForce RS150-S GE and RadiForce RS150-ST5/ST8 GE

do not duplicate

Copyright© 2014 by General Electric Co.

Anti-Seismic Kit Mounting Instruction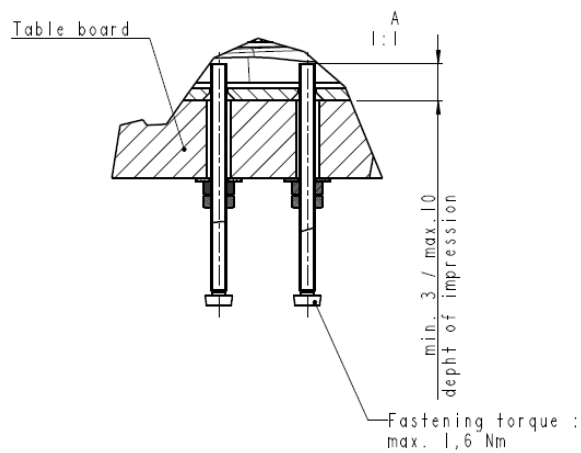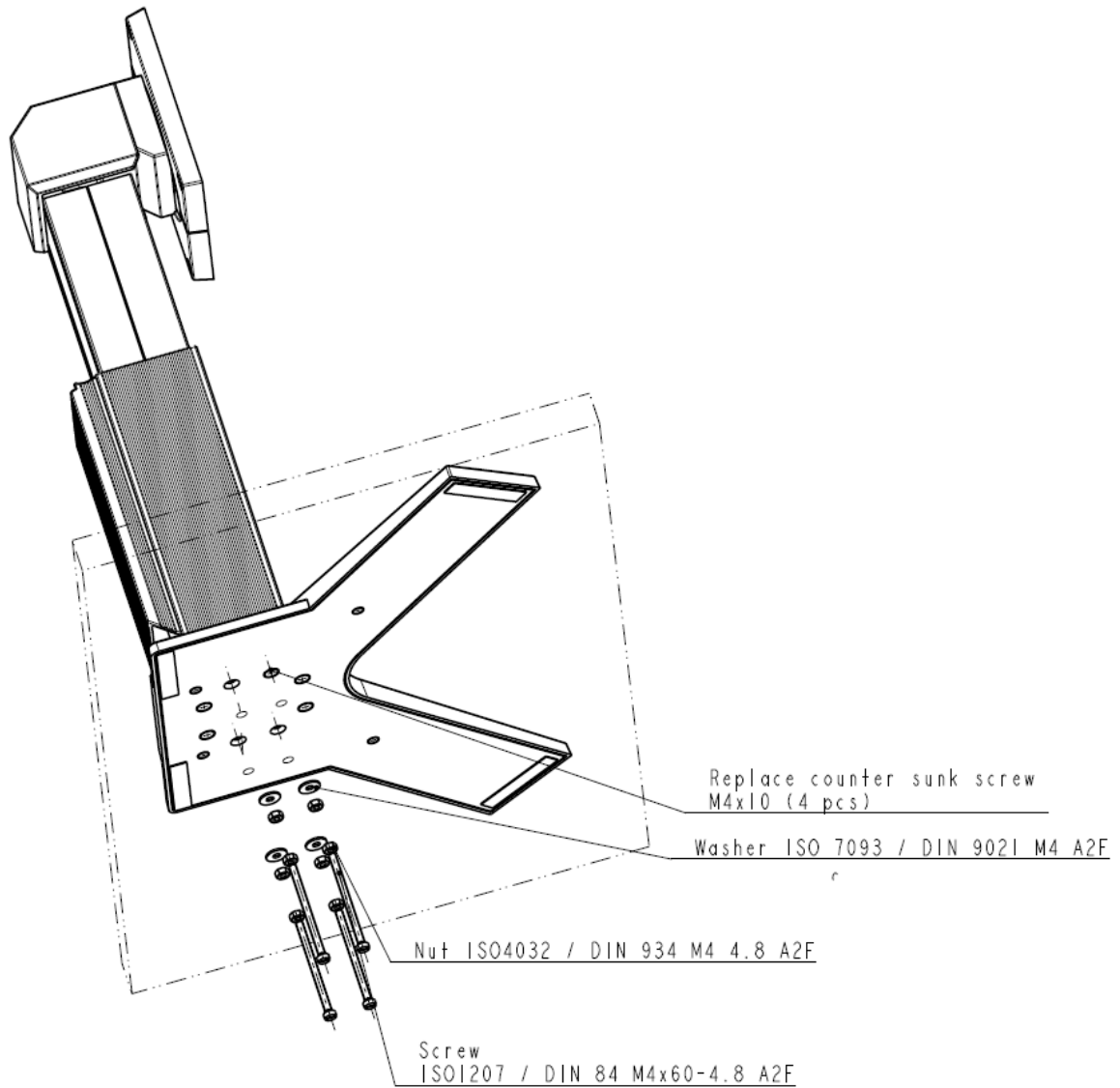

English	EN
French	FR
German	DE
Chinese	ZH
Czech	CS
Danish	DA
Dutch	NL
Hungarian	HU
Italian	IT
Japanese	JA
Norwegian	NO
Portuguese (Brazil)	PT
Russian	RU
Spanish	ES
Swedish	SV
Greek	EL
Bulgarian	BG
Estonian	ET

Finnish	FI
Indonesian	ID
Korean	KO
Latvian	LV
Lithuanian	LT
Romanian	RO
Serbian	SR
Slovakian	SK
Turkish	TR
Ukrainian	UA
Kroatian	HR

1 Safety Precautions

WARNING

X-RAY EQUIPMENT IS DANGEROUS TO BOTH PATIENT AND OPERATOR UNLESS MEASURES OF PROTECTION ARE STRICTLY OBSERVED


Though this equipment is built to the highest standards of electrical and mechanical safety, the useful xray beam becomes a source of danger in the hands of the unauthorized or unqualified operator. Excessive exposure to x-radiation causes damage to human tissue.

Therefore, adequate precautions must be taken to prevent unauthorized or unqualified persons from operating this equipment or exposing themselves or others to its radiation.


Before operation, persons qualified and authorized to operate this equipment should be familiar with the Recommendations of the International Commission on Radiological Protection, contained in Annals Number 60 of the ICRP, and with applicable national standards.

2 Monitor Installation

1. Remove the monitor from the plastic bag.
2. Unscrew the fixation screw and remove cover of the monitor.
3. Connect the video cable(s) (Display Port/ DVI / VBS) to the monitor and to the video source.
4. Plug the female end of the power cord into the power plug of the monitor and the male end into a power outlet.
5. Fix the cable cover on the monitor rear and screw it with the fixation screw.
6. Adjust the tilt angle and the monitor height, to a suitable position (on the desktop versions).
7. Switch on the monitor.
8. For optimal performance, it is recommended that you use the monitor under SXGA Resolution (1280 x 1024, refresh rate: 60 Hz).
9. If the picture is not displayed correctly, use the OSD Menu for adjustment.
10. Please refer to the service manual that is included on CD-ROM for further details about the OSD menu and functions.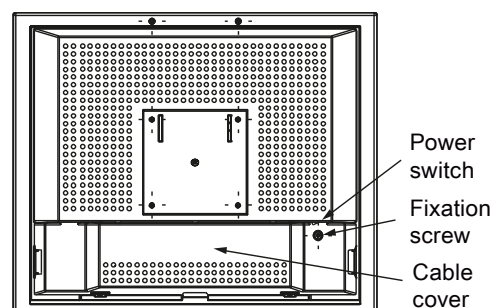


3 Connections

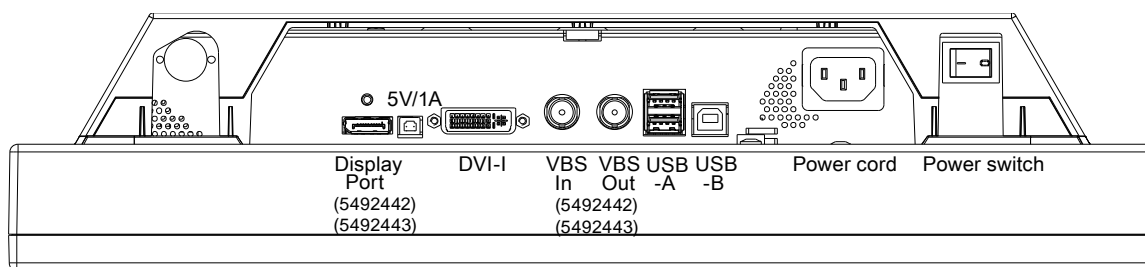EN

4 Control Elements

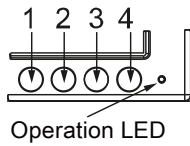Key	Action
1	<ul style="list-style-type: none">• Open OSD menu• Open main menu• Toggle between submenu
2	<ul style="list-style-type: none">• Toggle between main menu• Decrease/change value
3	<ul style="list-style-type: none">• Toggle between main menu• Increase/change value
4	Leave OSD or submenu (settings are retained)

Caution Information for end customer

No changes must be carried out on site by the user.
This also applies to settings made using the monitor keys.
These are therefore locked for certain applications. If settings have to be changed,
please contact the responsible servicing department.

5 Safety / Regulatory Specification

Safety standards	EN60601, IEC 60601
Approvals	CAN/CSA C22.2 No. 60601-1-8, CSA/us mark
Protection class	I
Degree of protection to DIN 40 050	IP20 Tested against IPX1
Conformity	CE to MPG 93/42/EC
CCC	Yes

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