# **NGPDU Series ABB AF116 Kxg Contactor Replacement Instruction**

**Service Documentation**

5505564-1EN

Rev 4

© 2020, GE Company,  
All Rights Reserved.

# ***LEGAL NOTES***

---

## **TRADEMARKS**

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

## **COPYRIGHTS**

All Material, Copyright © 2020 by GE Company, Inc. All rights reserved.

This page intentionally left blank.

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### DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent write "Damage In Shipment" on ALL copies of the freight or express bill BEFORE delivery is accepted or "signed for" by a GE representative or hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 6.
- Fill out the GIQ workflow for any items missing, damaged, OBF/FOI for in process installs:  
[http://supportcentral.ge.com/ProcessMaps/form\\_new\\_request.asp?prod\\_id=268679&form\\_id=573167&node\\_id=1916016&map\\_id=&reference\\_id=&reference\\_type](http://supportcentral.ge.com/ProcessMaps/form_new_request.asp?prod_id=268679&form_id=573167&node_id=1916016&map_id=&reference_id=&reference_type)
- Contact your local service coordinator for more information on this process.

### CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

## IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The GE Company, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the GE Company, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

## LITHIUM BATTERY CAUTIONARY STATEMENTS

**CAUTION**  
Risk of  
Explosion

**Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.**

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## OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the GEMS CQA Process to report all omissions, errors, and defects in this publication.

# Revision History

Revision	Date	Reason for change
1	02/19/14	Initial release.
2	03/20/15	Add step for cables routing.
3	06/13/19	Updated Figure 2-10 and it's Table on page 25 Added Table 2-1 for Kxg Module
4	04/21/20	Added 5505564-2 information per SPR HCSDM00600224

This page intentionally left blank.

# Table of Contents

## Chapter 1

**Introduction..... 17**

**Section 1.0  
System Requirement..... 17**

**Section 2.0  
Material List ..... 17**

**Section 3.0  
Flowchart ..... 18**

## Chapter 2

**Installation Procedure..... 19**

**Section 1.0  
Overview ..... 19**

**Section 2.0  
Replacement Procedure ..... 19**

2.1 Prerequisite..... 19

2.2 Preparation ..... 19

2.3 Remove the Old Kxg Contactor, related Cables and Bracket..... 20

2.3.1 For GECL10..... 20

2.3.2 For ABB A110..... 21

2.4 Install the new Kxg Contactor and Reconnect Cables..... 22

**Section 3.0  
Completion..... 27**

# Chapter 1

## Introduction

### Section 1.0 System Requirement

This instruction is applied for all NGPDU Series system with Kxg GECL10 and ABB A110 contactor.

- Kxg GECL10 contactor: 2351488
- Kxg ABB A110 contactor: 5337228

### Section 2.0 Material List

Verify that the contents of this service kit are correct according to the Material List below.

#### NGPDU ABB AF116 Contactor service kit (5505564)

ITEM	PART NO.	DESCRIPTION	QTY.
1	5438062	ABB AF116 Contactor	1.0
2	5321401-2	ABB AF140 Bracket	1.0
3	1107-M5C014-14	Cross Recessed Pan Head Screw, M5-0.8 X14mm Long, Grade 4.8 Steel, Zinc plate RoHS Fastener	4.0
4	2000-M5-03	Flat Washer, ISO7089, M5, Steel 200HV, Zinc Plated, RoHS Fastener	4.0
5	5505564-1EN	NGPDU Series ABB AF116 Kxg Contactor Replacement Instruction	1.0
6	U0201AC	Clamp CV_100	5.0
7	U0203AC	Clamp CV_200	5.0


#### NGPDU ABB AF116 Contactor and CEL19 Auxiliary Contact service kit (5505564-2)

ITEM	PART NO.	DESCRIPTION	QTY.
1	5438062-2	ABB AF116 Contactor	1.0
2	5321401-2	ABB AF140 Bracket	1.0
3	1107-M5C014-14	Cross Recessed Pan Head Screw, M5-0.8 X14mm Long, Grade 4.8 Steel, Zinc plate RoHS Fastener	4.0
4	2000-M5-03	Flat Washer, ISO7089, M5, Steel 200HV, Zinc Plated, RoHS Fastener	4.0
5	5505564-1EN	NGPDU Series ABB AF116 Kxg Contactor Replacement Instruction	1.0
6	U0201AC	Clamp CV_100	5.0
7	U0203AC	Clamp CV_200	5.0

# Section 3.0

## Flowchart

The Illustration below shows the main flow of the Kxg contactor replacement procedures.
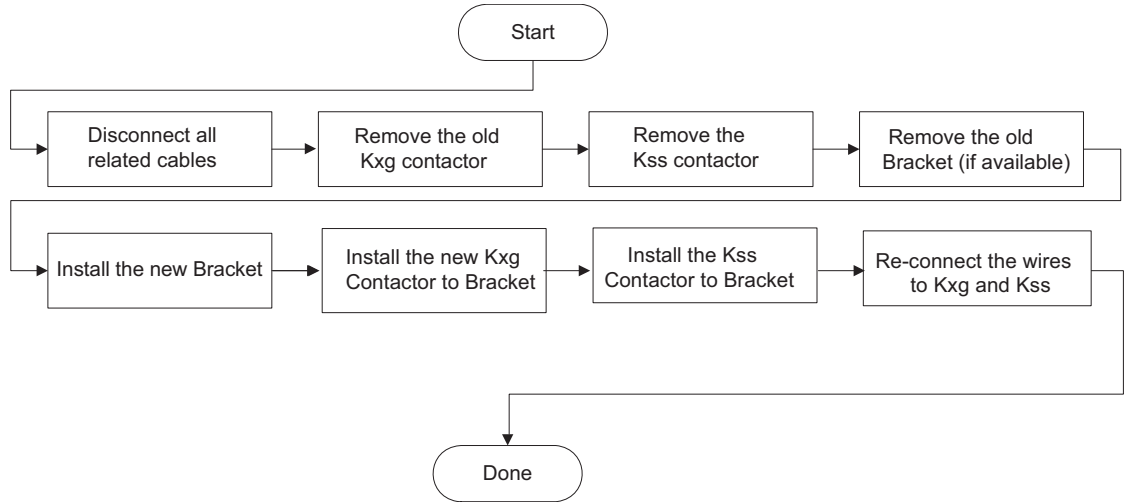
# Chapter 2

## Installation Procedure

---

### Section 1.0 Overview

New ABB AF116 contactor, which has much more reliability, is introduced into NGPDU series to replace old contactors (GECL10 or ABB A110).

### Section 2.0 Replacement Procedure

#### 2.1 Prerequisite

N/A

#### 2.2 Preparation

Understand and follow the Safety Guidelines Manual.

**WARNING**

**POTENTIAL FOR SHOCK.  
VOLTAGE MAY BE PRESENT.  
USE PROPER LOCKOUT/TAGOUT PROCEDURES BEFORE INSPECTING  
COMPONENTS.**

- 1.) Switch OFF the OC main power.
- 2.) **Switch OFF the breaker in the Power Distribution Box (hospital supply).**


Note: If there is UPS, switch OFF the UPS output.

- 3.) Remove power to the NGPDU using proper Lockout/Tagout procedures.
- 4.) Remove the Top and Front Covers from the NGPDU.
- 5.) Switch OFF the CB1 at the front side of the NGPDU.
- 6.) Remove the Front Plastic Covers. (See [Figure 2-1](#))

Figure 2-1 Remove the Plastic Covers

GECL10

ABB A110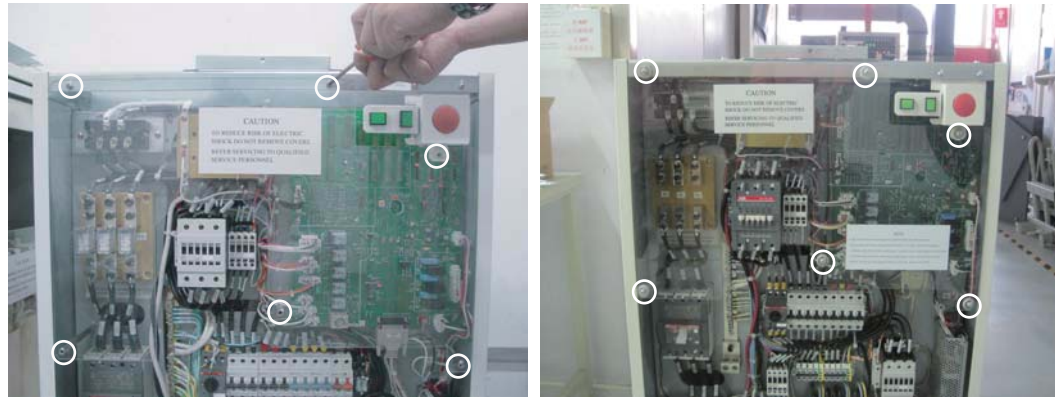


## 2.3 Remove the Old Kxg Contactor, related Cables and Bracket

### 2.3.1 For GECL10

- 1.) Disconnect wires from the Kxg and Kss Contactors, please pay attention to the cable label and location, mark labels if necessary to avoid mis-wiring when restore back.

Figure 2-2 Kxg and Kss of GECL10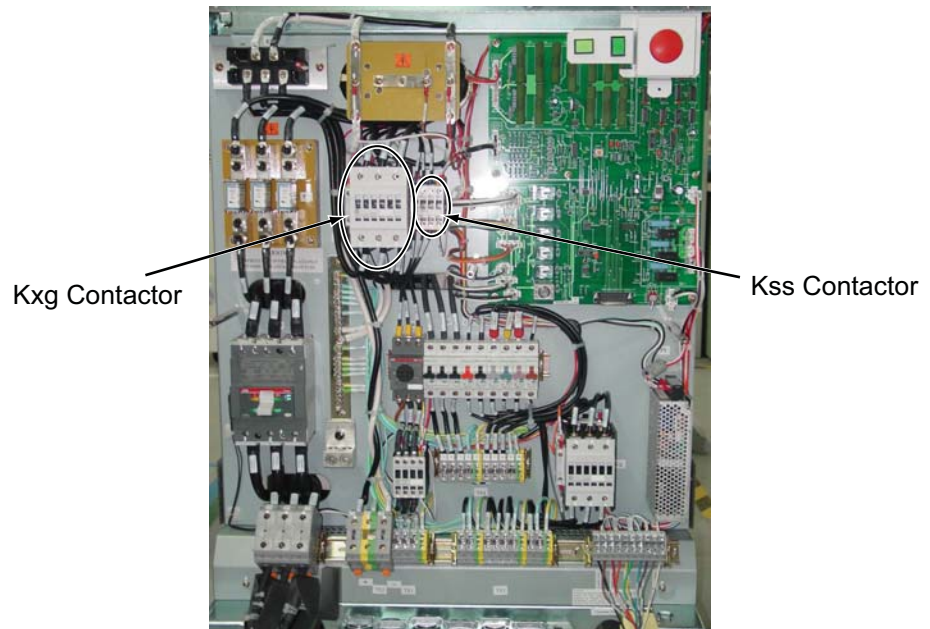


- 2.) Remove the Kxg and Kss Contactors by unscrewing their screws.

### 2.3.2 For ABB A110


- 1.) Disconnect wires from the Kxg and Kss Contactors, please pay attention to the cable label and location, remark labels if necessary to avoid mis-wiring when restore back.

Figure 2-3 Kxg and Kss of ABB A110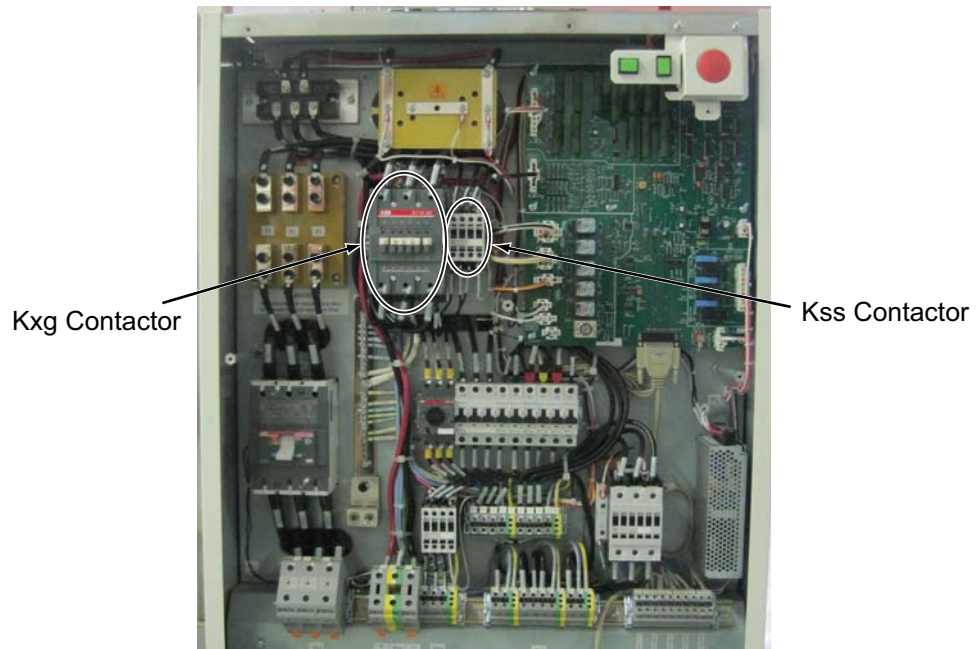

- 2.) Remove the Kxg and Kss Contactors by unscrewing their screws on the corners.
- 3.) Remove the Bracket by unscrewing its four screws.

Figure 2-4 Remove the Contactor Bracket

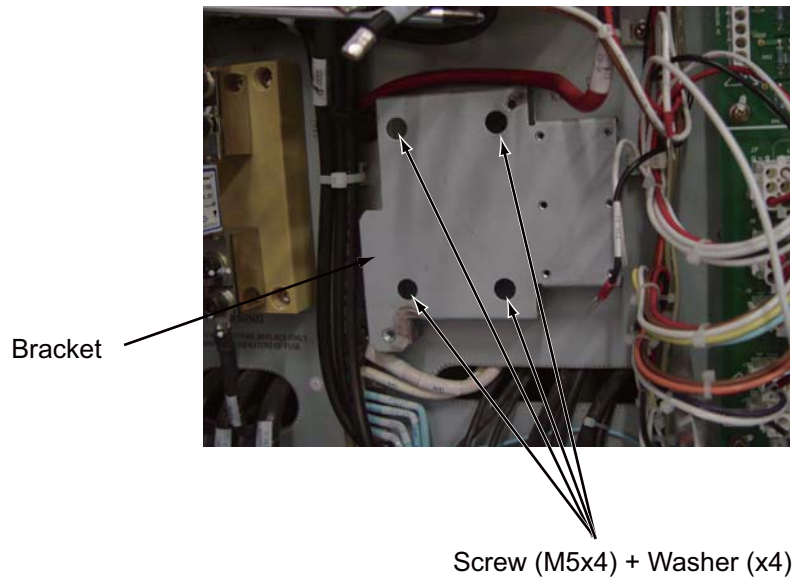## 2.4 Install the new Kxg Contactor and Reconnect Cables


- 1.) Install the new Bracket to NGPDU frame by screwing its four screws and washers (shipped with kit **5505564** or **5505564-2**).

**Figure 2-5 New Bracket**


- 2.) Confirm the Auxiliary contactor of the new Kxg contactor assembles and works well. Push and release the movable part of the Kxg contactor several times, confirm the movable part of the Auxiliary contactor follows the actions of the Kxg contactor.

**Figure 2-6 New Contactor Assembly**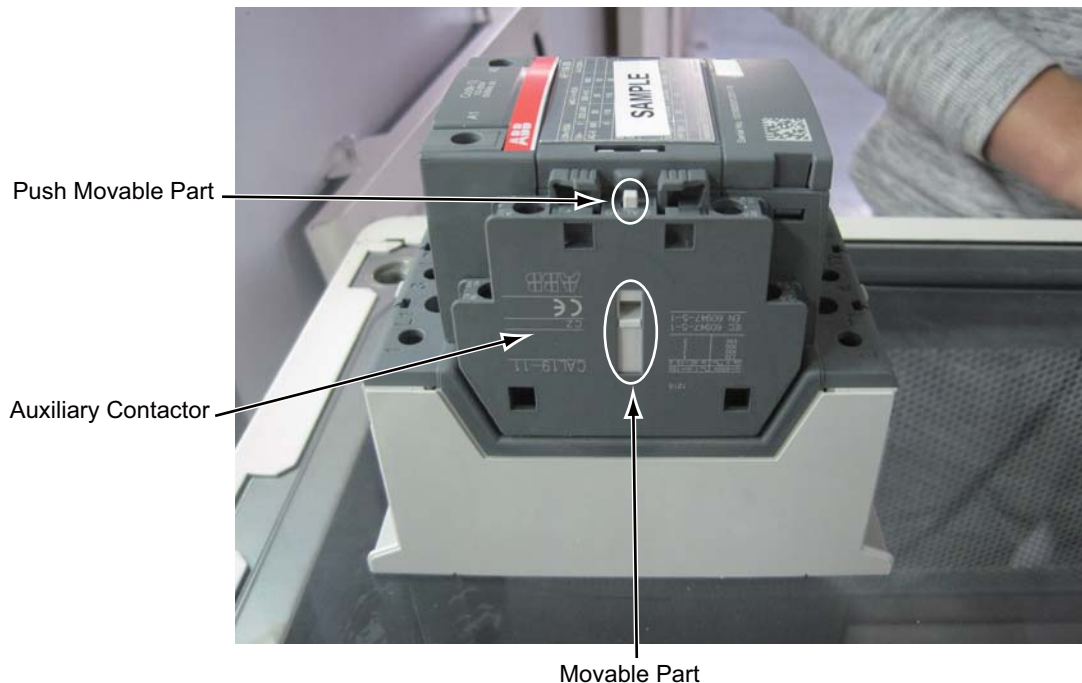

- 3.) Install the Kss contactor on the Bracket by screwing its old screws.
- 4.) Install the new Kxg contactor assy on the Bracket by screwing its four screws (shipped with ABB AF116 contactor ).


Figure 2-7 Install the Kxg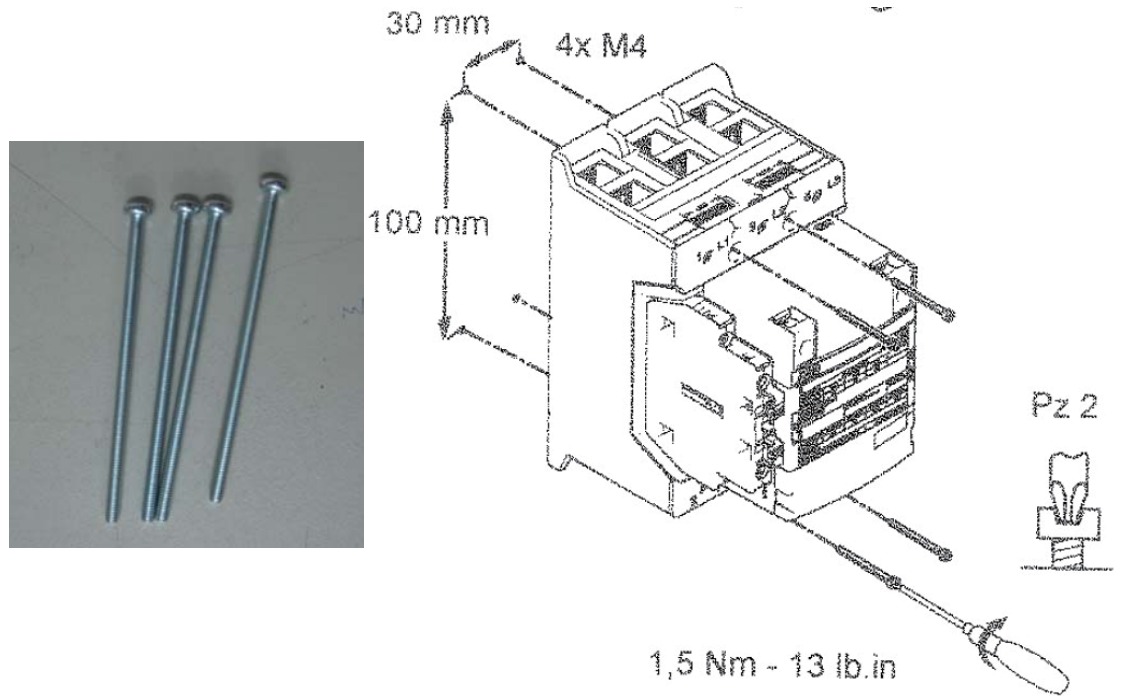

2 - INSTALLATION PROCEDURE


- 5.) Loosen all the screws that fix wires of Kxg contactor (Note the highlighted circle in Figure 2-8).

Figure 2-8 Kxg and Kss contactors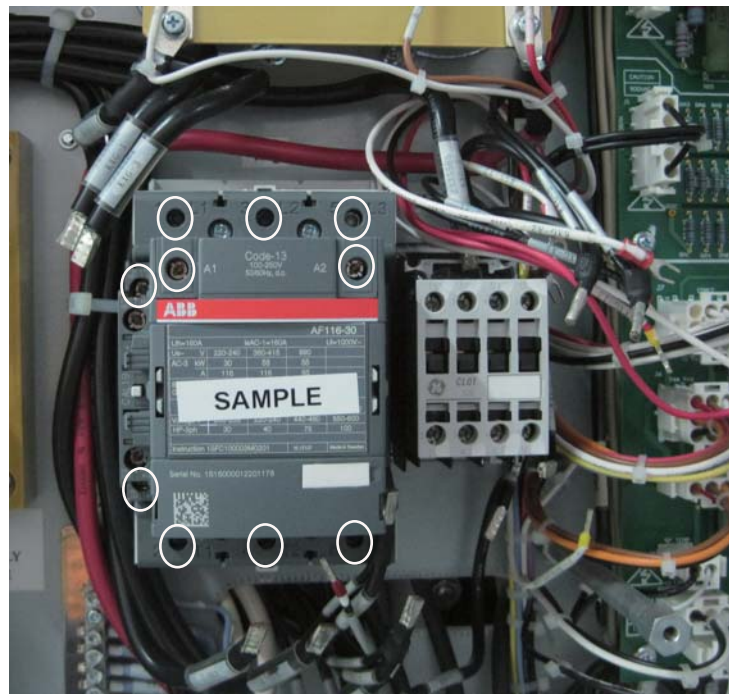

**Note:** As the following figure shows, connect wires to the Kxg contactor to avoid loosening.

**Figure 2-9 Connect the wires**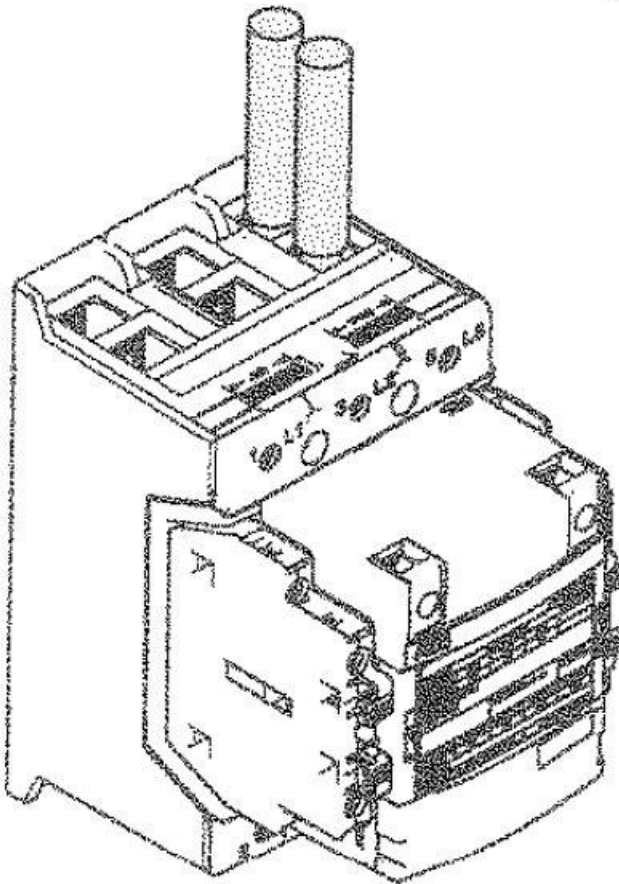


- 6.) Reconnect all of the wires to the Kxg and Kss according to remark label. (See [Figure 2-8](#) and [Figure 2-10](#))
- 7.) Check all connections for tightness.

**Cable installation sequence:**

- Kxg-L1, L2, L3
- Kxg-A1, A2
- Other wires

Figure 2-10 Reconnect the wires

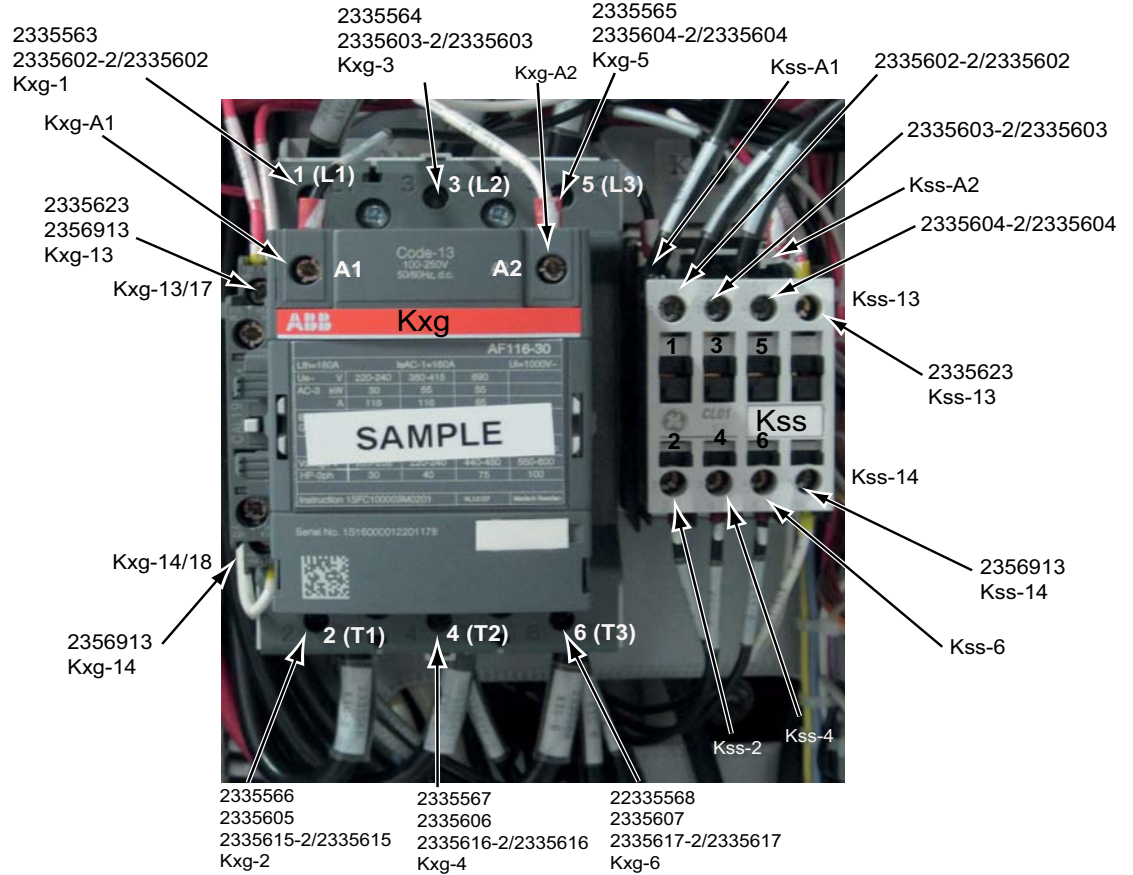2 - INSTALLATION PROCEDURE

	Contactor	Cable Label (Cable 1)	Cable Label (Cable 2)	Cable Label (Cable 3)
<b>KXG</b>	KXG 1 (L1)	2335563 (KXG 1)	2335602-2/2335602	KXG-1
	KXG 3 (L2)	2335564 (KXG 3)	2335603-2/2335603	KXG-3
	KXG 5 (L3)	2335565 (KXG 5)	2335604-2/2335604	KXG-5
	KXG 2 (T1)	2335566 (KXG 2)	2335605	2335615-2/2335615
	KXG 4 (T2)	2335567 (KXG 4)	2335606	2335616-2/2335616
	KXG 6 (T3)	2335568 (KXG 6)	2335607	2335617-2/2335617
	KXG A1	KXG-A1		
<b>KXG Aux</b>	KXG 13/17	2335623 (KXG-13)	2356913 (KXG-13)	
	KXG 14/18	2356913 (KXG-14)		
<b>KSS</b>	KSS-1	2335602-2/2335602		
	KSS-3	2335603-2/2335603		
	KSS-5	2335604-2/2335604		
	KSS-2	KSS-2		
	KSS-4	KSS-4		
	KSS-6	KSS-6		
	KSS A1	KSS-A1		
	KSS A2	KSS-A2		
	KSS 13	2335623 (KSS-13)		
	KSS 14	KSS-14		

**Table 2-1 List of Kxg Model & Aux Contact No.**

<b>Contactor Type</b>	<b>Kxg Aux Contact No.</b>
GECL10	Kxg 13, 14
ABB A110	Kxg 13, 14
ABB AF116 (CAL 19)	Kxg 13, 14
ABB AF116 (CEL 19)	Kxg 17, 18

8.) Route cables with tie-wraps as shown below.

Figure 2-11 Cable Routing

2 – INSTALLATION  
PROCEDURE

### Section 3.0 Completion

- 1.) Make sure the wires are correctly connected and in good connection.
- 2.) Restore the NGPDU to original configuration.
- 3.) Re-power ON system and verify correct operation.

[www.gehealthcare.com](http://www.gehealthcare.com)