GE Healthcare

# Accessory Interface Hardware Installation Manual

OPERATING DOCUMENTATION

5317931-1EN

Rev 2

© 2013 General Electric Company.

All rights reserved.

# **LEGAL NOTES**

## **TRADEMARKS**

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

## **COPYRIGHTS**

All Material Copyrighted (c) 2013 by the GE Company, All rights reserved.

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## **DAMAGE IN TRANSPORTATION**

All packages should be closely examined at time of delivery. If damage is apparent, have notation "damage in shipment" written on all copies of the freight or express bill before delivery is accepted or "signed for" by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 8.
- Fill out a report on <http://us44hdd21/sctq/InstallFulfill/InstalFulfillment.htm>
- Contact your local service coordinator for more information on this process.

Rev. Jan. 5, 2005

## **CERTIFIED ELECTRICAL CONTRACTOR STATEMENT**

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

## **IMPORTANT...X-RAY PROTECTION**

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

## LITHIUM BATTERY CAUTIONARY STATEMENTS

### CAUTION

**Risk of Explosion.**

**Danger of explosion if battery is incorrectly replaced.**

**Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.**

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## OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the Healthcare PQR Process to report all omissions, errors, and defects in this publication.

# Revision History

Revision	Date	Reason for change
1	Nov 07, 2012	Initial release.
2	Jan 29, 2013	Updated Left Accessory Panel. Delete figure for forward tilt button. Add step for EKG/ECG LAN Port connection.

*Page Left Intentionally Blank*

# Table of Contents

---

## Preface

**Publication Conventions ..... 17**

**Section 1.0**

**Safety & Hazard Information ..... 17**

1.1 Text and Character Representation..... 17

1.2 Graphical Representation ..... 18

**Section 2.0**

**Publication Conventions ..... 18**

2.1 General Paragraph and Character Styles..... 18

2.2 Page Layout..... 19

2.3 Computer Screen Output/Input Text Character Styles ..... 19

2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)..... 20

## Chapter 1

**Overview..... 21**

**Section 1.0**

**Introduction ..... 21**

**Section 2.0**

**Material List ..... 22**

## Chapter 2

**Accessory Interface Hardware Installation ..... 25**

**Section 1.0**

**Installation Procedure..... 25**

1.1 Base Cover Removal..... 25

1.2 Install IPC and Accessory Panel..... 26

1.3 IPC Harness Assembly Connections (5316429-3) ..... 31

**Section 2.0**

**Finish Installation..... 36**

2.1 LED Inspection of IPC Board..... 36

2.2 LED Inspection of LAN Switch..... 37

2.3 Finalization..... 37

*Page Left Intentionally Blank*

# Preface

## Publication Conventions

Purpose: Standardize conventions for representing information in a uniform way of communicating information to a reader in a consistent manner. Conventions are used so that the reader can easily recognize the actions or decisions that must be made. There are a number of character and paragraph styles used in this publication to accomplish this task. Please become familiar with them before proceeding forward.

## Section 1.0 Safety & Hazard Information

### 1.1 Text and Character Representation

Within this publication, different paragraph and character styles have been used to indicate potential hazards. Paragraph prefixes, such as hazard, caution, danger and warning, are used to identify important safety information. Text (Hazard) styles are applied to the paragraph contents that is applicable to each specific safety statement. Words describe the type of potential hazard that may be encountered and are placed immediately before the paragraph it modifies. Safety information will normally include:

- Type of potential Hazard
- Nature of potential injury
- Causative condition
- How to avoid or correct the causative condition

#### EXAMPLES OF HAZARD STATEMENTS USED

It's important that you read and understand hazard statements, and not just ignore them.

**DANGER  
SEVERE  
PERSONAL  
INJURY**

**DANGER IS USED WHEN A HAZARD EXISTS WHICH WILL CAUSE SEVERE PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:**

- **ELECTROCUTION**
- **CRUSHING**
- **RADIATION**

**WARNING  
SERIOUS  
PERSONAL  
INJURY**

**WARNING IS USED WHEN A HAZARD EXISTS WHICH COULD OR CAN CAUSE SERIOUS PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:**

- **POTENTIAL FOR SHOCK**
- **EXPOSED WIRES**
- **FAILURE TO TAG AND LOCKOUT SYSTEM POWER COULD ALLOW FOR UN-COMMANDED MOTION.**

**CAUTION  
Minor  
Personal  
Injury**


**Caution is used when a hazard exists which can or could cause minor injury to self or others if instructions are ignored. They include for example:**

- **Loss of critical patient data**
- **Crush or pinch points**
- **Sharp objects**

- NOTICE** Notice is used when a hazard is present that can cause property damage but has absolutely no personal injury risk. They can include:
- Property Damage Only**
- Disk drive will be destroyed
  - Internal mechanical damage, such as to the x-ray tube
  - Coasting the rotor through resonance.

## 1.2 Graphical Representation

Important information will always be preceded by the exclamation point  contained within a triangle, as seen throughout the Preface chapter. In addition to text, several different graphical icons (symbols) may be used to make you aware of specific types of hazards that could possibly cause harm.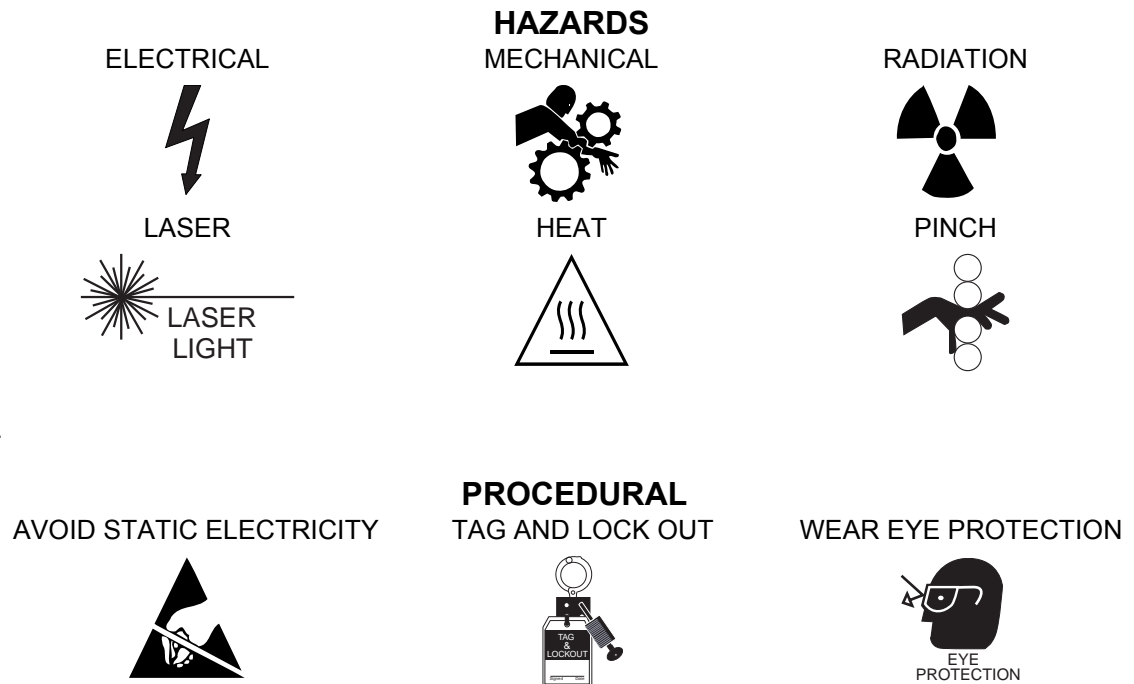

## Section 2.0 Publication Conventions

### 2.1 General Paragraph and Character Styles

Prefixes are used to highlight important non-safety related information. Paragraph prefixes (such as purpose, example, comment or note) are used to identify important but non-safety related information. Text styles are also applied to text within each paragraph modified by the specific prefix.

#### EXAMPLES OF PREFIXES USED FOR GENERAL INFORMATION


**Purpose:** Introduces and provides meaning as to the information contained within the chapter, section or subsection, Such as used at the beginning this chapter for example.

**Note:** Conveys information that should be considered important to the reader.

**Example:** Used to make the reader aware that the paragraph(s) that follow are examples of information possibly stated previously.

**Comment:** Represents “additional” information that may or may not be relevant to your situation.

## 2.2 Page Layout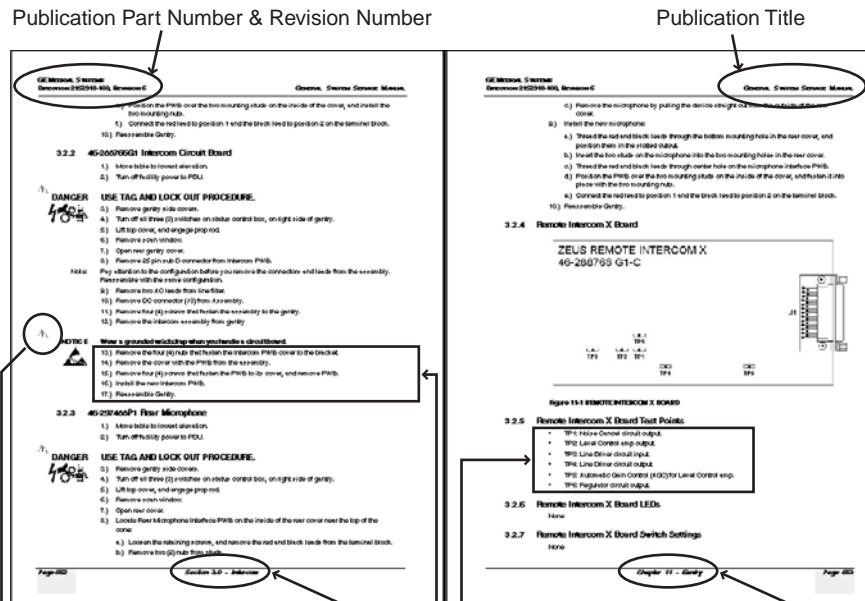

The current section and its title are always shown in the footer of the left (even) page.

An exclamation point in a triangle is used to indicate important information to the user.

Paragraphs preceded by **Alphanumeric** characters (e.g. numbers) contain information that must be followed in a **specific order**.

The current chapter and its title are always shown in the footer of the right (odd) page.

Paragraphs preceded by a **symbol** (e.g. bullets) contain information that has **no specific order**.

Headers and footers in this publication are designed to allow you to quickly identify your location. The document’s part number and revision number appears in every header on every page. Odd numbered page footers indicate the current chapter, its title and current page number. Even page footers show the current section and its title, as well current page number.

## 2.3 Computer Screen Output/Input Text Character Styles

Within this publication, mono-spaced character styles (fonts) are used to indicate computer text that’s either screen input and output. Mono spaced fonts, such as courier, are used to indicate text direction. When you type at your keyboard, you are generating computer input. Occasionally you will see the math operators “greater than” and “less than” symbols used to indicate the start and finish of variable output. When reading text generated by the computer, you are reading it as computer generated output. In addition to direction, characters are italicized (e.g. *italics*) to indicate information specific to your system or site.

**Example:**  
Fixed Output

This paragraph’s fonts represents computer generated screen “fixed” output. It’s output is fixed from the sense that it does not vary from application to application. It’s the most commonly used style used to indicate filenames, paths and text that does not change from system to system. The character style used is a fixed width such as courier.

**Example:** *This paragraph's font represents computer screen output that is "variable". Its used to represent output that varies from application to application or system to system. Variable output is sometimes found placed between greater than and lesser than operators for clarification. For example: <variable\_ouput> or <3.45.120.3>. In both cases, the < and > operators are not part of the actual input.*

**Example:** **This paragraph's font represents fixed input. It's computer input that is typed-in via the keyboard. Typed input that does not vary from application to application or system to system. Fixed text the user is required to supply as input. For example: cd /usr/3p**

**Example:** *This paragraph's font represents computer input that can vary from application to application or system to system. With variable text, the user is required to supply system dependent input or information. Variable input sometimes is placed between greater-than and less-than operators. For example: <variable\_input>. In these cases, the (<>) operators would be dropped prior to input. For example: ypcat hosts | grep <3.45.120.3> would be typed into the computer as ypcat hosts | grep 3.45.120.3 without the greater-than and less-than operators.*

## 2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)

Different character styles are used to indicate actions requiring the reader to press either a hard or soft button, switch or key. Physical hardware, such as buttons and switches, are called hard keys because they are hard wired or mechanical in nature. A keyboard or on/off switch would be a hard key. Software or computer generated buttons are called soft keys because they are software generated. Software driven menu buttons are an example of such keys. Soft and hard keys are represented differently in this publication.

**Example:** A power switch **ON/OFF** or a keyboard key like **ENTER** is indicated by applying a character style that uses both over and under-lined bold text that is bold. This is a hard key.

**Example:** Whereas the computer **MENU** button that you would click with your mouse or touch with your hand uses over and under-lined regular text. This is a soft key.

**END OF SECTION**

# **Chapter 1**

## **Overview**

### **Section 1.0**

#### **Introduction**

The Accessory Interface Hardware option provides interface for the following options.

- Cardiac Gating
- Advantage 4D Respiratory Gating
- Smart Step
- Xtream Injector

The Accessory Interface Hardware option is shipped ready for immediate installation. The following installation procedure describes how to install that hardware on the Gantry.

Refer to [Table 1-1](#) for a listing of the parts in the kit.

## Section 2.0 Material List

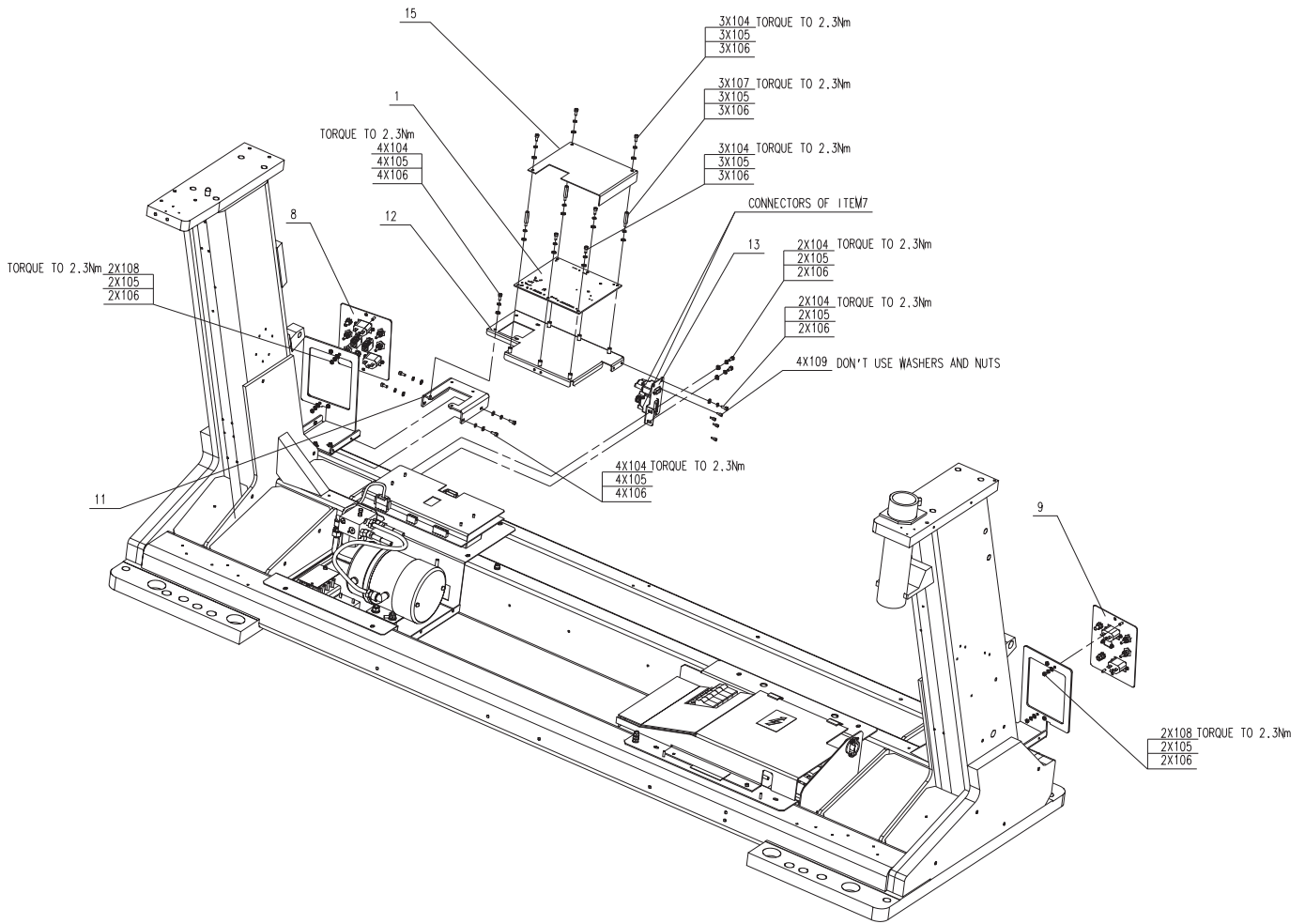
Figure 1-1 Accessory Interface Hardware Assembly 5317931-3ADW

Mark	Part Number	Quantity	Description
1	5136002-2	1.0	Integrated Peripheral Controller PWA
7	5316429-3	1.0	IPC Harness Assembly - Support Fuse Interface
8	5306106-3	1.0	Accessory Panel LH ASM, with Fuse
9	5306107-3	1.0	Accessory Panel RH ASM, with Fuse
10	5306201	1.0	Cable, Cardiac Pulmonary, CT HD
11	5309358	1.0	BSD V3 IPC MNT brkt cushion
12	5309359	1.0	BSD V3 IPC MNT brkt
13	5309361	1.0	BSD V3 connector MNT brkt
15	5304664	1.0	Cover IPC Board, Kitty Hawk
17	5394300-3	20.0	Nylon Cable Tie, 290mm Long, 3.70mm Wide, RoHS Compliant
104	1000-M4C010-04	18.0	Hexagon Socket Head Cap Screw, ISO 4762, M4-0.7, X10mm Long, Grade 8.8 Steel, Zinc Plated
105	2203-M4-07	25.0	Curved Lock Washer, DIN 128A, M4, Spring Steel, Zinc Plated, RoHS Fastener
106	2000-M4-03	25.0	Washer Plain - Large 4.3 mm 9 mm, HV200, Steel, Zinc Plated, RoHS Fastener
107	5214786-52	3.0	STANDOFF-SPACER MALE-FEMALE M4-0.7 X 15 mm Long, RoHS Stud
108	3000-M4C-04	4.0	Hexagon Nut, ISO4032, M4-0.7, Grade 8 Steel, Zinc Plated, RoHS Fastener
109	46-221417P4	4.0	FEMALE SCREWLOCKS FOR HD CONNECTORS. 4-40 THREAD, 0.470 OVERALL LENGTH YELLOW CHROMATE, TWO PER KIT. 0.312 BOTTOM, 0.094 PANEL THK. MAX KIT INCLUDES NUTS AND WASHERS See drawing for approved vendor
200	5317931-1EN	1	Instruction

**Table 1-1 5317931-3 Material List**

*Page Left Intentionally Blank*

# Chapter 2

## Accessory Interface Hardware Installation

### Section 1.0 Installation Procedure

**DANGER RISK OF ELECTRIC SHOCK. USE PROPER LOCKOUT/TAGOUT PROCEDURES.**

#### 1.1 Base Cover Removal

- 1.) Remove the two gantry side covers. Refer to your System Service Manual for any needed instruction.
- 2.) Turn **OFF** all 3 switches (Axial Drive, HVDC, 120 VAC) on the service switch panel.
- 3.) Shutdown system software and turn off the console power switch.
- 4.) Remove all system power at the main disconnect (A1) panel. Perform proper Lockout/Tagout power control procedures.
- 5.) Remove both left and right Gantry side base covers (items 6 & 7, respectively, in [Figure 2-1](#)) and then both rear base covers (items 8 & 9, in [Figure 2-1](#)).


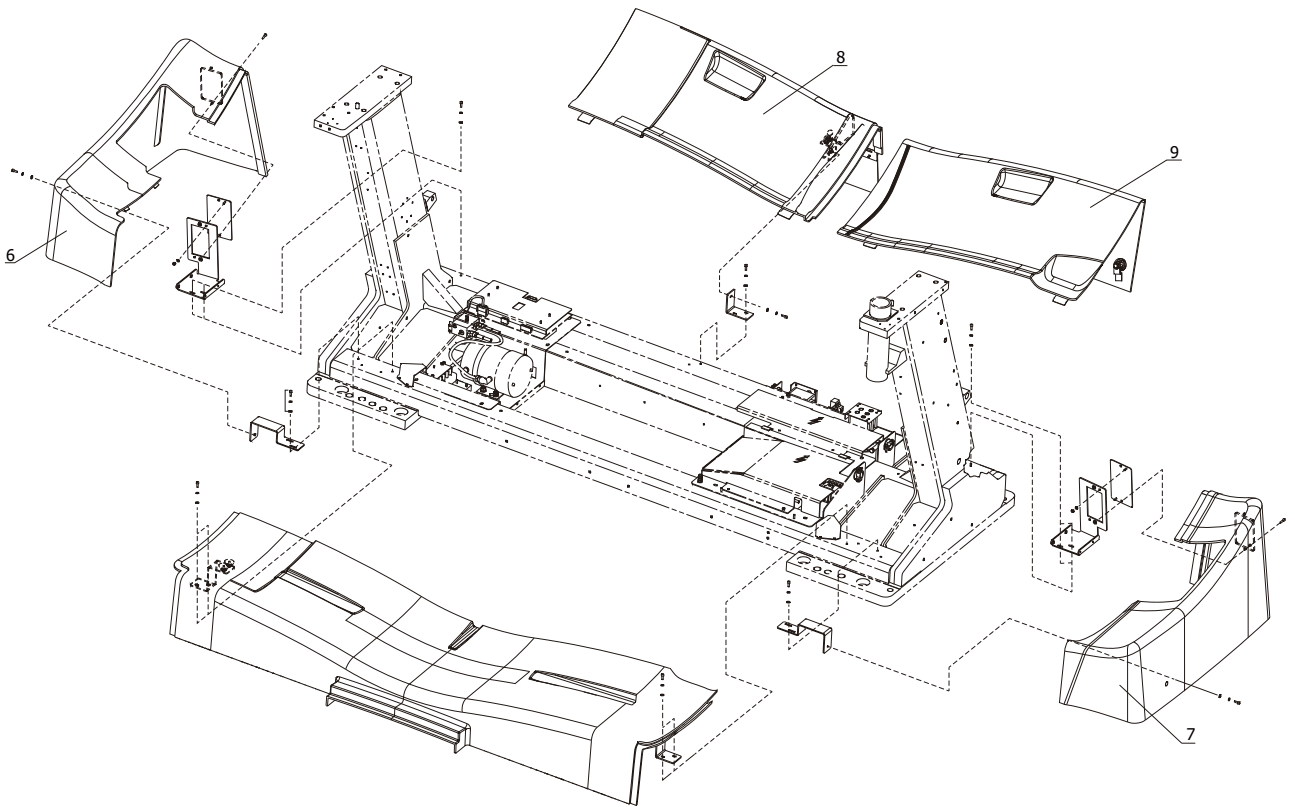
Figure 2-1 Gantry Base Covers

## 1.2 Install IPC and Accessory Panel


- 1.) Assemble the IPC and the left accessory panel.

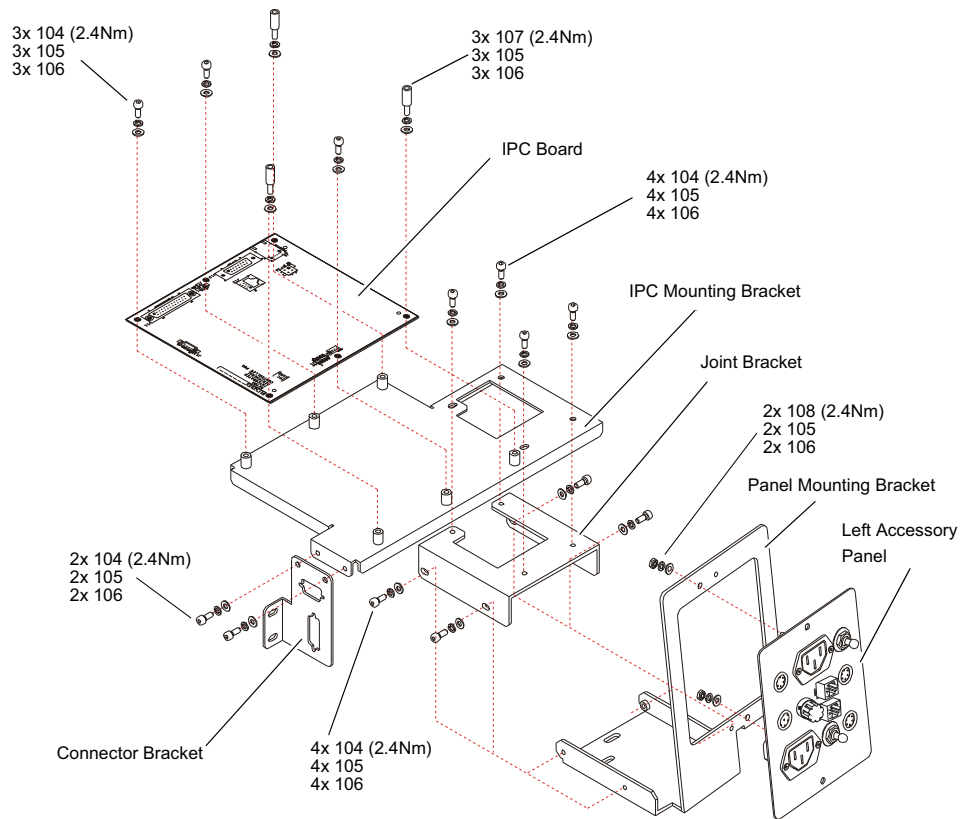**Figure 2-2 IPC Board and Left Accessory Panel**


- Install the IPC board to the IPC mounting bracket by screwing its three screws with washers.
- Screw three studs with washers to the IPC board.
- Install the IPC plastic cover by screwing its three screws.
- Install the connector bracket to the IPC mounting bracket by screwing its two screws with washers.
- Install the left accessory panel to the left panel mounting bracket by screwing its two screws with washers.

**Note:** Pay attention to the correct installation for the left/right accessory panel and panel mounting bracket. The definition of “Left and Right” are based on the facing gantry front cover.

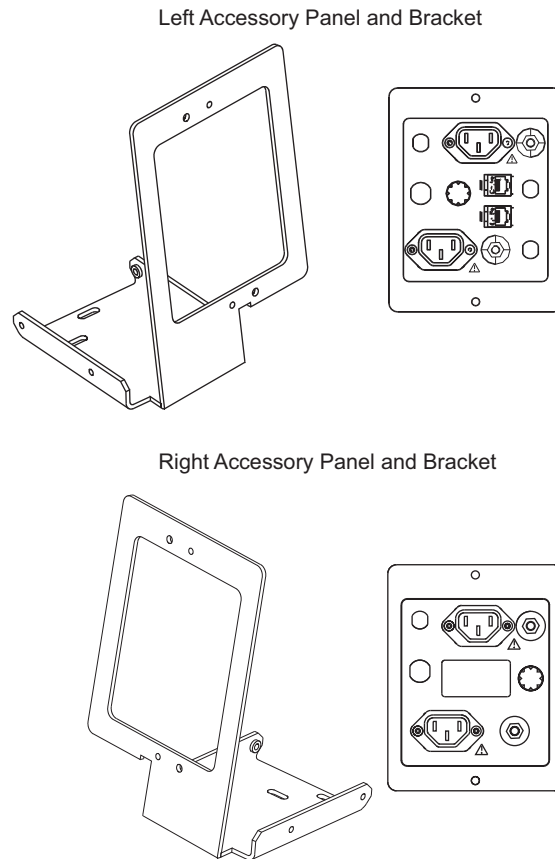
**Figure 2-3 Left and Right Accessory Panel and Bracket**

- f.) Install the joint bracket to the left accessory panel mounting bracket by screwing its four screws with washers.
- g.) Fasten the joint bracket to the IPC mounting bracket by screwing its four screws with washers.

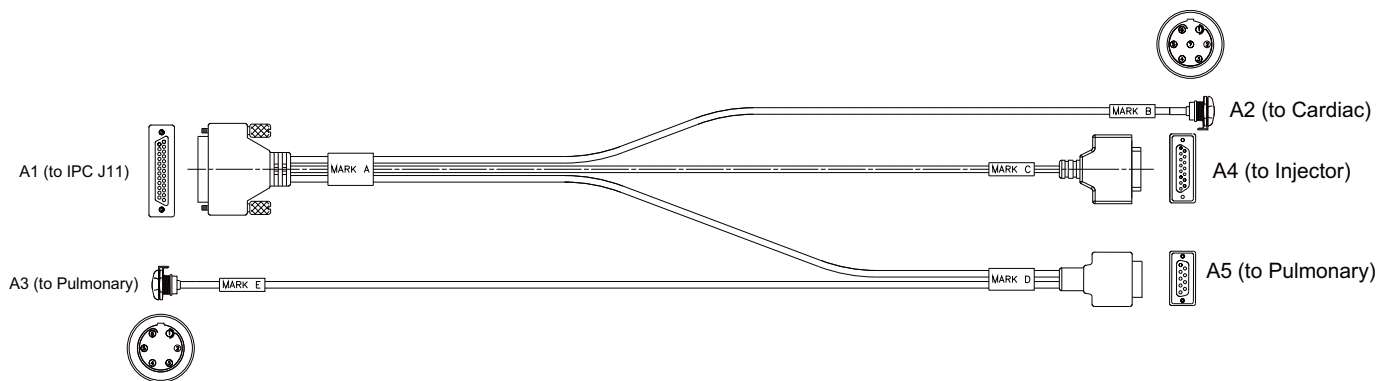
**Figure 2-4 Cardiac/Pulmonary Cable (5306201)**

- h.) Connect the Cardiac/Pulmonary cable (5306201).
  - I.) Connect the cable A1 to the IPC board J11.
  - II.) Connect the cable A2 to the left accessory panel cardiac.
  - III.) Connect the cable A3 to the left accessory panel pulmonary.
  - IV.) Fix the cable A4 and A5 to the connector bracket by screwing female screw-locks.  
(Do not use washers and nuts)

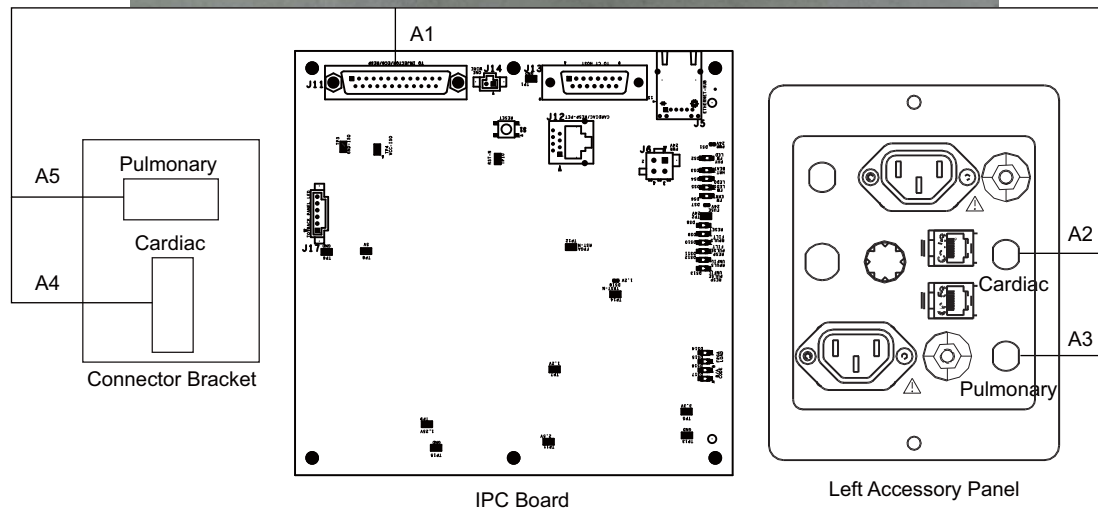
**Figure 2-5 IPC Board and Left Accessory Panel Assembly**

- 2.) Install the IPC board and left accessory panel assembly to the Gantry rear base by screwing its four screws with washers.

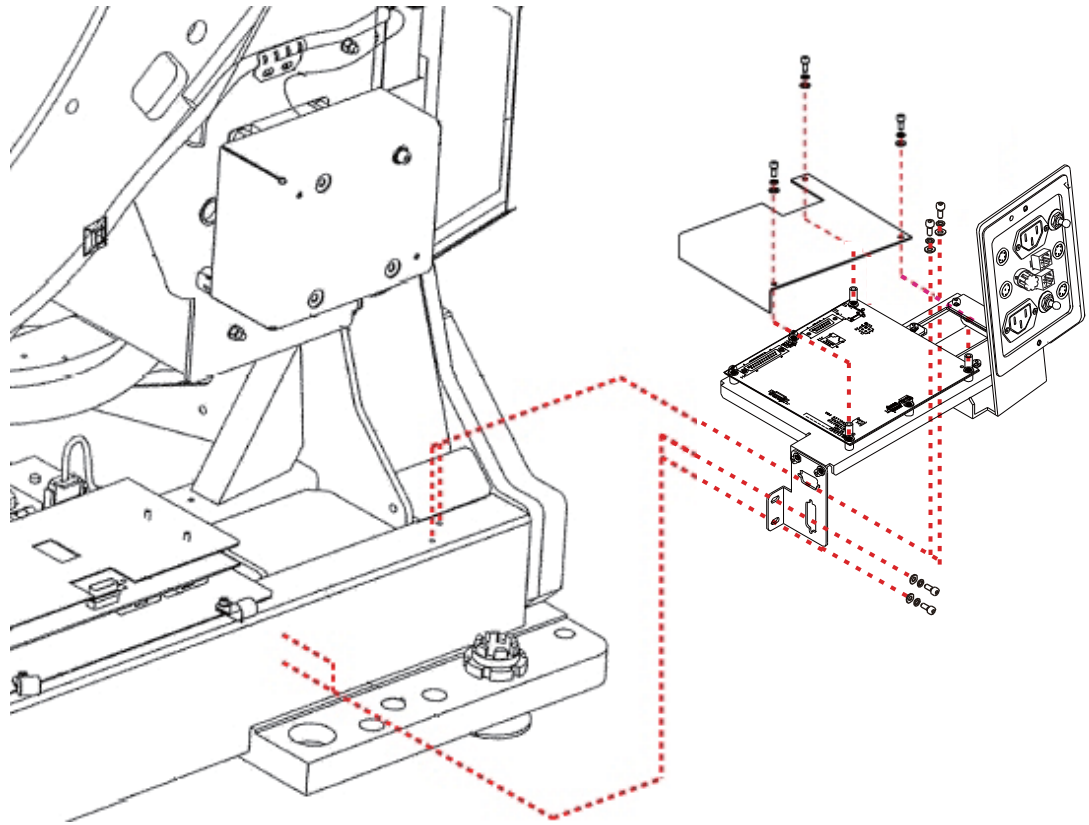**Figure 2-6 IPC and Left Accessory Panel Installation**

3.) Assemble the right accessory panel, and install it to the Gantry rear base.

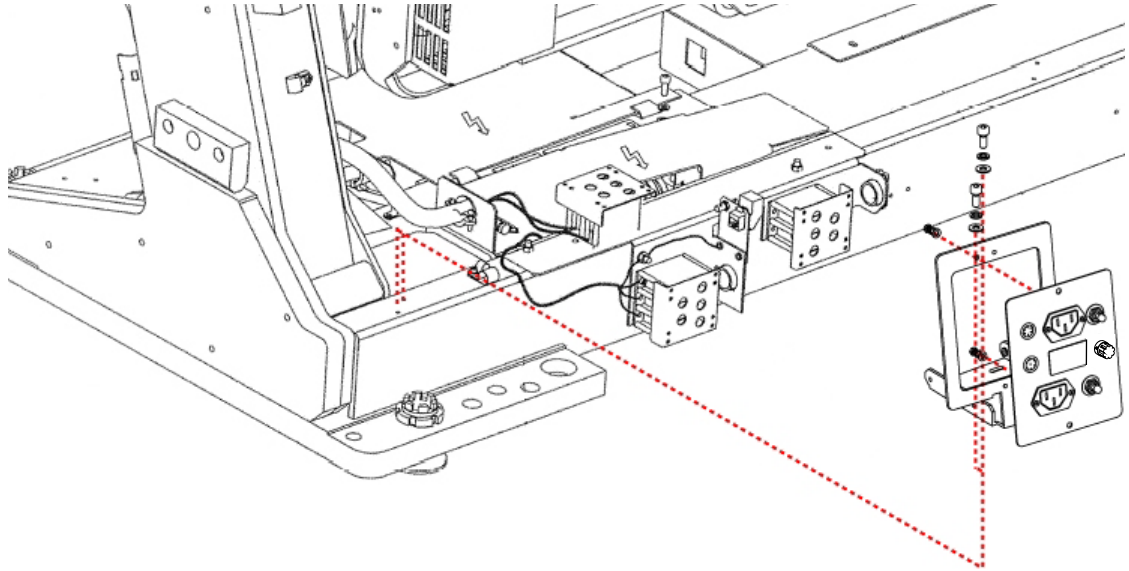**Figure 2-7 Right Accessory Panel Installation**

### 1.3 IPC Harness Assembly Connections (5316429-3)

Mark	Part Number	Quantity	Description
1	5444335	1.0	Cable--LAN to hub SW2
2	5456206	1.0	Cable, Cardiac-Pulmonary_PathFinder
3	5441132	1.0	Smart Step Cable
4	5444336	1.0	Cable IPC to Gantry PS
5	5306481-3	1.0	Power Cable, IPC Accessory Panel (added fuse on panel)

Table 2-1 IPC Harness Assembly (5316429-3)

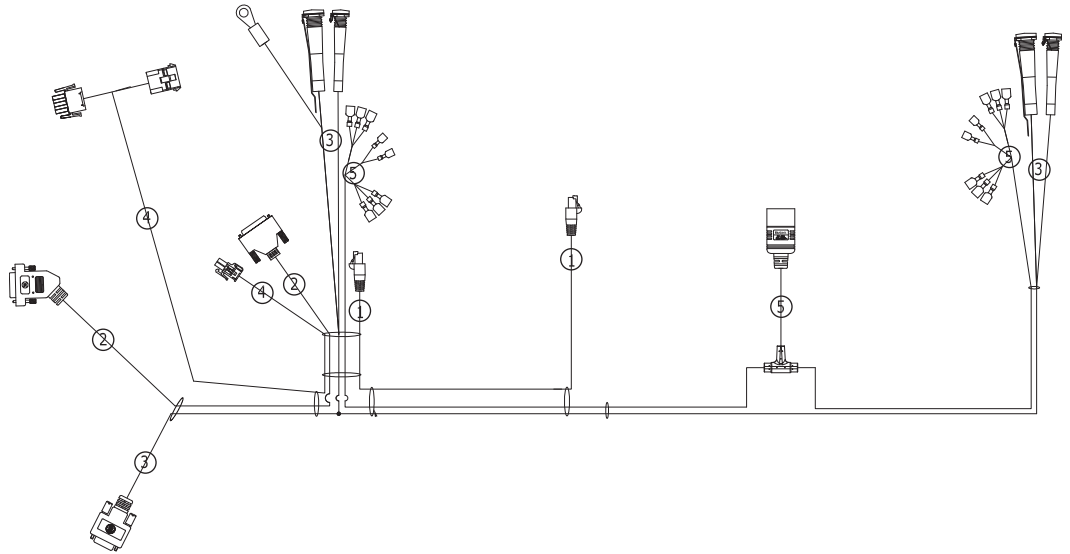
Figure 2-8 IPC Harness Assembly (5326429-3)

1.) Carefully route the IPC harness assembly as shown in illustration [Figure 2-9](#).



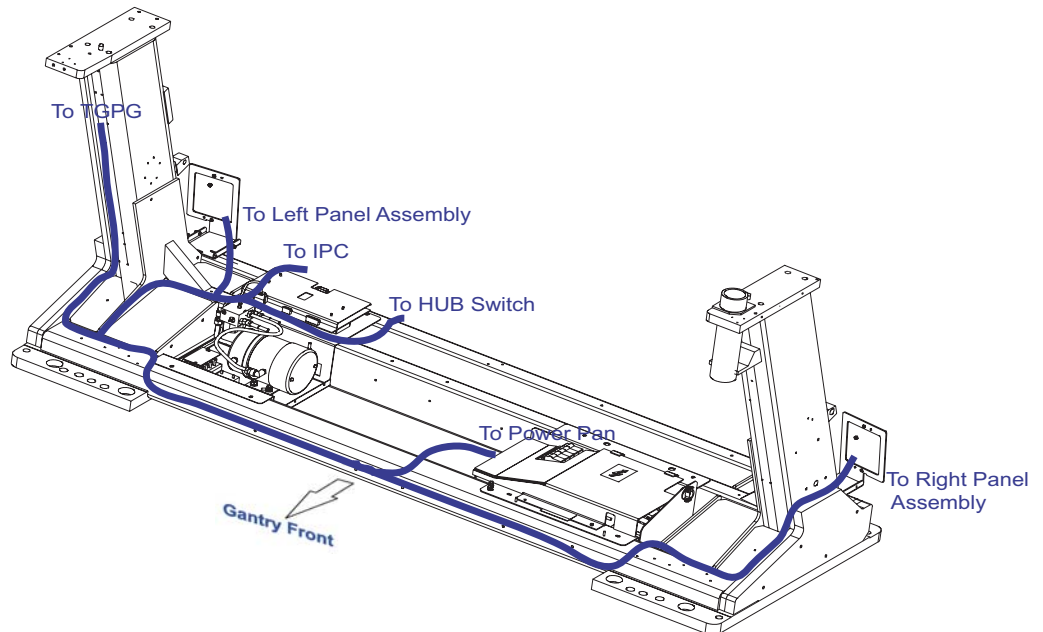
Figure 2-9 Cable Wiring

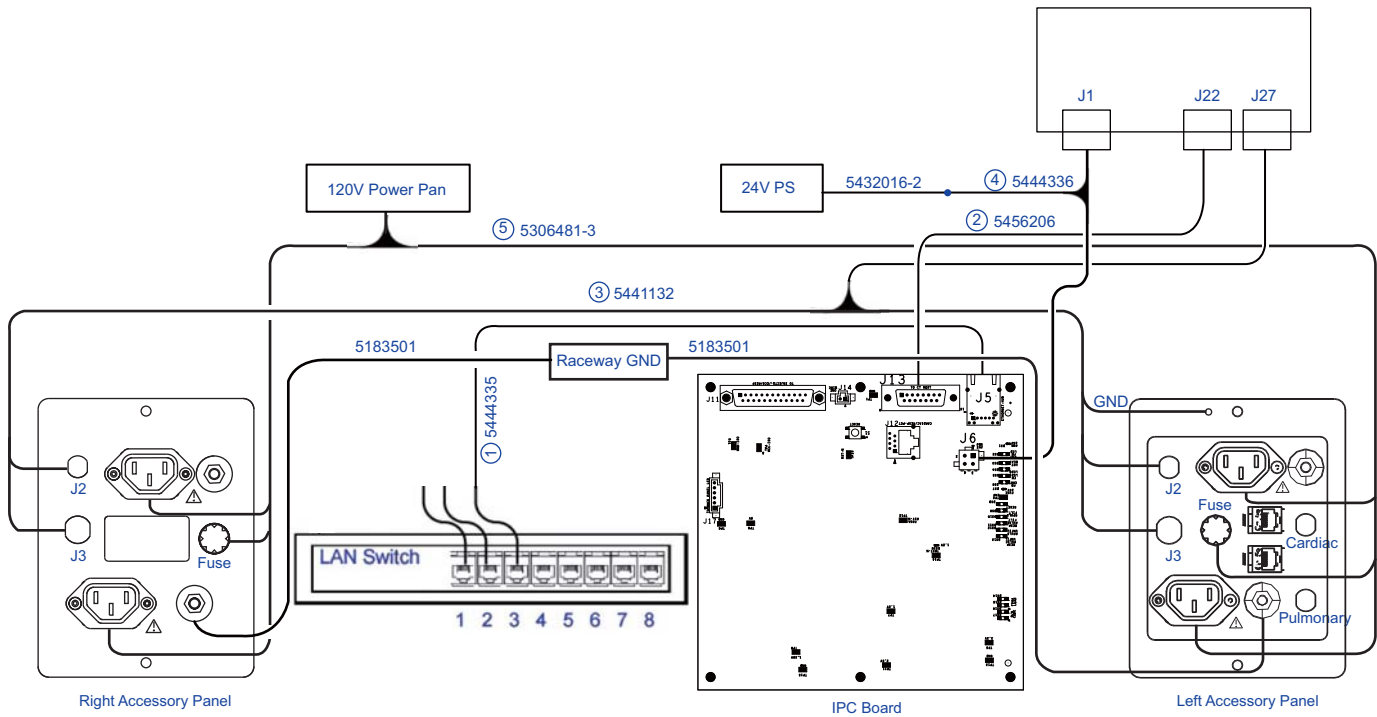
**Figure 2-10 IPC Harness Assembly (5326429-3) Connection**

- 2.) Connect the LAN cable (5444335) between the IPC board J5 and the Hub switch 2.
- 3.) Connect the Cardiac/Pulmonary Pathfinder cable (5456206) between the IPC board J13 and the TGPG J22. (See [Figure 2-10](#) and [Figure 2-13](#))
- 4.) Connect the Smart Step cable (5441132) between the TGPG J27 and the Left/Right Accessory Panel. (See [Figure 2-10](#) and [Figure 2-13](#))
- 5.) Connect the IPC 24V power cable (5444336)

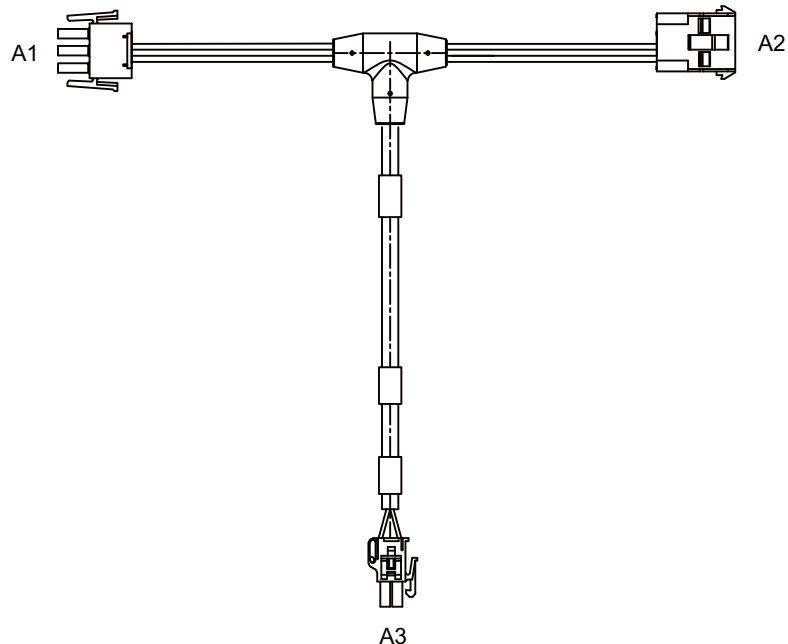**Figure 2-11 IPC Power Cable (5444336)**

- a.) Connect the IPC power cable (24V, 5444336) A3 to the IPC board J6.
  - b.) Disconnect the 24V power cable (5432016-2) connector from the TGPG board J1 and connect it to the IPC power cable (5444336) A2. (See [Figure 2-11](#) and [Figure 2-13](#))
  - c.) Connect the IPC power cable (5444336) A1 connector to the TGPG board J1. (See [Figure 2-13](#))
- 6.) Connect the ground cables (5183501) between the Left/Right Accessory Panel and the gantry raceway. (See [Figure 2-10](#))
  - 7.) Connect the 120V power cable (5306481-3).

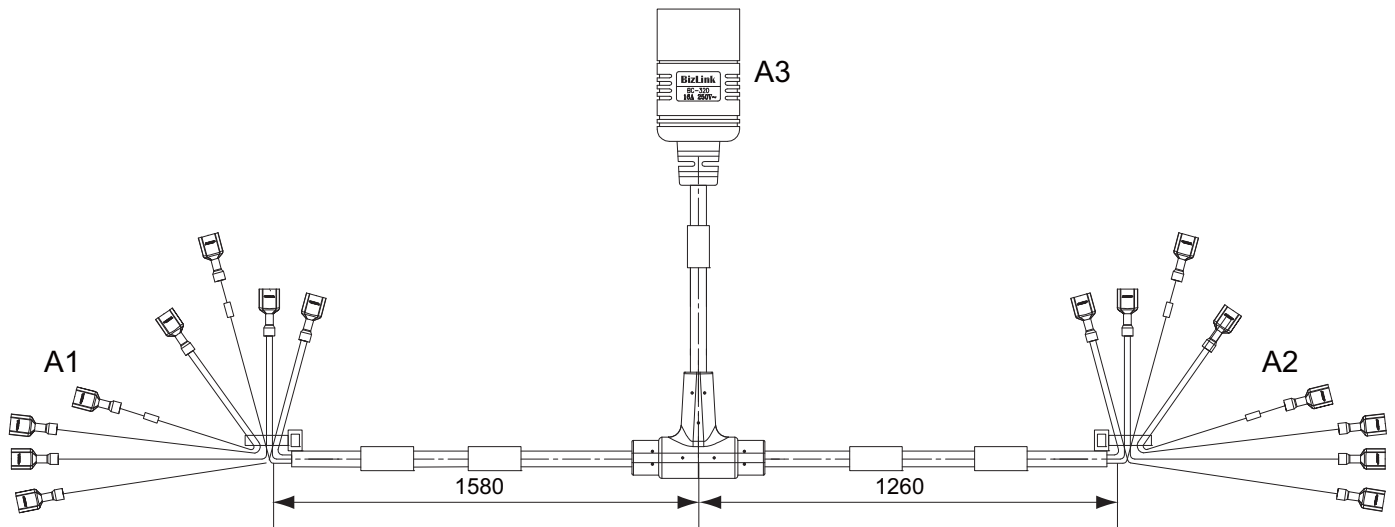**Figure 2-12 120V Power Cable (5306481-3)**

- a.) Connect the 120V power cable (5306481-3) A1 to Left Accessory Panel and A2 to the Right Accessory Panel. (See [Figure 2-14](#) and [Figure 2-15](#))
- b.) Turn **ON** "120VAC" switch from the service control panel.
- c.) Press and hold the forward tilt button from the service control panel, until tilt the gantry forward to expose the 120VAC power pan.
- d.) Turn **OFF** "120VAC" switch, and connect the 120V power cable (5306481-3) A3 to 120VAC power pan. (See [Figure 2-16](#))
- e.) Restore the gantry to zero degrees tilt.

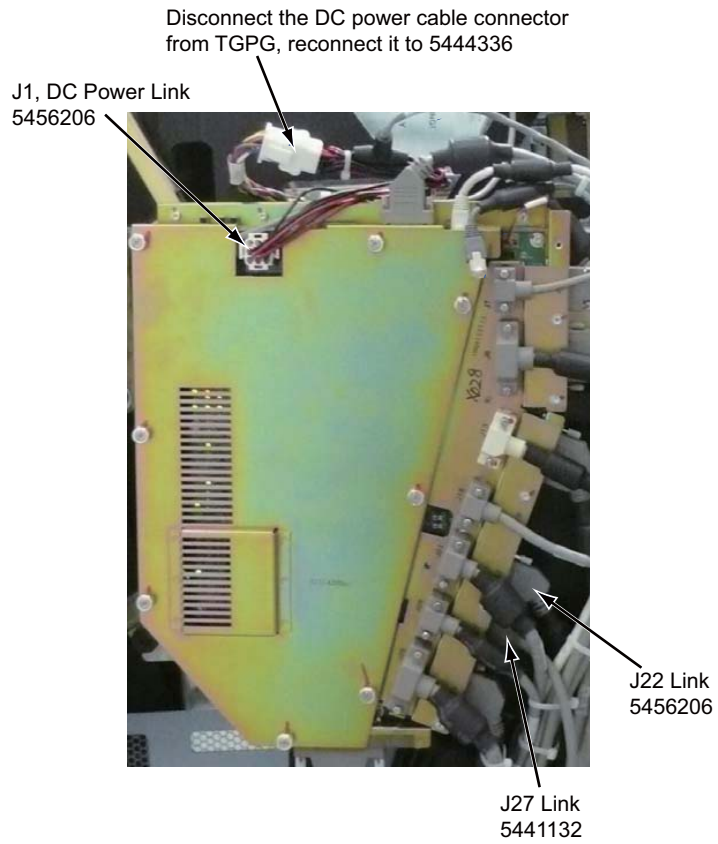
Figure 2-13 TGPG Connection


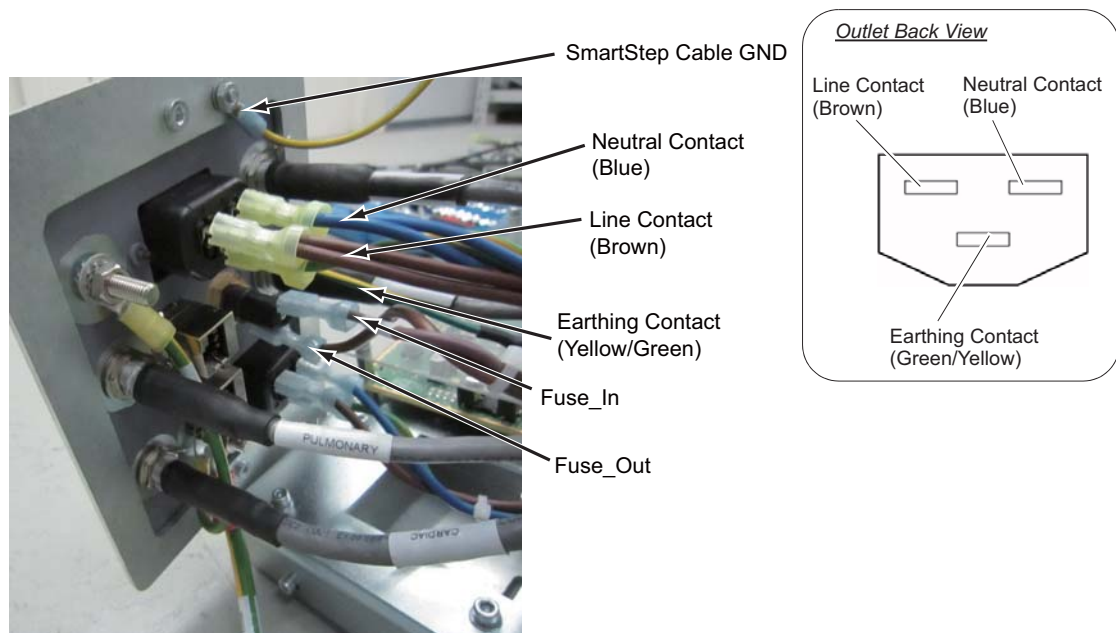
Figure 2-14 Left Accessory Panel Connection

**Figure 2-15 Right Accessory Panel Connection**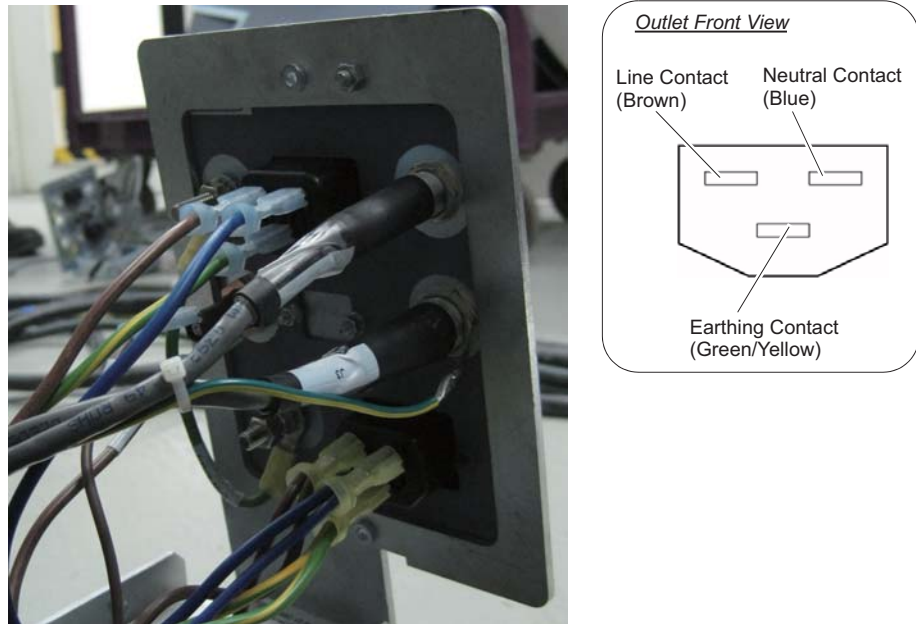

**Figure 2-16 120VAC Power Pan**

8.) Secure the cables with tie wraps.

Note: If you encounter the following issues when installing the cable harness (P/N 5316429-3) to the Gantry, cut some tie-wraps and route the cables appropriately.

- Harness interferes with Gantry Left Base Cover thus cover can not be installed.
- Too much force is applied to the LAN switch connectors when Gantry Rear Base Cover is installed.

9.) Restore power to the system.

## Section 2.0 Finish Installation

### 2.1 LED Inspection of IPC Board

- 1.) Turn on main power to system at A1 panel or PDB.
- 2.) Turn ON "Axial Drive", "HVDC" and "120VAC" switches on the service switch panel.
- 3.) To make sure the IPC board is functioning, an LED inspection should be made on the board.  
 The LED configuration on the board should be as follows with power on but no options connected to the options interface panels:

LED	Color	Label	Status (without options connected)
DS1	Green	24V_PWR	On Solid
DS2	Green	PHY_FW_LED	On Solid
DS3	Green	HRTBT	Off (Blinking when Injector option is installed)
DS4	Green	FW_LED_0	Off
DS5	Green	FW_LED_1	Off
DS6	Green	FW_ERROR	Off
DS7	Green	24V_FUSE	On Solid
DS8	Green	RESET	On Solid
DS9	Yellow	RESET	Off
DS10	Green	R_PULSE_FILTERED	Off
DS11	Green	RESP_PULSE_FILTERED	Off
DS12	Green	R_PULSE_UNFILTERED	Off
DS13	Green	RESP_PULSE_UNFILTERED	Off
DS14	Yellow	FPGA_LOAD	Off
DS15	Green	FPGA_LOAD	On Solid
DS16	Yellow	B/A_CODE	Off
DS17	Green	B/A_CODE	On Solid
DS18	Green	+1.2V_OK	On Solid
Ethernet	Orange/Green	Ethernet-Hub	Blinking with network activity

**Table 2-2 IPC Board LED Status**

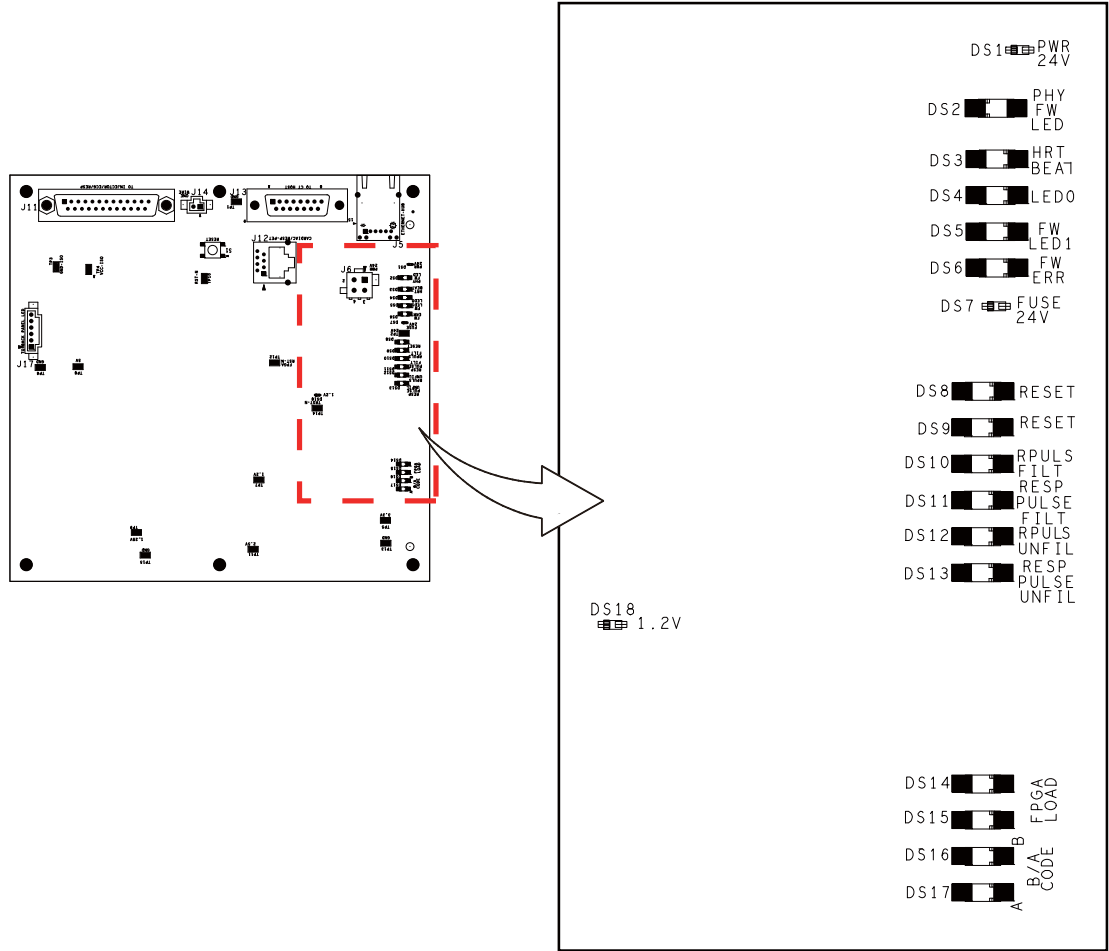
Figure 2-17 IPC Board LED locations

## 2.2 LED Inspection of LAN Switch

- 1.) Visually verify the front panel Power LED of the switch is illuminated.
- 2.) Visually verify that all appropriate Port LEDs show active links.

## 2.3 EKG/ECG LAN Port

Refer to System Installation manual, Option Installation section for the cable connection of EKG/ECG LAN Port.

## 2.4 Finalization

- 1.) Perform an application scan to ensure proper system operation before turning the system over to the customer.
- 2.) Turn OFF all 3 switches (Axial Drive, HVDC, 120 VAC) on the service switch panel.
- 3.) Reinstall both rear base covers (see [Figure 2-1](#)).
- 4.) Reinstall the gantry side covers. Refer to your System Service Manual for any needed instruction.

*Page Left Intentionally Blank*

[www.gehealthcare.com](http://www.gehealthcare.com)

**Imagination at work**