GE Healthcare

# Optima CT660 / Revolution EVO MAR Upgrade Kit Installation

OPERATING DOCUMENTATION

5727755-1EN

Rev 1

© 2015 General Electric Company.

All rights reserved.

---

# **LEGAL NOTES**

---

## **TRADEMARKS**

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

## **COPYRIGHTS**

All Material Copyrighted (c) 2015 by the GE Company, All rights reserved.

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## **DAMAGE IN TRANSPORTATION**

All packages should be closely examined at time of delivery. If damage is apparent, have notation "damage in shipment" written on all copies of the freight or express bill before delivery is accepted or "signed for" by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 8.
- Fill out a report on <http://us44hdd21/sctq/InstallFulfill/InstalFulfillment.htm>
- Contact your local service coordinator for more information on this process.

Rev. Jan. 5, 2005

## **CERTIFIED ELECTRICAL CONTRACTOR STATEMENT**

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

## **IMPORTANT...X-RAY PROTECTION**

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

## LITHIUM BATTERY CAUTIONARY STATEMENTS

### CAUTION

**Risk of Explosion.**

**Danger of explosion if battery is incorrectly replaced.**

**Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.**

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## OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the Healthcare PQR Process to report all omissions, errors, and defects in this publication.

---

# ***Revision History***

---

<b>Revision</b>	<b>Date</b>	<b>Reason for change</b>
1	November 30, 2015	Initial Release

---

# List of Effected Pages

---

PAGES	REVISION	PAGES	REVISION	PAGES	REVISION
1 through 30	1				

# Table of Contents

---

## Preface

**Publication Conventions ..... 19**

**Section 1.0**

**Safety & Hazard Information ..... 19**

1.1 Text and Character Representation..... 19  
1.2 Graphical Representation ..... 20

**Section 2.0**

**Publication Conventions ..... 21**

2.1 General Paragraph and Character Styles..... 21  
2.2 Page Layout..... 21  
2.3 Computer Screen Output/Input Text Character Styles ..... 22  
2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)..... 22

## Chapter 1

**Introduction..... 23**

**Section 1.0**

**Purpose ..... 23**

**Section 2.0**

**Personnel Requirements ..... 23**

**Section 3.0**

**Preliminary Requirements ..... 23**

3.1 Tools and Test Equipment ..... 23  
3.2 Consumables ..... 23  
3.3 Safety..... 23  
3.4 Required Conditions ..... 24

**Section 4.0**

**Material List ..... 24**

## Chapter 2

**Upgrade Procedure ..... 25**

**Section 1.0 Overview ..... 25**

1.1 Preparation ..... 25  
1.2 Procedure steps..... 25

**Section 2.0 Procedure..... 26**

2.1 Record the following setting information..... 26  
2.2 Save System State ..... 26  
2.3 15HW13.x Load From Cold ..... 26  
2.4 Z Alignment Cal. Creation..... 27  
2.5 InSite and iLinq Check out..... 27

---

<b>Section 3.0</b>	<b>Finalization .....</b>	<b>28</b>
3.1	Protocol update .....	28
3.2	Quality Assurance Test .....	28
3.3	Save System State.....	28
3.4	Operator manuals .....	28

---

# Preface

## Publication Conventions

---

Purpose: Standardize conventions for representing information in a uniform way of communicating information to a reader in a consistent manner. Conventions are used so that the reader can easily recognize the actions or decisions that must be made. There are a number of character and paragraph styles used in this publication to accomplish this task. Please become familiar with them before proceeding forward.

## Section 1.0

### Safety & Hazard Information

#### 1.1 Text and Character Representation

Within this publication, different paragraph and character styles have been used to indicate potential hazards. Paragraph prefixes, such as hazard, caution, danger and warning, are used to identify important safety information. Text (Hazard) styles are applied to the paragraph contents that is applicable to each specific safety statement. Words describe the type of potential hazard that may be encountered and are placed immediately before the paragraph it modifies. Safety information will normally include:

- Type of potential Hazard
- Nature of potential injury
- Causative condition
- How to avoid or correct the causative condition

#### EXAMPLES OF HAZARD STATEMENTS USED

It's important that you read and understand hazard statements, and not just ignore them.

**DANGER  
SEVERE  
PERSONAL  
INJURY**

**DANGER IS USED WHEN A HAZARD EXISTS WHICH WILL CAUSE SEVERE PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:**

- **ELECTROCUTION**
- **CRUSHING**
- **RADIATION**

**WARNING  
SERIOUS  
PERSONAL  
INJURY**

**WARNING IS USED WHEN A HAZARD EXISTS WHICH COULD OR CAN CAUSE SERIOUS PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:**

- **POTENTIAL FOR SHOCK**
- **EXPOSED WIRES**
- **FAILURE TO TAG AND LOCKOUT SYSTEM POWER COULD ALLOW FOR UN-COMMANDED MOTION.**

 **CAUTION**  
**Minor Personal Injury**  
Caution is used when a hazard exists which can or could cause minor injury to self or others if instructions are ignored. They include for example:

- Loss of critical patient data
- Crush or pinch points
- Sharp objects

 **NOTICE**  
**Property Damage Only**  
Notice is used when a hazard is present that can cause property damage but has absolutely no personal injury risk. They can include:

- Disk drive will be destroyed
- Internal mechanical damage, such as to the x-ray tube
- Coasting the rotor through resonance.

## 1.2 Graphical Representation

Important information will always be preceded by the exclamation point  contained within a triangle, as seen throughout the Preface chapter. In addition to text, several different graphical icons (symbols) may be used to make you aware of specific types of hazards that could possibly cause harm.

<p><b>ELECTRICAL</b></p>  <p><b>LASER</b></p>  <p><b>LASER LIGHT</b></p>	<p><b>HAZARDS MECHANICAL</b></p>  <p><b>HEAT</b></p> 	<p><b>RADIATION</b></p>  <p><b>PINCH</b></p> 
<p><b>AVOID STATIC ELECTRICITY</b></p> 	<p><b>PROCEDURAL TAG AND LOCK OUT</b></p> 	<p><b>WEAR EYE PROTECTION</b></p>  <p><b>EYE PROTECTION</b></p>

# Section 2.0 Publication Conventions

## 2.1 General Paragraph and Character Styles

Prefixes are used to highlight important non-safety related information. Paragraph prefixes (such as purpose, example, comment or note) are used to identify important but non-safety related information. Text styles are also applied to text within each paragraph modified by the specific prefix.

### EXAMPLES OF PREFIXES USED FOR GENERAL INFORMATION

**Purpose:** Introduces and provides meaning as to the information contained within the chapter, section or subsection, Such as used at the beginning this chapter for example.

**Note:** Conveys information that should be considered important to the reader.

**Example:** Used to make the reader aware that the paragraph(s) that follow are examples of information possibly stated previously.

**Comment:** Represents "additional" information that may or may not be relevant to your situation.

## 2.2 Page Layout

The diagram illustrates the layout of two pages from a manual. The left page is titled 'Publication Part Number & Revision Number' and the right page is titled 'Publication Title'. Both pages show a header with 'GE Medical Systems' and 'Zelus Remote Intercom X Board' information. The left page contains sections 3.2.2 and 3.2.3, with a 'DANGER' symbol and a 'NOTE' symbol. The right page contains sections 3.2.4, 3.2.5, 3.2.6, and 3.2.7, with a diagram of the intercom board and a 'DANGER' symbol. Callouts point to the part number and revision number in the footer of the left page and the chapter title in the footer of the right page.

The current section and its title are always shown in the footer of the left (even) page.

An exclamation point in a triangle is used to indicate important information to the user.

Paragraphs preceded by **Alphanumeric** characters (e.g. numbers) contain information that must be followed in a **specific order**.

The current chapter and its title are always shown in the footer of the right (odd) page.

Paragraphs preceded by a **symbol** (e.g. bullets) contain information that has **no specific order**.

Headers and footers in this publication are designed to allow you to quickly identify your location. The document's part number and revision number appears in every header on every page. Odd

numbered page footers indicate the current chapter, its title and current page number. Even page footers show the current section and its title, as well current page number.

## 2.3 Computer Screen Output/Input Text Character Styles

Within this publication, mono-spaced character styles (fonts) are used to indicate computer text that's either screen input and output. Mono spaced fonts, such as courier, are used to indicate text direction. When you type at your keyboard, you are generating computer input. Occasionally you will see the math operators "greater than" and "less than" symbols used to indicate the start and finish of variable output. When reading text generated by the computer, you are reading it as computer generated output. In addition to direction, characters are italicized (e.g. *italics*) to indicate information specific to your system or site.

Example:  
Fixed Output

This paragraph's fonts represents computer generated screen "fixed" output. It's output is fixed from the sense that it does not vary from application to application. It's the most commonly used style used to indicate filenames, paths and text that does not change from system to system. The character style used is a fixed width such as courier.

Example:  
Variable Output

*This paragraph's font represents computer screen output that is "variable". Its used to represent output that varies from application to application or system to system. Variable output is sometimes found placed between greater than and lesser than operators for clarification. For example: <variable\_ouput> or <3.45.120.3>. In both cases, the < and > operators are not part of the actual input.*

Example:  
Fixed Input

**This paragraph's font represents fixed input. It's computer input that is typed-in via the keyboard. Typed input that does not vary from application to application or system to system. Fixed text the user is required to supply as input. For example: cd /usr/3p**

Example:  
Variable Input

***This paragraph's font represents computer input that can vary from application to application or system to system. With variable text, the user is required to supply system dependent input or information. Variable input sometimes is placed between greater-than and less-than operators. For example: <variable\_input>. In these cases, the (<>) operators would be dropped prior to input. For example: ypcat hosts | grep <3.45.120.3> would be typed into the computer as ypcat hosts | grep 3.45.120.3 without the greater-than and less-than operators.***

## 2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)

Different character styles are used to indicate actions requiring the reader to press either a hard or soft button, switch or key. Physical hardware, such as buttons and switches, are called hard keys because they are hard wired or mechanical in nature. A keyboard or on/off switch would be a hard key. Software or computer generated buttons are called soft keys because they are software generated. Software driven menu buttons are an example of such keys. Soft and hard keys are represented differently in this publication.

Example:  
Hard Keys

A power switch **ON/OFF** or a keyboard key like **ENTER** is indicated by applying a character style that uses both over and under-lined bold text that is bold. This is a hard key.

Example:  
Soft Keys

Whereas the computer MENU button that you would click with your mouse or touch with your hand uses over and under-lined regular text. This is a soft key.

**END OF SECTION**

# Chapter 1

## Introduction

### Section 1.0 Purpose

This document describes the upgrade procedure to Optima CT660 M40/Revolution EVO Phase3.5 software for IB system.

This upgrade is effective for the following software.

- 13HW38.14 for Optima CT660 M40 (with any Service Pack)
- 14HW17.4 for Revolution EVO

### Section 2.0 Personnel Requirements

Required Persons	Preliminary Reqs	Procedure	Finalization
1	Timing Not Applicable	100 mins	40 mins

### Section 3.0 Preliminary Requirements

#### 3.1 Tools and Test Equipment

Item	Qty.	Effectivity	Part No.	Manufacturer
Standard FE Tool Kit	1	-	-	-

#### 3.2 Consumables

Item	Qty.	Effectivity	Part No.	Manufacturer
None	1	-	-	-

#### 3.3 Safety

No general safety information.

### 3.4 Required Conditions

Condition	Reference	Effectivity
Only trained service personnel should service the GE CT Scanner.	-	-

## Section 4.0 Material List

#### For Optima CT660 M40 (Cj Phase3.5 MAR Upgrade)

<b>B76972FJ CT660 7.0M MAR UPGRADE</b>	
Name	Description
B76982FJ	Smart MAR
B77852FJ	Cj_M40 P35 UPG
5693083-40	Cj_M40 Phase3.5 Upgrade
5727755-1EN	Optima CT660 and Revolution EVO MAR Upgrade Manual
5389067-10	Cj_M40 Phase3.5 Software Collector with Rating Plate
5375974-9	Cj_M40 P3.5 Doc Collector
5611030	120sec Cine SW Key with DVD
5405579	DVD RAM Bare Media
5468600	CT System e-User Leaflet Multi-language Operator Documentation Introduction_Multi-languages
5458039-51	15HW13.7 Service Pack 1

#### For Revolution EVO (Cj Phase3.5 MAR Upgrade)

<b>B75342RE REVO EVO MAR UPGRADE</b>	
Name	Description
B76982FJ	Smart MAR
B75352RE	Revolution EVO P35 UPG
5693083	Revolution EVO Phase3.5 Upgrade
5727755-1EN	Optima CT660, Revolution EVO MAR Upgrade Manual
5050553-2	Revolution EVO Phase3.5 Software Collector with Rating Plate
5375974-31	Revolution P3.5 EVO Doc Collector
5611030	120sec Cine SW Key with DVD
5405579	DVD RAM Bare Media
5468600	CT System e-User Leaflet Multi-language Operator Documentation Introduction_Multi-languages
5458039-51	15HW13.7 Service Pack 1

---

# **Chapter 2**

## **Upgrade Procedure**

---

### **Section 1.0 Overview**

#### **1.1 Preparation**

The following preparation shall be done before upgrade.

- SSA Gen2. Class M Key (Renew it at SSA Web site before use it, it expires from one month after last renewal).

#### **1.2 Procedure steps**

The following shows the brief steps of upgrade.

- 1.) Save System State.
- 2.) Load From Cold.
- 3.) Install Class C/M.
- 4.) Restore System State.
- 5.) Install Service Pack.
- 6.) Execute TIC.
- 7.) Flash Download.
- 8.) Install options.
- 9.) Z Alignment cal. Vector creation.
- 10.) InSite and iLinq Check out.
- 11.) Update Protocol about NV (Notification Value).
- 12.) IQ check
- 13.) Save System State (with another media from original)
- 14.) Hand User manuals and Feature Release Overview to customer.

## Section 2.0 Procedure

### 2.1 Record the following setting information

- Connect Pro (IP Address, Port Number, Etc.)
- Exam Split (HES/VES) If `/usr/g/ctuser/ves/.hesMode` exists, it's HES mode.
- EKG Monitor (model)
- Injector (pressure Unit)

### 2.2 Save System State

- 1.) Go to Service Desktop → Utilities
- 2.) Execute Save System State.

### 2.3 15HW13.x Load From Cold

Perform the following tasks according to (15HW13.x) Load From Cold procedure in Optima CT660 or Revolution EVO Service Method CD-ROM.

- Disconnect hospital network cable from Console bulkhead.
- OS Load
- Apps Load
- Start LFC Menu
  - \* Set Date and Time
  - \* Install Class C & M
  - \* Restore System State
  - \* Install Options (MAR and Cine-120-sec)
  - \* Check System State

Note: If Check System State is failure about TZ (Time Zone) mismatching, ignore it.

- Reboot the System
- Tube Install Certification
- Flash download
- Reconnect hospital network cable.
- Service Pack Install.

---

## 2.4 Z Alignment Cal. Creation

Note: If the following steps are not performed, abnormal image will be reconstructed.

Create Z Alignment Cal. vector with following steps,

- 1.) Tilt the gantry to zero degree position.
- 2.) Move the cradle (and IMS) to out limit position.
- 3.) Remove mylar window.
- 4.) Execute FRDM wizard.
- 5.) Press "Start Process" button.
- 6.) Press "Mini Check" button, and follow the displayed instruction.
- 7.) Confirm that the Z Alignment plot is displayed, then close it.
- 8.) DAS Tools is initiated, dismiss DAS Tools.
- 9.) Abort FRDM wizard and select "Yes" to close.
- 10.) Install mylar window.

## 2.5 InSite and iLinq Check out

If the system has class M software and previously InSite and/or iLinq was enabled, perform the checkout at this time.

Request Remote Engineer to Check out the system with new model type "\_SUSEIIP45".

---

## Section 3.0 Finalization

### 3.1 Protocol update

After software upgrade, the displayed dose values will be changed from the old software in some scan parameters setting.

Ask customer to update NV (Notification Value) of protocol that uses PedBody SFOV at Protocol Management according to the UPDATE OF DOSE CALCULATION METHOD in provided Tips and Work Around.

### 3.2 Quality Assurance Test

- 1.) Perform FastCal.
- 2.) Perform Quality Assurance Test according to **Quality Assurance Test** procedure in Optima CT660 Service Methods CD-ROM or Revolution EVO Service Methods CD-ROM.

### 3.3 Save System State

- 1.) Perform the System State Save and Restore procedure and save a System State Backup. Keep the original System state media for back up.
- 2.) Save the System State media in a safe and secure location for future service activity.

### 3.4 Operator manuals

Hand the provided Operator manuals to customer.

© 2015, General Electric Company.  
GE Medical Systems, a General Electric Company, going to market as GE Healthcare.  
3000 N. Grandview Boulevard  
Waukesha, Wisconsin 53188  
USA

[www.gehealthcare.com](http://www.gehealthcare.com)

**Imagination at work**