DIN6868-157 19inch LCD Monitor Option Installation

Service Documentation

5798612-1EN

Rev 2

© 2018, General Electric Company,
All Rights Reserved.

LEGAL NOTES

TRADEMARKS

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions. Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation. All other products and their name brands are trademarks of their respective holders.

COPYRIGHTS

All Material, Copyright © 2018 by General Electric Company, Inc. All rights reserved.

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WARNING (EN)

- This Service Manual is available in English only.
- If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services.
- Do not attempt to service the equipment unless this service manual has been consulted and is understood.
- Failure to heed this warning may result in injury to the service provider, operator, or patient, from electric shock or from mechanical or other hazards.

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DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent write "Damage In Shipment" on ALL copies of the freight or express bill BEFORE delivery is accepted or "signed for" by a GE representative or hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 6
- Fill out the GIQ workflow for any items missing, damaged, OBF/FOI for in process installs:
http://http://supportcentral.ge.com/ProcessMaps/form_new_request.asp?prod_id=268679&form_id=573167&node_id=1916016&map_id=&reference_id=&reference_type
- Contact the local service coordinator.

CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

LITHIUM BATTERY CAUTIONARY STATEMENTS

CAUTION
Risk of
Explosion

Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.

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OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the GEMS CQA Process to report all omissions, errors, and defects in this publication.

Revision History

Revision	Date	Reason for change
1	05/03/18	Initial release.
2	12/03/18	Add TG18 patterns installation procedure.

This page intentionally left blank.

Table of Contents

Chapter 1	
Introduction	17
Section 1.0	
Overview	17
Section 2.0	
Material List	17
Section 3.0	
Flowchart	18
Chapter 2	
Installation Procedure	19
Section 1.0	
Overview	19
Section 2.0	
Installation Procedure	19
2.1 Operator-Room Monitor Installation.....	19
2.2 In-Room Monitor Installation.....	21
Section 3.0	
OSD Settings	24
3.1 Operator-Room Monitor Settings.....	24
3.2 In-Room Monitor Settings.....	25
Section 4.0	
TG18 Pattern Installing and Viewing	27
Section 5.0	
FRU Request	28
Section 6.0	
Function Check	28

This page intentionally left blank.

Chapter 1

Introduction

Section 1.0 Overview

The option kit (5798612) provides a 19inch monitor solution complies to German DIN6868-157. Customers can choose to purchase this option if they use their CT system for the primary diagnostic image review by the radiologist.

This option is planning to be available for the following forward products (please follow the proper PCM configuration):

- Optima CT520 with DoD
- Optima CT540
- Discovery RT
- Revolution EVO
- Revolution Discovery CT
- Revolution Frontier

And this option is also available for the customer purchased the Digital In-Room Monitor Kit B7710AN.

Section 2.0 Material List


Verify that the contents of this option kit are correct according to the Material List below.

Part Number	Description	Qty
5746368	19inch LCD HB color Monitor W Stand	1
5797891	Boom in Room DIN Monitor Spacer	1
1000-M4C020-04	Hexagon Socket Head Cap Screw, M4-0.7, X20mm Long, Grade 8.8 Steel, Zinc Plated, RoHS Fastener	4
2000-M4-02	Flat Washer, ISO7089, M4, Steel 140HV, Zinc Plated, RoHS Fastener	4
2203-M4-07	Curved Lock Washer, DIN 128A, M4, Spring Steel, Zinc Plated, RoHS Fastener	4
5798612-1EN	DIN6868-157 19inch LCD Monitor Option Installation	1

Table 1-1 DIN6868-157 19inch LCD Monitor Option Kit (5798612)

Section 3.0 Flowchart

The Illustration below shows the main flow of the installation procedures of the option.

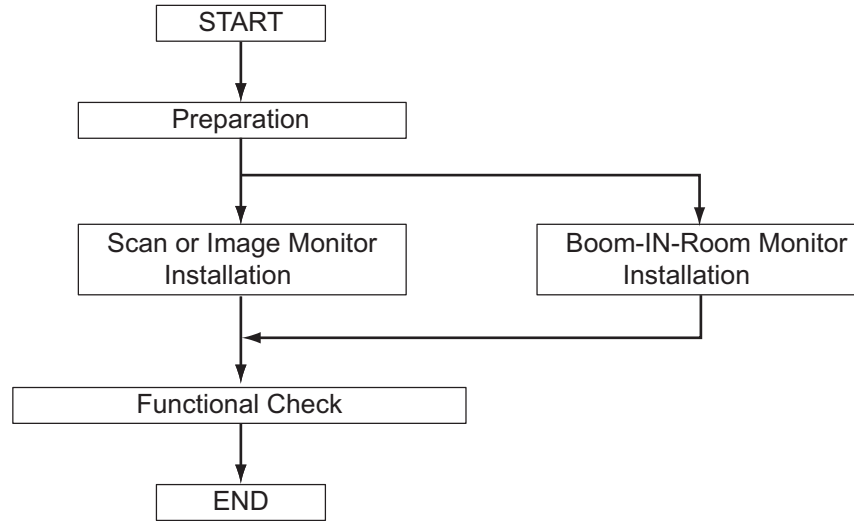Chapter 2

Installation Procedure

Section 1.0 Overview

This instruction describes the hardware installation procedure.

Section 2.0 Installation Procedure

2.1 Operator-Room Monitor Installation

- 1.) Unpack the monitor option kit package to take out the LCD monitor and place it on desk.
- 2.) Make sure the Operator Console is power off.
- 3.) Remove the rear little cover from the back of the monitor by unscrewing its one screw.


Figure 2-1 DIN LCD Monitor

Rear Little Cover


- 4.) Take out the signal and power cables shipped with Console, connect them to the monitor (DVI and Power Cord).

Figure 2-2 Remove Rear Little Cover to Connect Cables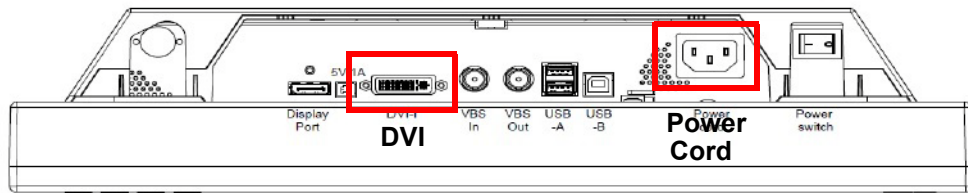

- 5.) Reinstall the rear litter cover and route two monitor cables in the cable duct.

Figure 2-3 Cables Routing

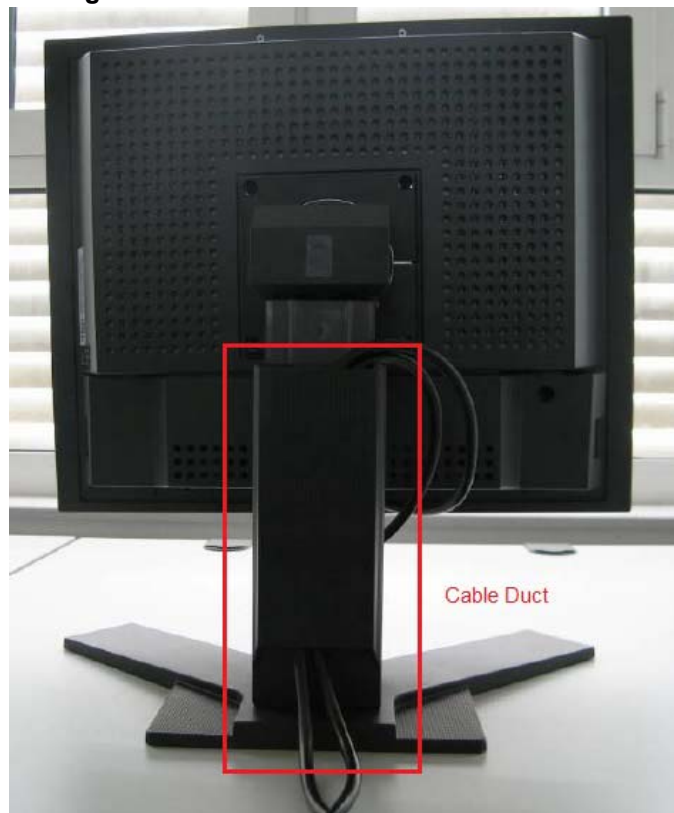- 6.) Connect monitor cables to Console for the **Image Monitor**.
 - Signal Cable: Console Host DVI-D
 - Power Cable: Console AC Box (meet the proper System Installation Manual)

NOTICE If the scan monitor is **DIN6868-157 19inch LCD monitor**, please refer the proper **System Installation Manual for the scan monitor (EIZO or HP) cables connection**.

2.2 In-Room Monitor Installation

Note: If customer has been purchased **Boom-In-Room Kit** and **DIN6868-157 19inch LCD Monitor Option Kit**, FE should use **DIN6868-157 19inch LCD monitor** for In-Room monitor installation.

- 1.) Please refer **Boom-In-Room Installation Manual** for the Boom assembly installation.
- 2.) Make sure the Operator Console is power off.
- 3.) Unpack the monitor option kit package to take out the LCD monitor.
- 4.) Remove the monitor stand by unscrewing four screws.

Figure 2-4 Monitor Stand Removal

- 5.) Remove the rear litter cover from the back of the monitor by unscrewing its one screw.

Figure 2-5 Stand and Rear Litter Cover Removal

- 6.) Tear off the protective paper from the spacer (5797891). Make the adhesive side face to handle. Pass four M4x20mm screws (1000-M4C020-04, shipped with DIN option kit) through four screw holes on LCD Bracket, LCD Handle and Spacer.

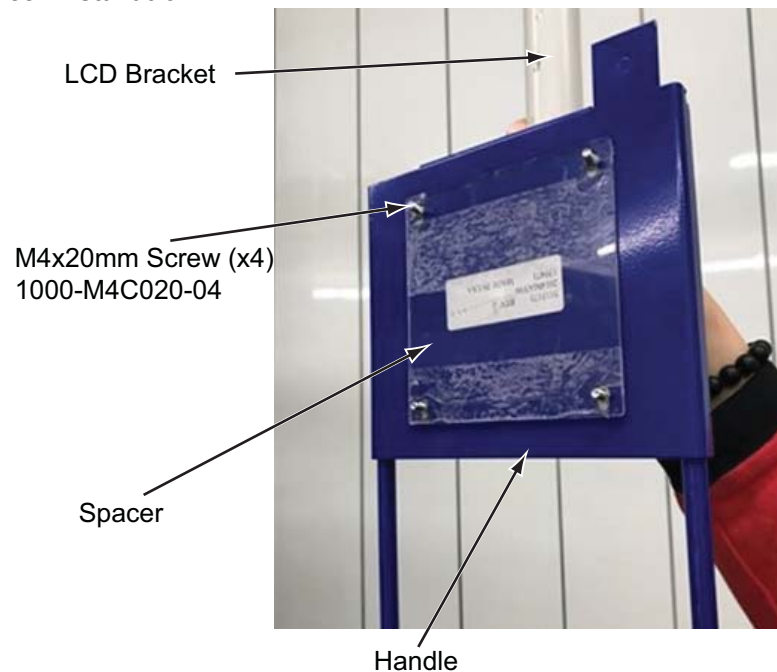
 **CAUTION** **DO NOT use four M4 screws shipped with BOOM-In-Room Kit to secure DIN6868-157 LCD monitor, ONLY use four M4x20mm screws (1000-M4C020-04, shipped with DIN option kit) to secure the monitor.**

Figure 2-6 Spacer Installation

- 7.) Fix the DIN monitor on the handle by using four M4x20mm screws (1000-M4C020-04, shipped with DIN option kit), apply torque of 2.3Nm+/-10%.


Figure 2-7 Monitor Installation

- 8.) Please refer Boom-In-Room Installation Manual for the In-Room Monitor cable connections.
- 9.) Reinstall the rear litter cover.

Note: If there is interference between the rear litter cover and the handle, you can loosen two screws (as shown in [Figure 2-8](#)) to install the rear litter cover, then fasten two screws by torque of 2.3Nm+/-10%.

Figure 2-8 Monitor Cables Connection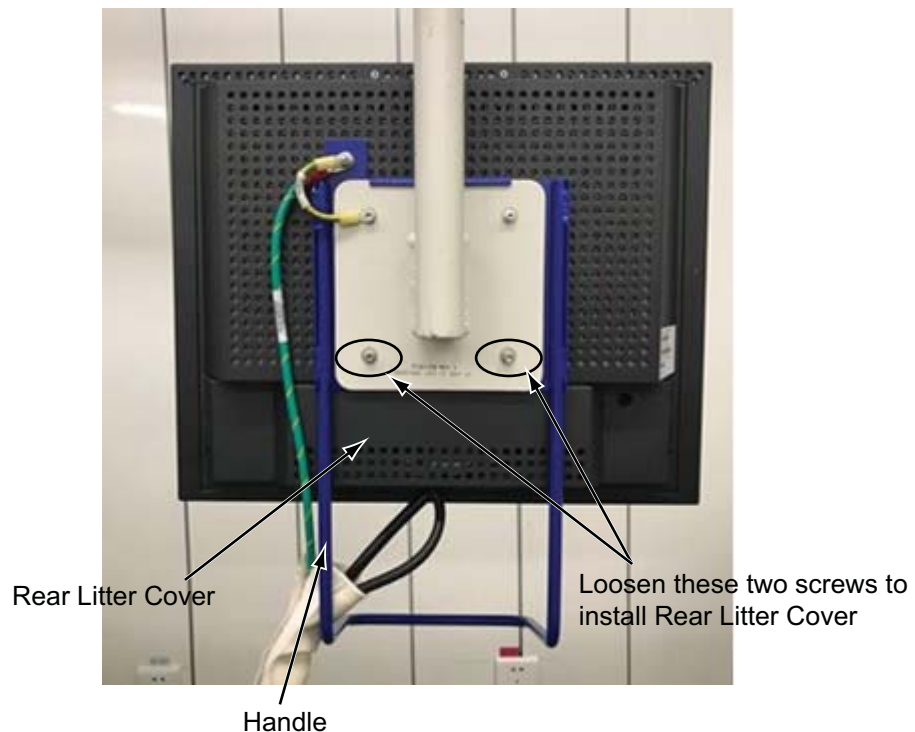


Section 3.0 OSD Settings

3.1 Operator-Room Monitor Settings


- 1.) Keep default setting: "LUT 3" in **LUT** menu.
- 2.) Checking methods:

Figure 2-9 LCD Monitor Operation Key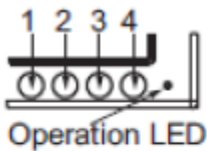

Key	Action
1	<ul style="list-style-type: none"> • Open OSD menu • Open main menu • Toggle between submenu
2	<ul style="list-style-type: none"> • Toggle between main menu • Decrease/change value
3	<ul style="list-style-type: none"> • Toggle between main menu • Increase/change value
4	Leave OSD or submenu (settings are retained)

- a.) Unlock the OSD by the key sequence: press key 4 once and key 2 three times.
- b.) Press key 3, move to **LUT**.
- c.) Check if the settings are as below.

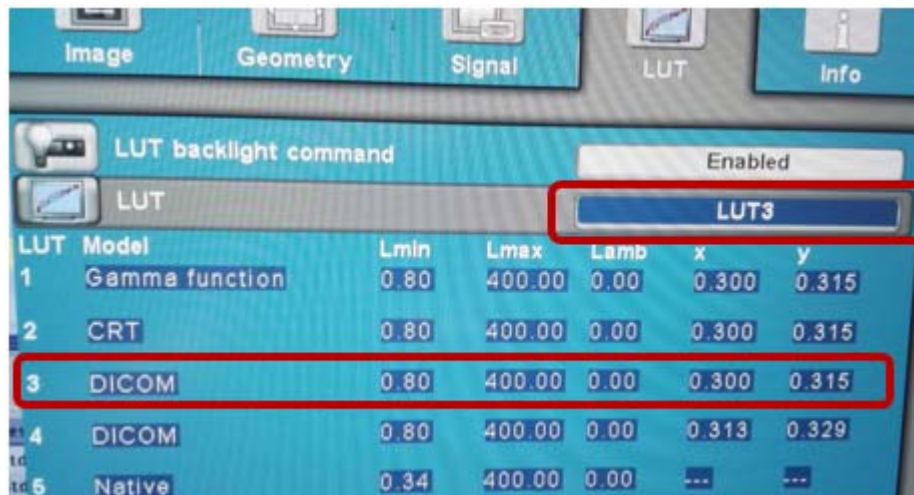- d.) Press key 4 to exit.
- e.) Press key 2 to "reject all changes in the OSD".
- f.) Lock the OSD by the key sequence: press key 4 once and key 2 three times.


NOTICE If the following two cases happened on the Site, please follow [Section 3.2 In-Room Monitor Settings](#) to set the operator room image monitor.

- The ambient illuminance in operator room is high and the operator room image monitor can't pass DIN6868-157 test
- Or, the operator room ambient illuminance is 200Lx to 500Lx during DIN6868-157 test


3.2 In-Room Monitor Settings

Figure 2-10 LCD Monitor Operation Key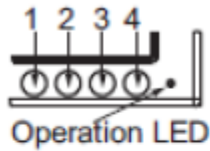


Key	Action
1	<ul style="list-style-type: none"> • Open OSD menu • Open main menu • Toggle between submenu
2	<ul style="list-style-type: none"> • Toggle between main menu • Decrease/change value
3	<ul style="list-style-type: none"> • Toggle between main menu • Increase/change value
4	Leave OSD or submenu (settings are retained)

- 1.) Unlock the OSD by the key sequence: press key 4 once and key 2 three times.
- 2.) Press key 3, move to **LUT**.
- 3.) Press key 1 enter to sub menu, then use key 3 to select **User** for LUT.
- 4.) Press key 4 back to main menu.
- 5.) Press key 3 to **Adjust LUT**.
- 6.) Use key 1 move to **Lamb**, press key 3 to set as "060", check if the "Current target value (on the OSD lower left corner)" = 1.20cd/sqm.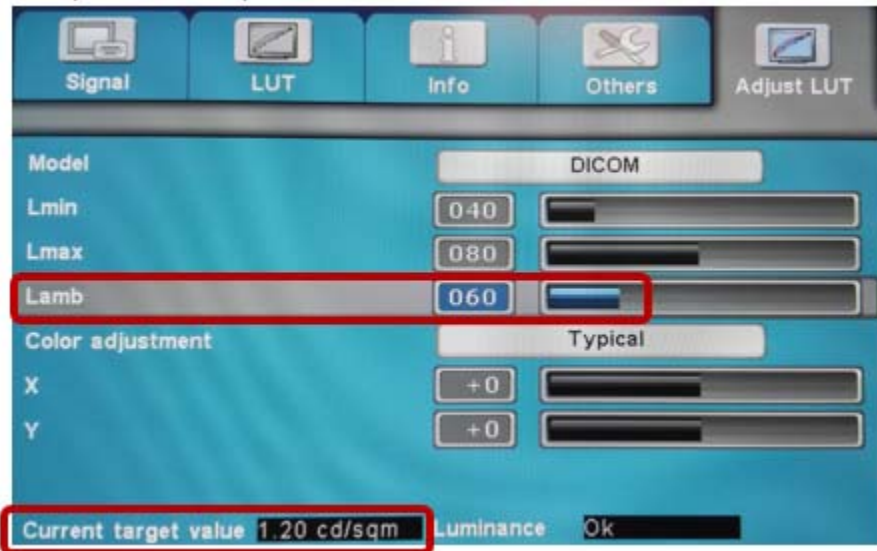


- 7.) Press key 1 to **Color Adjustment**, use key 3 to select **Typical**.
- 8.) Press key 4 back to main menu. Use key 2 back to **LUT**, check if the settings are the same as below.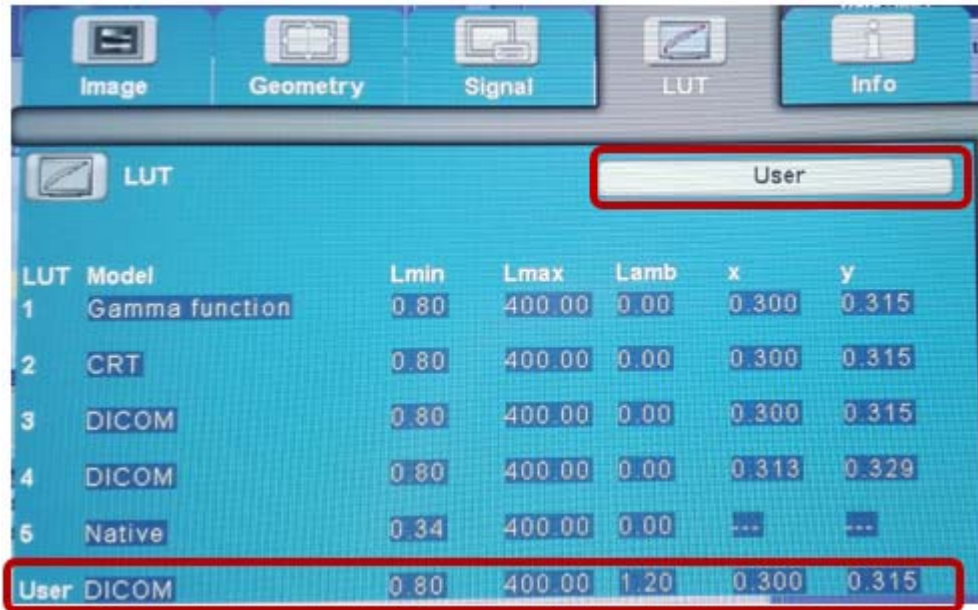


- 9.) Press key 4 to exit.
- 10.) Press key 1 to "Save all changes in the OSD".
- 11.) Lock the OSD by the key sequence: press key 4 once and key 2 three times.

Section 4.0 TG18 Pattern Installing and Viewing

- 1.) Click [Tools Chest] button, then select "Install TG18 Pattern" in pop up window.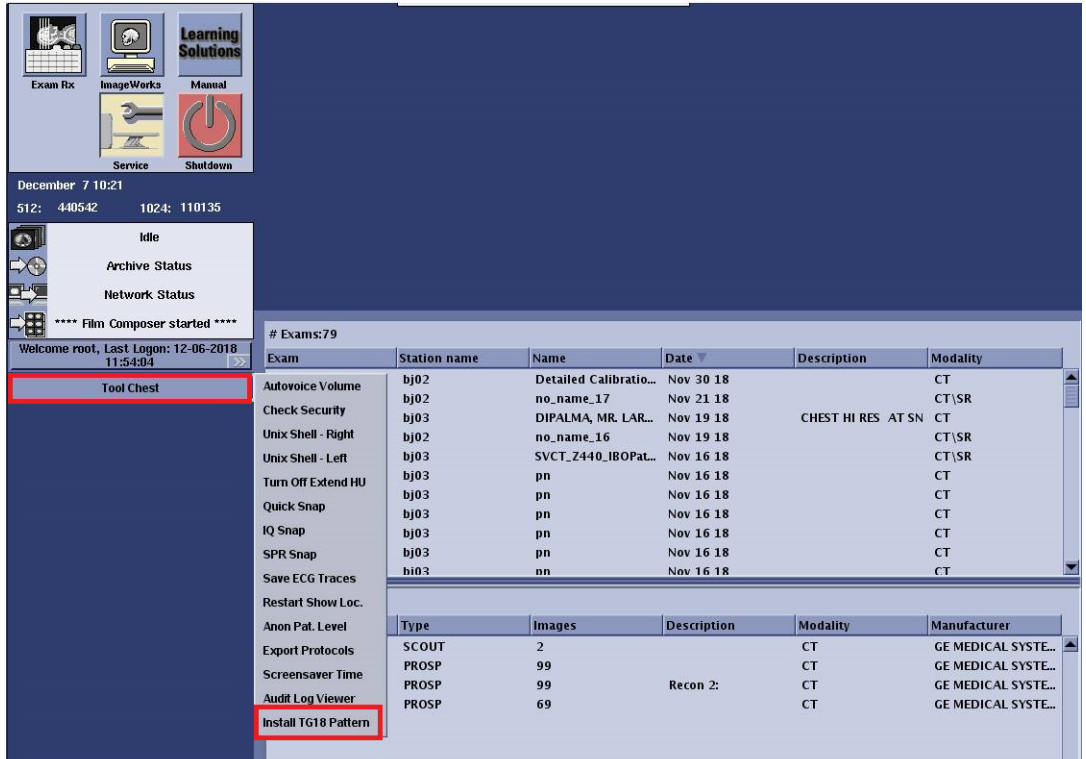

- 2.) Switch to ImageWorks (1) window and search "AAPM" (2, image name) in the search bar, then TG18 Pattern will be found in the exam list.
- 3.) Choose TG18 Pattern (3) and click "Viewer" (4) to enter the image window.

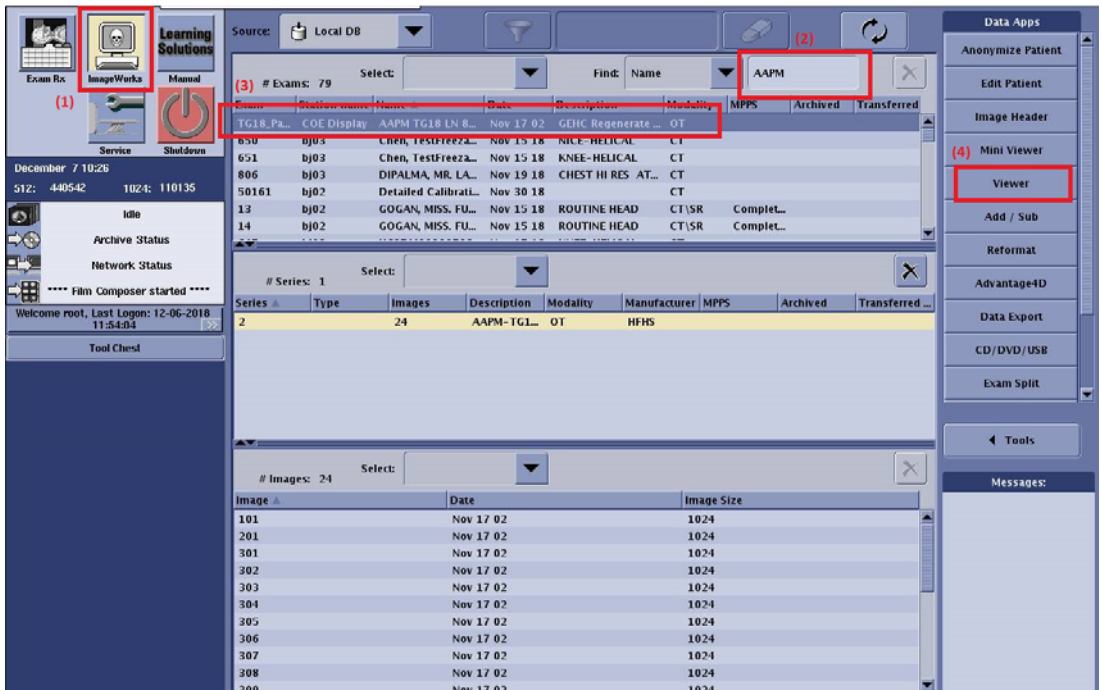2 - INSTALLATION PROCEDURE

Section 5.0 FRU Request

If DIN6868-157 19inch LCD Monitor is defective, please order new monitor according to the below list.


Failure	FRU	Picture
Operator-Room Monitor	5746371 (Display) + 5561139 (Stand)	
In-Room Monitor	5746371 (Display)	

Table 2-1 DIN Monitor FRU List

Section 6.0 Function Check

- 1.) Perform Power On Self Test:
 - a.) Switch ON the main switch of the OC.
 The test progress information is displayed on the monitor.
 - b.) Verify that the system starts up normally.
- 2.) Perform the normal test scanning and check that the images are normal.

www.gehealthcare.com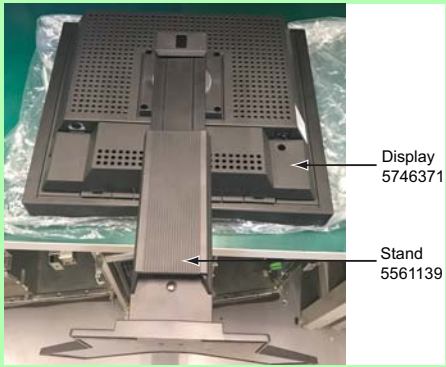