












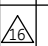
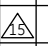
Global Table Interconnect

This drawing is applicable for


- 5121647 GT LONG TABLE ASSY
- 5122080 GT MIDDLE TABLE ASSY
- 5121647-2 HD-GT2000 WITH SHIP COLLECTOR
- 5122080-2 HD-GT1700 WITH SHIP COLLECTOR
- 5174269-3 GT PET TABLE ASSY
- 5121647-3 GT2000 N9 ASSY
- 5122080-3 GT1700 N9 ASSY
- 5121647-4 GT2000 N9 ED3 ASSY
- 5122080-4 GT1700 N9 ED3 ASSY
- 5380966 GT675 table
-  5272966-2 Heavy Patient table-650lb Patient For Freedom
-  5272966-3 Heavy patient table ED3 Assy
-  6121647-3 GT2000N9 ASSY, Russia
-  6121647-4 GT2000N9 ED3 ASSY, Russia
-  6121647-40 GT2000N9 ASSY, Russia

Note:

- REFER TO 5127766 FOR FUNCTIONAL DRAWINGS.
- (*) Not included in GT PET TABLE ASSY

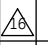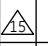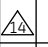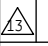								
	p.1	Add 6121647-40 to physical ID..		2238299	Sep - 22 -2017	Imaizumi	Masaura	
	p.1	Add 6121657-3 and 6121647-4 o physical ID..		2134070	May - 25 -2012	Ariyama	Takeshima	
	p.1	Add 5272966-2 and 5272966-3 to physical ID.		2128673	Jan - 24 -2012	K.Sato	N.Ariyama	
	All	Add page1 and all sheets revised to add RoHS P/N.		2087537	Nov - 15 - 2011	K.Odagiri	N.Ariyama	
REV.	PAGE	DESCRIPTION	—	ECR/ECO	—	DATE	REV. BY	CH.
	1	Sep-22-2017	DR. 有山	Elec eng	GLOBAL TABLE INTERCONNECT			
	1	May-25-2012	CH. 高市					
	1	Jan-24-2012	APP. 薩た	Nov/05/2004	5121647SCH			PAGE OF 1/5

OVER VIEW

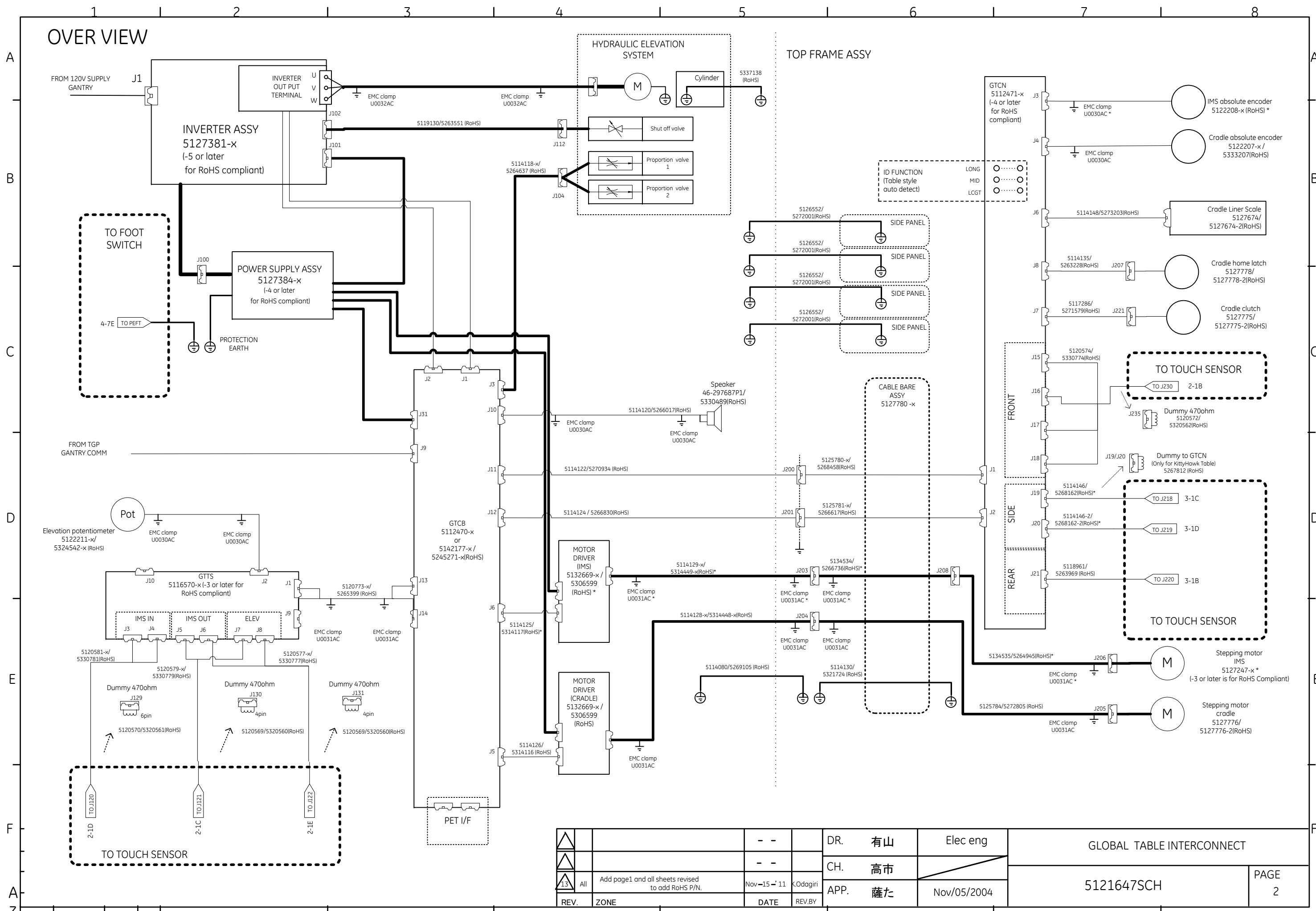


DR.	有山	Elec eng	GLOBAL TABLE INTERCONNECT	
CH.	高市		5121647SCH	
APP.	薩た	Nov/05/2004	PAGE 2	
REV.	ZONE	DATE	REV.BY	
13	All	Add page1 and all sheets revised to add RoHS P/N.	Nov-15-11	K.Odagiri


State: RELEASE - Document is released and under formal Change Control. Changes are subject to the ECR/ECO Process.
 See the GEHC Myworkshop System to determine the status of this document.

TOUCH SENSOR


TOP COVER FRONT ASSY 5127465 - x
(-5 or later is for RoHS compliant)

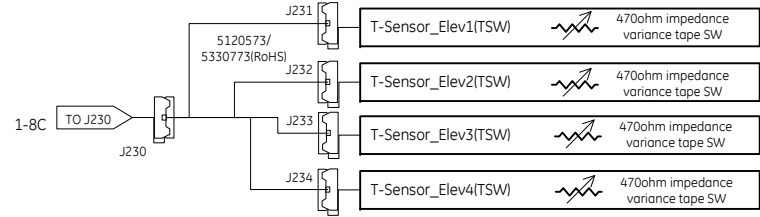



IMS LEFT SIDE COVER 5191722-x

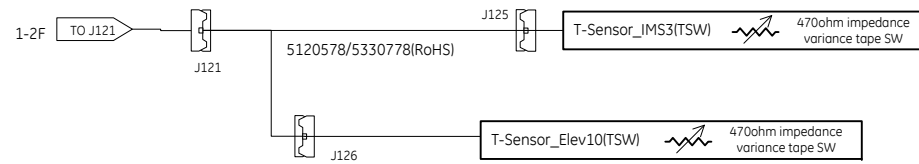

IMS REAR UPPER COVER

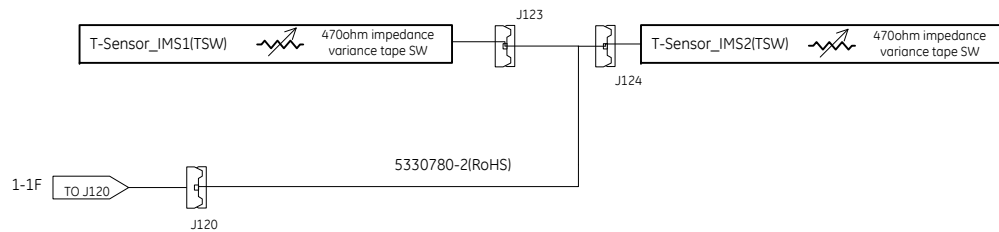





IMS RIGHT SIDE COVER 5191736-x

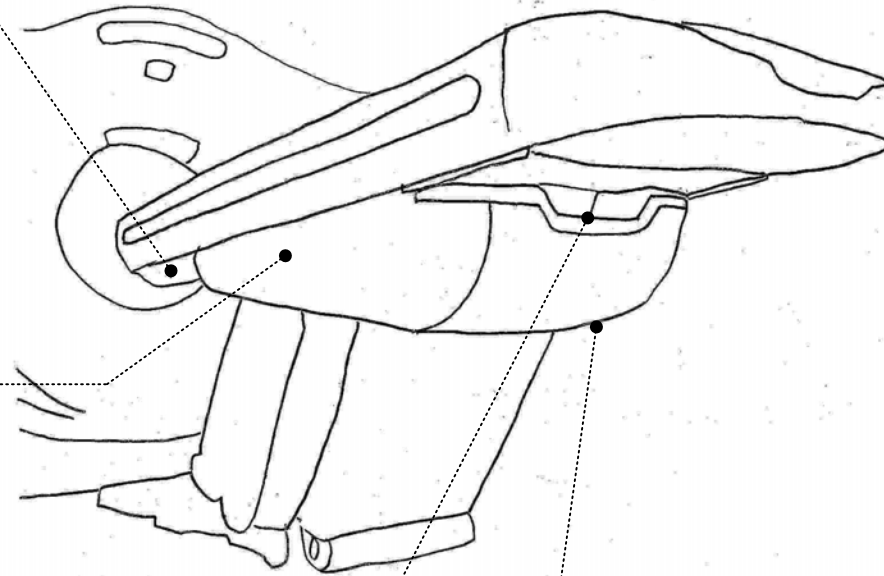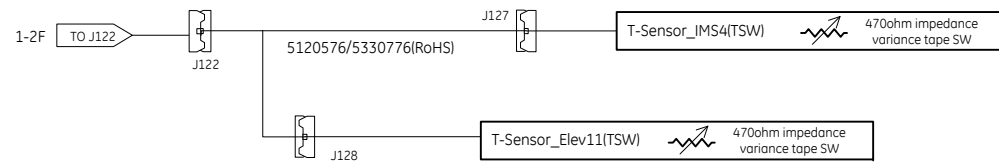

△		--		DR.	有山	Elec eng	GLOBAL TABLE INTERCONNECT	
△		--		CH.	高市		PAGE	
13	All	Add page1 and all sheets revised to add RoHS P/N.		Nov-15-11	K.Odagiri	APP.	薩た	Nov/05/2004
REV.	ZONE			DATE	REV.BY		5121647SCH	3


State: RELEASE - Document is released and under formal Change Control. Changes are subject to the ECR/ECO Process.
 See the GEHC Myworkshop System to determine the status of this document.

TOUCH SENSOR

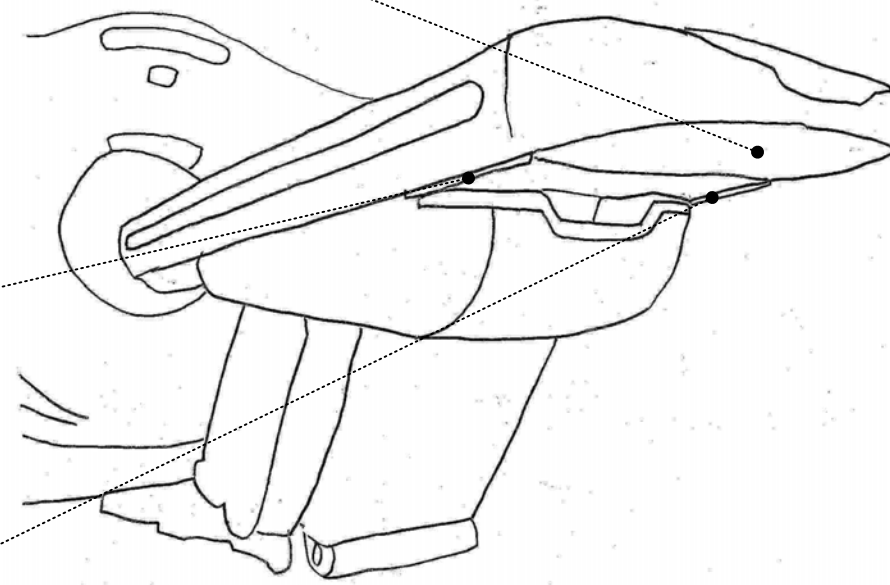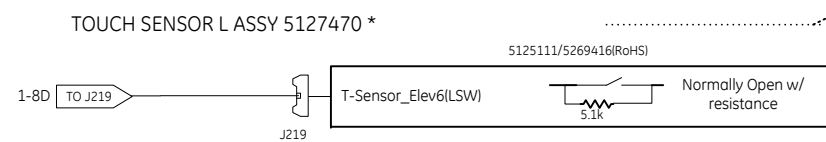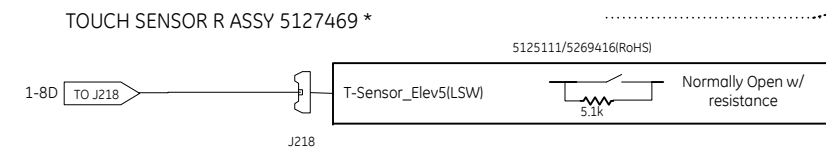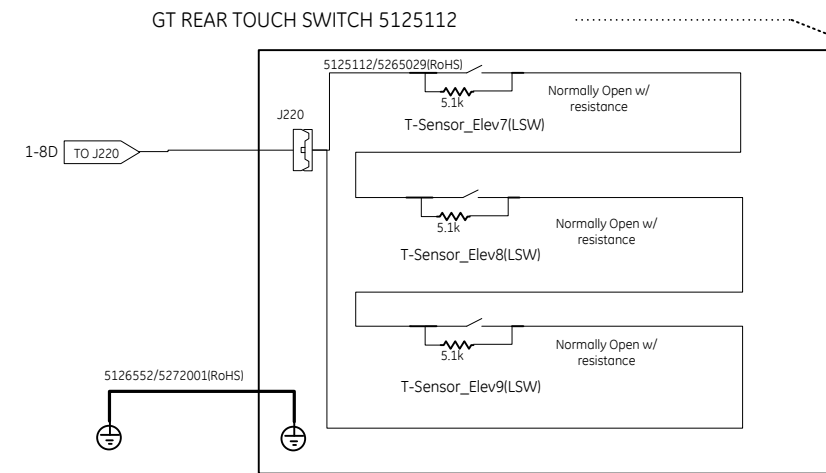

△		--		DR.	有山	Elec eng	GLOBAL TABLE INTERCONNECT	
△		--		CH.	高市		5121647SCH	
13	All	Add page1 and all sheets revised to add RoHS P/N.		Nov-15-11	K.Odagiri	APP.	薩た	Nov/05/2004
REV.	ZONE			DATE	REV.BY			PAGE 4

State: RELEASE - Document is released and under formal Change Control. Changes are subject to the ECR/ECO Process. See the GEHC Myworkshop System to determine the status of this document.

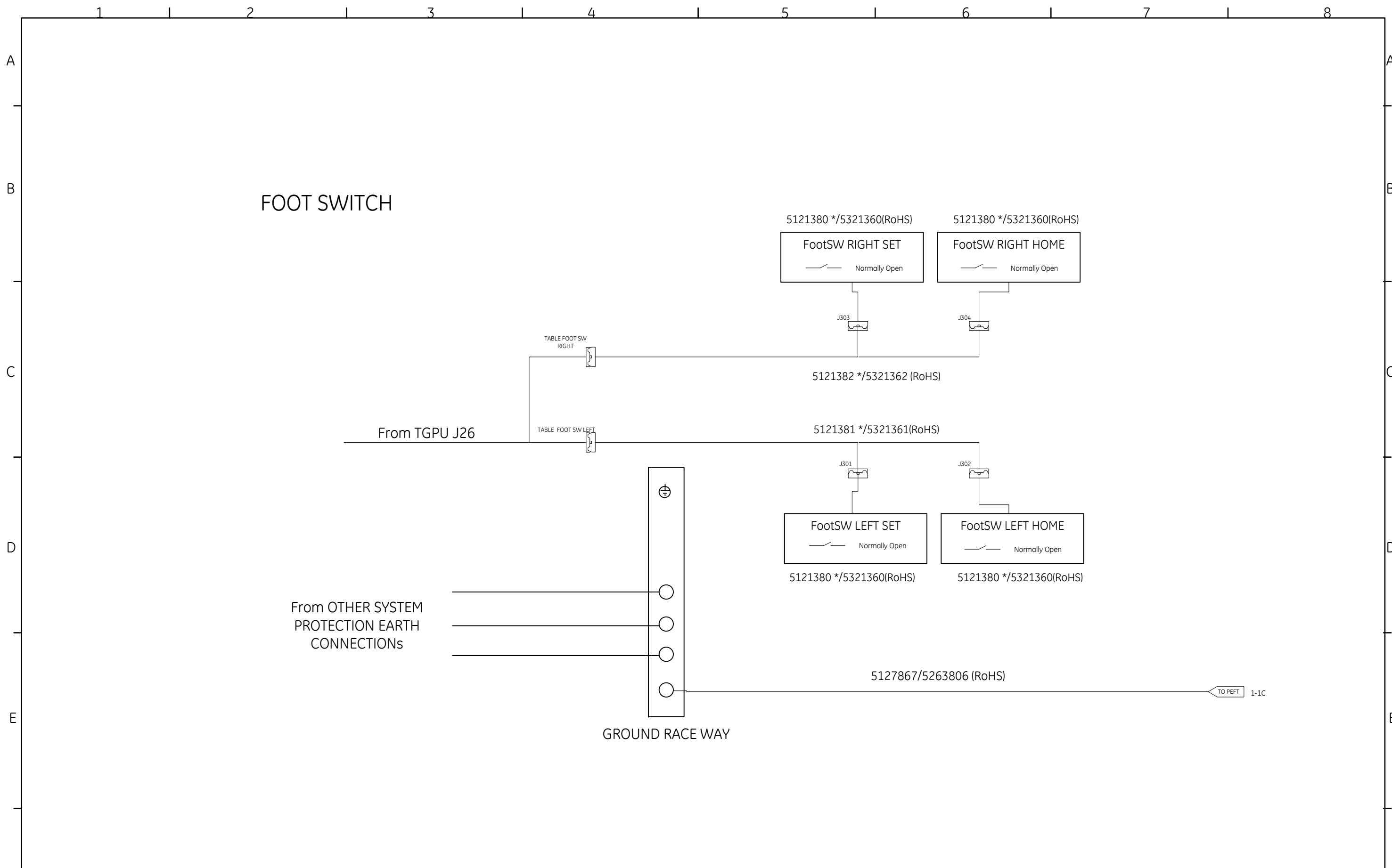

△		--		DR.	有山	Elec eng	GLOBAL TABLE INTERCONNECT	
△		--		CH.	高市		PAGE	
13	All	Add page1 and all sheets revised to add RoHS P/N.		Nov-15-11	K.Odagiri	APP.	薩た	Nov/05/2004
REV.	ZONE			DATE	REV.BY		5121647SCH	5

State: RELEASE - Document is released and under formal Change Control. Changes are subject to the ECR/ECO Process. See the GEHC Myworkshop System to determine the status of this document.