



# NIO CONSOLE OUTLET AC BOX SCH



## POWER INLET ASSY

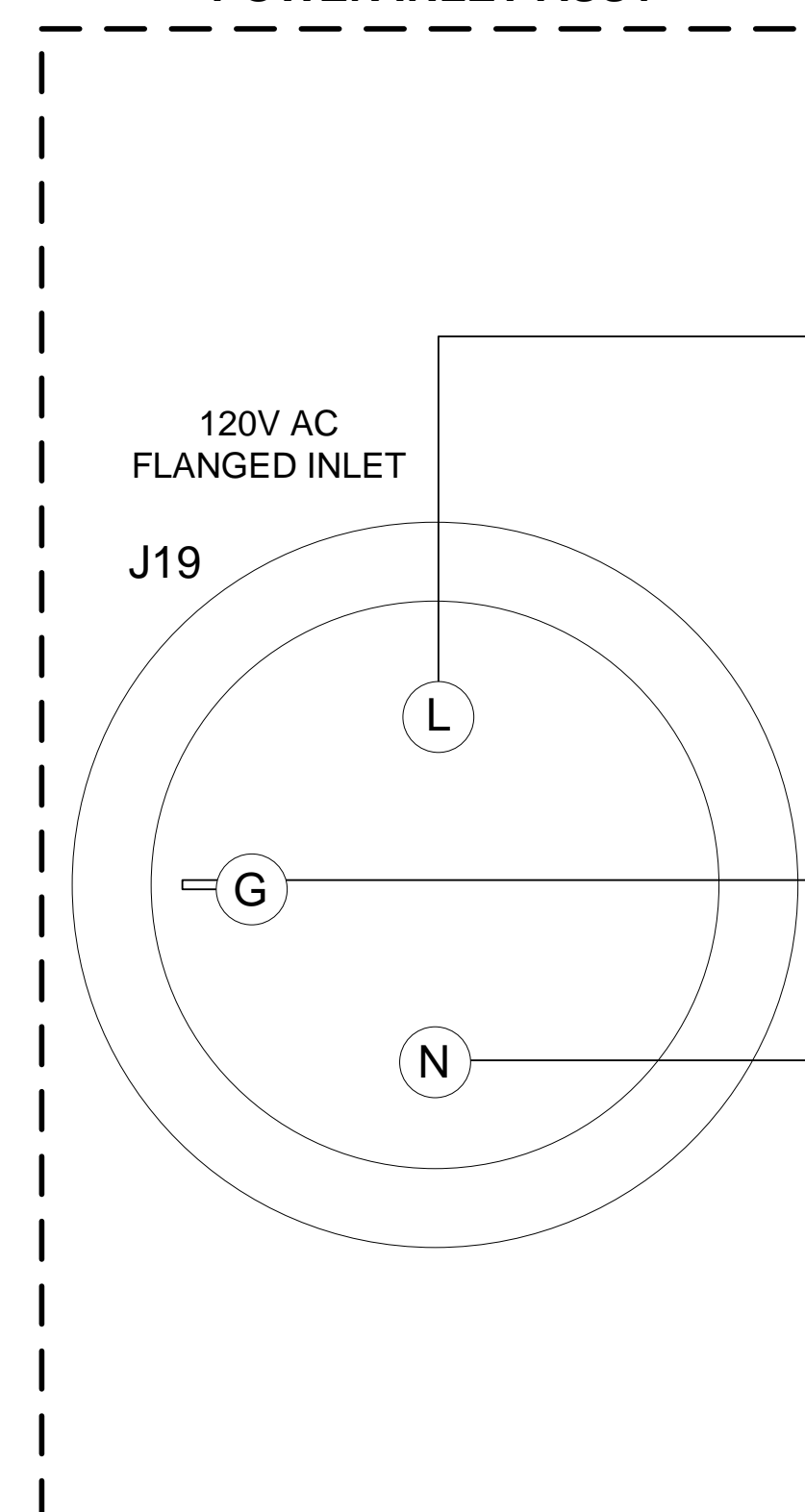


## POWER SWITCH ASSY

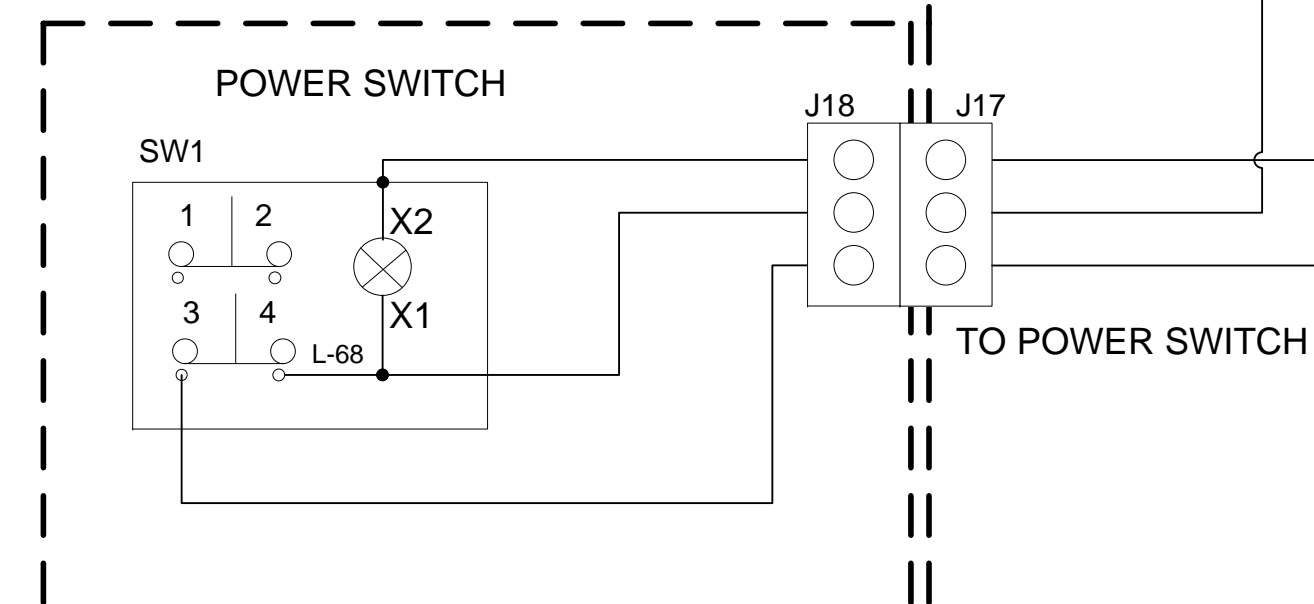


CIRCUIT BREAKER  
SIGNAL POLE SERIES  
15AMPS 50/60Hz  
CB1

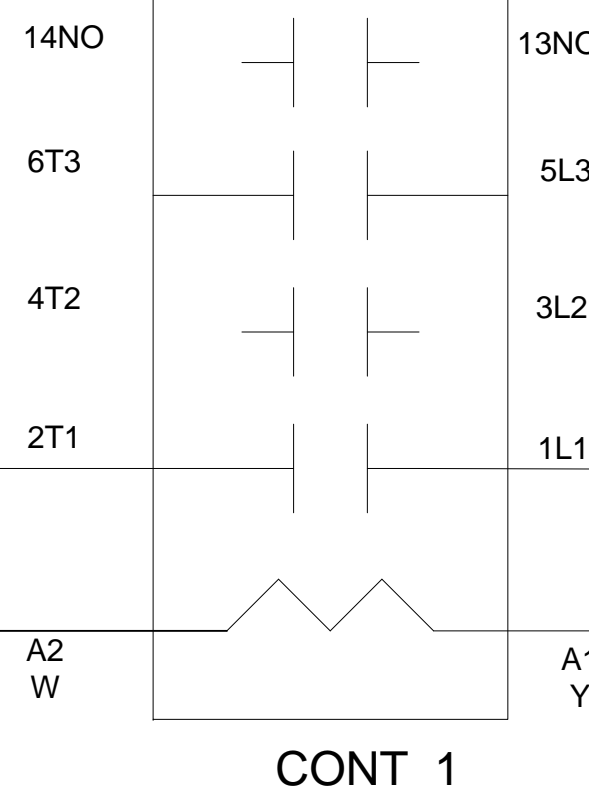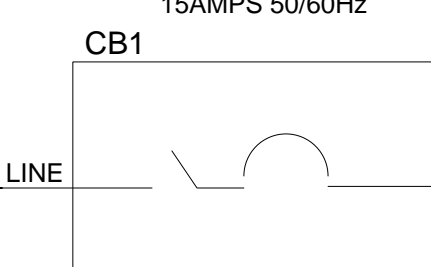


PERFORMANCE EMC FILTER  
250V AC 16AMPS 50/60Hz  
FL\_1

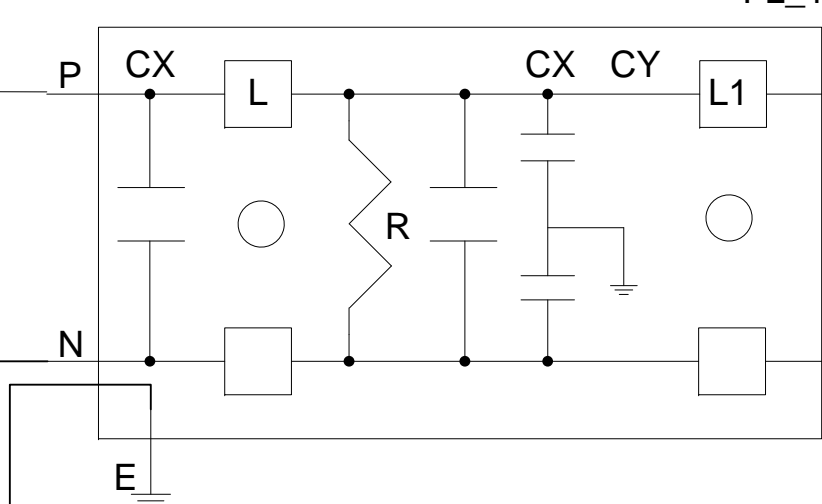


IEC POWER CONNECTOR, FEMALE, 250V 15A

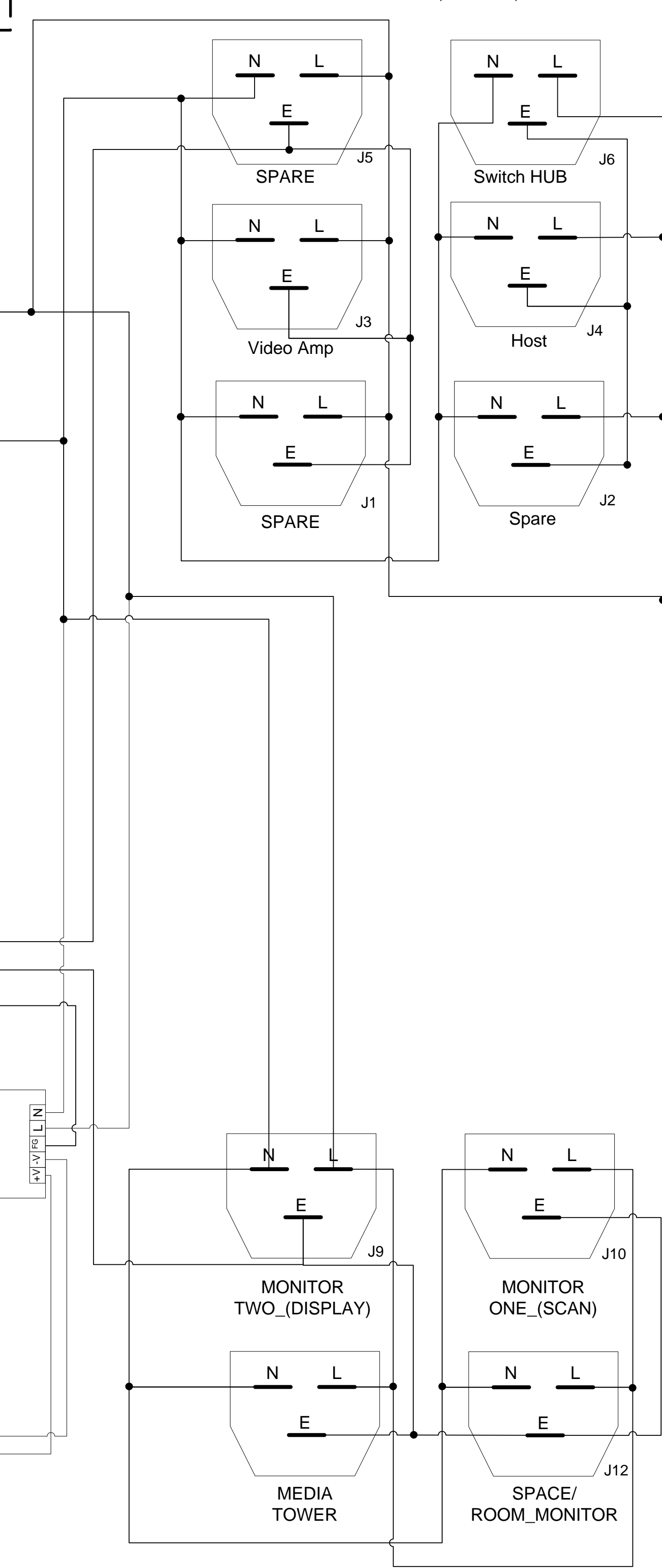

GROUND BAR  
GB1

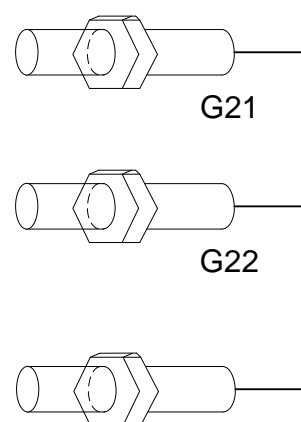


IEC POWER CONNECTOR  
FEMALE, 250V 15A

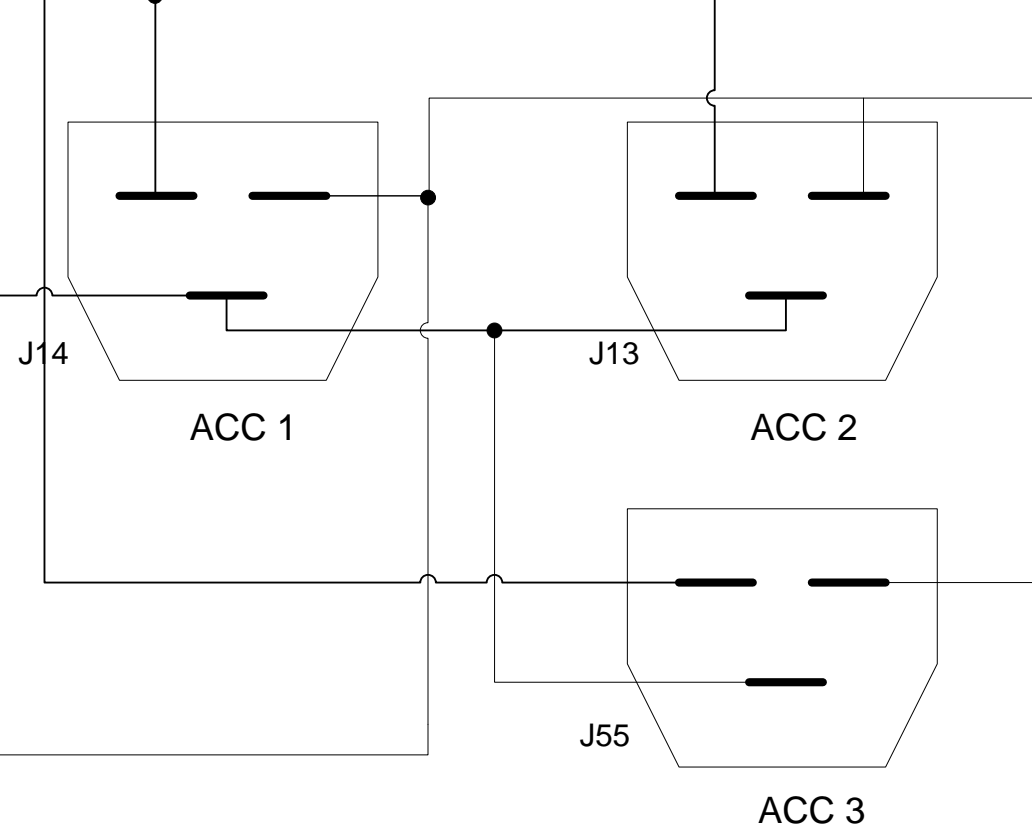


POWER SUPPLY

GSCB PS

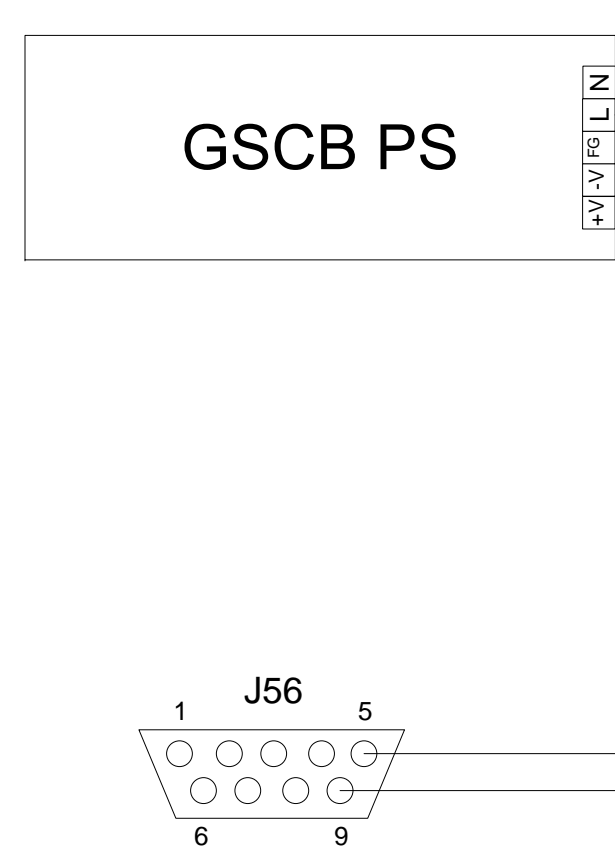

CIRCUIT BREAKER  
SIGNAL POLE SERIES  
15AMPS 50/60Hz  
CB2

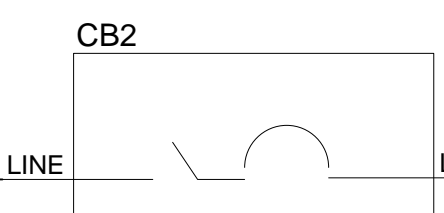

REV 2 ECO 2156887  
MADE BY Wu Wendong DATE 2013-Oct-10

DESCRIPTION Update the J2, J4 and J6 connector information  
APP BY DATE

MADE BY: WU WENDONG  
DATE: FEB09, 2012

ELECTRONIC RELEASE PROCESS  
DESIGN RELEASED



GE Healthcare  
China Corporation

DESIGN TITLE NIO OUTLET AC BOX ASSY  
FIRST MADE FOR

5412524-2SCH

REV 2

SIZE D

SHEET

1/1