Optima CT520 DoD IB Upgrade Instruction

Service Documentation

5745295-1EN

Rev 1

© 2016, GE Company,
All Rights Reserved.

LEGAL NOTES

TRADEMARKS

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

COPYRIGHTS

All Material, Copyright © 2016 by GE Company, Inc. All rights reserved.

This page intentionally left blank.

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DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent write "Damage In Shipment" on ALL copies of the freight or express bill BEFORE delivery is accepted or "signed for" by a GE representative or hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

Fill out a report on <http://egems.med.ge.com/edq/home.jsp>

Contact the local service coordinator.

CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

LITHIUM BATTERY CAUTIONARY STATEMENTS

CAUTION
Risk of
Explosion

Danger of explosion if battery is incorrectly replaced. Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.

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OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the GEMS CQA Process to report all omissions, errors, and defects in this publication.

Revision History

Revision	Date	Reason for change
1	03/22/2016	Initial release.

This page intentionally left blank.

Table of Contents

Chapter 1	
Upgrade Overview	17
Section 1.0	
Flowchart	17
Section 2.0	
Material List	18
Chapter 2	
Upgrade Procedure	19
Section 1.0	
Preparation in Site.....	19
1.1 Record System INFO Information.....	19
1.1.1 Record All Remote Host Network Information	19
1.1.2 Record Product Network Filters (PNF) Information	20
1.1.3 Record AW IP for Direct Information	20
1.2 Save System State	21
Section 2.0	
NIO16 Console Upgrade	22
2.1 Old Console (Z800) Removal	22
2.2 New Console (Z820) Installation.....	23
2.2.1 Preparative	23
2.2.2 Unpack NIO16 Console with Z820 PC	26
2.2.3 GPU Card Installation	27
2.2.4 NIO16 Console Cable Connections.....	28
2.2.5 Console Cover Installation.....	31
Section 3.0	
System Configuration	32
3.1 Restore System State.....	32
3.2 Configure System Information on Z820 (SUSE).....	32
Section 4.0	
Flash Download.....	35
Section 5.0	
ASiR Option Installation	36
5.1 Check GPU Card.....	36
5.2 Option Installation.....	36
Section 6.0	
Finalization.....	37

This page intentionally left blank.

Chapter 1

Upgrade Overview

The **Upgrade Kit** is only available for Optima CT520 DoD IB system with NIO16 Console Z800 Host Computer. The upgrade instruction is to guide FE how to upgrade NIO16 Console Host Computer from Z800 (App version: 12BW31.xx) to Z820 (App version: 14BW25.xx) on CT-only Optima CT520 DoD system.


NOTICE

If the Boom-In-Room monitor has been installed on Site, customer **MUST** order B7710AU to match OC upgrade. The Boom-In-Room Upgrade Kit Instruction (5719432-1EN, shipped with B7710AU) also applies to Optima CT520 DoD.

Section 1.0

Flowchart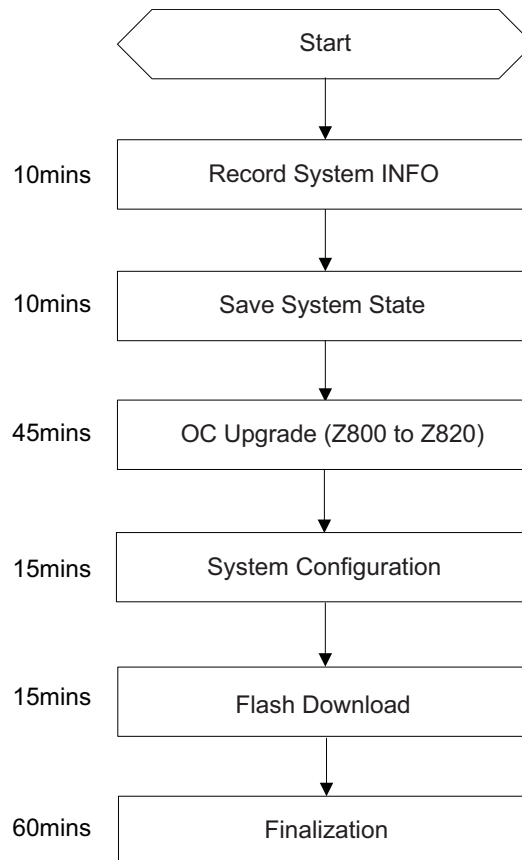

Section 2.0 Material List

Verify that the contents of upgrade kit are correct according to the material list below.

Description	Number
CT520 TP/SW Collector	B77052CB
NIO Console without Cover	B76992CB
CT520 IB Upgrade Kit	B78442CB

Table 1-1 Optima CT520 DoD IB Upgrade Package (B78432CB)

Description	Number
Upgrade Instruction	5745295-1EN
DP to DVI Cable	5408703

Table 1-2 CT520 IB Upgrade Kit (B78442CB)

Description	Number	
	ASiR	Non ASiR
Console Cover Kit	B77092CB	B70762RT

Table 1-3 Console Cover Kit

Chapter 2

Upgrade Procedure


Section 1.0 Preparation in Site

1.1 Record System INFO Information

Write down all of the system INFO information in Console Information Sheets, please refer to the Service Methods CD 5457941-2EN service manual.

1.1.1 Record All Remote Host Network Information


- 1.) Select [ImageWorks] from the desktop.
- 2.) Select in sequence: Tools -> Network Configuration.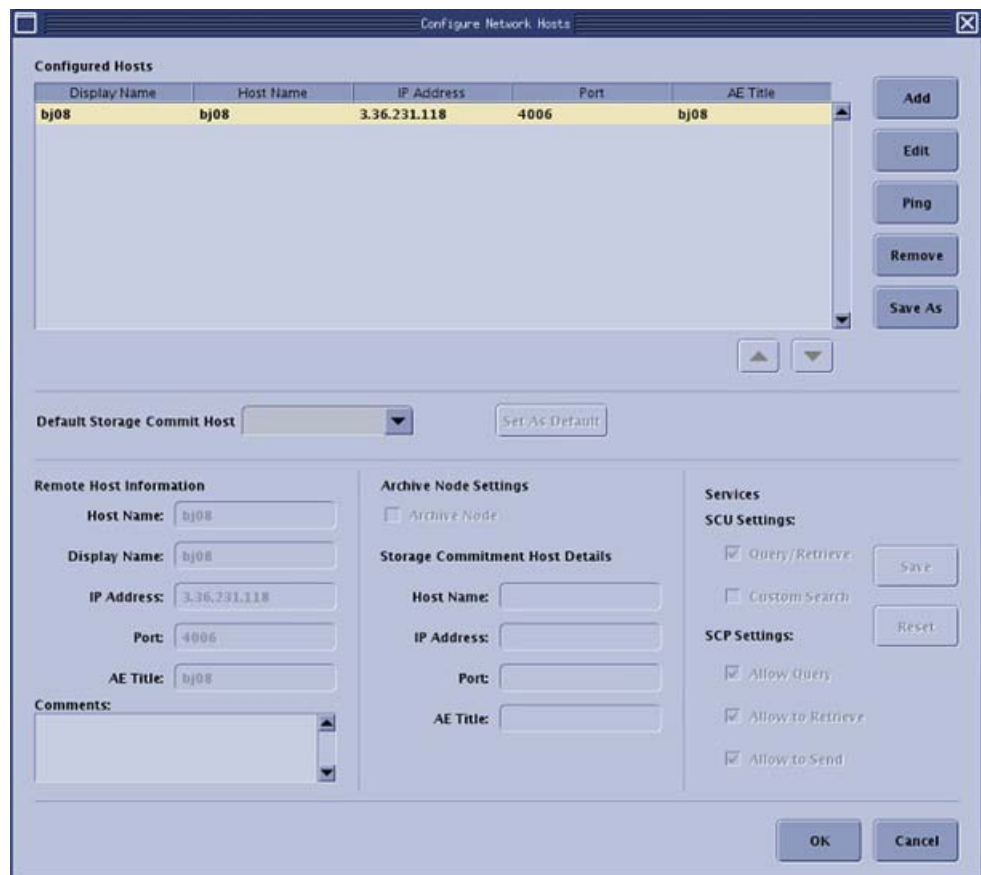

- 3.) Select host name to display remote host information.
- 4.) Record the network information.


1.1.2 Record Product Network Filters (PNF) Information

- 1.) Select [Service] from the desktop.
- 2.) Select in sequence: Configuration -> Configure PNF.
- 3.) Record PNF information.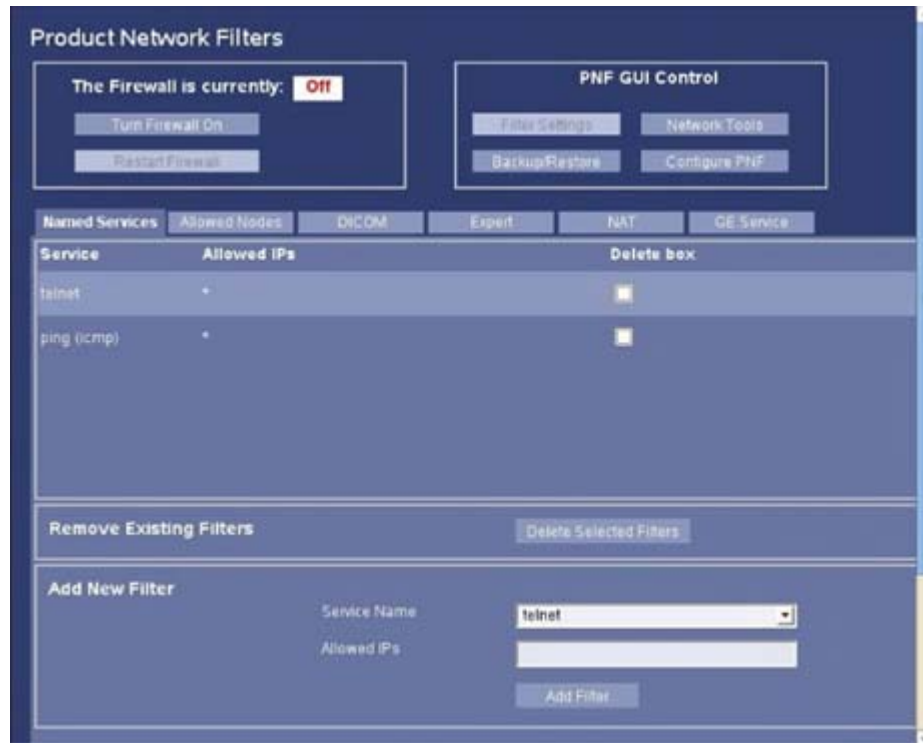

1.1.3 Record AW IP for Direct Information

- 1.) Select [Service] from the desktop.
- 2.) Select in sequence: Configuration -> Configure AW DirectConnect and record AW IP address.

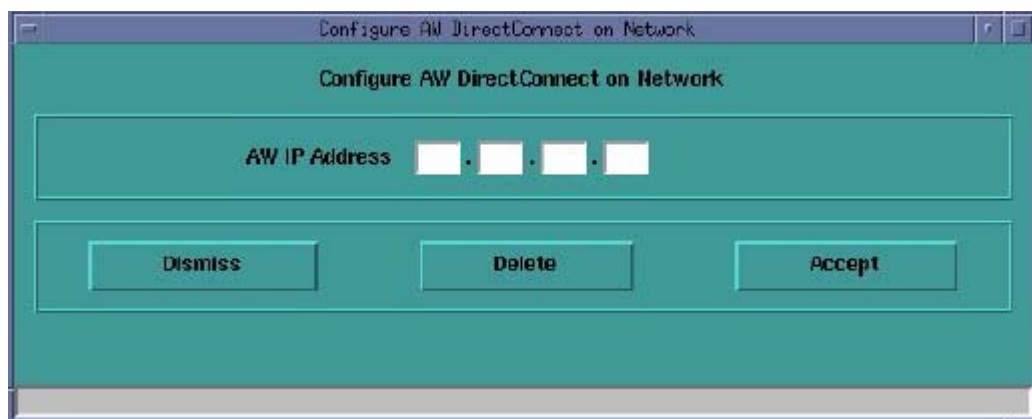- 3.) Select in sequence: Configuration -> Configure AW Server Client and record AW server INFO.

1.2 Save System State

Refer to the latest service methods (5457941-2EN) to understand the detailed Save State Procedure and other information saving procedure.

Note: If a Save State was just completed, it can be omitted here.

Note: When performing Save/Restore System State, make sure there is **ONLY** one USB storage device plugged in USB ports of both Console and Media Tower simultaneously, otherwise it may result in incorrect data storage.

- 1.) Insert the System State DVD-RAM media into the peripheral tower drive or USB media in any of the console's USB ports.
- 2.) Select [Service Desktop] -> [Utilities].
- 3.) Select [System State] for DVD-RAM media or [System State - USB] for USB media. The System State Save/Restore screen appears.
- 4.) Select [All] to select all the calcs, characterization, etc.
- 5.) Select [Save].
- 6.) If DVD-RAM or USB is ready, select [Yes].
- 7.) When completed select [Dismiss].
- 8.) Remove System State DVD-RAM media from the peripheral tower or USB media from USB port.

Section 2.0

NIO16 Console Upgrade

2.1 Old Console (Z800) Removal

NOTICE Make Sure to follow all Lockout/Tagout requirements while performing this procedure.

- 1.) Move the cradle all the way out of the gantry and lower to its lowest point.
- 2.) Shut down the system and power off the Operator Console power switch.
- 3.) Switch OFF the breaker in the Power Distribution Box (hospital supply).


Note: If there is UPS, switch OFF the UPS output.

- 4.) Visually verify proper labeling of each cable and disconnect all console rear cables.
- 5.) Disconnect the cables between the old console and system (Gantry, PDU and local hospital network).
- 6.) Remove the old OC aside.

Table 2-1 System Interconnect Cables for Console

No.	Description	Long Cable	Short Cable
53	LVAC Power Cable - PDU to Operator Console	2343531	2343531-2
56	Ground, Raceway to Console	2371450-3	2371450-4
101	Signal Cable - Gantry to Console	5419981	5419981-2
102	Signal Cable (Ethernet) - Gantry to Console	2373436-2	2373436-3
103	Data Cable (Fiber Optic) - Gantry to Console	5432019	

Figure 2-2 NIO16 OC Interconnect (with Z820)

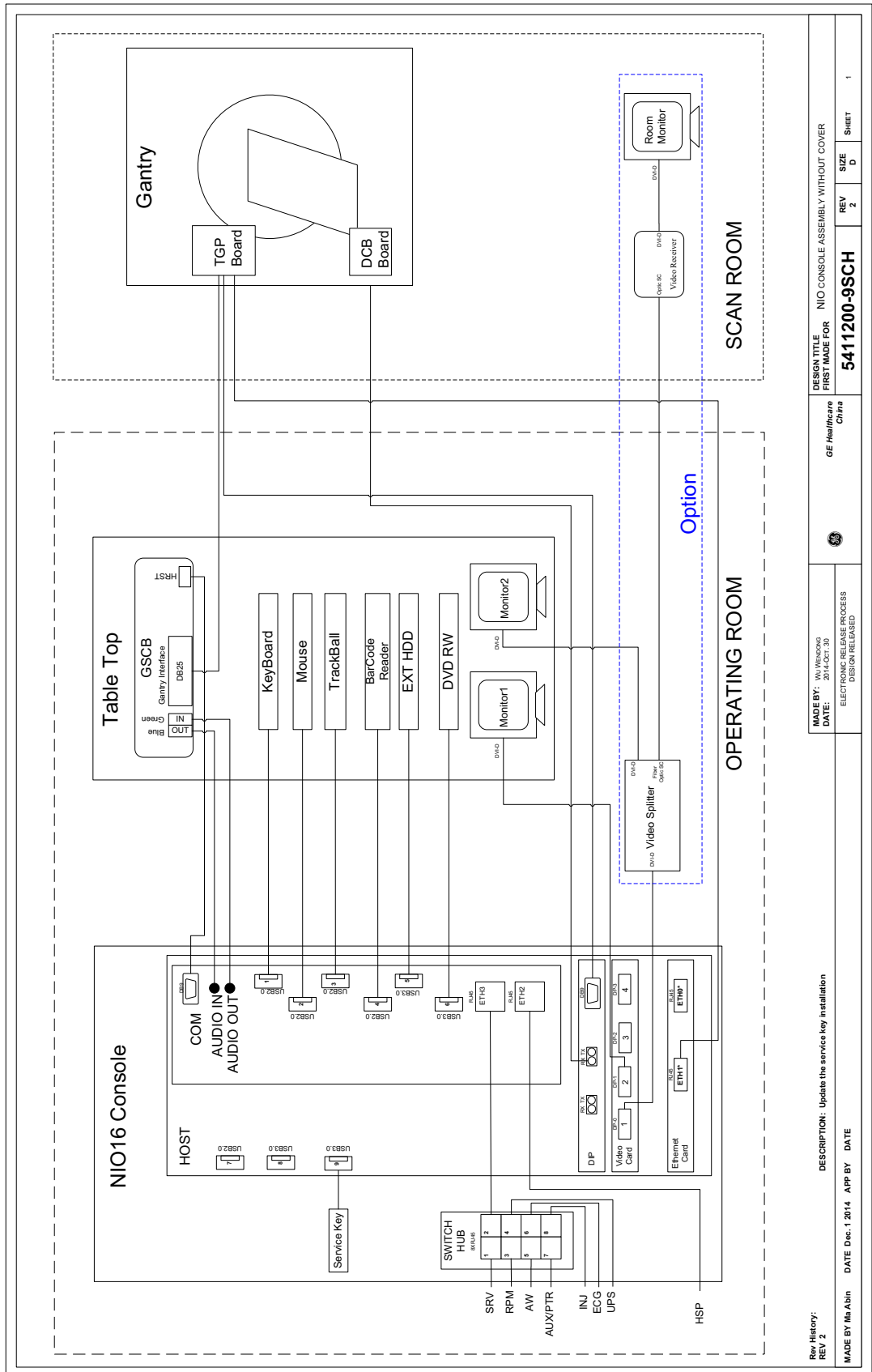2.2.2 Unpack NIO16 Console with Z820 PC

Please read details about NIO16 installation procedure in Service Manuals.

- 1.) Unpack console packaging.
 - a.) Remove all items from the console.
 - b.) Remove all packaging and discard per local requirement.
 - c.) Place the step-board under the front edge of the skid and step on it to raise the front edge of the skid as in

Figure 2-3 Step-Board Used to Raise Front Edge of Skid

- d.) Remove the two front cushions from the bottom of the skid.


Figure 2-4 Cushion on Bottom of Skid

- e.) Lift up on the strap on the front of the step-board ([Figure 2-3](#)) to lower the skid. Remove the step-board.
- f.) Ensure the console stabilizers are in line with the notched portion at the front of the skid. This will allow enough clearance to smoothly roll the console down the ramps.

- 2.) Move and place the console to the operator room.
- 3.) Adjust two rubber footer to prevent console from sliding.

Figure 2-5 Rubber Footer Adjustment

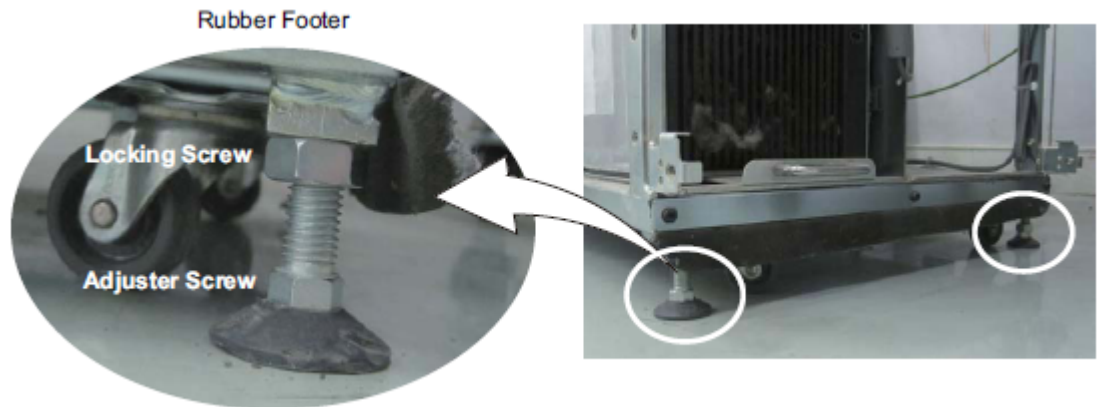- 4.) Remove the Host PC support bracket from left side of Z820.

Figure 2-6 Host PC Support Bracket

2.2.3 GPU Card Installation

Note: If customer does not purchase ASiR option, please ignore this section.
Please refer to ASiR Upgrade Installation Manual (5344359-1EN_r5 or later) to install GPU card (shipped with B77092CB) in Z820 Host Computer.

2.2.4 NIO16 Console Cable Connections

Figure 2-7 Z820 Host Computer Connections



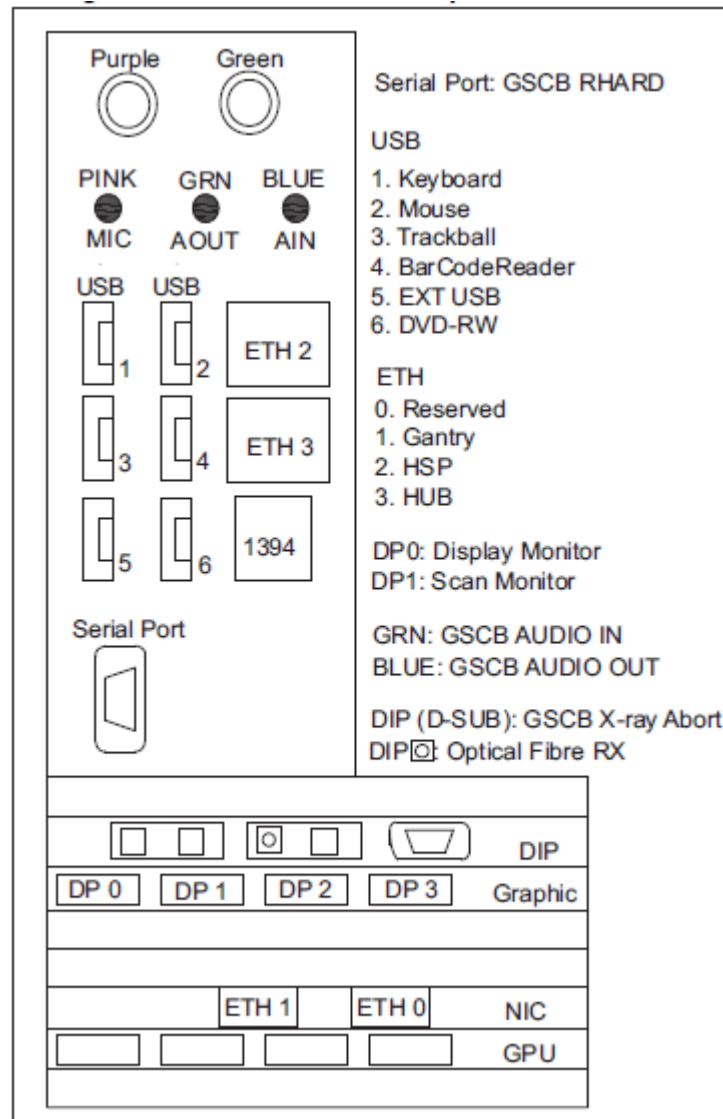
Figure 2-8 NIO16 Console AC Box Assignments

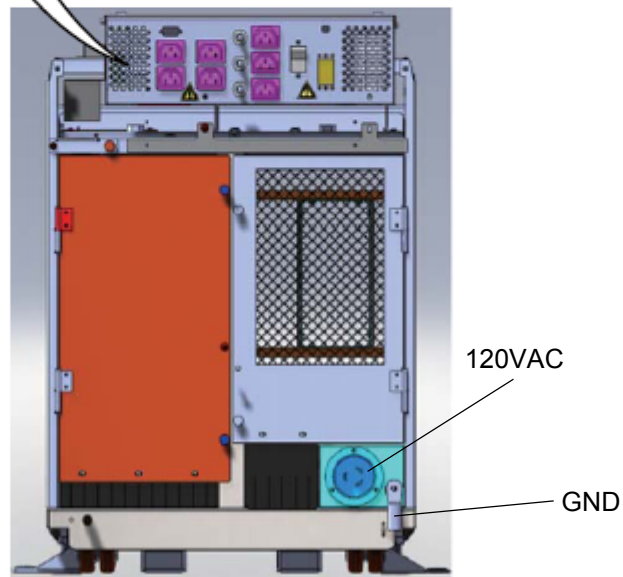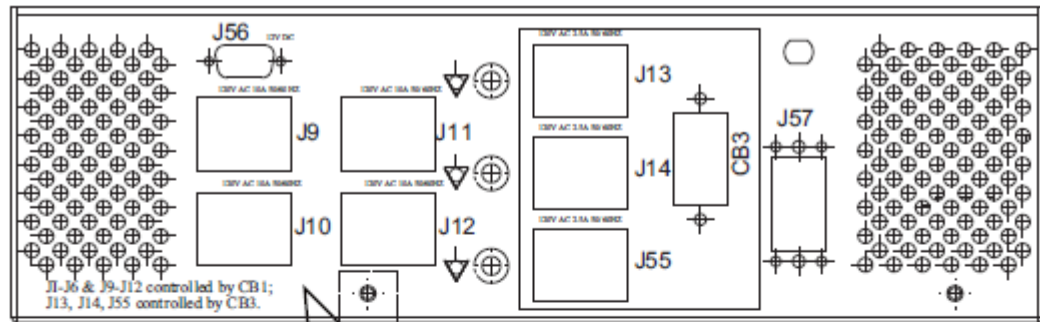Number	Description
J9	Display Monitor Power Connection
J10	Scan Monitor Power Connection
J11	Peripheral Media Tower Power Connection
J12	In-Room Monitor Connection
J13	Injector Power Connection
J14	RPM Power Connection
J56	GSCB Power Connection

- 1.) Connect the Keyboard and Mouse cables to NIO16 Console Z820 Host Computer.
- 2.) Connect GSCB cable connectors to NIO16 Console AC Box and Z820 Host Computer according to [Figure 2-9](#) and [Table 2-2](#).

Figure 2-9 GSCB and Cable

Table 2-2 GSCB Cable Connectors

PN	Description	From	To
5404262	GSCB Cable	GSCB	DB9/M Black - AC Box J56 Voice Blue - Host Computer Audio In Voice Green - Host Computer Audio Out DB9/F Gray - Host Computer RS232 DB9/F Black - Host Computer DIP


- 3.) Connect Media Tower cable connectors to NIO16 Console AC Box and Z820 Host Computer.
- 4.) Connect the LCD Monitors for Z820 Host Computer.
 - a.) Remove the old Video cable from the LCD Monitors.
 - b.) Connect the new Video cable (5408703, shipped with upgrade kit) between Z820 Host Computer and LCD Monitor.

Figure 2-10 Video Cable and Power Cable for LCD Monitor


- c.) Connect the LCD Monitor power cables to NIO16 Console AC Box.
 - Scan Monitor
Video Cable: from Console Host DP1 to Monitor DVI
Power Cable: from Console AC Box J10 to Monitor
 - Image Monitor
Video Cable: from Console Host DP0 to Monitor DVI
Power Cable: from Console AC Box J9 to Monitor
- d.) Route LCD video cable and power cable.

Figure 2-11 Cable Routing and Keeper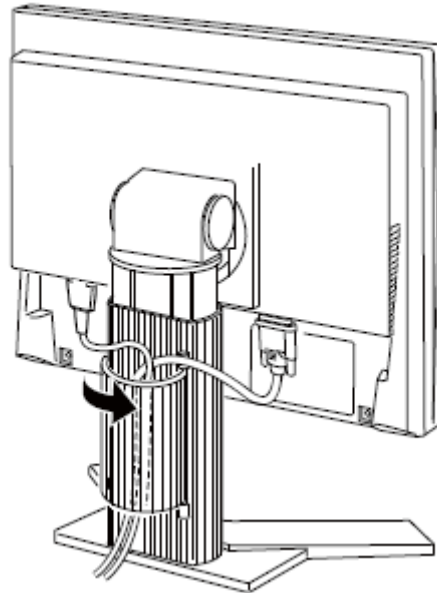

- 5.) Plug cables into Switch Hub on console if request.
- 6.) Connect other cables to Z820 Host Computer according to [Figure 2-7](#).

Figure 2-12 Switch Hub Connections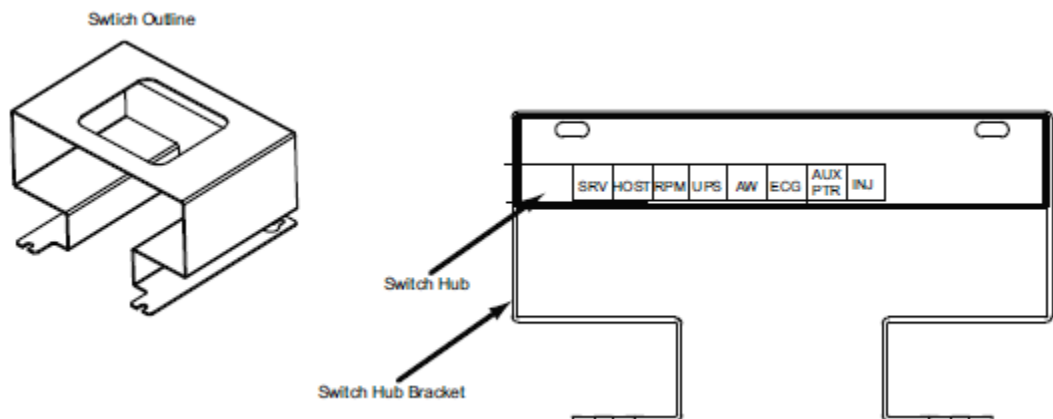

- 7.) Connect the NIO16 Console power cable and Ground cable to the Console Power Panel.

2.2.5 Console Cover Installation

Refer to Console Cover Installation procedure in 5350500-2EN rev15 or later service manual.

Note: If customer purchases ASiR option, the Console Covers are black or else they are white.

Section 3.0 System Configuration

3.1 Restore System State

- 1.) Insert a previously saved system state DVD-RAM media into the peripheral tower drive or USB media in any of the console's USB ports.
- 2.) Wait until the media is ready.
- 3.) Select [Service] -> [Utilities].
- 4.) Select [System State] for DVD-RAM media or [System State - USB] for USB media. The System State Save/Restore screen appears.
- 5.) Select [All].
- 6.) Select [Restore]. The Restore System State box appears.
- 7.) Select [Yes]. A message at the end of the System State Log window should display: Restore System State Completed Successfully.

Note: If the site installed 6 options (refer to [Section 1.1.5](#)). Error message will be pop-up during this time, please click [ok] to continue. Option Name is different between CTT system and SUSE system.

- 8.) When completed, select [Cancel].


Note: If Does Check option is installed, Dose Audit tool starts up when Reconfig Info or Protocol is selected. That shows configuration and Protocol summary. Press [Quit] to continue.

- 9.) Select [Yes] when the Scan Hardware Reset pop-up appears.
- 10.) When completed, select [Dismiss].
- 11.) Remove System State media and place in a safe location.
- 12.) Verify Restore System State.


3.2 Configure System Information on Z820 (SUSE)

- 1.) Configure all remote network information.
 - a.) Select [ImageWorks] from the desktop.


b.) Select in sequence: Tools -> Network Configuration.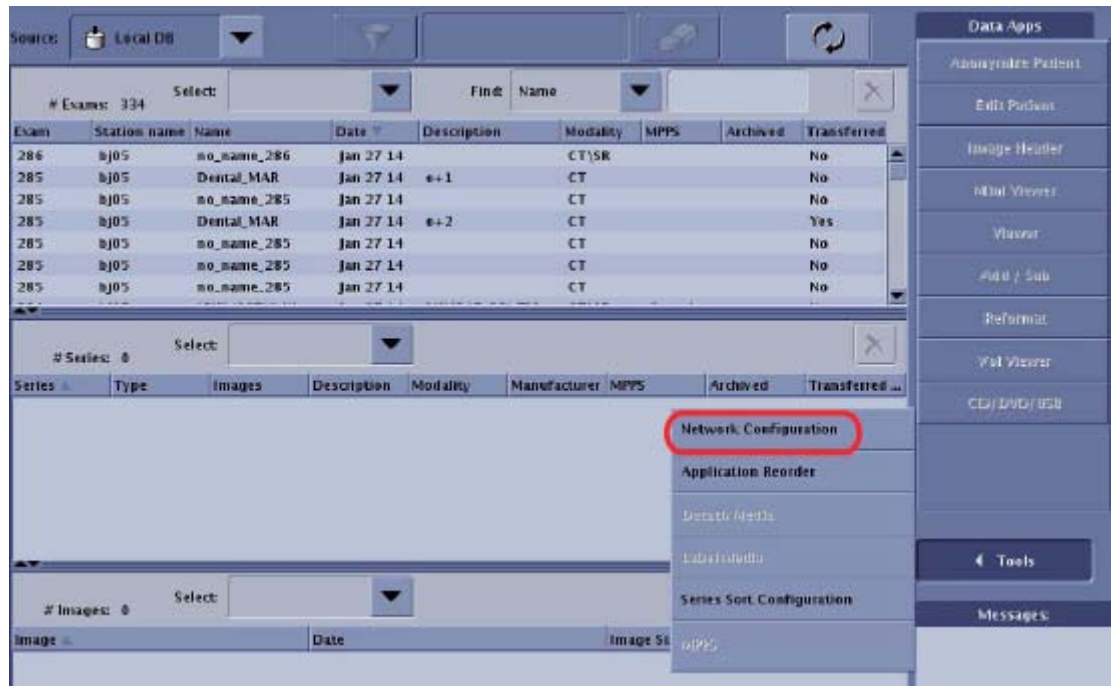

c.) Select [Add].

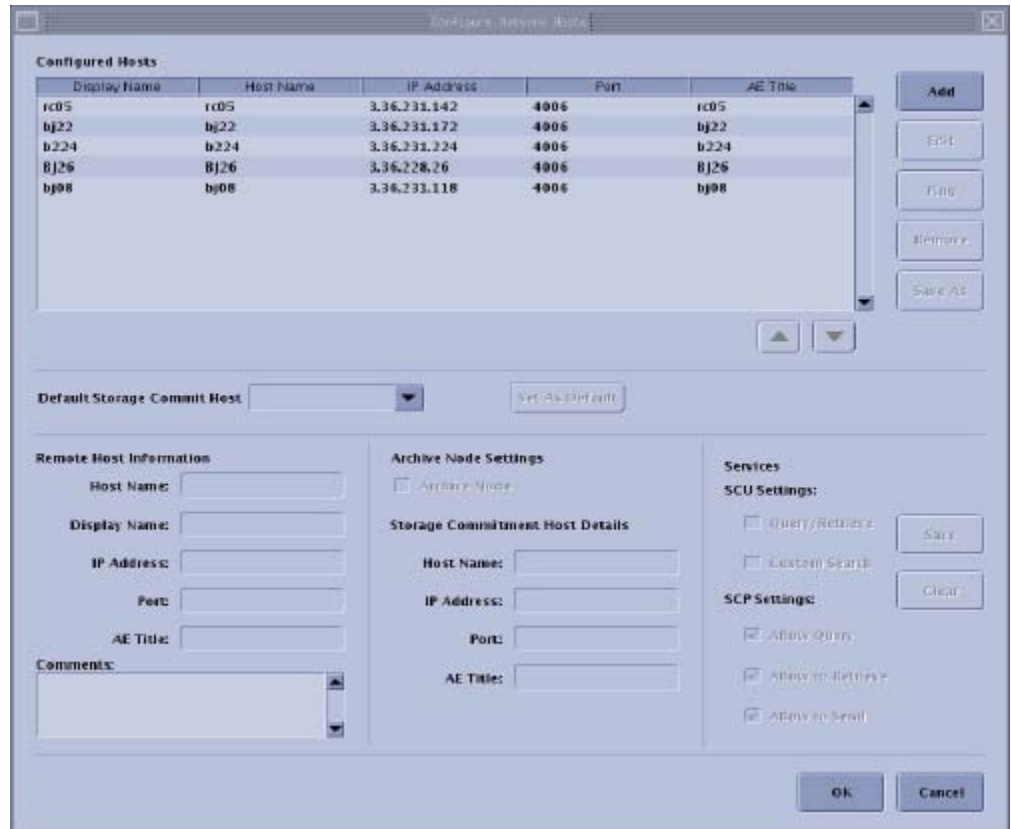d.) Input the Host Name then press [Enter].

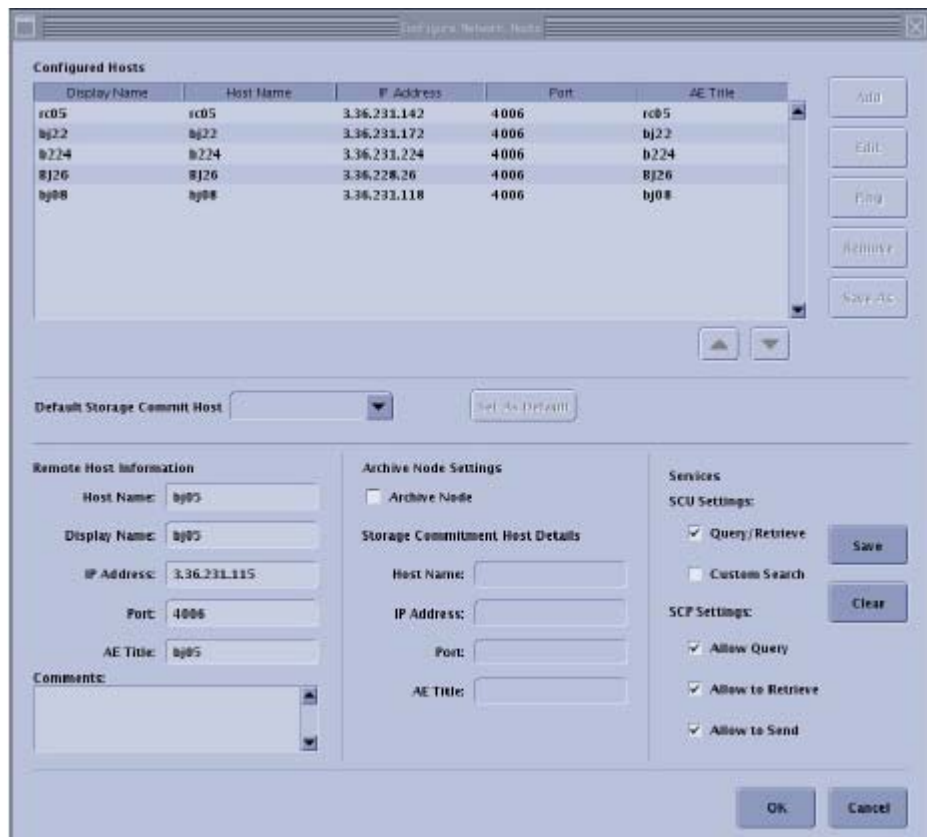- 2.) Configure PNF information, please refer to [Section 1.1.2](#).
- 3.) Configure AW IP for direct information, please refer to [Section 1.1.3](#).

Section 4.0 Flash Download

Note: The Flash Download takes anywhere from 5-30 minutes, depending on which subsystems require updates.


- 1.) Perform the Flash Download Utility found on the Common Service Desktop - Utilities Tab, select [Flash Download].

Figure 13 Common Service Desktop - Utilities Tab, Flash Download

- 2.) When the Flash Download Window opens, Select [Update].

Figure 14 Flash Download Window5

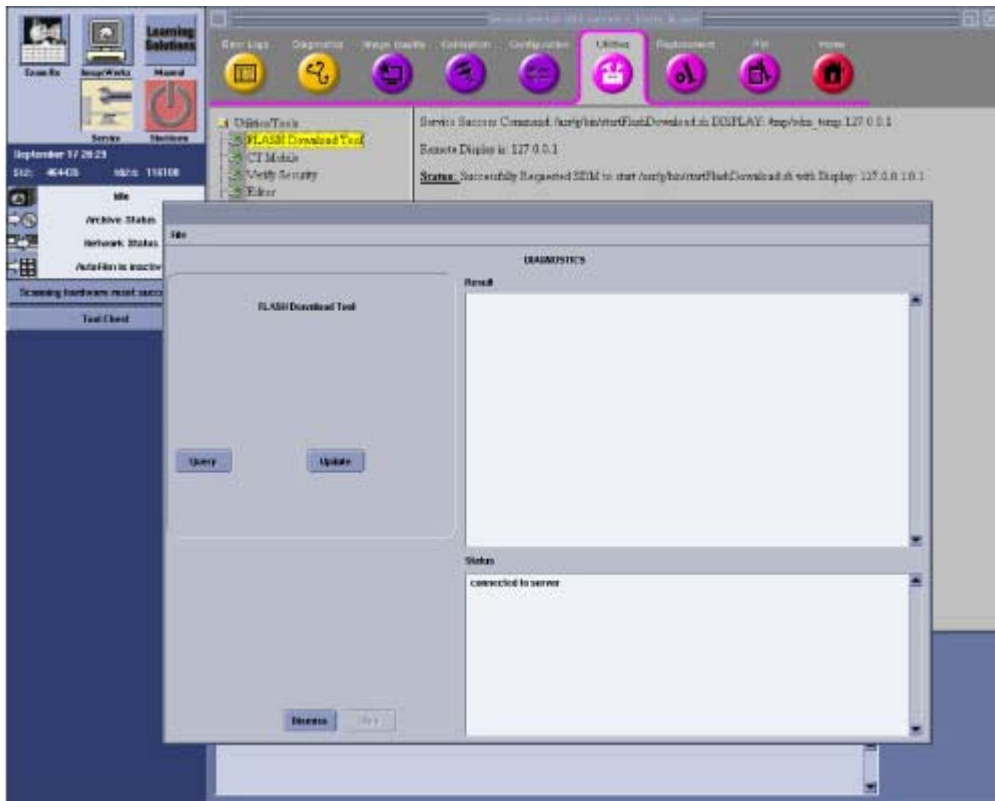- 3.) Once the Gantry Hardware Flash Downloads successfully, select [Dismiss].
- 4.) Close the Common Service Desktop.

Section 5.0 ASiR Option Installation

Note: If customer does not purchase ASiR option, please ignore this section.

5.1 Check GPU Card

Please refer to ASiR Upgrade Installation Manual (5344359-1EN_r5 or later) to check GPU card.

5.2 Option Installation

- 1.) Un-install Neuro 3D Filter option if needed. (This option conflicts with ASiR Application.)
- 2.) Install ASiR option software (shipped with B77092CB) using Option Install menu.
- 3.) Confirm that system started up normally.
- 4.) Install the Recon Enhancement Option software (shipped with B77092CB) using Option Install menu.

Section 6.0 Finalization

- 1.) Perform *Quality Assurance Test* of Functional Checks on Service Methods (5457941-2EN), make sure all of them pass.
- 2.) Update GIB with the Product Locator Card information. Go to the following address to access the site: http://gib.gehealthcare.com/gib/gib_entry.jsp
- 3.) Insite Check out, refer to 5457941-2EN Service Methods CD.
- 4.) Save System State
 - a.) Insert the Save System State DVD into the Peripheral Tower DVD-RW drive.
 - b.) Select [SERVICE DESKTOP].
 - c.) If reloading software, select [UTILITIES].
 - d.) Select [SYSTEM STATE].
 - e.) Select [ALL] to select all the calcs, characterization, etc.
 - f.) Select [SAVE].
 - g.) If applicable, select [OK] when the following message appears: System State Media Status: Please insert a DVD into the drive and press Save again
 - h.) When completed select [DISMISS].
 - i.) Close the Service Desktop window in the upper left corner of the screen.

This page intentionally left blank.

www.gehealthcare.com