

Rail Heater check (Detector level check list) -3

Top heater check – If one of tests fail, top heater has been damaged.

- Disconnect bottom heater, connect top heater
- Disconnect J1, J2 and J3 cable comes from CFHC

Routing board	Location	Measuring point A	Measuring point B	Normal Resistance
Low channel side	Top Heater center zone	J1-2	Detector frame	Open
	Top Heater Left	J1-1	J7-3	26~31Ω
High channel side	Top Heater center zone	J1-4	Detector frame	Open
	Top Heater Right	J1-3	J6-3	26~31Ω
Between Low-High Ch.	Top Heater center zone	High ch J1-J4	Low ch J1-J2	20~25Ω