GE Healthcare

Table Protection Pad Installation Manual

OPERATING DOCUMENTATION

5786404-1EN

Rev 1.0

© 2017 General Electric Company.

All rights reserved.

LEGAL NOTES

TRADEMARKS

Adobe, the Adobe logo, Acrobat, the Acrobat logo, Exchange, and PostScript are trademarks of Adobe Systems Incorporated or its subsidiaries and may be registered in certain jurisdictions.

Microsoft is a registered trademark and Windows is a trademark of Microsoft Corporation.

All other products and their name brands are trademarks of their respective holders.

COPYRIGHTS

All Material Copyrighted (c) 2017 by the GE Company, All rights reserved.

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DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent, have notation "damage in shipment" written on all copies of the freight or express bill before delivery is accepted or "signed for" by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage MUST be reported to the carrier immediately upon discovery, or in any event, within 14 days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this 14 day period.

To file a report:

- Call 1-800-548-3366 and use option 8.
- Fill out a report on <http://us44hdd21/sctq/InstallFulfill/InstalFulfillment.htm>
- Contact your local service coordinator for more information on this process.

Rev. Jan. 5, 2005

CERTIFIED ELECTRICAL CONTRACTOR STATEMENT

All electrical Installations that are preliminary to positioning of the equipment at the site prepared for the equipment shall be performed by licensed electrical contractors. In addition, electrical feeds into the Power Distribution Unit shall be performed by licensed electrical contractors. Other connections between pieces of electrical equipment, calibrations and testing shall be performed by qualified GE Medical personnel. The products involved (and the accompanying electrical installations) are highly sophisticated, and special engineering competence is required. In performing all electrical work on these products, GE will use its own specially trained field engineers. All of GE's electrical work on these products will comply with the requirements of the applicable electrical codes.

The purchaser of GE equipment shall only utilize qualified personnel (i.e., GE's field engineers, personnel of third-party service companies with equivalent training, or licensed electricians) to perform electrical servicing on the equipment.

IMPORTANT...X-RAY PROTECTION

X-ray equipment if not properly used may cause injury. Accordingly, the instructions herein contained should be thoroughly read and understood by everyone who will use the equipment before you attempt to place this equipment in operation. The General Electric Company, Medical Systems Group, will be glad to assist and cooperate in placing this equipment in use.

Although this apparatus incorporates a high degree of protection against x-radiation other than the useful beam, no practical design of equipment can provide complete protection. Nor can any practical design compel the operator to take adequate precautions to prevent the possibility of any persons carelessly exposing themselves or others to radiation.

It is important that anyone having anything to do with x-radiation be properly trained and fully acquainted with the recommendations of the National Council on Radiation Protection and Measurements as published in NCRP Reports available from NCRP Publications, 7910 Woodmont Avenue, Room 1016, Bethesda, Maryland 20814, and of the International Commission on Radiation Protection, and take adequate steps to protect against injury.

The equipment is sold with the understanding that the General Electric Company, Medical Systems Group, its agents, and representatives have no responsibility for injury or damage which may result from improper use of the equipment.

Various protective materials and devices are available. It is urged that such materials or devices be used.

LITHIUM BATTERY CAUTIONARY STATEMENTS

CAUTION

Risk of Explosion.

Danger of explosion if battery is incorrectly replaced.

Replace only with the same or equivalent type recommended by the manufacturer. Discard used batteries according to the manufacturer's instructions.

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OMISSIONS & ERRORS

Customers, please contact your GE Sales or Service representatives.

GE personnel, please use the Healthcare PQR Process to report all omissions, errors, and defects in this publication.

Revision History

Revision	Date	Reason for change
1.0	August 24, 2017	Initial Release

List of Effected Pages

PAGES	REVISION	PAGES	REVISION	PAGES	REVISION
1 through 32	1.0				

Table of Contents

Preface

Publication Conventions 19

Section 1.0

Safety & Hazard Information 19

1.1 Text and Character Representation..... 19
1.2 Graphical Representation 20

Section 2.0

Publication Conventions 21

2.1 General Paragraph and Character Styles..... 21
2.2 Page Layout..... 21
2.3 Computer Screen Output/Input Text Character Styles 22
2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)..... 22

Chapter 1

Introduction..... 23

Section 1.0

Overview 23

Section 2.0

Personnel Requirements 23

Section 3.0

Material List 23

Section 4.0

Preliminary Requirements..... 24

4.1 Tools and Test Equipment..... 24
4.2 Consumables..... 24
4.3 Safety..... 24
4.4 Required Conditions 24

Chapter 2

Installation Procedure..... 25

Section 1.0

Installation Procedure..... 25

Section 2.0

Finalization..... 29

Preface

Publication Conventions

Purpose: Standardize conventions for representing information in a uniform way of communicating information to a reader in a consistent manner. Conventions are used so that the reader can easily recognize the actions or decisions that must be made. There are a number of character and paragraph styles used in this publication to accomplish this task. Please become familiar with them before proceeding forward.

Section 1.0

Safety & Hazard Information

1.1 Text and Character Representation

Within this publication, different paragraph and character styles have been used to indicate potential hazards. Paragraph prefixes, such as hazard, caution, danger and warning, are used to identify important safety information. Text (Hazard) styles are applied to the paragraph contents that is applicable to each specific safety statement. Words describe the type of potential hazard that may be encountered and are placed immediately before the paragraph it modifies. Safety information will normally include:

- Type of potential Hazard
- Nature of potential injury
- Causative condition
- How to avoid or correct the causative condition

EXAMPLES OF HAZARD STATEMENTS USED

It's important that you read and understand hazard statements, and not just ignore them.

**DANGER
SEVERE
PERSONAL
INJURY**

DANGER IS USED WHEN A HAZARD EXISTS WHICH WILL CAUSE SEVERE PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:

- **ELECTROCUTION**
- **CRUSHING**
- **RADIATION**

**WARNING
SERIOUS
PERSONAL
INJURY**

WARNING IS USED WHEN A HAZARD EXISTS WHICH COULD OR CAN CAUSE SERIOUS PERSONAL INJURY OR DEATH IF INSTRUCTIONS ARE IGNORED. THEY CAN INCLUDE:

- **POTENTIAL FOR SHOCK**
- **EXPOSED WIRES**
- **FAILURE TO TAG AND LOCKOUT SYSTEM POWER COULD ALLOW FOR UN-COMMANDED MOTION.**

 **CAUTION**
Minor Personal Injury
Caution is used when a hazard exists which can or could cause minor injury to self or others if instructions are ignored. They include for example:

- Loss of critical patient data
- Crush or pinch points
- Sharp objects

 **NOTICE**
Property Damage Only
Notice is used when a hazard is present that can cause property damage but has absolutely no personal injury risk. They can include:

- Disk drive will be destroyed
- Internal mechanical damage, such as to the x-ray tube
- Coasting the rotor through resonance.

1.2 Graphical Representation

Important information will always be preceded by the exclamation point  contained within a triangle, as seen throughout the Preface chapter. In addition to text, several different graphical icons (symbols) may be used to make you aware of specific types of hazards that could possibly cause harm.

<p>ELECTRICAL</p>  <p>LASER</p>  <p>LASER LIGHT</p>	<p>HAZARDS MECHANICAL</p>  <p>HEAT</p> 	<p>RADIATION</p>  <p>PINCH</p> 
<p>AVOID STATIC ELECTRICITY</p> 	<p>PROCEDURAL TAG AND LOCK OUT</p> 	<p>WEAR EYE PROTECTION</p>  <p>EYE PROTECTION</p>

Section 2.0 Publication Conventions

2.1 General Paragraph and Character Styles

Prefixes are used to highlight important non-safety related information. Paragraph prefixes (such as purpose, example, comment or note) are used to identify important but non-safety related information. Text styles are also applied to text within each paragraph modified by the specific prefix.

EXAMPLES OF PREFIXES USED FOR GENERAL INFORMATION

Purpose: Introduces and provides meaning as to the information contained within the chapter, section or subsection, Such as used at the beginning this chapter for example.

Note: Conveys information that should be considered important to the reader.

Example: Used to make the reader aware that the paragraph(s) that follow are examples of information possibly stated previously.

Comment: Represents "additional" information that may or may not be relevant to your situation.

2.2 Page Layout

The diagram illustrates the layout of two pages from a manual. The left page is titled 'Publication Part Number & Revision Number' and the right page is titled 'Publication Title'. Both pages show a header with 'GE Medical Systems Intercom 2625F98-901, Revision 1.0' and 'GE Medical Systems Service Manual'. The left page contains sections 3.2.2 and 3.2.3, with a 'DANGER' symbol and a 'NOTE' symbol. The right page contains section 3.2.4, which includes a diagram of the 'ZEUS REMOTE INTERCOM X 46-288769-01-C' and a table of test points. Callouts point to the part number and revision number in the header of both pages, and the title in the header of the right page. The footer of the left page shows 'Page 882' and 'Section 3.0 - Intercom'. The footer of the right page shows 'Page 883' and 'Chapter 11 - Safety'.

The current section and its title are always shown in the footer of the left (even) page.

An exclamation point in a triangle is used to indicate important information to the user.

Paragraphs preceded by **Alphanumeric** characters (e.g. numbers) contain information that must be followed in a **specific order**.

The current chapter and its title are always shown in the footer of the right (odd) page.

Paragraphs preceded by a **symbol** (e.g. bullets) contain information that has **no specific order**.

Headers and footers in this publication are designed to allow you to quickly identify your location. The document's part number and revision number appears in every header on every page. Odd

numbered page footers indicate the current chapter, its title and current page number. Even page footers show the current section and its title, as well current page number.

2.3 Computer Screen Output/Input Text Character Styles

Within this publication, mono-spaced character styles (fonts) are used to indicate computer text that's either screen input and output. Mono spaced fonts, such courier, are used to indicated text direction. When you type at your keyboard, your are generating computer input. Occasionally you will see the math operators "greater than" and "less than" symbols used to indicate the start and finish of variable output. When reading text generated by the computer, you are reading it as computer generated output. In addition to direction, characters are italicized (e.g. *italics*) to indicate information specific to your system or site.

Example:
Fixed Output

This paragraph's fonts represents computer generated screen "fixed" output. It's output is fixed from the sense that it does not vary from application to application. It's the most commonly used style used to indicate filenames, paths and text that does not change form system to system. The character style used is a fixed width such as courier.

Example:
Variable Output

This paragraph's font represents computer screen output that is "variable". Its used to represent output that varies from application to application or system to system. Variable output is sometimes found placed between greater than and lesser than operators for clarification. For example: <variable_ouput> or <3.45.120.3>. In both cases, the < and > operators are not part of the actual input.

Example:
Fixed Input

This paragraph's font represents fixed input. It's computer input that is typed-in via the keyboard. Typed input that does not vary from application to application or system to system. Fixed text the user is required to supply as input. For example: cd /usr/3p

Example:
Variable Input

This paragraph's font represents computer input that can vary from application to application or system to system. With variable text, the user is required to supply system dependent input or information. Variable input sometimes is placed between greater-than and less-than operators. For example: <variable_input>. In these cases, the (<>) operators would be dropped prior to input. For example: ypcat hosts | grep <3.45.120.3> would be typed into the computer as ypcat hosts | grep 3.45.120.3 without the greater-than and less-than operators.

2.4 Buttons, Switches and Keyboard Inputs (Hard & Soft Keys)

Different character styles are used to indicate actions requiring the reader to press either a hard or soft button, switch or key. Physical hardware, such as buttons and switches, are called hard keys because they are hard wired or mechanical in nature. A keyboard or on/off switch would be a hard key. Software or computer generated buttons are called soft keys because they are software generated. Software driven menu buttons are an example of such keys. Soft and hard keys are represented differently in this publication.

Example:
Hard Keys

A power switch **ON/OFF** or a keyboard key like **ENTER** is indicated by applying a character style that uses both over and under-lined bold text that is bold. This is a hard key.

Example:
Soft Keys

Whereas the computer MENU button that you would click with your mouse or touch with your hand uses over and under-lined regular text. This is a soft key.

END OF SECTION

Chapter 1

Introduction

Section 1.0 Overview

This procedure describes the installation of the Table Protection Pad for VT1700 / 2000 series Table.

- After the Table Protection Pad has been installed , It is necessary to confirm that the function of Side Touch Sensor Left and Right is normal, if it is equipped in the table.

Section 2.0 Personnel Requirements

Required Persons	Preliminary Reqs	Procedure	Finalization
1	Timing Not Applicable	1.5 hours	15 mins

Section 3.0 Material List

Part number	Quantity	Description
5732313	1	Cushion pad for GT top side cover kit
5786404-1EN	1	VT Table Protection Pad Installation Manual

Table 1-1 : 5782417(Cushion pad for GT top side cover kit)

Section 4.0

Preliminary Requirements

4.1 Tools and Test Equipment

Same as ones required for System Installation. Refer to Installation manual 5786404-1EN

4.2 Consumables

Isopropyl Alcohol (As Required)

4.3 Safety

No general safety information.

4.4 Required Conditions

This kit must be installed by GE field engineer.

Chapter 2

Installation Procedure

Section 1.0 Installation Procedure

- 1.) Raise the Table to Maximum height.

Note: **If the system is installing Liquid Bearing Tube, check the x-ray tube temperature status before removing the gantry and table power**

- 2.) Check the *Xray Tube type* on the system.

- a.) Open unix shell and type `swhwinfo`
If **PN** or **NV** was included in the output strings

<For Example>

15HW25.2.H40_PN_SS64_G_GTL

13HW31.8_SP2-0-1.V40_NV_HS64_G_GTL

- b.) Check the status color is Not Green of Tube Care indication, before performing a system shutdown.


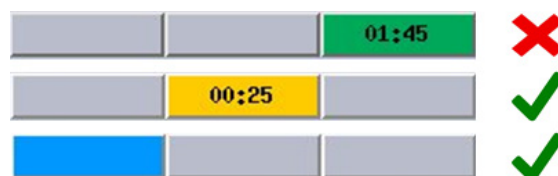
Figure 2-1 : Tube Care indication

NOTICE If the present status color is “Green”, it’s necessary to wait until it becomes “Yellow” or “Blue”

- 3.) Remove power from Table by turning off “120VAC”, “Axial Drive” and “HVDC” switch on Service Switch Panel

- 4.) Determine the mounting position of Protection pad.
It can be adjust by user's request if it is within the "Mounting possible range"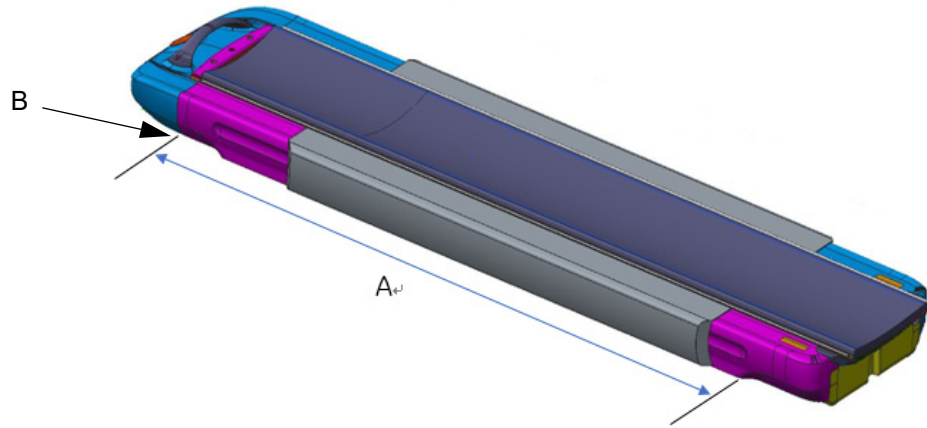


A: Mounting possible range
B: Do not overlap "Rear Top Cover"

Figure 2-2 : Mounting possible range : A

- 5.) Clean and dirt off the attaching surface of the Table Side Covers using alcohol.
6.) Remove paper Liner ① from VELCRO Tape

Note: Do not be remove yet the paper liner ②

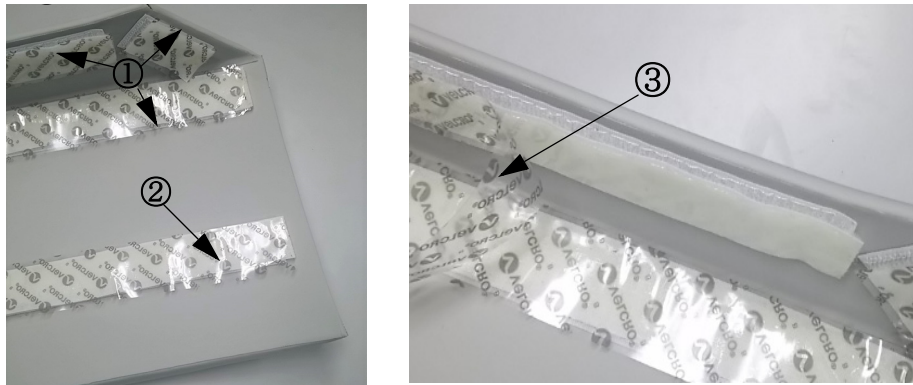- ① VELCRO Tape
② VELCRO Tape
③ Paper liner removal

Figure 2-3 : Remove paper liner

Note: **The adhesive on surface of this tape is very strong, so it is difficult to adjust the position after attaching Protection Pad once**

- 7.) Paste the adhesive surface ① to the top of the side cover after attachment position was determined.

Figure 2-4 : Paste the adhesive surface ①

- 8.) Remove paper liner ② from VELCRO Tape and paste the adhesive surface ② to the side of Table Side Cover
- 9.) Push the Protection Pad against the table side cover to secure attachment.

Figure 2-5 : Paste adhesive surface ②

Section 2.0 Finalization

- 1.) Power up the Table from the Service Switch Panel

Turn on all 3 switches (Axial Drive, HVDC, 120VAC)

- 2.) Verify that the Left and Right Side Touch Sensor function is operation normally.

Note: **it is necessary to confirm that normal operating properly, if Side Touch Sensor Left and Right is equipped on the table. (VT2000 only)**


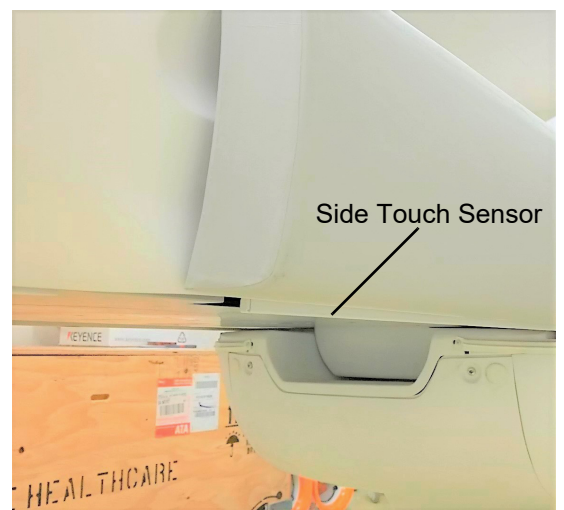
Figure 2-6 : Side Touch sensor

This Page Intentionally Blank.

© 2017, General Electric Company.

www.gehealthcare.com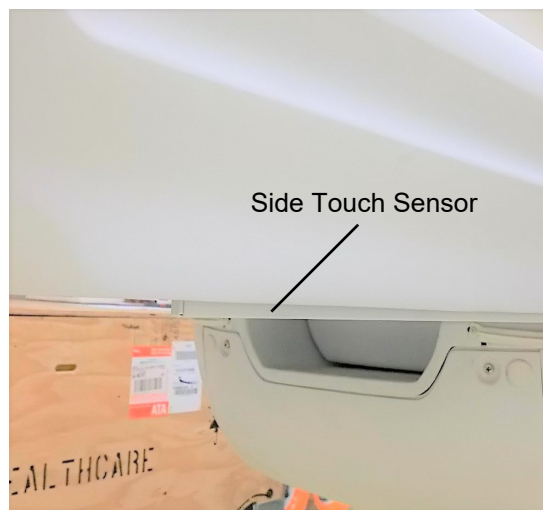

Imagination at work