MR Service Manual

Pre-11.x Service Methods Addendum

OPERATING DOCUMENTATION

5407677
Revision 2.0

WARNING

STRONG MAGNETIC FIELD

NO PACEMAKERS*
NO NEUROSTIMULATORS*
NO CONDUCTIVE/METALLIC IMPLANTS*

**Persons with pacemakers,
neurostimulators or metallic
implants must not enter this area.**

Serious injury may result.

* In general, patients with conductive (e.g. metallic) implants are contraindicated for MR scans. For patients with implants that are labeled as 'MR Safe' or 'MR Conditional', consult the implant device manufacturer's documentation.

* **WARNING:** Only use quadrature transmit for 'MR Conditional' devices.

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**WARNING
(EN)**

This service manual is available in English only.

- If a customer's service provider requires a language other than English, it is the customer's responsibility to provide translation services.
- Do not attempt to service the equipment unless this service manual has been consulted and is understood.
- Failure to heed this warning may result in injury to the service provider, operator or patient from electric shock, mechanical or other hazards.

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This page left intentionally blank.

Revision History

Date	Description	Revision
Dec 14, 2018	<p>To resolve NC # 17087417, previous Revision 2 and 3 are being combined into revision 2 , NC # 17087417 documents a revision mismatch, Revision 1 as shown in My Work Shop for PN 5407677 versus revision 3 as shown in document central by DOC1709911.</p> <p>Combining the revisions, creates a specification/file which shows the part revision of PN 5407677 as revision 2.</p> <p>This document/file is released as DOC1709911 revision 2</p> <p>Specification 5407677DDW revision 2 will point to DOC1709911, revision 2 , for PN 5407677, per ECR/ECO 2259457, creating and releasing revision 2 for PN 5407677.</p> <p>Resolving the revision mismatch.</p> <p>The following defines the changes and the associated verification for the previous revision 2 and revision 3:</p> <p>Updated for 1R15. Changed scope to all pre-11.x systems.</p> <ol style="list-style-type: none"> 1. Added reference to MR Service Safety manual. 2. Added check for MRU 3. Added check for latest software and patch installation. <p>Document review and approval per DOC1709911</p> <p>AND</p> <p>Updated to match Arbortext/eCLS (Version Set ID 273218) in the Service Methods manuals, and changed first bullet in Section 3.3.1 to P/N 5116158 per SPR HCSDM00079622.</p>	2
Oct. 15, 2010	Initial Release	1

This page left intentionally blank.

Table of Contents

1 Pre-11.x System Updates.....	15
1.1 Safety Updates.....	15
1.2 MRU PM Checks.....	15
1.3 Software Revision and Service Pack Check and Recording	16

This page left intentionally blank.

1 Pre-11.x System Updates

This addendum applies to two types of Pre-11.x system magnets

- Superconductive Magnets: Lx and earlier systems, MR Max, Vectra, Sierra, and Contour 1-4 systems
- Permanent Magnets: Ovation HD, Ovation 5, Ovation 4 Ovation 1-3, Profile HD, Profile 5, Profile 1-4, Profile HSR

1.1 Safety Updates

WARNING

PERSONNEL INJURY, EQUIPMENT DAMAGE

THE SAFETY INFORMATION IN THE SERVICE METHODS COVERED BY THIS ADDENDUM HAS BEEN SUPERSEDED BY THE MR MAGNET SAFETY MANUAL, PN 5452735.

BEFORE PERFORMING ANY SERVICE PROCEDURE, READ THE ENTIRE PROCEDURE TO UNDERSTAND ANY SAFETY REQUIREMENTS. REFER TO THE MR SERVICE SAFETY MANUAL, PN 5452735, AVAILABLE FROM THE ONLINE DOCUMENTATION LIBRARY, FOR THE LATEST SAFETY INFORMATION.

This update applies to all systems.

The MR Service Safety manual supersedes any safety information in the service methods manuals listed in this addendum. Before entering the Magnet room or performing service on any of the systems listed in this addendum:

1. Fully read the MR Service Safety manual to understand the latest MR safety requirements.
2. Read the entire service procedure.
3. If, at any time, you believe any of the safety procedures or requirements conflict or you are not sure how to proceed safely, contact your supervisor, GE Zone Support engineer, or GE Service for clarification.

1.2 MRU PM Checks

This update applies to superconductive magnets only.

1. When performing MRU PM checks use the table below to identify the MRU and perform the check indicated.
2. If a check is not listed in any PM schedule, perform the check while performing schedule B actions (for example, if Remote MRU Testing is not listed on any schedule, perform this test the next time schedule B activities are done).

The model number is listed on the side of the MRU, ERU, or EDU. Refer to the appropriate vendor manual available from the online documentation library for applicable testing procedures.

NOTE: For systems with six yearly PM schedules, the quarterly test is performed six times per year. Yearly test always performed once per year.

Table 1: MRU, ERU, or EDU PM Testing Matrix

Type	Model Number	Service Manual	Quarterly Test	Yearly Test	Remote MRU Testing (if installed)
Type I	5196918 or 5196918-2	5265188TPH	Perform quarterly service test	Perform yearly service test	Ensure the remote MRU cable is connected at both ends
Type II	46-294231G1	46-318393	Perform quarterly service test	Perform yearly service test	Perform quarterly and yearly test
Type III	46-252138G4, G3, G2	46-015120	Perform weekly functional check (Installation, Section 4, Emergency/Magnet Rundown Unit). Note: Quarterly test and Yearly test are the same.		Remote MRU not available
Type IV	2295525-8	2295400	Perform checks in Chapter 4, Section 4.2, Steps 1, 2, and 3. Note: Quarterly test and Yearly test are the same.		Ensure the remote MRU cable is connected

1.3 Software Revision and Service Pack Check and Recording

This update applies to all systems.

Perform the following at each PM and when installing host system software or service packs.

1. **IMPORTANT:** Make sure that the latest Service Pack is loaded. See the latest revision of *MR Service Pack Software Matrix*, DOC1667089, available from the online documentation library.
2. Record software revision and service pack (if necessary):
 - a. Obtain the GE Equipment Maintenance sticker (part number 5661793).
 - b. Use the procedure appropriate to your system, below, to check the software version.

Table 2: Software Version Checks

Lx and earlier systems, Ovation HD, Ovation 5, Ovation 4 Ovation 1-3, Profile HD, Profile 5	<ol style="list-style-type: none"> 1. Open a C-shell, type getver and click [enter]. 2. Check the software version.
MR Max, Vectra, Sierra, Contour 1-4, Profile 1-4 and Profile HSR systems	<ol style="list-style-type: none"> 1. Open command window. 2. Enter the following command: /use/g/bin/sw_config_status[Enter]. 3. Check the software version.
MR Max, Vectra, Sierra	<ol style="list-style-type: none"> 1. Turn the system power off. 2. Turn the system power on. 3. Check the software version displayed on CRT.

- c. Record the software revision and service pack on the sticker, with the date, Employee ID and initials and place the sticker on GOC (or host PC if the system does not have a GOC). The sticker should be placed on the either side of the GOC (or host PC) near the front.

© 2010-2015 General Electric Company. All Rights Reserved.
General Electric Company
3200 N. Grandview Boulevard
Waukesha, Wisconsin 53188
USA

www.gehealthcare.com