*GE Medical Systems*

---

# **Technical Publications**

**Direction 2284360  
Revision 1**

# **Signa<sup>®</sup> Release LX BodyPak Software Feature Installation**

Copyright<sup>©</sup> 2001 by General Electric Co.

**Operating Documentation**

## DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent, have notation "**damage in shipment**" written on **all** copies of the freight or express bill **before** delivery is accepted or "signed for" by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage **MUST** be reported to the carrier **immediately** upon discovery, or in any event, within **14** days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this **14** day period.

**Immediately** complete a "Damage Loss Claim Form", available via MS Exchange Mail, after the damage is found.

MS Exchange Path:

Outlook/Public Folder/All Public Folders/Medical Systems/!Global Initiatives/Information Management/Forms/Common Forms/DAMAGE LOSS CLAIM FORM.

Send the completed form to the email address listed in the form.

For more information about the Transportation Claim Procedure, access the GE Medical Systems Intranet and enter the following URL address (case sensitive):

<ftp://3.87.40.2/globepro/qualsys/Docs/190016MF.PDF>

Rev. 11/15/2000

## Direction 2128126 - Language Policy For Service Documentation, Rev 0

### WARNING

- THIS SERVICE MANUAL IS AVAILABLE IN ENGLISH ONLY.
- IF A CUSTOMER'S SERVICE PROVIDER REQUIRES A LANGUAGE OTHER THAN ENGLISH, IT IS THE CUSTOMER'S RESPONSIBILITY TO PROVIDE TRANSLATION SERVICES.
- DO NOT ATTEMPT TO SERVICE THE EQUIPMENT UNLESS THIS SERVICE MANUAL HAS BEEN CONSULTED AND IS UNDERSTOOD.
- FAILURE TO HEED THIS WARNING MAY RESULT IN INJURY TO THE SERVICE PROVIDER, OPERATOR OR PATIENT FROM ELECTRIC SHOCK, MECHANICAL OR OTHER HAZARDS.

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# REVISION HISTORY

<u>REV</u>	<u>DATE</u>	<u>PRIMARY REASON FOR CHANGE</u>
A.....	Sep 19, 2000	Initial version for Release.
B.....	Dec 14, 2000	Change procedure t follow patch precess
0 .....	Jan 11, 2001	Rev 0 Release
1 .....	Aug 23, 2001	Corrected unmount comand per SPR MRIge68531

<u>PAGE</u>	<u>REV</u>	<u>PAGE</u>	<u>REV</u>	<u>PAGE</u>	<u>REV</u>	<u>PAGE</u>	<u>REV</u>
Title Page .....	1						
8.....	1						

\* This revision/letter corresponds to the indicated document's revision control system.

## TABLE OF CONTENTS

1- INSTALLATION INSTRUCTIONS .....	7
2 - SECURITY KEY INSTALLATION.....	8

## 1- INSTALLATION INSTRUCTIONS

1. From a C Shell login as root by typing: **su root <Enter>**.
2. password: **operator**
3. Insert the BodyPak CD into the CD-ROM Drive.
4. Type **mediad <Enter>**
5. Type **swmgr <Enter>**
6. Wait for Software Manager to come up.
7. Type **/CDROM/dist** in the **[Software location Edit Window]**.
8. Push the **[Lookup]** button to the right of the **[Software location Edit Window]**.
9. Push the **[Customize Installation]** button at the top of the screen.
10. All available patches & software should be visible at this time in the selection window. All patches that apply to your system and are not loaded will be selected (check marked). BodyApps will not load without Patch 70 and 71.
11. The **[Start]** button will activate (The letters in "Start" will change colors or shades). If the **[Start]** button activates, push it and the patch load will begin. A progress bar will open and count up from 0 to 100%. The load takes less than 2 minutes. When the progress bar goes away, a confirmation window will be displayed. Select **[Exit]** to quit the software manager.
12. After the patch is selected, if there is a problem, the **[Conflicts]** button will activate, (The letters in "Conflicts" will change colors or shades). If this happens, this indicates that your system does not need this patch or that there are certain patches on the system that are not compatible with the new patches. Push the **[Conflicts]** button to view the conflicts. For BodyPak, Patch70 and Patch71 it is required that you uninstall the following
  - 53: "83 M5 patches"
  - 59: "Cardiac view copy patch"
  - 62: "efgre3d SmartPrep UCERD receiver selection patch"
  - 69: "efgre3d contrast issue"
13. After the conflicts are resolved, choose **[Close]** to proceed to the installation.
14. After the installation select **Exit, Installation Completed** checkbox. Then select **[OK]**.

15. Type the following in the C Shell to eject the CDROM

**umount /CDROM <Enter>** (unmounts the CD from the system)

**Eject <Enter>** (opens the CD Drawer)

16. The following files are loaded on your system

CERD: cerd.P3.i40 (/usr/g/bin)

PSD: efgre3d psd files

TOOLS: rft tool psd files

BODYPAK Sequences:

- fseopt
- fseopt.psd.o
- ssfse\_etl
- ssfse\_etl.psd.o
- fgre-dual
- fgre-dual.psd.o

## 2 - SECURITY KEY INSTALLATION

1. To obtain the license for BodyPak, you need the Host ID to generate the license key.

- to get the Host ID open a C Shell and type **sysinfo -v**

2. You will receive a license number that is unique to the site.

3. Open a C shell.

4. Type **cd /usr/g/caldir <Enter>**

5. Create a new file called "secure.bodypak" in this directory. (This can be done with the command **touch secure.bodypak <Enter>**)

6. Open the file using an editor (vi or jot) and enter the license number in the first line.

7. Save and exit from the file

The three type-in PSDs frfseopt, ssfse\_etl, and fgre-dual are now ready to use.

### NOTE:

The new Pulse Sequences will not be saved as part of the Save/Restore Process. After a Load From Cold you will have to re-install the CD as indicated in the procedures above. The license number that is in the /usr/g/caldir directory will be saved during the save info process. But it is still highly recommended that you write the license number down.