

TABLE OF CONTENTS

TABLE OF CONTENTS	1
1- PNEUMATIC PATIENT ALERT SYSTEM	2
REVISION HISTORY	3

1- PNEUMATIC PATIENT ALERT SYSTEM

The Pneumatic Patient Alert System is a stand-alone system that allows the patient to contact the operator even when the console volume is turned down. The patient squeezes a hand-held bulb that activates an audible and visual alarm near the operator. The operator must reset the control box in order to stop the alarms. Either a PULSE or STEADY audible tone can be selected at the control box.

REVISION HISTORY

REV	DATE	AUTHOR	PRIMARY REASONS FOR CHANGE
A	April 27, 1998	W. Pasciak	Initial release converted to Word
0	Oct. 15, 1999	M. Keber	Added correct proprietary heading to document; removed redundant description information.