

APPENDIX D - DISK DRIVE FIX-UP

TABLE OF CONTENTS

<u>SECTION</u>	<u>TITLE</u>	<u>PAGE</u>
D-1	Disk Drive Fix-Up Horizon 5.X.....	D-2

D-1 DISK DRIVE FIX-UP (HORIZON 5.X)

This section applies to Signa Horizon 1.5T and 1.0T systems only. If proper computer shutdown procedures were not followed or if during the course of execution, the computer inadvertently crashed, files may remain open. The operating system will not function unless the files are closed. In these instances, a program must be run to perform the file clean up operation on the disk.

This program is contained in the System Startup Software. Bring the system down and restart as described in Appendix B. If this does not clear the problem, perform a Load From Cold.