

## TABLE OF CONTENTS

o.3.1 Spike Noise (Head Axial, A/P).....	3
o.3.2 Spike Noise (Head Axial, R/L).....	5
o.3.3 Spike Noise (Head Sagittal).....	6
o.3.4 Spike Noise (Head Coronal).....	7
o.3.5 Spike Noise (Body Coronal).....	8
o.3.6 Spike Noise (Body Sagittal).....	9
o.3.7 Spike Noise (Body Axial, R/L).....	10
o.3.8 Spike Noise (Body Axial, A/P).....	11
o.5.1 Grafidy.....	12
o.12.1 Head Slice Thickness (2D, 3mm).....	13
o.12.2 Head Slice Thickness (2D, 5mm).....	15
o.12.3 Head Slice Thickness (2D, 10mm).....	16
o.12.4 Head Slice Thickness (2D, Grad Echo, 5mm).....	17
o.12.5 Head Slice Thickness (3D, Localizer).....	18
o.12.6 Head Slice Thickness (3D, Grad Echo, 1.5mm).....	19
o.13.1 SNR Check (Body, Axial).....	20
o.13.2 SNR Check (Body, Sagittal).....	22
o.13.3 SNR Check (Body, Coronal).....	23
o.13.4 SNR Check (Head, Axial).....	24
o.13.5 SNR Check (Head, Sagittal).....	26
o.13.6 SNR Check (Head, Coronal).....	27
o.13.7 Surface Coil SNR.....	28
o.14.1 Multicoil Spike Noise (CTLTOP Coils).....	30
o.14.2 Multicoil Spike Noise (CTLBOT Coils).....	32
o.15.1 Slice Offset (2D, 3mm, Table 50mm).....	33
o.15.2 Slice Offset (2D, 3mm, Table -50mm).....	35
o.15.3 Slice Offset (3D, Grad Echo Localizer).....	36
o.15.4 Slice Offset (3D, Grad Table 50mm).....	37
o.15.5 Slice Offset (3D, Grad Table -50mm).....	38
o.17.1 Body F/R Quadrature.....	39
o.17.2 Head F/R Quadrature.....	41
o.18.1 Dummy Load.....	43
o.19.1 LVShim (Localizer).....	45
o.19.2 LVShim Scan.....	47
o.20.1 First Image.....	48
o.21.1 T2 Uniformity (Body, Axial).....	50
o.21.2 T2 Uniformity (Body, Sagittal).....	52
o.21.3 T2 Uniformity (Body/Head, Coronal).....	53
o.22.1 Isocenter Cal (DQA-III).....	54
o.23.1 Head Power Monitor.....	56
o.23.2 Body Power Monitor.....	58
o.23.3 Amp Shutdown Verification.....	60
o.24.1 System Gain Cal (Body).....	62
o.24.2 System Gain Cal (Head).....	64
o.28.1 Full Field Distortion (Sagittal).....	66
o.28.2 Full Field Distortion (Coronal Localizer).....	68
o.28.3 Full Field Distortion (Coronal).....	70
o.29.1 Noise Floor (Head).....	71
o.29.2 Noise Floor (Body).....	73
o.30.1 Carrier Leakage.....	74
o.31.1 Respiratory Compensation.....	76
o.32.1 Gradcal-X (DQA-III).....	77
o.32.2 Gradcal-Y (DQA-III).....	79
o.32.3 Gradcal-Z (DQA-III).....	81

o.36.1 Correlated Noise (256 matrix).....83

o.36.2 Correlated Noise (512 matrix).....85

o.36.3 Multicoil Correlated Noise .....86

o.36.4 EPI Correlated Noise .....88

o.37.1 Body Slice Thickness (Sagittal, 3mm).....90

o.37.2 Body Slice Thickness (Sagittal, 5mm).....92

o.37.3 Body Slice Thickness (Sagittal, 10mm).....93

o.37.4 Body Slice Thickness (Localizer).....94

o.37.5 Body Slice Thickness (Coronal, 3mm) .....95

o.37.6 Body Slice Thickness (Coronal, 5mm) .....96

o.37.7 Body Slice Thickness (Coronal, 10 mm) .....97

o.38.1 Geometry Verification (Coronal) .....98

o.38.2 Geometry Verification (Axial) .....100

o.39.1 FID Tuning.....101

o.40.1 Gradient Waveforms .....103

o.41.1 Body APB Cal.....105

o.41.2 Head APB Cal.....107

o.42.1 Bandpass Asymmetry Correction Compensation (BACC).....109

o.43.1 Center Frequency .....110

o.44.1 Head B0 Dither.....112

o.45.1 Body B<sub>0</sub> Dither & EPI Group Delay.....114

o.56.1 SNR - Probe S .....116

o.57.1 SNR - Probe P .....118

### o.3.1 Spike Noise (Head Axial, A/P)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>spike noise test</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential]</b>
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[10]</b>
TR.	<b>[20]</b>
Flip Angle	<b>[10]</b>
Bandwidth	<b>[15.63]</b> (1.0T)
	<b>[15.00]</b> (1.5T)

SCANNING RANGE

FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
S/I Start	<b>[160]</b>
S/I End	<b>[S60]</b>
# of Slices	20 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:45)
# of Locs Before Pause	<b>0</b>
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

<b>[Research Operations]</b>	<b>[Setup Params]</b>
R1	<b>11</b>
R2	<b>15</b>
TG	<b>0</b>
Number of Frames	<b>2</b>
	<b>[Done]</b>

**o.3.2 Spike Noise (Head Axial, R/L)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note

<u>ACQUISITION TIMING</u>	
Freq Dir	<b>[&gt;] [R/L]</b>

(lowest window)                      **[Save Series]**

RX MANAGER                              **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

### o.3.3 Spike Noise (Head Sagittal)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note

#### IMAGING PARAMETERS

Plane **[>] [Sagittal]**

#### SCANNING RANGE

R/L Start **[R60]**

R/L End **[L60]**

P/A Center 0 (default)

I/S Center 0 (default)

#### ACQUISITION TIMING

Freq Dir **[>] [A/P]**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

### o.3.4 Spike Noise (Head Coronal)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note

#### IMAGING PARAMETERS

Plane **[>] [Coronal]**

#### SCANNING RANGE

A/P Start **[P60]**

A/P End **[A60]**

S/I Center 0 (default)

R/L Center 0 (default)

#### ACQUISITION TIMING

Freq Dir **[>] [R/L]**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

**o.3.5 Spike Noise (Body Coronal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note
<u>PATIENT POSITION</u>	
Coil	<b>[...] [Bodyd] [Accept]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

### o.3.6 Spike Noise (Body Sagittal)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
<u>SCANNING RANGE</u>	
R/L Start	<b>[R60]</b>
R/L End	<b>[L60]</b>
P/A Center	0 (default)
I/S Center	0 (default)
<u>ACQUISITION TIMING</u>	
Freq Dir	<b>[&gt;] [S/I]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

**o.3.7 Spike Noise (Body Axial, R/L)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
<u>SCANNING RANGE</u>	
S/I Start	<b>[I60]</b>
S/I End	<b>[S60]</b>
L/R Center	0 (default)
P/A Center	0 (default)
<u>ACQUISITION TIMING</u>	
Freq Dir	<b>[&gt;] [R/L]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

**o.3.8 Spike Noise (Body Axial, A/P)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note
<u>ACQUISITION TIMING</u>	
Freq Dir	<b>[&gt;] [A/P]</b>
( <u>lowest window</u> )	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series

**o.5.1 Grafidy**

<b><u>SCAN OPTION</u></b>	<b><u>INPUT</u></b>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>grafidy</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>grafidy</b>
Protocol	no entry
<u>(lowest window)</u>	<b>[Save Series]</b>
<b>[Research Operations]</b>	<b>[Setup Params]</b>
R1	<b>13</b>
R2.	<b>14</b>
TG	<b>0</b> (TG <u>must</u> be set to 0.)
Number of Frames.	<b>4</b>
	Window 1
	Frame 1 + Frame 0
	Window 2
	Frame 3 + Frame 0
	<b>[Done]</b>
<b>[Manual Prescan]</b>	
Type	<b>P</b>
Gain	<b>1</b>
Windows menu	<b>[Two Windows]</b>
	<b>[Done]</b>

**o.12.1 Head Slice Thickness (2D, 3mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	head slice thick
Weight (Lb))	111
	[Landmark]
Landmark	[>] [Nasion]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Head] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Axial]
Mode	[>] [2D]
Pulse Seq	[...] [Spin Echo] [Accept]
Imaging Options	none
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	[25]
TR	[300]
Bandwidth	[7.81] (1.0T only)
<u>SCANNING RANGE</u>	
FOV	[20]
Slice Thickness	[3]
Spacing	[77]
S/I Start	[180]
S/I End	[S80]
# of Slices	3 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[4]</b> (1.0T)
	<b>[2]</b> (1.5T)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.12.2 Head Slice Thickness (2D, 5mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button On new series to activate it. See Note 1

IMAGING PARAMETERS

Imaging Options            **[...] [VBw] [Accept]**

SCAN TIMING

TE                            **[Min Full]**

TR                            **[800]**

Bandwidth                **[15.63]**

SCANNING RANGE

Slice Thickness           **[5]**

Spacing                    **[75]**

ACQUISITION TIMING

NEX                         **[4]**

(lowest window)           **[Save Series]**

RX MANAGER                **[Prepare to Scan]**

**Note: 1.** Only values different from previous series shown. All other values should be the same as the previous series.

**o.12.3 Head Slice Thickness (2D, 10mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note 1
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[10]</b>
Spacing	<b>[70]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note: 1.** Only values different from previous series shown. All other values should be the same as the previous series.

**o.12.4 Head Slice Thickness (2D, Grad Echo, 5mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note 1
<u>IMAGING PARAMETERS</u>	
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
<u>SCAN TIMING</u>	
TE	<b>[9]</b>
TR	<b>[100]</b>
Flip Angle	<b>[30]</b>
Bandwidth	<b>15.63</b>
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[5]</b>
Spacing	<b>[77]</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	<b>1</b>
<u>ACQUISITION TIMING</u>	
Phase	<b>[128]</b>
NEX	<b>[1] (0:16)</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note: 1.** Only values different from previous series shown. All other values should be the same as the previous series.

### o.12.5 Head Slice Thickness (3D, Localizer)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note 1

#### IMAGING PARAMETERS

Plane	<b>[&gt;] [Sagittal]</b>
Imaging Options	none

#### SCAN TIMING

TR	<b>[50]</b>
----	-------------

#### SCANNING RANGE

FOV	<b>[24]</b>
Spacing	<b>[1.5]</b>

#### ACQUISITION TIMING

Freq Dir	<b>[&gt;] [S/I]</b>
----------	---------------------

<u>(lowest window)</u>	<b>[Save Series]</b>
------------------------	----------------------

<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>
-------------------	--------------------------

**Note:** Only values different from previous series are shown. All other values should be the same as the previous series.

**o.12.6 Head Slice Thickness (3D, Grad Echo, 1.5mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it

IMAGING PARAMETERS

Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [3D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>

SCAN TIMING

TR	<b>[34]</b>
Bandwidth	1.0T only <b>15.63</b>

SCANNING RANGE

Slice Thickness	<b>[1.5]</b>
# of Slices	<b>[28]</b>

ADDITIONAL PARAMETERS

**[Graphic Rx]**  
See procedure for instructions.

ACQUISITION TIMING

Freq Dir	<b>[&gt;] [A/P]</b>
----------	---------------------

(lowest window)                      **[Save Series]**

RX MANAGER                              **[Prepare to Scan]**

**Note:** Only values different from previous series are shown. All other values should be the same as the previous series.

**o.13.1 SNR Check (Body, Axial)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body snr</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[20]</b>
TR	<b>[2000]</b>
<u>SCANNING RANGE</u>	
FOV.	<b>[48]</b>
Slice Thickness.	<b>[3]</b>
Spacing	<b>[1.5]</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (8:52)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.13.2 SNR Check (Body, Sagittal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note.

IMAGING PARAMETERS

Plane **[>] [Sagittal]**

SCANNING RANGE

R/L Start **0**  
R/L End **0**  
P/A Center 0 (default)  
I/S Center 0 (default)

ACQUISITION TIMING

Freq Dir **[>] [S/I]**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.13.3 SNR Check (Body, Coronal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note.

IMAGING PARAMETERS

Plane **[>] [Coronal]**

SCANNING RANGE

A/P Start **[SEE PROCEDURE]**

A/P End **[SEE PROCEDURE]**

S/I Center 0 (default)

R/L Center 0 (default)

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.13.4 SNR Check (Head, Axial)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>head snr</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[20]</b>
TR	<b>[2000]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (8.52)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                    **[Save Series]**

**o.13.5 SNR Check (Head, Sagittal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note

IMAGING PARAMETERS

Plane **[>] [Sagittal]**

SCANNING RANGE

R/L Start **0**  
R/L End **0**  
P/A Center 0 (default)  
I/S Center 0 (default)

ACQUISITION TIMING

Freq Dir **[>] [S/I]**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.13.6 SNR Check (Head, Coronal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note

IMAGING PARAMETERS

Plane **[>] [Coronal]**

SCANNING RANGE

A/P Start **[SEE PROCEDURE]**

A/P End **[SEE PROCEDURE]**

S/I Center 0 (default)

R/L Center 0 (default)

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.13.7 Surface Coil SNR**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>surface coil snr</b>
Weight (Lb)	<b>300</b>
	<b>[Landmark]</b>
Landmark	<b>[Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Surface] Select appropriate surface coil [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[20]</b>
TR	<b>[2000]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>0</b>
I/S Start	<b>0</b>
I/S End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (8.52)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b> See Note 1.
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

Note 1: Depends on surface coil selected. Some that require R/L for the Freq Dir setting are: Extrem, QuadKnee Neurovas. Most require the A/P setting.

**o.14.1 Multicoil Spike Noise (CTLTOP Coils)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>mc noise test</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Phased Array] Select [CTLTOP] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential] [Variable BW]</b>
Psd Name	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[10]</b>
TR.	<b>[20]</b>
Flip Angle	<b>[10]</b>
Bandwidth	<b>15.63 (1.0T)</b> <b>15.00 (1.5T)</b>

SCANNING RANGE

FOV	<b>[48]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
R/L Start	<b>[L15]</b> (0 if Pelvic Coil is used)
R/L End	<b>[R15]</b> (0 if Pelvic Coil is used)
# of Slices	6 (default)
P/A Center	0 (default)
I/S Center	<b>[S120.0]</b>
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (0:32)
Phase FOV	1.00 (default)
# Locs efore pause	<b>0</b>
Freq Dir	<b>[&gt;] [S/I]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

<b>[Research Operations]</b>	<b>[Setup Params]</b>
R1	<b>11</b>
R2	<b>15</b>
TG	<b>0</b>
Number of Frames	<b>2</b>
	<b>[Done]</b>

**o.14.2 Multicoil Spike Noise (CTLBOT Coils)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button on new series to activate it. See Note
<u>PATIENT POSITION</u>	
Coil	<b>[...] [Phased Array] Select [CTLBOT] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
<u>SCANNING RANGE</u>	
I/S Center	<b>[I120.0]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<b>[Research Operations]</b>	<b>[Setup Params]</b>
R1	<b>11</b>
R2	<b>15</b>
TG	<b>0</b>
Number of Frames	<b>2</b>
	<b>[Done]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.15.1 Slice Offset (2D, 3mm, Table 50mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>head slice thick</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [SpinEcho] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[300]</b>
Bandwidth	<b>15.63</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[20]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[97]</b>
I/S Start	<b>I50</b>
I/S End	<b>S50</b>
# of Slices	2 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	<b>50</b>

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[2]</b> (2:35)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.15.2 Slice Offset (2D, 3mm, Table –50mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>SCANNING RANGE</u>	
Table Delta	<b>[-50]</b>
( <u>lowest window</u> )	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.15.3 Slice Offset (3D, Grad Echo Localizer)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
<u>SCAN TIMING</u>	
TE	<b>[9]</b>
TR	<b>[50]</b>
Flip Angle	<b>[30]</b>
Bandwidth	<b>[15.63]</b>
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
R/L Start	<b>0</b>
R/L End	<b>0</b>
# of Slices	<b>1</b>
P/A Center	0 (default)
I/S Center	0 (default)
Table Delta	0.00 (default)
<u>ACQUISITION TIMING</u>	
Phase	<b>[128]</b>
NEX	<b>[1] (0:10)</b>
Freq Dir	<b>[&gt;] [S/I]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.15.4 Slice Offset (3D, Grad Table 50mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note

IMAGING PARAMETERS

Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [3D]</b>

SCAN TIMING

TR	<b>[34]</b>
----	-------------

SCANNING RANGE

Slice Thickness	<b>[3]</b>
# of Slices	<b>[60]</b>
Table Delta	<b>[50]</b>

ADDITIONAL PARAMETERS

**[Graphic Rx]**  
See procedure for instructions.

ACQUISITION TIMING

Freq Dir	<b>[&gt;] [A/P]</b>
----------	---------------------

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.15.5 Slice Offset (3D, Grad Table –50mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	Place cursor in Series list area Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>SCANNING RANGE</u> Table Delta	<b>[50]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.17.1 Body F/R Quadrature**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body f/r quad</b>
Weight (Lb))	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	<b>[...] [EDR]</b>
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[84]</b> for 1.5T <b>[92]</b> for 1.0T
Band Width	<b>[15.63]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[2]</b> (0:43)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.17.2 Head F/R Quadrature**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>hd f/r quad</b>
Weight (Lb))	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	<b>[...] [EDR]</b>
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[84]</b> for 1.5T <b>[92]</b> for 1.0T
Bandwidth	<b>[15.63]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[2]</b> (0:43)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.18.1 Dummy Load**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>dummy load cal</b>
Weight (Lb))	<b>300</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>cal</b>
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[200]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>0</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq **[256]**  
 Phase **[128]**  
 NEX **[2] (0:52)**  
 Freq Dir **[>] [R/L]**  
 Auto Center Freq **[>] [Peak]**

(lowest window) **[Save Series]**

**[Research Operations], [Display CVs]**

Modify the following:

Calmode **2** (trapezoid pulse)  
 p2\_ramp **1** (1  $\mu$ sec ramptime)  
 t2 **50000** (50 msec tr)  
 pismode **1** (exec service)  
 pmode **1** (data collection)  
 daqm **1** (data collection)

**[Accept]**

**[Research Operations] [Setup Params]**

R1 **7**  
 R2 **14**  
 Number of Frames **[2]**  
 Window 1  
 Frame **1** + Frame **0**

**[Done]**

**[Manual Prescan]**

Windows (From Menu Bar) **2**  
 Window Type **I Channel**  
**[Done]**

**o.19.1 LVShim (Localizer)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u> .	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	LV shim
Weight (Lb))	300
	[Landmark]
Landmark	[>] [Sternal]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Body] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Coronal]
Mode	[>] [2D]
Pulse Seq	[...] [Spin Echo] [Accept]
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	[20]
TR	[300]
Bandwidth	[15.63] (1.0T only)
<u>SCANNING RANGE</u>	
FOV	[48]
Slice Thickness	[10]
Spacing	[1.5]
Start	0
End	0
# of Slices	1 (default)
S/I Center	0 (default)
R/L Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:23)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [S/I]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.19.2 LVShim Scan**

**SCAN OPTION                      INPUT**

PATIENT REGISTER .      **[New Pt]**

**PATIENT POSITION**

Patient Position                      **[>] [Supine]**

Patient Entry                              **[>] [Head First]**

Coil    **[...] [Body] [Accept]**

Series Description                      Note 1

**IMAGING PARAMETERS**

Plane    **[>] [Axial]**

Mode    **[>] [2D]**

Pulse Seq                                      **[...] [Spin Echo] [Accept]**

Imaging Options                      none (default)

Psd Name                                      **LVshim**

Protocol                                      no entry

**ADDITIONAL PARAMETERS [USER CVs]**

**USER CONTROL VARIABLES**

No. of scan planes      **[6]**                      [2.0 to 16.0]

Bandwidth in Hz      **[Note 2]**                      [100.0 to 20000.0]

**[Accept]**

(lowest window)

**[Save Series]**

**Note 1:** Refer to procedure for description name to use.

**Note 2:** Refer to procedure for proper Bandwidth to use.

**o.20.1 First Image**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>first image</b>
Weight (Lb))	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>[1]</b> (default)
TE	<b>[20]</b>
TR	<b>[400]</b>
Band Width	<b>[15.63]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[2]</b> (3:26)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.21.1 T2 Uniformity (Body, Axial)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body T2</b>
Weight (Lb))	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>[4]</b>
TE	<b>[30]</b>
TR	<b>[1000]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (2:20)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

### o.21.2 T2 Uniformity (Body, Sagittal)

RX MANAGER Place cursor in Series list area.  
Press right mouse button, then  
**[Copy Series], [Paste Series]**.  
Double-click left mouse button  
on new series to activate it.  
See Note

#### IMAGING PARAMETERS

Plane .[>] **[Sagittal]**

#### SCANNING RANGE

R/L Start **0**  
R/L End **0**  
P/A Center 0 (default)  
I/S Center 0 (default)

#### ACQUISITION TIMING

Freq Dir **[>] [S/I]**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

### o.21.3 T2 Uniformity (Body/Head, Coronal)

RX MANAGER Place cursor in Series list area.  
Press right mouse button, then  
**[Copy Series], [Paste Series]**.  
Double-click left mouse button  
on new series to activate it.  
See Note

#### IMAGING PARAMETERS

Plane **[Coronal]**

#### SCANNING RANGE

A/P Start see procedure  
A/P End see procedure  
S/I Center 0 (default)  
R/L Center 0 (default)

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.22.1 Isocenter Cal (DQA-III)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u> .	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	iso cal
Weight (Lb))	111
	[Landmark]
Exam Description	dqa iso cal (see Note 1)
	[Landmark]
Landmark	[>] [Nasion]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Head] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Axial]
Mode	[>] [2D]
Pulse Seq	[...] [Spin Echo] [Accept]
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	[25]
TR	[300]
Bandwidth	[15.63] (1.0T only)
<u>SCANNING RANGE</u>	
FOV	[24]
Slice Thickness	[3.0]
Spacing	[1.5]
S/I Start	0
S/I End	0
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[512]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:20)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                    **[Save Series]**

**Note 1:** The Exam Description name must be exact because it is used by the automated analysis tool (Daily Quality).

**o.23.1 Head Power Monitor**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>head pm</b>
Weight (Lb))	<b>300 ← IMPORTANT</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>cal</b>
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[200]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>0</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[2]</b> (0:52)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **Save Series]**

**o.23.2 Body Power Monitor**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body pm</b>
Weight (Lb))	<b>300 ← IMPORTANT</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>cal</b>
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[200]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>0</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	[256]
Phase	[128]
NEX	[2] [R/L]
Auto Center Freq	[>] [Peak]

(lowest window)                      **Save Series]**

**[Research Operations], [Display CVs]**

Modify the following:

dcset	<b>255</b>
t3	<b>20000</b>
	<b>[Accept]</b>

**o.23.3 Amp Shutdown Verification**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	pm check
Weight (Lb)	300 ← <i>IMPORTANT</i>
	[Landmark]
Landmark	[>] [Sternal Notch]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Body] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Axial]
Mode	[>] [2D]
Pulse Seq	[...] [Spin Echo] [Accept]
Imaging Options	none (default)
Psd Name	cal
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	[25]
TR	[200]
<u>SCANNING RANGE</u>	
FOV	[24]
Slice Thickness	[10]
Spacing	0
Start	0
End	0
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq. [256]  
 Phase [128]  
 NEX [2] (0:52)  
 Freq Dir [>] [R/L]  
 Auto Center Freq. [>] [Peak]

(lowest window) [Save Series]

[Research Operations], [Display CVs]

Modify the following:

dcset 255  
 aset 30  
 calmode 2  
 trig 1

For RF/PEN Cabinet only, modify the following additional CVs:

pwset 255  
 aset 120  
 p1 3100  
 p3 2400

[Accept]

**o.24.1 System Gain Cal (Body)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>bd sys gain</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b> for 1.5T <b>[21]</b> for 1.0T
TR	<b>[700]</b>
Bandwidth	(1.0T only) <b>15.63</b>
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (1:38)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq.	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.24.2 System Gain Cal (Head)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>hd sys gain</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[700]</b>
Bandwidth	<b>15.63</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (1:38)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq.	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.28.1 Full Field Distortion (Sagittal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>ffd</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[25]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[20]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
P/A Center	0 (default)
I/S Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:23)
Phase FOV	1:00 (default)
Freq Dir	<b>[&gt;] [S/I]</b>
Auto Center Freq.	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.28.2 Full Field Distortion (Coronal Localizer)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	<b>[New Series]</b> (If this procedure is Being performed immediately after The Sagittal series, otherwise click <b>[New Pt]</b> )
 <u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
 <u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
 <u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[20]</b>
TR.	<b>[150]</b>
Bandwidth	<b>15.63</b> (1.0T only)
 <u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[20]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
R/L Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (0:24)
Phase FOV	1:00 (default)
Freq Dir	<b>[&gt;]</b> <b>[R/L]</b>
Auto Center Freq.	<b>[&gt;]</b> <b>[Peak]</b>

(lowest window)            **[Save Series]**

RX MANAGER                **[Prepare to Scan]**

**o.28.3 Full Field Distortion (Coronal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u>	<b>[New Series]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Coronal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[25]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[20]</b>
Spacing	<b>[1.5]</b>
Start	See procedure
End	See procedure
# of Slices	1 (default)
S/I Center	0 (default)
R/L Center	0 (default)
Table Delta	0.00 (default)
<u>ACQUISITION TIMING</u>	
Freq.	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:23)
Phase FOV	1:00 (default)
Freq Dir	<b>[&gt;] [S/I]</b>
Auto Center Freq.	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**o.29.1 Noise Floor (Head)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>noise floor</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential]</b>
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[9]</b>
TR.	<b>[20]</b>
Flip Angle	<b>[10]</b>
Bandwidth	<b>15.63</b> (1.0T) <b>15.00</b> (1.5T)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (0:05)
Phase FOV	1:00 (default)
Freq Dir	<b>[&gt;]</b> <b>[A/P]</b>
Auto Center Freq.	<b>[&gt;]</b> <b>[Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.29.2 Noise Floor (Body)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER .</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series].</b> Double-click left mouse button On new series to activate it  See Note
<u>PATIENT POSITION</u>	
Coil	<b>[...] [Body] [Accept]</b>
<u>(lowest window</u>	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.30.1 Carrier Leakage**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>carrier leak</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential]</b>
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[16]</b>
TR.	<b>[35]</b>
Flip Angle	<b>[30]</b>
Bandwidth	<b>15.63</b> (1.0T)
	<b>15.00</b> (1.5T)
<u>ADDITIONAL PARAMETERS [SAT]</u>	
Saturation Parameters	
A Thickness	<b>[80]</b> (default Band)
P Thickness	<b>[80]</b> (default Band)

SCANNING RANGE

FOV	<b>[40]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[2.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq.	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (0:05)
Phase FOV	1:00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq.	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

### o.31.1 Respiratory Compensation

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	resp bellows
Weight (Lb)	111
	[Landmark]
Landmark	[>] [Sternal Notch]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Body] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Sagittal]
Mode	[>] [2D]
Pulse Seq	[...] [Spin Echo] [Accept]
Imaging Options	[...] [Respiratory Compensation] [Accept]
Psd Name	none
Protocol	no entry

It is not necessary to enter the remainder of scan protocol for this procedure.

**o.32.1 Gradcal-X (DQA-III)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>gradcal</b>
Weight (Lb)	<b>111</b>
Extra Description	<b>dqa xfield cal (See Note 1)</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Coronal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[25]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63 (1.0T only)</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[512]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:20)
Phase FOV	1 (default)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

**Note 1:** The Exam Description name must be exact because it is used by the automated analysis tool (Daily Quality).

**o.32.2 Gradcal-Y (DQA-III)**

<b><u>SCAN OPTION</u></b>	<b><u>INPUT</u></b>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>gradcal</b>
Weight (Lb)	<b>111</b>
Exam Description	<b>dqa yfield cal (See Note 1)</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[25]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63 (1.0T only)</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[512]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:20)
Phase FOV	1 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

**Note 1:** The Exam Description name must be exact because it is used by the automated analysis tool (Daily Quality).

**o.32.3 Gradcal-Z (DQA-III)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>gradcal</b>
Weight (Lb)	<b>111</b>
Exam Description	<b>dqa zfield cal (See Note 1)</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Coronal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[25]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63 (1.0T only)</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[512]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (1:20)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [S/I]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

**Note 1:** The Exam Description name must be exact because it is used by the automated analysis tool (Daily Quality).

**o.36.1 Correlated Noise (256 matrix)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>corr noise</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential]</b>
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[10]</b>
TR	<b>[20]</b> (1.0T) <b>[25]</b> (1.5T)
Flip Angle	<b>[10]</b>
Bandwidth	<b>[&gt;] 15.63</b> (1.0T) <b>[&gt;] 16.00</b> (1.5T)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (0:05)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>

(lowest window)                      **[Save Series]**

<b>[Research Operations]</b>	<b>[Setup Params]</b>
R1	<b>11</b>
R2	<b>15</b>
TG	<b>0</b>
Number of Frames	<b>[2]</b>
	<b>[Done]</b>

**o.36.2 Correlated Noise (512 matrix)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER .</u>	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>SCAN TIMING</u>	
Bandwidth	<b>[&gt;] 31.25 (1.0T)</b> <b>[&gt;] 32.00 (1.5T)</b>
<u>SCANNING RANGE</u>	
FOV	<b>[20]</b>
<u>ACQUISITION TIMING</u>	
Freq	<b>[512]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series shown. All other values should be the same as the previous series.

**o.36.3 Multicoil Correlated Noise**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>mc corr noise</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Phased Array] [CTLMID] or [Pelvic] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Gradient Echo] [Accept]</b>
Imaging Options	<b>[...] [Sequential]</b>
Psd Name	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[10]</b>
TR	<b>[31]</b> (1.0T) <b>[25]</b> (1.5T)
Flip Angle	<b>[10]</b>
Bandwidth	<b>[&gt;] 31.25</b> (1.0T) <b>[&gt;] 32.00</b> (1.5T)
<u>SCANNING RANGE</u>	
FOV	<b>[20]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq [512]  
 Phase [256]  
 NEX [1] (0:06)  
 Phase FOV 1.00 (default)  
 Freq Dir [>] [A/P]  
 Auto Center Freq [>] [Peak]

(lowest window) [Save Series]

[Research Operations] [Setup Params]

R1 11  
 R2 15  
 TG 0  
 Number of Frames [2]

[Done]

[Research Operations] [Display CVs]

CV Name: saveinter <Enter>

Set value to: 1 <Enter> (Note 1)

[Accept]

**Note 1:** Setting this CV to 1 generates five images for each single multi-coil scan. Images 1 to 4 will correspond to data from Receivers 0 to 3. Therefore, Image 1 is reconstructed from Receiver 0 data. Image 5 is the combined and processed data from all 4 Receivers. This is the same image normally observed when saveinter = 0.



ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b> (0:02)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
Autoshim	<b>no</b> (Verify)
Phase Correct	<b>no</b> (Verify)
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.37.1 Body Slice Thickness (Sagittal, 3mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body slice thick</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none
Psd Name	no entry
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[300]</b>
Bandwidth	<b>[7.81]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[20]</b>
Slice Thickness	<b>[3]</b>
Spacing	<b>[77]</b>
R/L Start	<b>[R80]</b>
R/L End	<b>[L80]</b>
# of Slices	3 (default)
P/A Center	0 (default)
I/S Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[4]</b> (1.0T)
	<b>[2]</b> (1.5T)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [A/P]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.37.2 Body Slice Thickness (Sagittal, 5mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it  See Note 1
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[5]</b>
Spacing	<b>[75]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANGER</u>	<b>[Prepare to Scan]</b>

**Note:** 1. Only values different from previous series are shown. All other values  
Should be the same as the previous series.

**o.37.3 Body Slice Thickness (Sagittal, 10mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button On new series to activate it  See Note 1
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[10]</b>
Spacing	<b>[70]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>
<u>RX MANGER</u>	<b>[Prepare to Scan]</b>

**Note:** 1. Only values different from previous series are shown. All other values Should be the same as the previous series.

**o.37.4 Body Slice Thickness (Localizer)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note
<u>SCANNING RANGE</u>	
Slice Thickness	<b>[3]</b>
Spacing	<b>[1.5]</b>
Start	<b>0</b>
End	<b>0</b>
# of Slices	<b>1</b>
<u>AQUISTION TIMING</u>	
Freq	<b>[256]</b>
Phase	<b>[128]</b>
NEX	<b>[1]</b>
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [S/I]</b>
(lowest window)	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series are shown. All other values  
Should be the same as the previous series.

**o.37.5 Body Slice Thickness (Coronal, 3mm)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note.

IMAGING PARAMETERS

Plane **[>] [Coronal]**

SCANNING RANGE

Spacing **[77]**

ADDITIONAL PARAMETERS [Graphic Rx]

Position cursor at P= 0 mm

This should coincide with the center of the phantom.

**[Start]**

Position cursor 80 mm away (posterior) from list location.

**[End]**

See procedure for additional information.

AQUISTION TIMING

Freq **[256]**

Phase **[256]**

NEX **[4]** (1.0T)

**[2]** (1.5T)

Phase FOV 1.00 (default)

Freq Dir **[>] [S/I]**

Auto Center Freq **[>] {Peak}**

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note:** Only values different from previous series are shown. All other values  
Should be the same as the previous series.

**o.37.6 Body Slice Thickness (Coronal, 5mm)**

<b><u>SCAN OPTION</u></b>	<b><u>INPUT</u></b>
---------------------------	---------------------

<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note1.
---------------------	--

**ADDITIONAL PARAMETERS [Graphic Rx]**

Position cursor at P= 0 mm

This should coincide with the center of the phantom.

**[Start]**

Position cursor 80 mm away (posterior) from list location.

**[End]**

See procedure for additional information.

**SCANNING RANGE**

Slice Thickness	<b>[5]</b>
Spacing	<b>[75]</b>

(lowest window) **[Save Series]**

RX MANAGER **[Prepare to Scan]**

**Note 1:** Only values different from previous series are shown. All other values Should be the same as the previous series.

**o.37.7 Body Slice Thickness (Coronal, 10 mm)**

<b><u>SCAN OPTION</u></b>	<b><u>INPUT</u></b>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it See note 1.

**ADDITIONAL PARAMETERS [Graphic Rx]**

Position cursor at P= 0 mm  
This should coincide with the center of the phantom.  
**[Start]**  
Position cursor 80 mm away (posterior) from list location.  
**[End]**  
See procedure for additional information.

**SCANNING RANGE**

Slice Thickness	<b>[10]</b>
Spacing	<b>[70]</b>
(lowest window)	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note 1:** Only values different from previous series are shown. All other values  
Should be the same as the previous series.

**o.38.1 Geometry Verification (Coronal)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>gradient polarity</b> (See Note 1)
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Coronal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE.	<b>[20]</b>
TR.	<b>[300]</b>
Bandwidth	<b>15.63</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[1.5]</b>
A/P Start	<b>0</b>
A/P End	<b>0</b>
# of Slices	1 (default)
S/I Center	0 (default)
R/L Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	[256]
Phase	[256]
NEX	[1]
Phase FOV	1:00 (default)
Freq Dir	[>] [S/I]
Auto Center Freq	[>] [Peak]

(lowest window)                      **[Save Series]**

**Note 1:** The Exam Description name must be exact because it is used by the automated analysis tool (Daily Quality).

### o.38.2 Geometry Verification (Axial)

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>RX MANAGER</u> .	Place cursor in Series list area. Press right mouse button, then <b>[Copy Series], [Paste Series]</b> . Double-click left mouse button on new series to activate it. See Note 1.
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
<u>SCANNING RANGE</u>	
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
<u>ACQUISITION TIMING</u>	
Freq Dir	<b>[&gt;] [R/L]</b>
(lowest window)	<b>[Save Series]</b>
<u>RX MANAGER</u>	<b>[Prepare to Scan]</b>

**Note:** Only values different from previous series are shown. All other values should be the same as the previous series.

**o.39.1 FID Tuning**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>fid</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>4</b>
TE	<b>[20]</b>
TR	<b>[250]</b>
Band Width	<b>[15.63]</b> (1.0T only)
Band Width 2	<b>[15.63]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[40]</b>
Slice Thickness	<b>[15]</b>
Spacing	<b>[1.5]</b>
R/L Start	<b>0</b>
R/L End	<b>0</b>
# of Slices	1 (default)
P/A Center	0 (default)
I/S Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	[256]
Phase	[256]
NEX	[2]
Phase FOV	1.00 (default)
Freq Dir	[>] [S/I]
Auto Center Freq	[>] [Peak]
<u>(lowest window)</u>	[Save Series]

**o.40.1 Gradient Waveforms**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>grad</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Nasion]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>[4]</b>
TE	<b>[25]</b>
TR	<b>[1000]</b>
Bandwidth	<b>[15.63]</b> (1.0T only)
Bandwidth 2	<b>[15.63]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[48]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>[1.5]</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[256]</b>
Phase	<b>[256]</b>
NEX	<b>[1]</b> (4:28)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq	<b>[&gt;] [Peak]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.41.1 Body APB Cal**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>body apb check</b>
Weight (Lb)	<b>300 ← IMPORTANT</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>cal</b>
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[55]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>0</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq [256]  
 Phase [128]  
 NEX [2] (0:52)  
 Freq Dir [>] [A/P]  
 Auto Center Freq [>] [Peak]

(lowest window) [Save Series]

[Research Operations] [Display CVs]

Modify the following:

Calmode 5 (normal rf sinc normal)  
 1a\_rf1 32766 (sets 90° pulse full-scale)  
 1a\_rf2 0 (turns off 180° pulse)

[Accept]

[Research Operations] [Setup Params]

TG 50

[Done]

**o.41.2 Head APB Cal**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>head apb check</b>
Weight (Lb)	<b>300 ← IMPORTANT</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>cal</b>
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	1 (default)
TE	<b>[25]</b>
TR	<b>[55]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[24]</b>
Slice Thickness	<b>[5]</b>
Spacing	<b>0</b>
S/I Start	<b>0</b>
S/I End	<b>0</b>
# of Slices	1 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq [256]  
 Phase [128]  
 NEX [2] (0:52)  
 Freq Dir [>] [A/P]  
 Auto Center Freq [>] [Peak]

(lowest window) [Save Series]

[Research Operations] [Display CVs]

Modify the following:

calmode 5 (normal rf sinc normal)  
 1a\_rf1 32766 (sets 90° pulse full-scale)  
 1a\_rf2 0 (sets 180° pulse to small-scale)

[Accept]

[Research Operations] [Setup Params]

TG 50

[Done]

**o.42.1 Bandpass Asymmetry Correction Compensation (BACC)**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>bacc</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none (default)
Psd Name	<b>bat</b>
Protocol	no entry
<u>(lowest window)</u>	<b>[Save Series]</b>

**o.43.1 Center Frequency**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>center frequency</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Body] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Sagittal]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	none
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>[4]</b>
TR	<b>[250]</b>
Bandwidth	<b>[15.00]</b> (1.0T only)
Bandwidth2	<b>[15.00]</b> (1.0T only)
<u>SCANNING RANGE</u>	
FOV	<b>[40]</b>
Slice Thickness	<b>[15]</b>
Spacing	<b>[1.5]</b>
R/L Start	<b>0</b>
R/L End	<b>0</b>
# of Slices	1 (default)
P/A Center	0 (default)
I/S Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	[256]
Phase	[256]
NEX	[2]
Phase FOV	1 (default)
Freq Dir	[>] [S/I]
Auto Center Freq	[>] [Peak]

(lowest window)                      [Save Series]

**o.44.1 Head B0 Dither**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER .</u>	<b>[New Pt]</b>
<u>PATIENT INFORMATION</u>	
Patient Id	<b>geservice</b>
Patient Name	<b>Head B0 Dither</b>
Weight (Lb)	<b>111</b>
	<b>[Landmark]</b>
Landmark	<b>[&gt;] [Sternal Notch]</b>
<u>PATIENT PROTOCOLS</u>	<b>[Patient Position]</b>
<u>PATIENT POSITION</u>	
Patient Position	<b>[&gt;] [Supine]</b>
Patient Entry	<b>[&gt;] [Head First]</b>
Coil	<b>[...] [Head] [Accept]</b>
<u>IMAGING PARAMETERS</u>	
Plane	<b>[&gt;] [Axial]</b>
Mode	<b>[&gt;] [2D]</b>
Pulse Seq	<b>[...] [Spin Echo] [Accept]</b>
Imaging Options	<b>[...] [Echo Planar]</b>
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Echoes	<b>[16]</b>
TE	<b>[Minimum]</b>
TR	<b>[6000]</b>
Bandwidth	<b>[62.50]</b>
<u>SCANNING RANGE</u>	
FOV	<b>[30]</b>
Slice Thickness	<b>[10]</b>
Spacing	<b>[10]</b>
S/I Start	<b>[120]</b>
S/I End	<b>[S20]</b>
# of Slices	3 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	[128]
Phase	[64]
NEX	[1] (0:06)
Freq Dir	[>] [R/L]
Auto Center Freq	[>] [Water]
<u>(lowest window)</u>	[Save Series]

**o.45.1 Body B<sub>0</sub> Dither & EPI Group Delay**

<u>SCAN OPTION</u>	<u>INPUT</u>
<u>PATIENT REGISTER</u>	[New Pt]
<u>PATIENT INFORMATION</u>	
Patient Id	geservice
Patient Name	Body B0 Dither
Weight (Lb)	111
	[Landmark]
Landmark	[>] [Sternal Notch]
<u>PATIENT PROTOCOLS</u>	[Patient Position]
<u>PATIENT POSITION</u>	
Patient Position	[>] [Supine]
Patient Entry	[>] [Head First]
Coil	[...] [Body] [Accept]
<u>IMAGING PARAMETERS</u>	
Plane	[>] [Axial]
Mode	[>] [2D]
Pulse Seq	[...] [Gradient Echo] [Accept]
Imaging Options	[...] [Echo Planar]
Psd Name	none
Protocol	no entry
<u>SCAN TIMING</u>	
# of Shots	[16]
TE	[Minimum]
TR	[6000]
Bandwidth	[62.50]
<u>SCANNING RANGE</u>	
FOV	[30]
Slice Thickness	[10]
Spacing	[10]
S/I Start	[120]
S/I End	[S20]
# of Slices	3 (default)
L/R Center	0 (default)
P/A Center	0 (default)
Table Delta	0.00 (default)

ACQUISITION TIMING

Freq	<b>[128]</b>
Phase	<b>[64]</b>
NEX	<b>[1]</b> (0:06)
Phase FOV	1.00 (default)
Freq Dir	<b>[&gt;] [R/L]</b>
Auto Center Freq	<b>[&gt;] [Water]</b>
<u>(lowest window)</u>	<b>[Save Series]</b>



SCANNING RANGE

FOV	[24]
Voxel Thickness	[20]
S/I Start	[S10]
S/I End	[I10]
R/L Start	[R10]
R/L End	[L10]
A/P Start	[A10]
A/P End	[P10]
Table Delta	0.00 (default)

ACQUISITION TIMING

NEX	[>] [2]
Freq Dir	[>] [A/P]
Auto Center Freq.	[>] [Water]

(lowest window)                      **[Save Series]**

**o.57.1 SNR - Probe P**

**SCAN OPTION                      INPUT**

PATIENT REGISTER            **[New Pt]**  
PATIENT INFORMATION  
 Patient Id                      **geservice**  
 Patient Name                  **probe p snr**  
 Weight (Lb)                   **111**  
                                      **[Landmark]**  
 Landmark                      **[>] [Nasion]**  
PATIENT PROTOCOLS        **[Patient Position]**

PATIENT POSITION  
 Patient Position              **[>] [Supine]**  
 Patient Entry                 **[>] [Head First]**  
 Coil                             **[...] [Head] [Accept]**

IMAGING PARAMETERS  
 Plane                          **[>] [Axial]**  
 Mode                           **[>] [MRS]**  
 Pulse Seq                     **[...] [Probe-P] [Accept]**  
 Imaging Options              **[...] [EDR] [Accept]**  
 Psd Name                      no entry  
 Protocol                        no entry

SCAN TIMING  
 # of Echos                    **1**  
 TE                              **[37]**  
 TR                              **[2000]**

ADDITIONAL PARAMETERS **[USER CVs]**

USER CONTROL VARIABLES

Scan Mode	<b>[1]</b>	[-1 to 1]
Total # of Scans	<b>[32]</b>	[1 to 4096]
AWS Optimization: 1=on	<b>[0]</b>	[0 or 1]
Spatial Sats: 1=on	<b>[0]</b>	[0 or 1]
	<b>[Accept]</b>	

SCANNING RANGE

FOV	[24]
Voxel Thickness	[20]
S/I Start	[S10]
S/I End	[I10]
R/L Start	[R10]
R/L End	[L10]
A/P Start	[A10]
A/P End	[P10]
Table Delta	0.00 (default)

ACQUISITION TIMING

NEX	[>] [2]
Freq Dir	[>] [A/P]
Auto Center Freq.	[>] [Water]

(lowest window)                    **[Save Series]**

## REVISION HISTORY

REV	DATE	AUTHOR	PRIMARY REASONS FOR CHANGE
0	Aug 30, 1998	L. Loehrer	Converted Toolbook to Word 7.0 document.
1	Nov 12, 1998	M. Keber	Updated protocol o.23.3 for additional CVs to modify for RF/PEN Cabinet.
2	Mar 3, 2000	G. Boerner	Updated all entries per 8.3 validation. Added "A" protocols 56 & 57. Deleted Protocol 46 which is Class "C".
3	May 19, 2000	M. Jones	Changed document file name to "PROTOCOLA.DOC" in header.