GE Healthcare

Optima MR360/Brivo MR355
Block Diagrams & Supplemental Schematics

Technical Publication

DAMAGE IN TRANSPORTATION

All packages should be closely examined at time of delivery. If damage is apparent, have notation “**damage in shipment**” written on **all** copies of the freight or express bill **before** delivery is accepted or “signed for” by a General Electric representative or a hospital receiving agent. Whether noted or concealed, damage **MUST** be reported to the carrier **immediately** upon discovery, or in any event, within **14** days after receipt, and the contents and containers held for inspection by the carrier. A transportation company will not pay a claim for damage if an inspection is not requested within this **14** day period.

File a report with

- Call 1–800–548–3366 and use option 8.
- Contact your local service coordinator for more information on this process.

Rev. 08/15/2003

© 2012, General Electric Company
GE Medical Systems, a General Electric Company, going to market as GE Healthcare
3200 N. Grandview Boulevard
Waukesha, Wisconsin 53188
USA

www.gehealthcare.com

WARNING

- THIS SERVICE MANUAL IS AVAILABLE IN ENGLISH ONLY.
- IF A CUSTOMER'S SERVICE PROVIDER REQUIRES A LANGUAGE OTHER THAN ENGLISH, IT IS THE CUSTOMER'S RESPONSIBILITY TO PROVIDE TRANSLATION SERVICES.
- DO NOT ATTEMPT TO SERVICE THE EQUIPMENT UNLESS THIS SERVICE MANUAL HAS BEEN CONSULTED AND IS UNDERSTOOD.
- FAILURE TO HEED THIS WARNING MAY RESULT IN INJURY TO THE SERVICE PROVIDER, OPERATOR OR PATIENT FROM ELECTRIC SHOCK, MECHANICAL OR OTHER HAZARDS.

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
REV 1

REVISION HISTORY

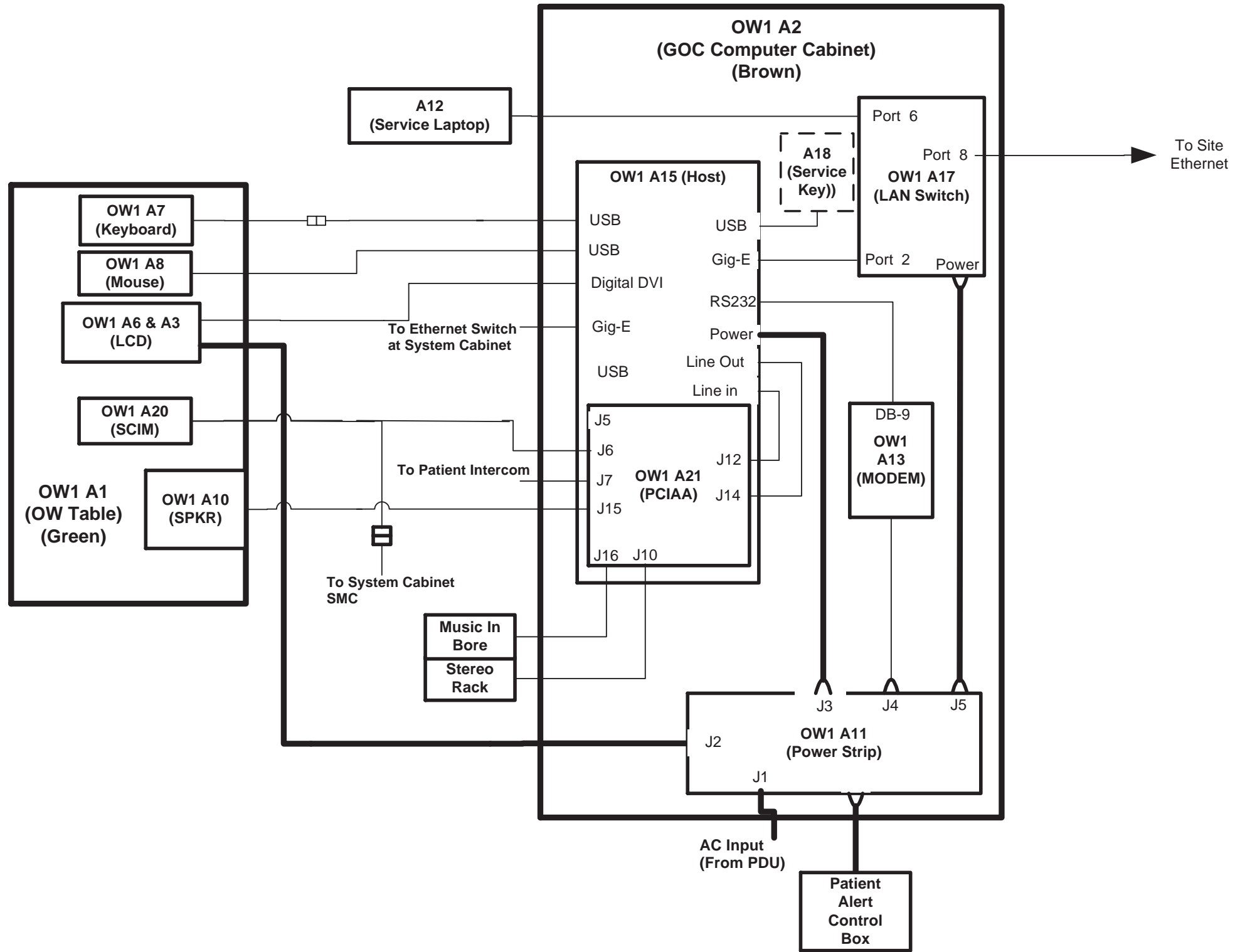REV	DATE	PRIMARY REASON FOR CHANGE
1.....	Aug 15, 2012	Initial Release



NOTES:

- 1 Network connection available for external suite connection
- 2 network connection available at front of goc assembly for the service laptop


Refer to the following functional block diagram for additional information:

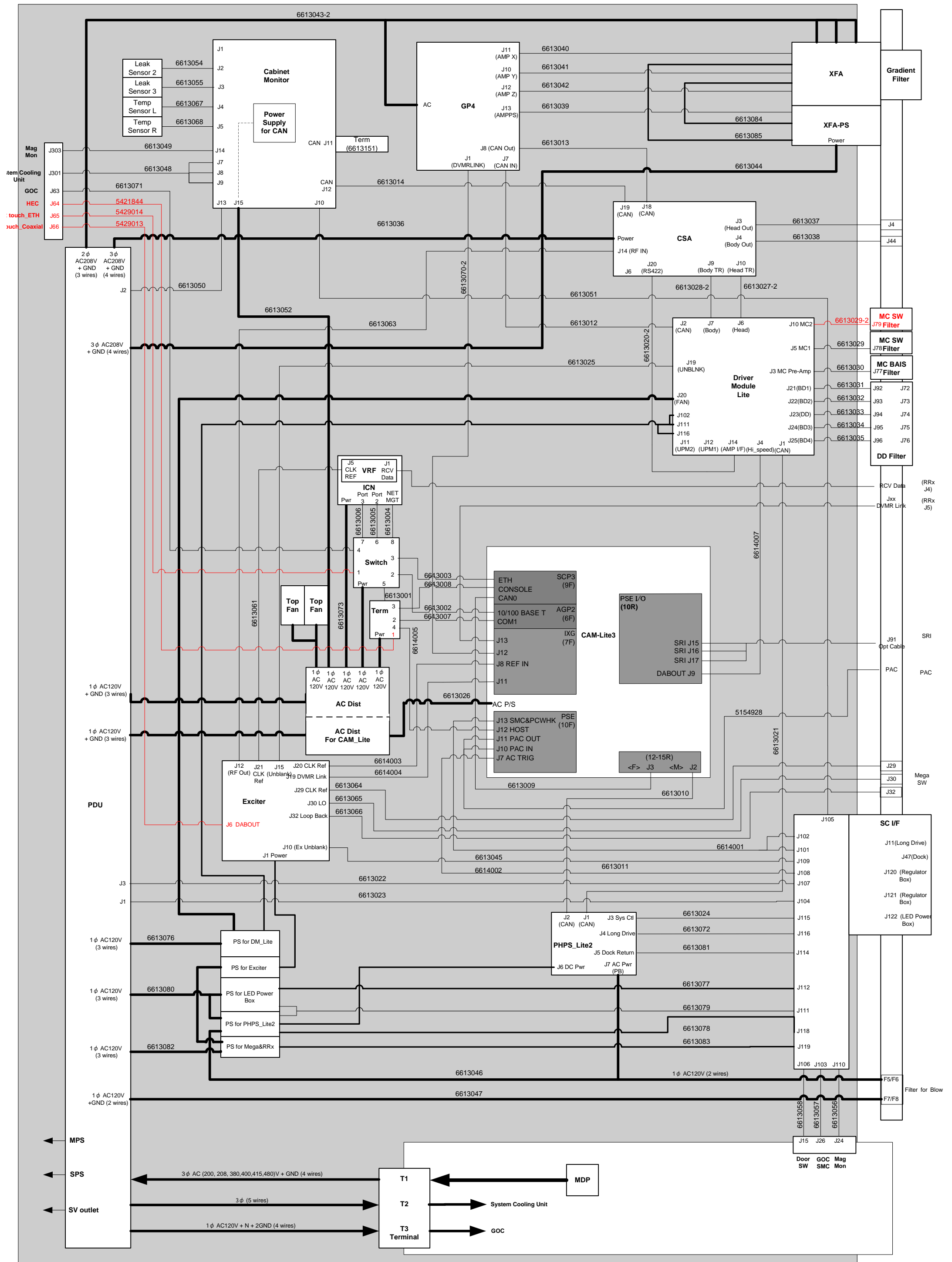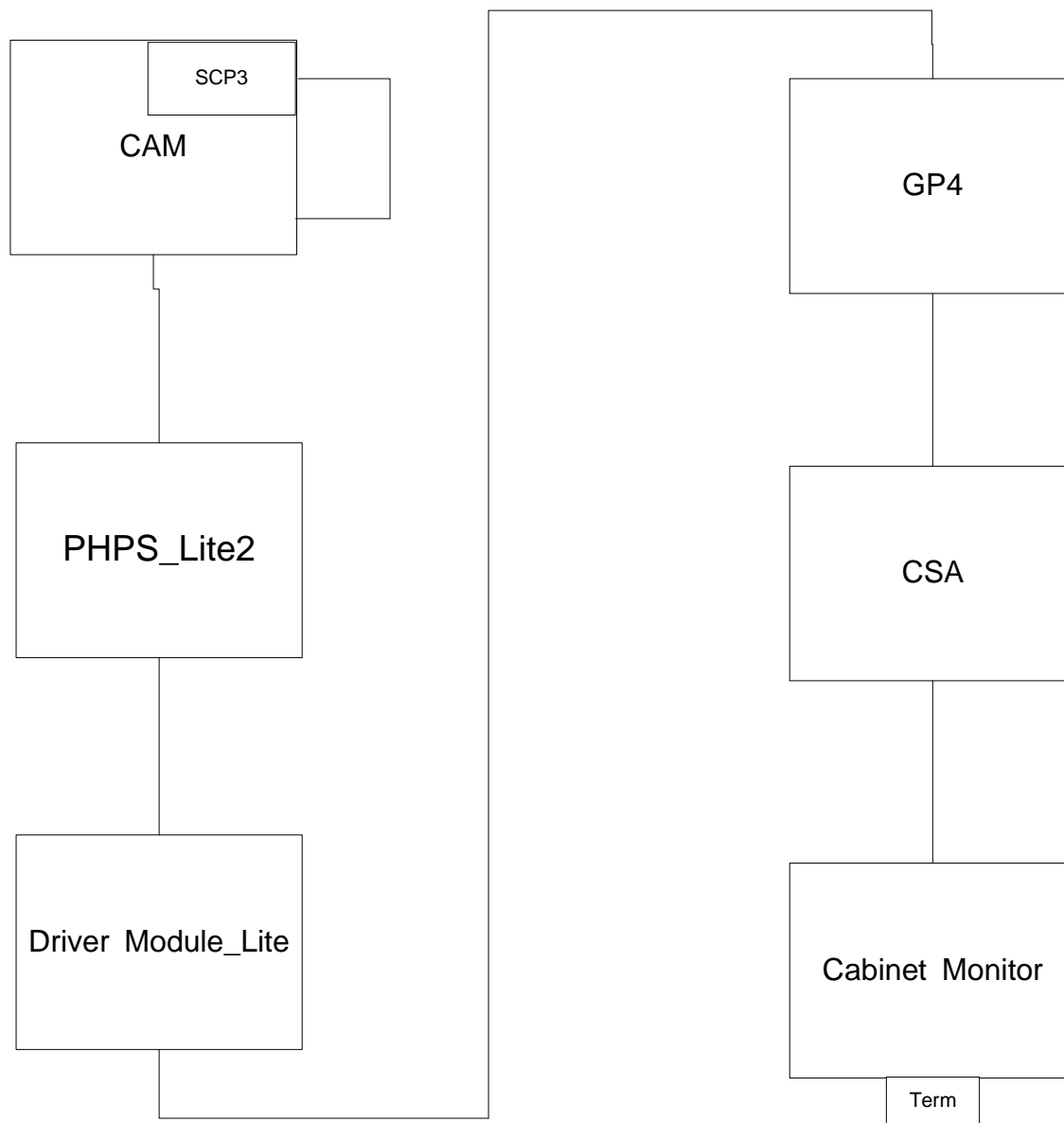
BAMBOO CABINET ASSY



CAN LOOP

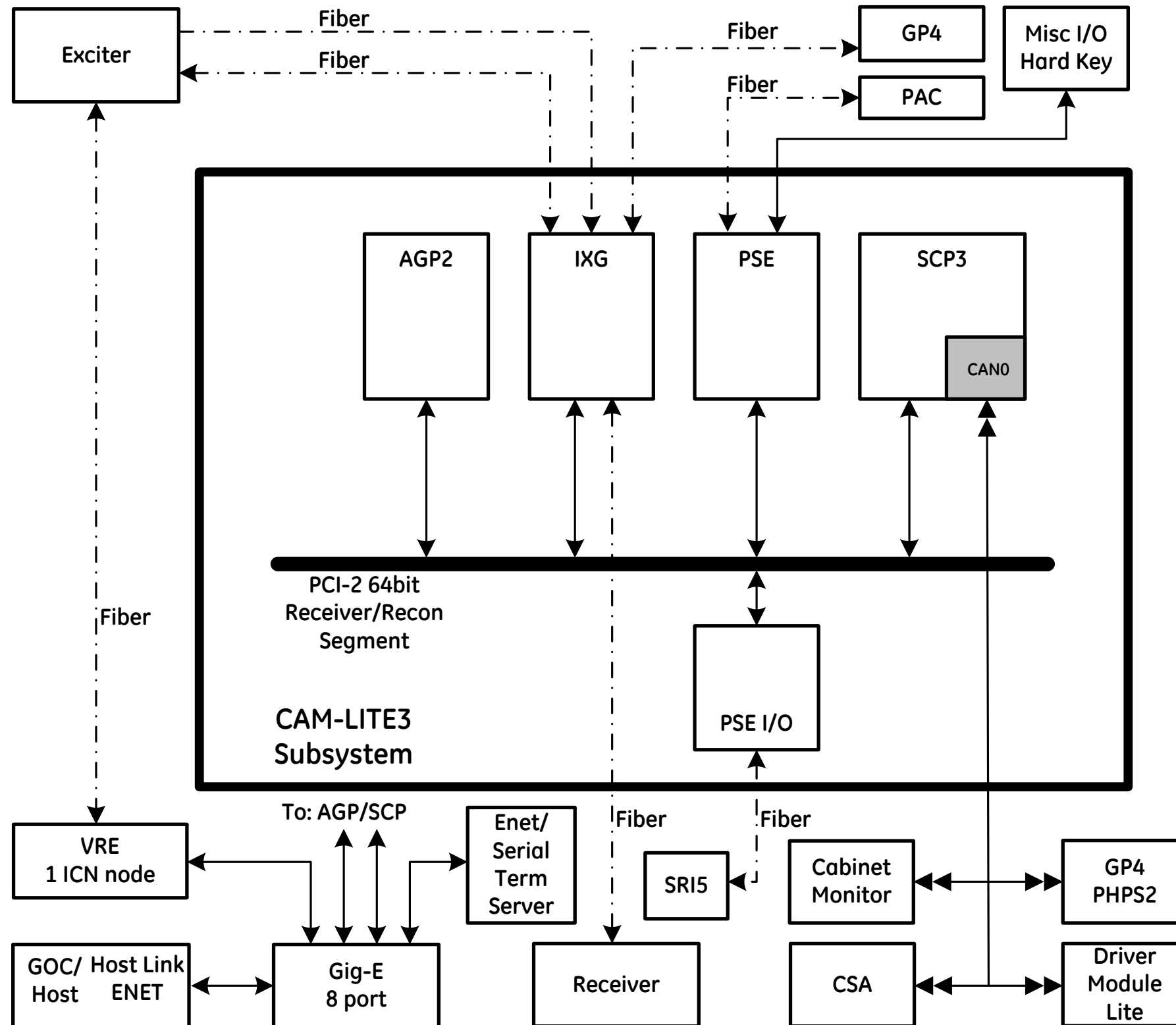
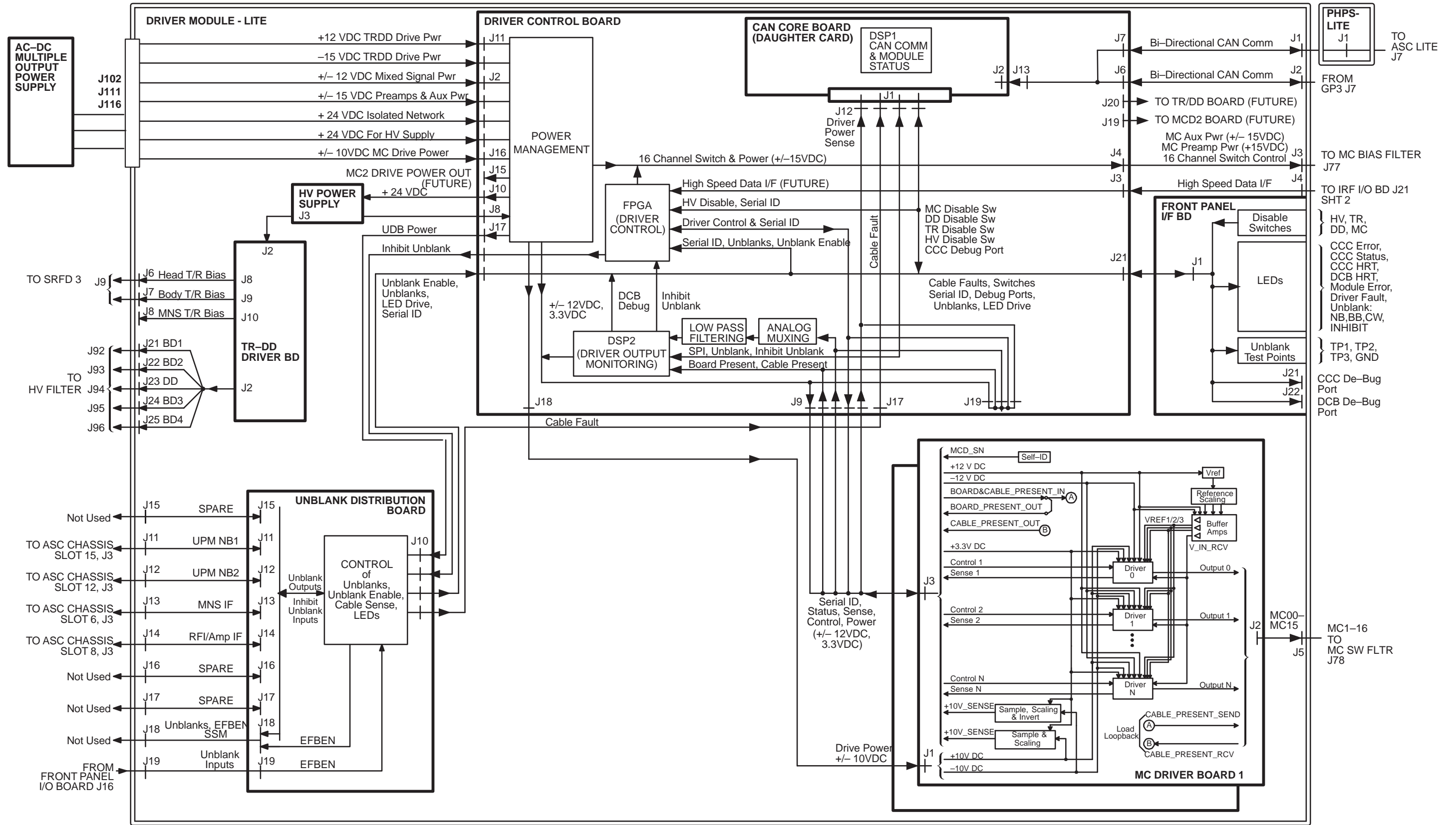
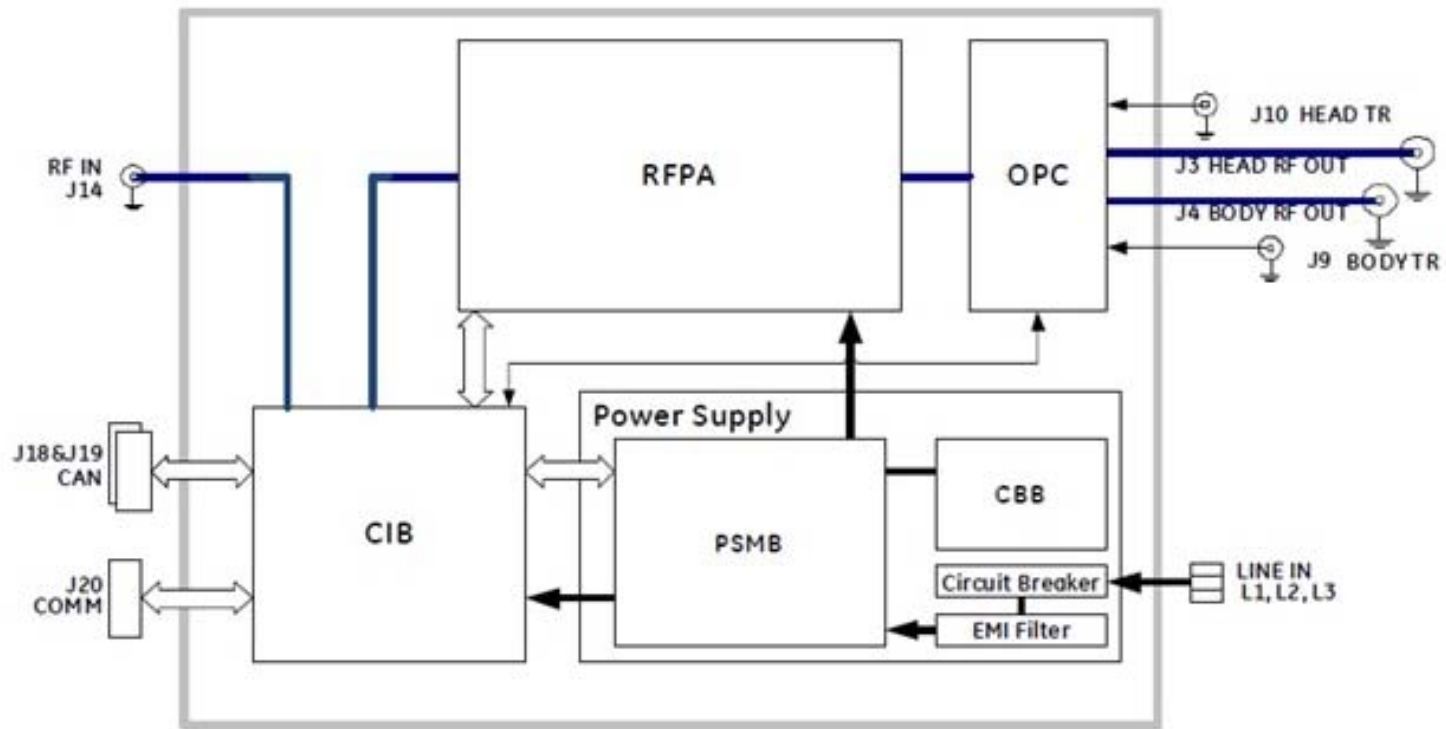
CSA(IRFA2)

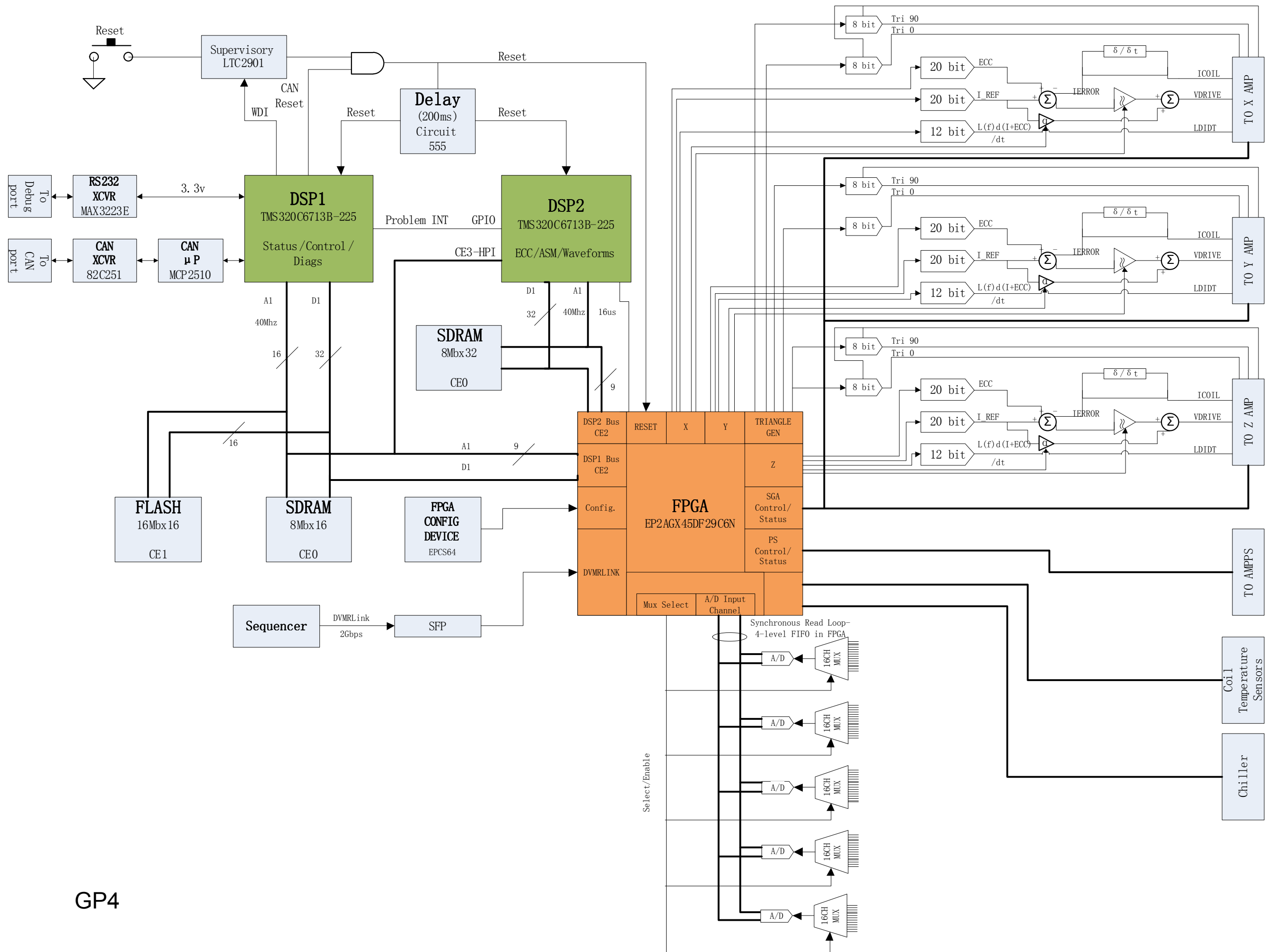
GP4

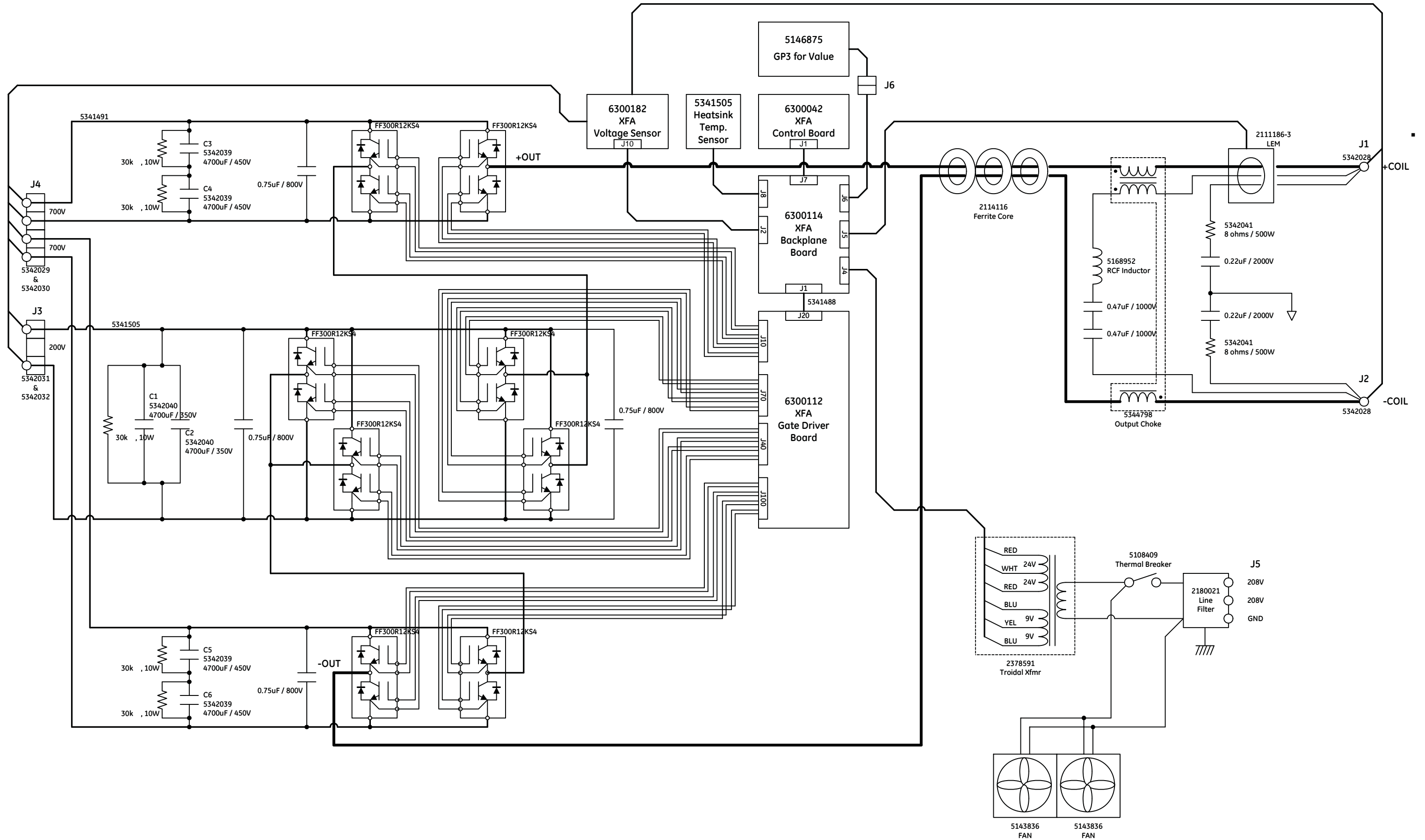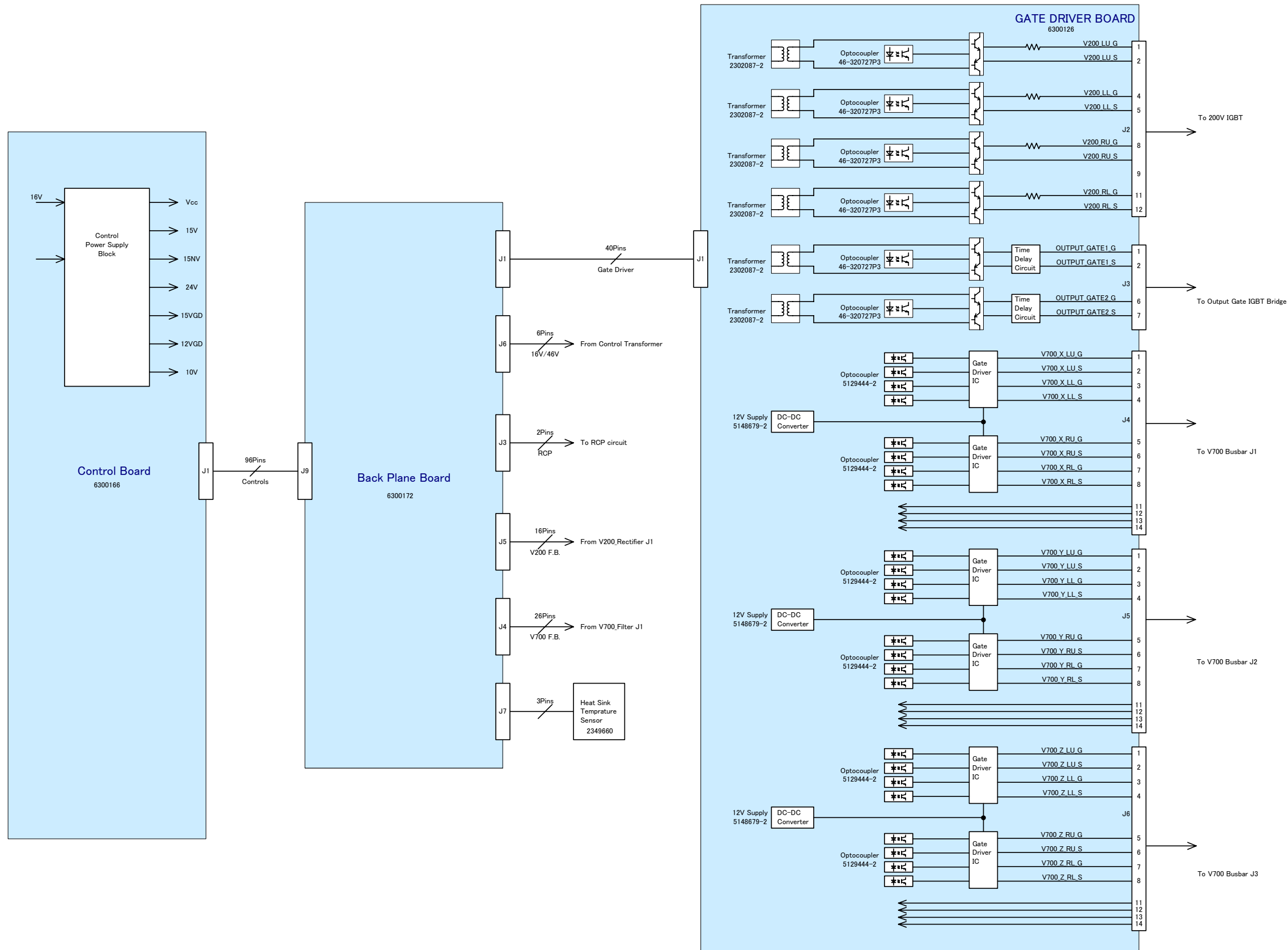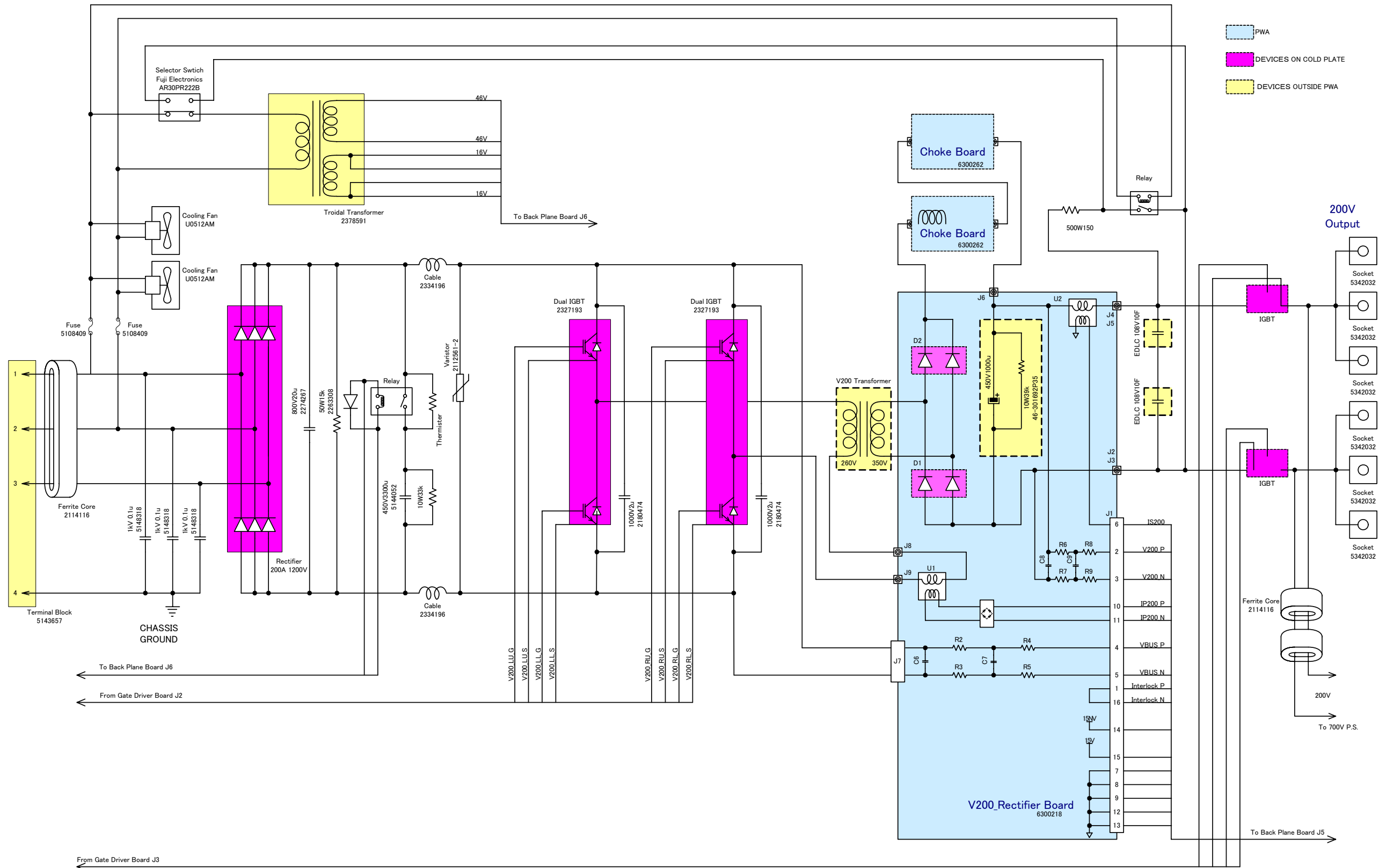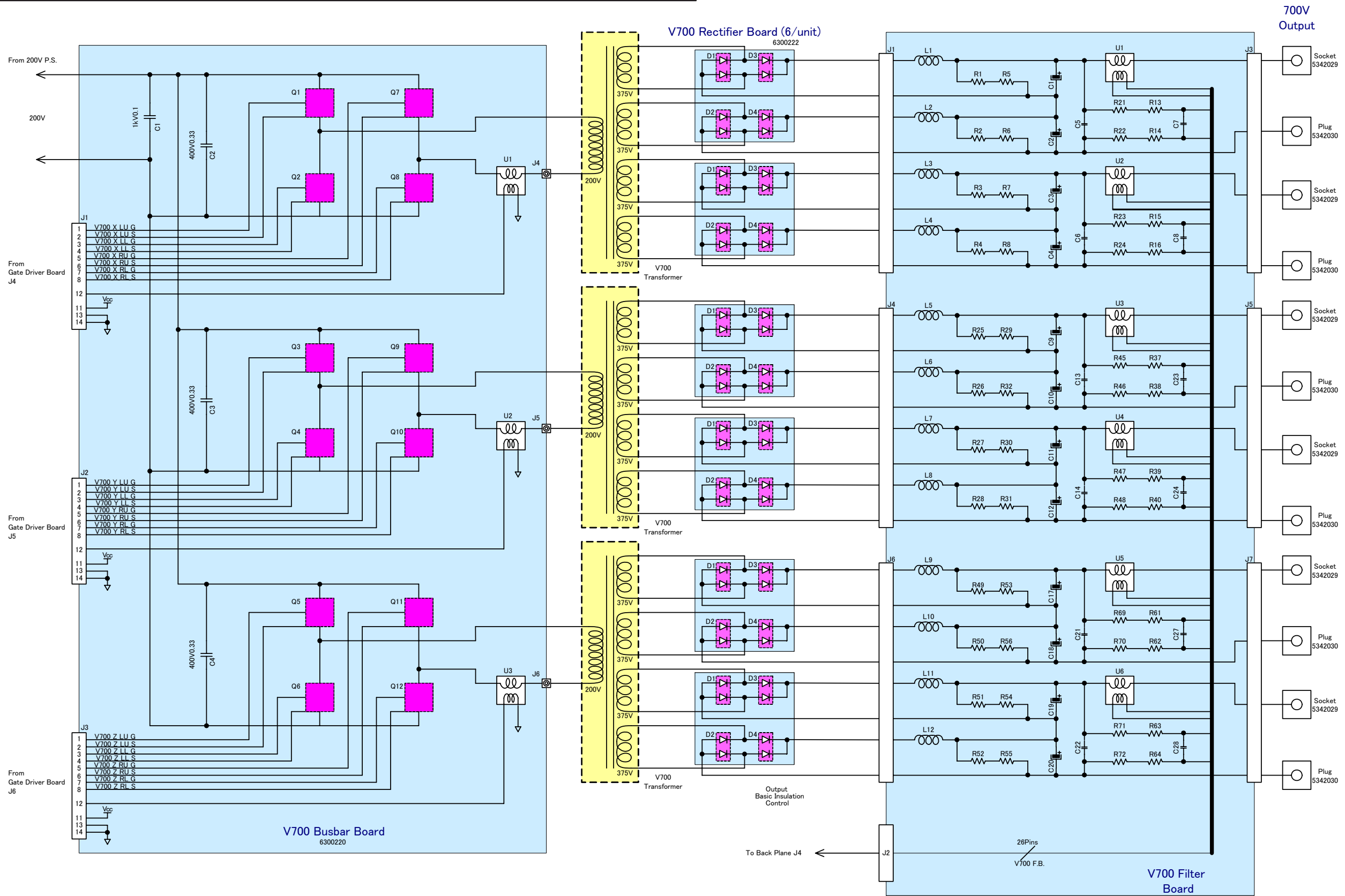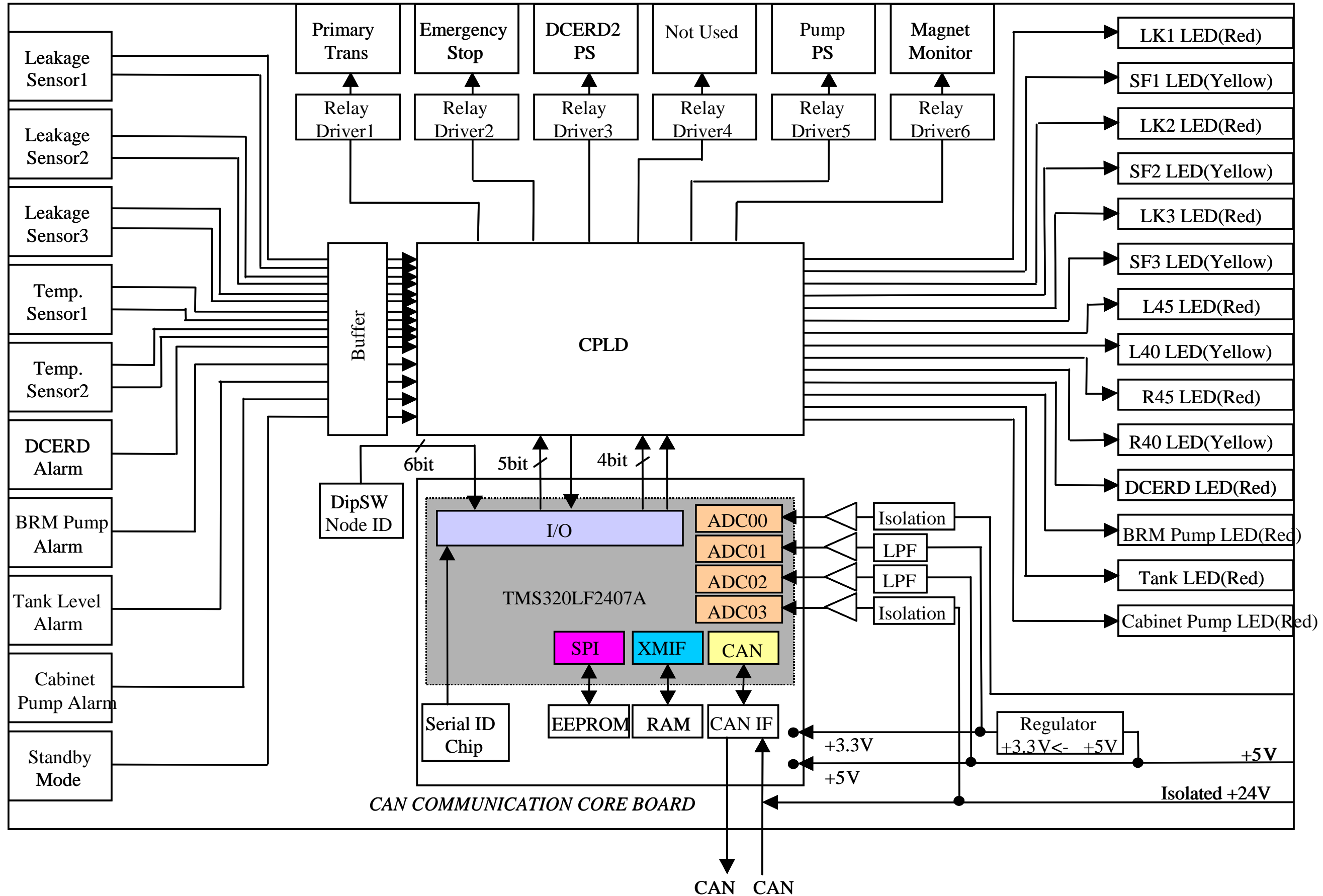
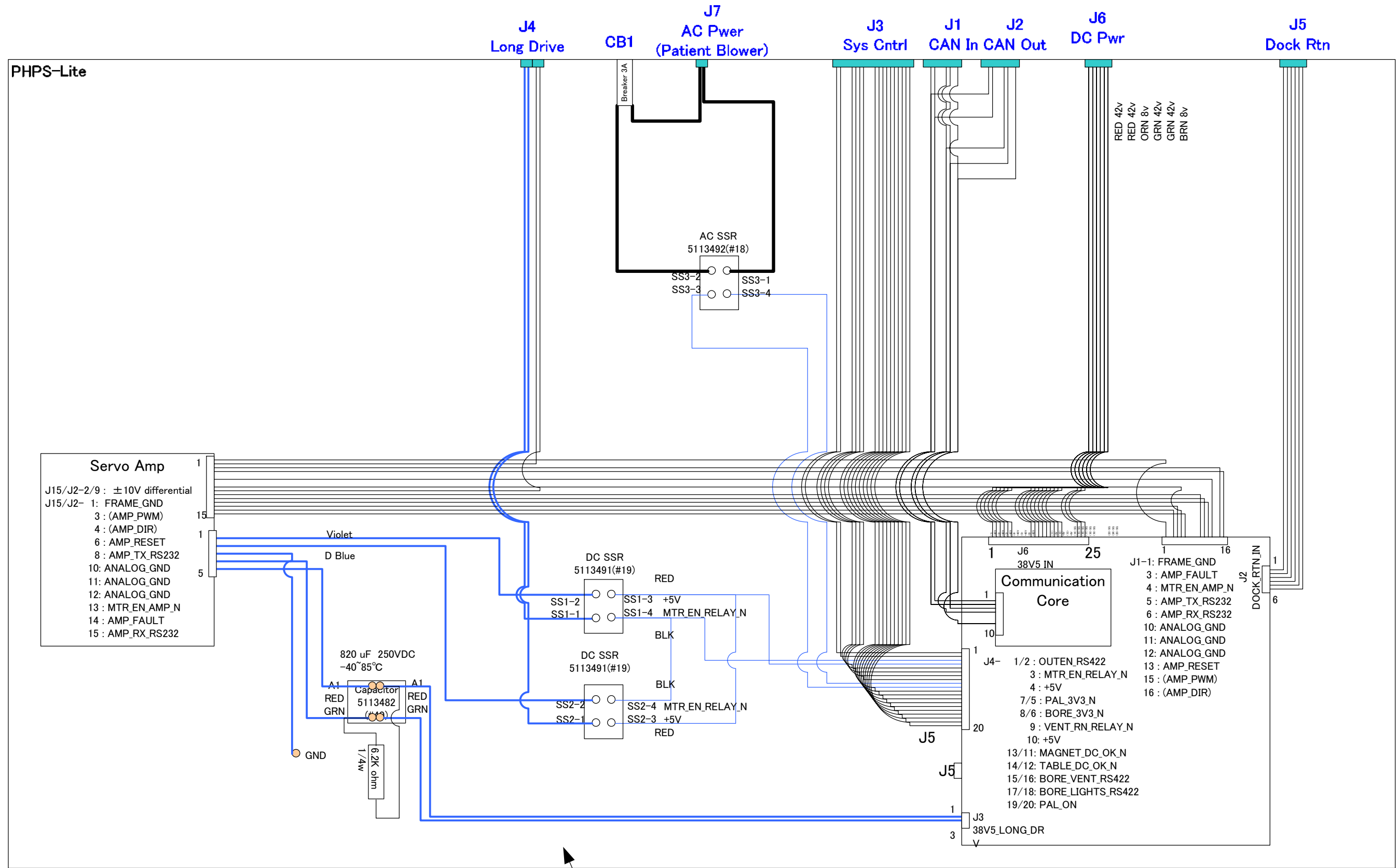
- PWA
- DEVICES ON COLD PLATE
- DEVICES OUTSIDE PWA



#25 5138536
 L=540

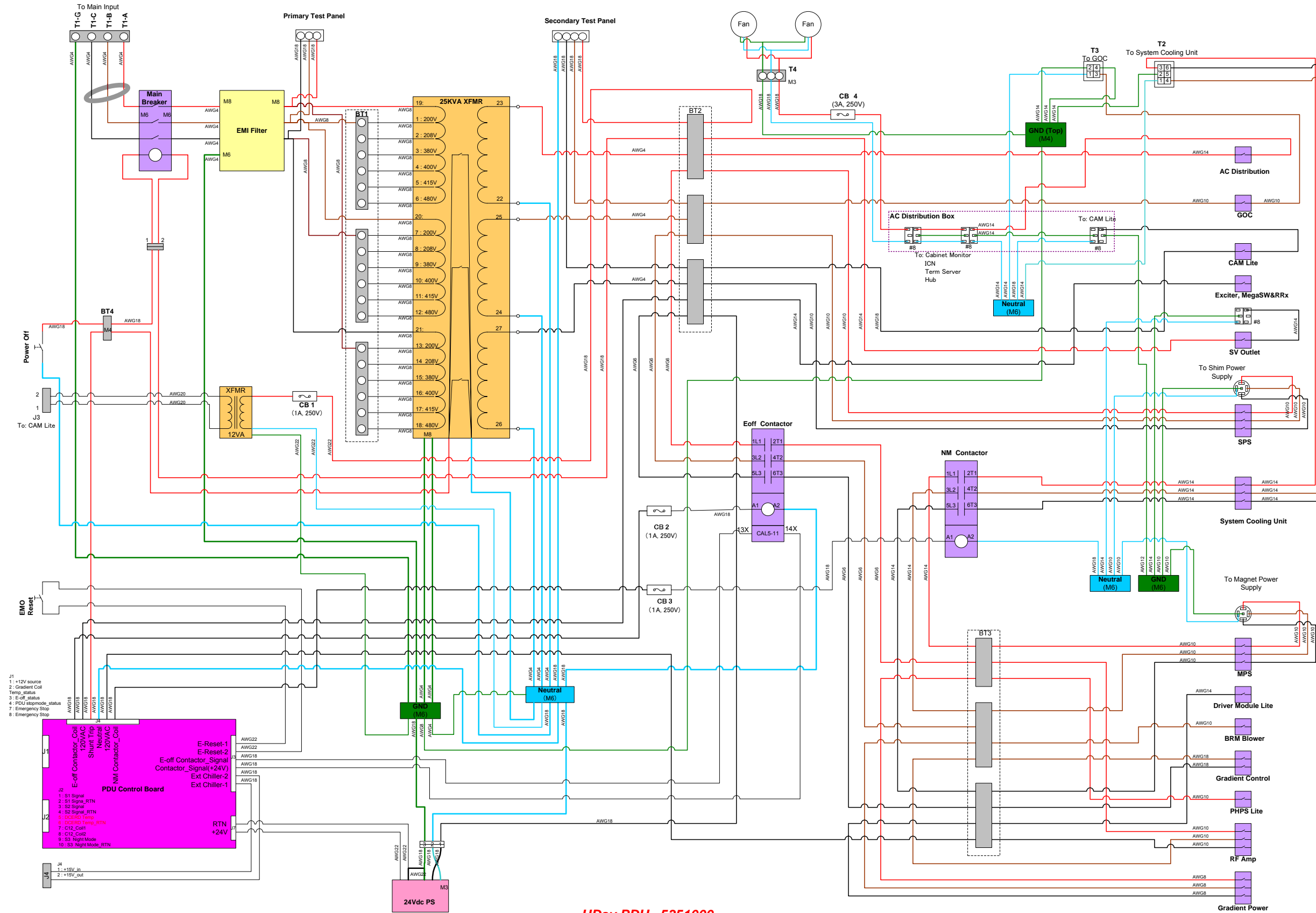Amp #350766-1
 MATE-N-LOK 3pin Plug
 Socket





HDsv PDU_ 5351000

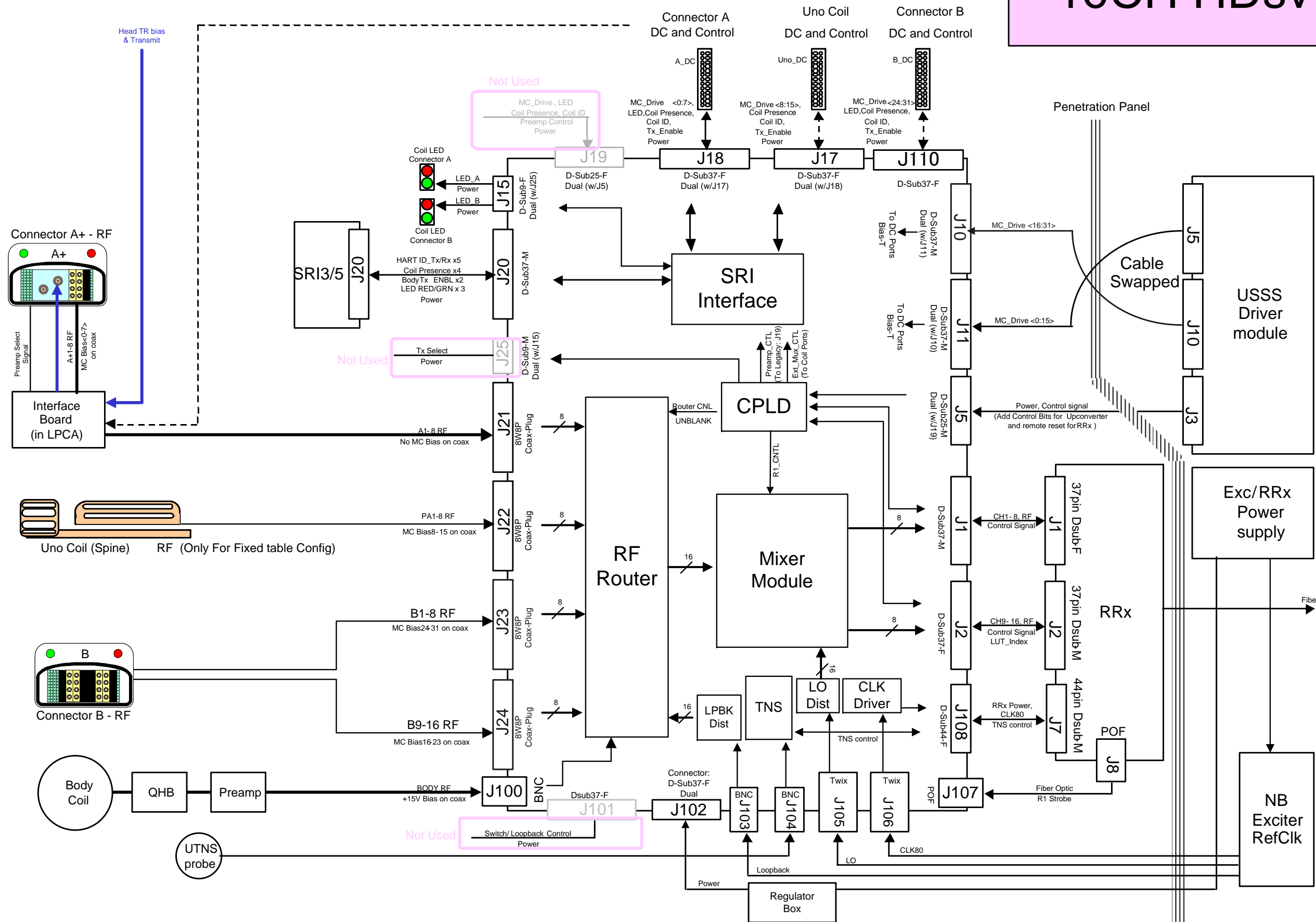16CH HDsv


8CH HDsv

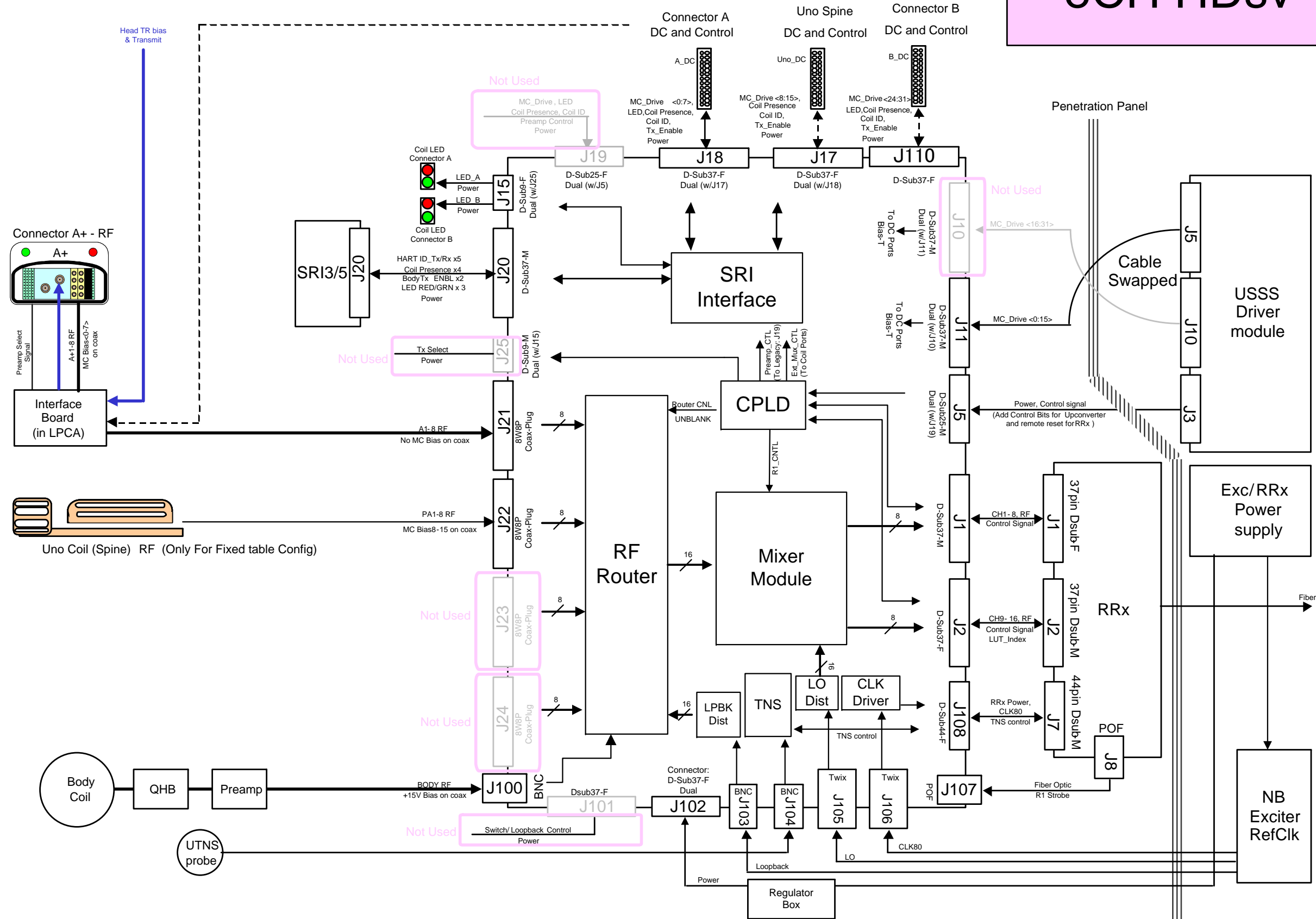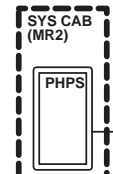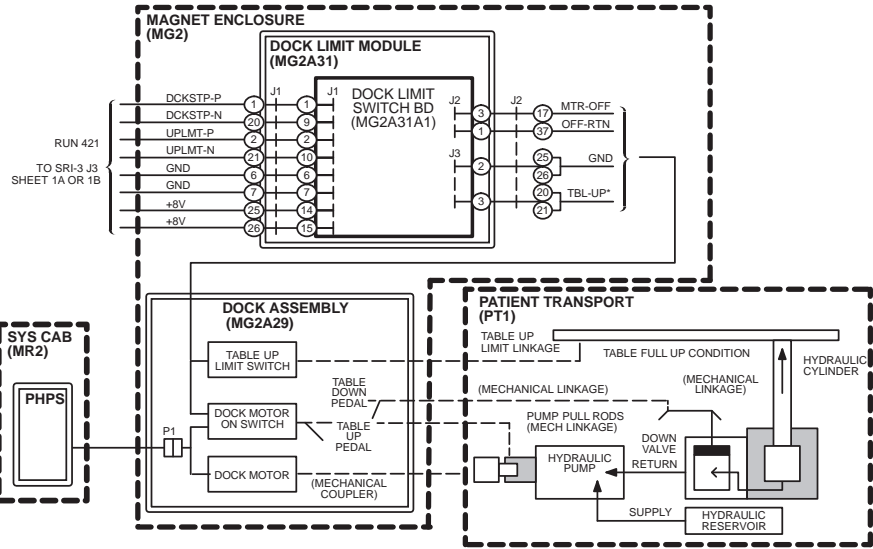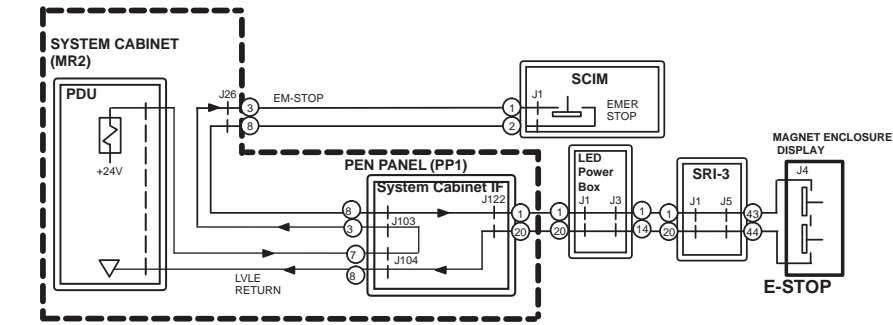
NOTES:

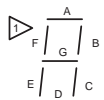Refer to the following functional block diagrams for additional information. :

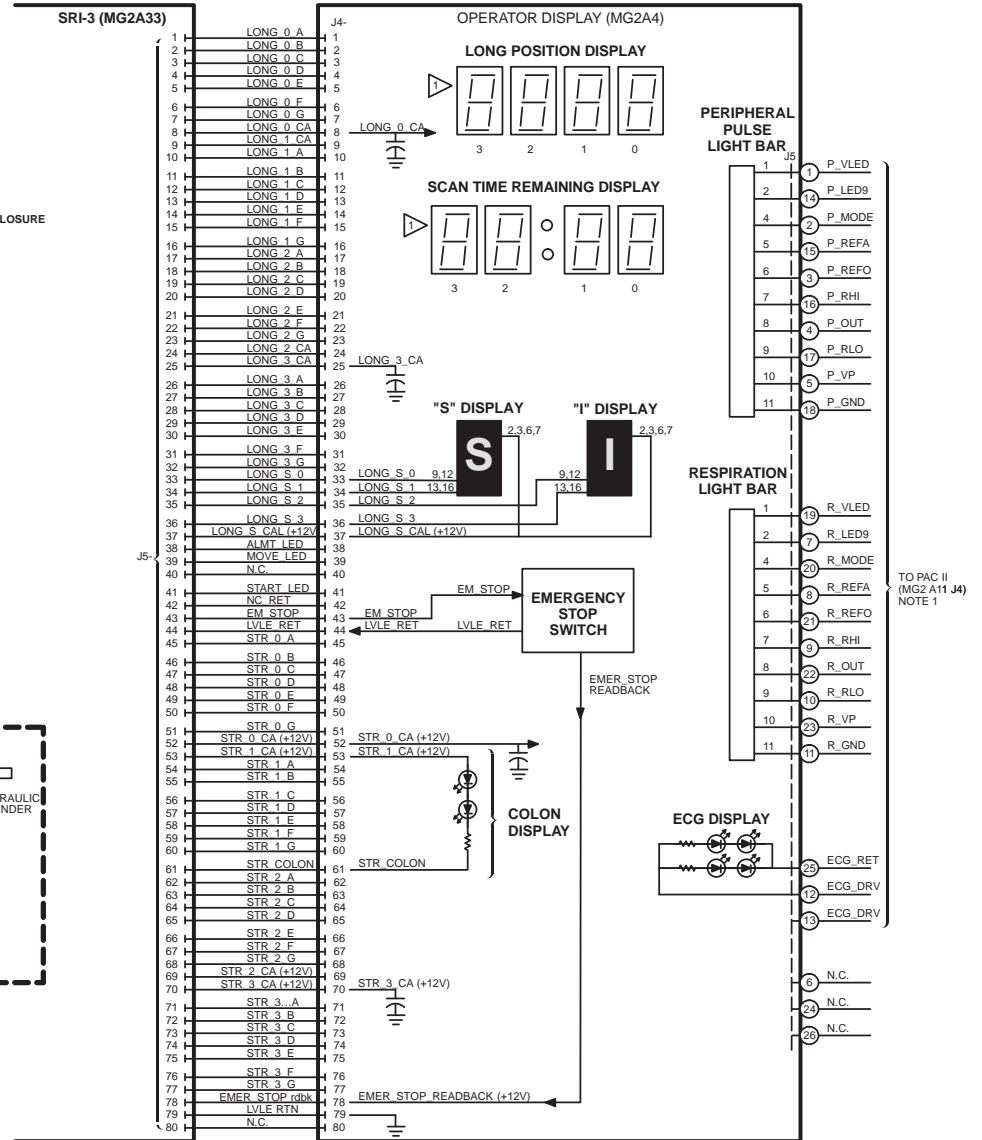
1) MGD/RF

NOTE:

Additional Signal/Cabling detail can be found under Interconnects on the Service Methods CDROM.
Refer to the following functional block diagrams for additional information.

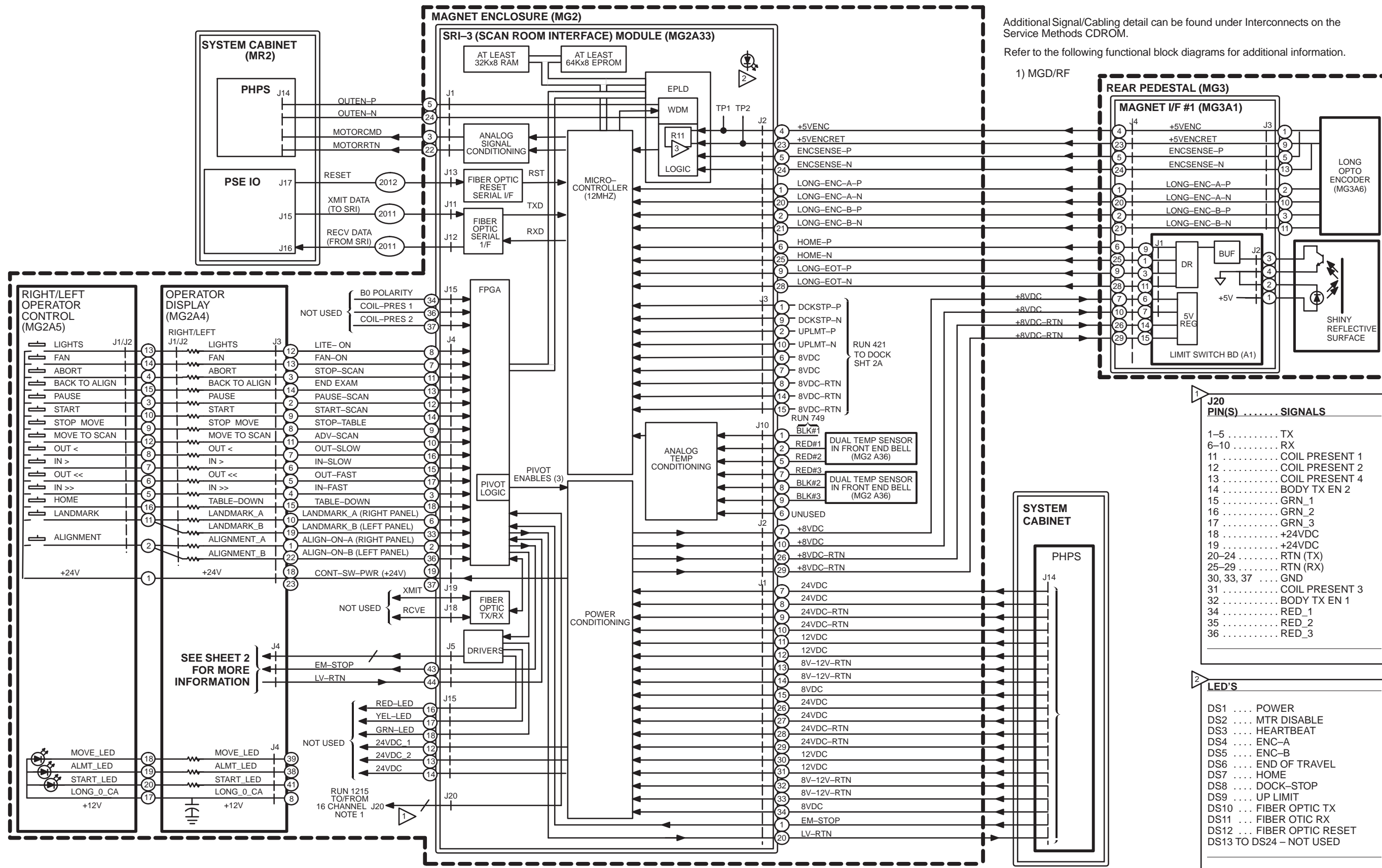1) MGD/RF


J20 PIN(S) SIGNALS

1-5	TX
6-10	RX
11	COIL PRESENT 1
12	COIL PRESENT 2
13	COIL PRESENT 4
14	BODY TX EN 2
15	GRN_1
16	GRN_2
17	GRN_3
18	+24VDC
19	+24VDC
20-24	RTN (TX)
25-29	RTN (RX)
30, 33, 37	GND
31	COIL PRESENT 3
32	BODY TX EN 1
34	RED_1
35	RED_2
36	RED_3

LED'S

DS1	POWER
DS2	MTR DISABLE
DS3	HEARTBEAT
DS4	ENC-A
DS5	ENC-B
DS6	END OF TRAVEL
DS7	HOME
DS8	DOCK-STOP
DS9	UP LIMIT
DS10	FIBER OPTIC TX
DS11	FIBER OPTIC RX
DS12	FIBER OPTIC RESET
DS13 TO DS24	NOT USED

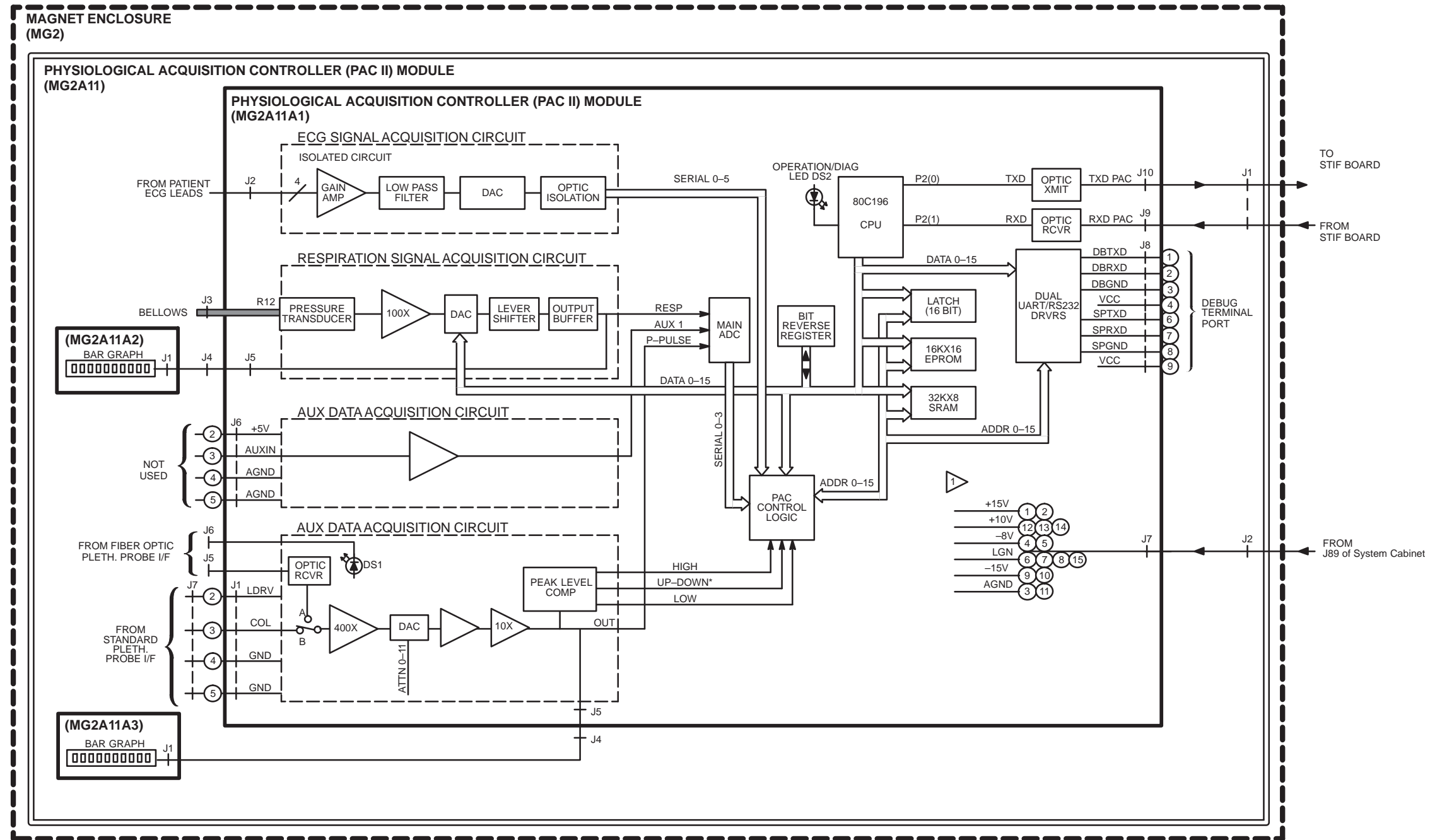DO NOT ADJUST R11

NOTES:

1 MULTIPLE WIRES/PINS WITH SAME SIGNAL ARE NOT SHOWN

REFER TO THE FOLLOWING FUNCTIONAL BLOCK DIAGRAM FOR ADDITIONAL INFORMATION:

NOTE:
1) PATIENT HANDLING

This page left intentionally blank.

©2012, General Electric Company.
GE Medical Systems, a General Electric Company, going to market as GE Healthcare.

[www. gehealthcare.com](http://www.gehealthcare.com)