

# RFA Loopback Setup for 3.0T with Dual-Drive RF Amplifier – Exciter

## Landmark:


One foot from head end of empty table.

## Note:

If monitoring Ch1, use cable going to J14 and disconnect and set aside cable going to J21.

If monitoring Ch2, use cable going to J21 and disconnect and set aside cable going to J14.

\* For DPP systems: Set attenuation to 20 dB.

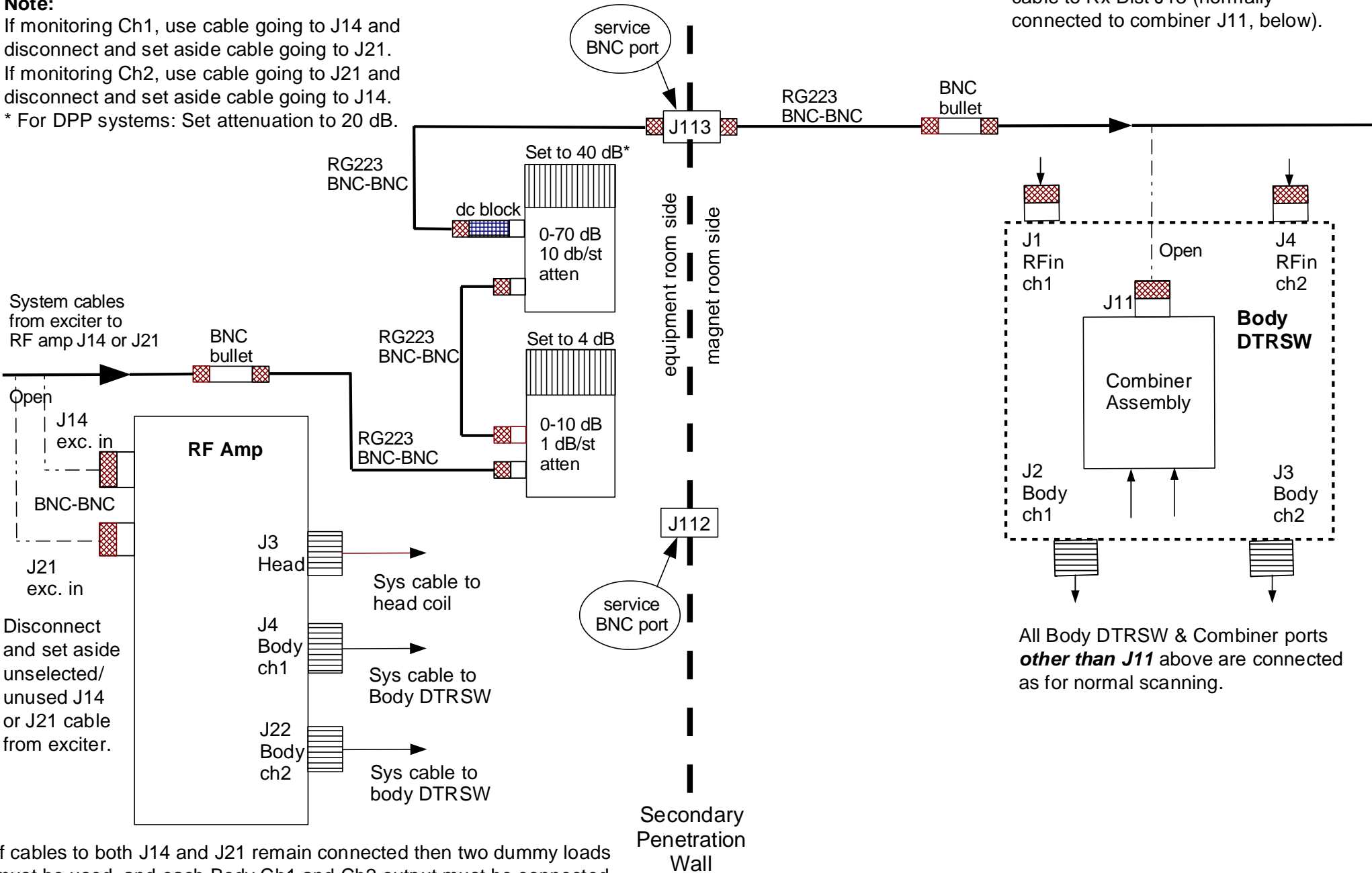

Connect the system cable to re-route box J7 or RFSBv1 B1 (normally connected to combiner J11, below).

For DPP systems: Connect the system cable to Rx Dist J13 (normally connected to combiner J11, below).